

TRANSFERRING YOUR OMNIPOD DASH™ SETTINGS

In order to set up your replacement Omnipod DASH™ Personal Diabetes Manager (PDM), you need your current insulin settings. To retrieve your insulin settings from your current Omnipod DASH™ PDM, **please follow steps 1-4 below.**

When you are ready to set up your replacement Omnipod DASH™ PDM please gather the following:


- + Omnipod DASH™ PDM currently in-use
- + Replacement Omnipod DASH™ PDM
- + A copy of your insulin settings. You can obtain your insulin settings by either:
 - Checking your current Omnipod DASH™ PDM
 - Contacting your healthcare provider
- + A new Pod
- + Insulin
- + Alcohol swabs

It is very important that you accurately enter your insulin pump settings. Entering the wrong settings could lead to serious harm.


Step 1 Write down your Blood Glucose Range from your current Omnipod DASH™ PDM

BG RANGE		
BG Goal Range:	Lower: _____ mmol/L	Upper: _____ mmol/L


Step 2 Write down your Basal Program Settings

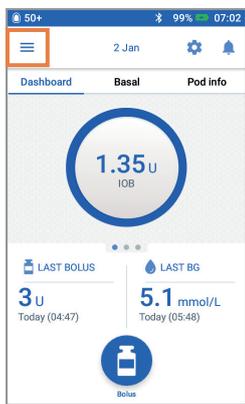


1. From the Home screen tap Menu icon

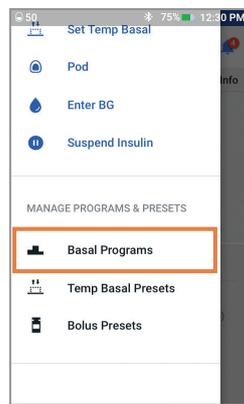

2. Tap "Basal Programs"

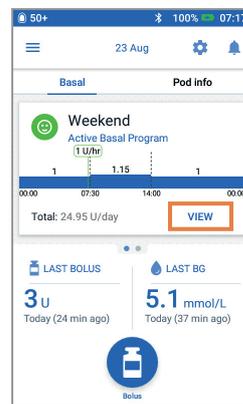

3. Tap "VIEW" to view the basal program. If you have an active Pod, you will need to tap "SUSPEND INSULIN" to view the active basal program. Review Program Name/Tag. Tap "NEXT"

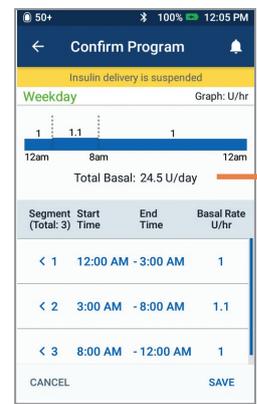

4. Review and write down Basal segments, rates and total basal amount found on this screen. Scroll down to include all segments for the entire 24 hour day. Tap "CANCEL". If you suspended insulin, you must tap "RESUME INSULIN" to restart your basal delivery.


BASAL SETTINGS		
Basal 1 Program (☰ >Settings>Basal Programs)	Time Segment	Basal Rate
Total Basal _____ U/day	00:00am - _____	_____ U/hr
	_____ - _____	_____ U/hr
	_____ - _____	_____ U/hr
	_____ - _____	_____ U/hr
Max Basal Rate (☰ >Settings>Basal & Temp Basal)	_____ U/hr	
Temporary Basal Rate (☰ >Settings>Basal & Temp Basal)	<input type="checkbox"/> ON <input type="checkbox"/> OFF	

Note: If you have additional Basal Programs, you can repeat the steps above for each program.


For your safety, when setting up your replacement Omnipod DASH™ PDM with Basal Programs double-check that the Total Basal amounts match in both Omnipod DASH™ PDMs.

Step 3 Write down your Bolus Settings


1. From the Home screen tap Menu icon

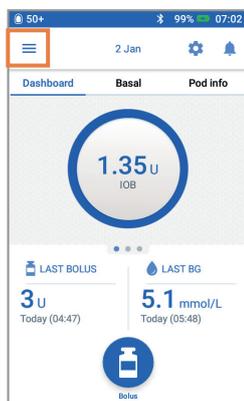

2. Tap "Settings" to expand list. Tap "Bolus"

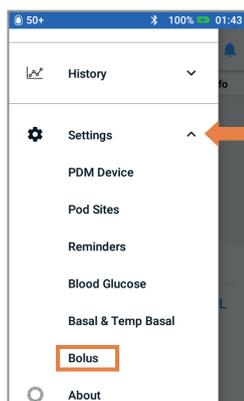

3. Tap on each Bolus setting. Write down all of the details for each of the settings listed on the table provided here. Remember to scroll down to include all of the Bolus settings.

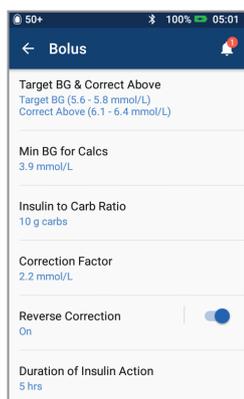


BOLUS SETTINGS (☰ >Settings>Bolus)			
Bolus Calculator	<input type="checkbox"/> ON <input type="checkbox"/> OFF		
Target BG & Correct Above	Time Segment	Target BG	Correct Above
	00:00am - _____	_____ mmol/L	_____ mmol/L
	_____ - _____	_____ mmol/L	_____ mmol/L
	_____ - _____	_____ mmol/L	_____ mmol/L
Min BG for Calcs	_____ mmol/L		
Insulin to Carb (IC) Ratio	Time Segment	1 unit of insulin covers	
	00:00am - _____	_____ g	
	_____ - _____	_____ g	
	_____ - _____	_____ g	
Correction Factor (CF)	Time Segment	1 unit of insulin decreases BG by	
	00:00am - _____	_____ mmol/L	
	_____ - _____	_____ mmol/L	
	_____ - _____	_____ mmol/L	
Reverse Correction	<input type="checkbox"/> ON <input type="checkbox"/> OFF		
Duration of Insulin Action	_____ hours		
Max Bolus	_____ units		
Extended Bolus	<input type="checkbox"/> ON <input type="checkbox"/> OFF		

Step 4 Additional settings – Presets and Reminders


If you're currently using the Presets, you will want to write these down from your Omnipod DASH™ PDM currently in-use so that you can enter them in your replacement Omnipod DASH™ PDM

Presets

1. From the Home screen tap Menu icon

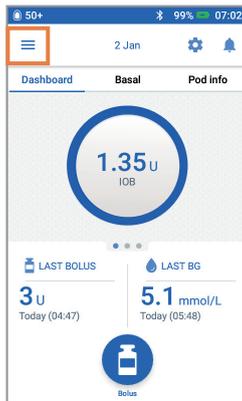


2. Scroll to “MANAGE PROGRAMS & PRESETS”. You can view your presets by tapping on “Temp Basal Presets” and/or “Bolus Presets”. Write down your Presets so you can enter them in your replacement PDM.

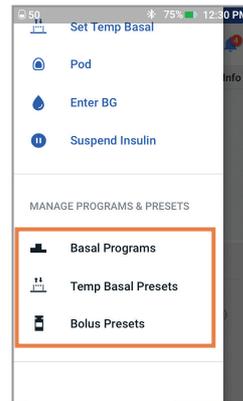


Reminders

It will be helpful for your routine to use the same Reminders in your replacement Omnipod DASH™ PDM


1. From the Home screen tap Menu icon

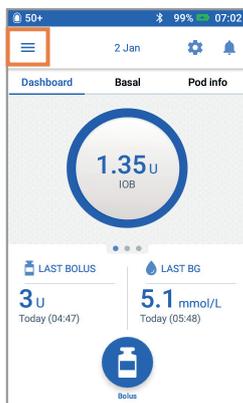

2. Tap “Settings”. Then Tap “Reminders”

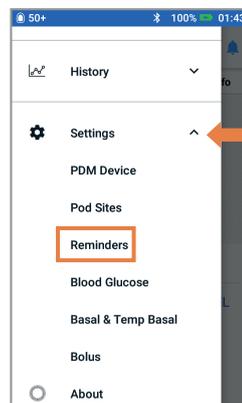

3. Tap on each reminder. Write down all the details of each reminder. Make sure to scroll down to include all the reminders listed

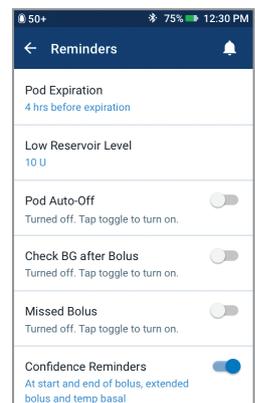

ADDITIONAL SETTINGS – Presets, Reminders

Presets (☰ > Settings > MANAGE PROGRAMS & PRESETS)

Reminders (☰ > Settings > Reminders)



omnipod[®]
DASH™

INSULIN MANAGEMENT SYSTEM



Now that you have your insulin pump settings written in this guide, you are ready to follow the *First Time Setup* steps in your replacement Omnipod DASH™ PDM. Remember, it is important that you accurately enter your insulin pump settings. Entering the wrong settings could lead to serious harm. You may also consider asking a friend or family member to double check that you have entered your settings accurately.

After you complete your *First Time Setup*, your replacement Omnipod DASH™ PDM will ask you to activate a new Pod. If you have not already done so, remember to deactivate your Pod from your currently in-use Omnipod DASH™ PDM.

- ▶ **If you have any further questions about your Omnipod DASH™ System, please contact our 24/7 Customer Care team at 0800 011 6132* (Outside the UK +44 800 011 6132*).**



INS-ODS-07-2020-00157 V1.0

All screen images are examples, for illustrative purposes only.
*Calls may be monitored and recorded for quality monitoring purposes. Calls to 0800 numbers are free from local landlines, but other networks may charge for these calls.

©2020 Insulet Corporation. Omnipod, the Omnipod logo and Podder are trademarks or registered trademarks of Insulet Corporation in the USA and other various jurisdictions. All rights reserved. 1 King Street, 5th Floor, Hammersmith, London W6 9HR.

Insulet