

Updating your Omnipod® 5 App for iPhone




When the Omnipod 5 App for iPhone that is compatible with FreeStyle Libre 3 Plus sensor is officially released, you will need to update your app from the TestFlight version to the version now available in the app store. Your settings and adaptivity will be transferred to the official version of the app. On your current Omnipod 5 App, you will get a notification telling you to update your app.


Caution: Do NOT delete your TestFlight-downloaded Omnipod 5 App before updating. The official app will replace the current app. If you delete the TestFlight version of the app, you will lose your saved settings and adaptivity and will have to complete first-time setup again.

To update from the notification:

It is recommended to update the app when you get the notification. If you tap Not Now, you will get a notification every 72 hours.

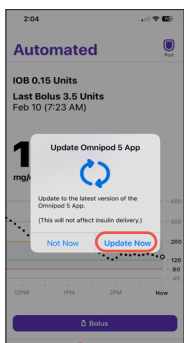

Tap Update Now

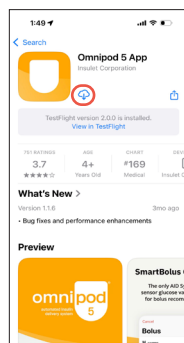


The app store will open to the Omnipod 5 App. Download the new app

Note: The app in the app store will show a different icon depending on whether or not you have downloaded the official app before. You will either see Get or a cloud icon.


Get




To update from the app store:

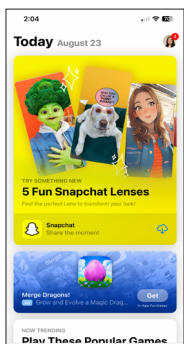


On your iPhone, open the app store

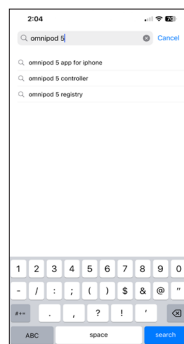

Search for Omnipod 5

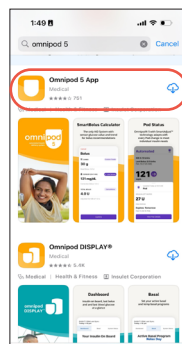

Select the app titled Omnipod 5 App

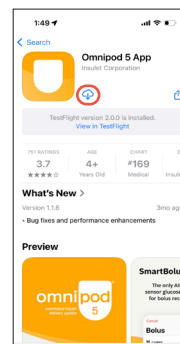

Download the new app