

Omnipod® 5 System for iPhone

TestFlight Instructions for Android/Controller Users



Thank you for taking part in the LMR to use a FreeStyle Libre 3 Plus sensor with the Omnipod 5 App for iPhone. To use a FreeStyle Libre 3 Plus sensor, you need to download the Omnipod 5 App using a unique method, which involves the TestFlight app. Once the Omnipod 5 App has been downloaded, you will need to go through first-time setup again. As a result, your data will be lost and adaptivity will restart. However, if you have previously used the Omnipod 5 App for iPhone, your existing data and adaptivity will be retained.




What is TestFlight?

Think of TestFlight as an early access version of the Apple App Store. It's a platform for downloading apps that are not yet publicly available, and it was created by Apple for this purpose.

Note: While TestFlight will work on iOS 16.0 and above, the **Omnipod 5 App requires iOS 18.0**. Update your phone to iOS 18.0 prior to downloading the Omnipod 5 App for iPhone.

Downloading TestFlight


Note: Before you download the new Omnipod 5 App in TestFlight, make sure you **remove any previous TestFlight versions of the Omnipod 5 App** on your phone indicated by the yellow dot next to the name. **Do NOT delete the App Store version of the Omnipod 5 App for iPhone!**

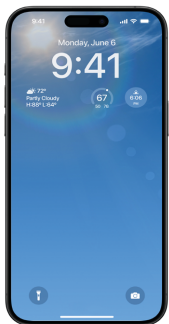


For the following steps, you **must use the iPhone** you plan to use with the Omnipod 5 App with!




You'll receive a personalized TestFlight invitation via email.

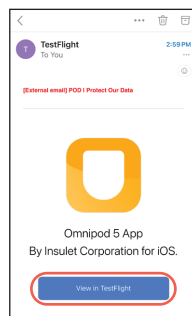


In the email, tap **View in TestFlight**. Your device's browser will open.

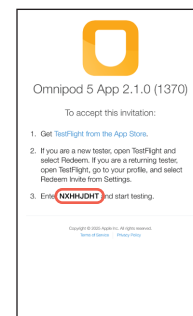


Write down the Redeem Code. You'll need to enter it later.

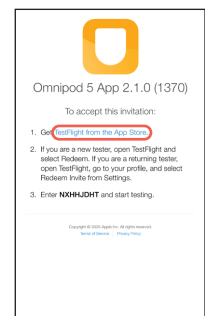


Tap Get TestFlight from the App Store.

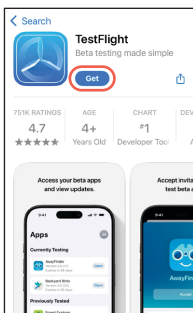


You'll be redirected to the Apple App Store. Tap **Get**.

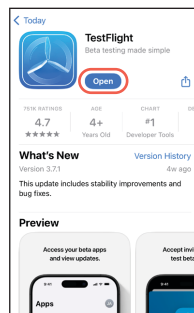

Once TestFlight has finished downloading, tap **Open**.

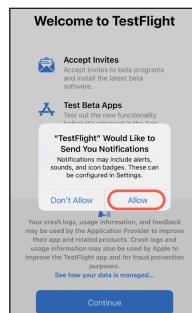

You'll be prompted to allow notifications. We recommend enabling them. Tap **Allow**.

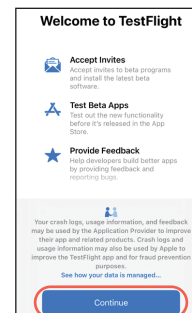


Carefully read TestFlight's Terms and Conditions. You must accept them to use the Omnipod 5 App. Tap **Continue**.

Omnipod® 5 System for iPhone

TestFlight Instructions for Android/Controller Users



Redeeming the invitation and installing the Omnipod 5 App for iPhone

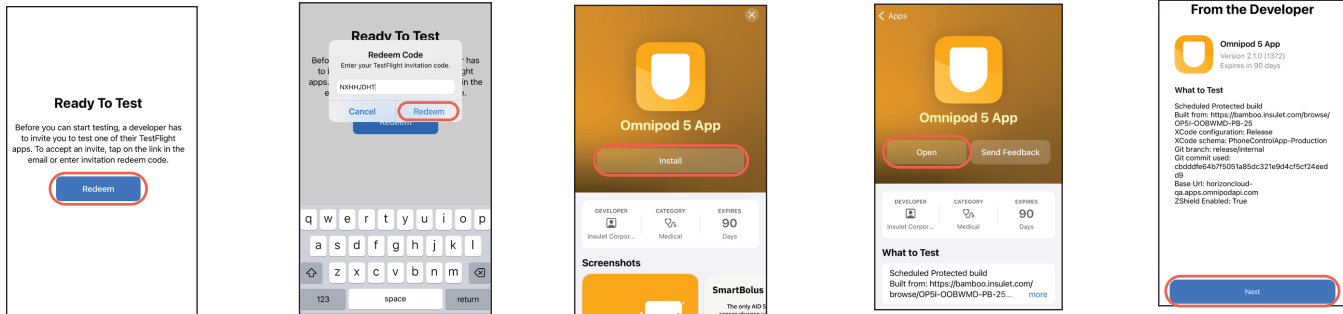

After you've accepted TestFlight's Terms and Conditions, you will see this screen. Tap **Redeem**.

If you have tested an app on TestFlight before, you will need to tap on your profile in the top right corner and tap **Redeem Invite**.

Enter the **Redeem Code** you wrote down earlier.
Tap **Redeem**.


Tap **Install**.

Once the Omnipod 5 App for iPhone has finished installing, tap **Open**.

If prompted to allow Bluetooth, Notifications, and Critical Alerts tap **Allow**.

Then tap **Next**. You will now go through first-time setup to start insulin therapy.

Updating the Omnipod 5 App for iPhone during the LMR

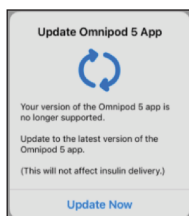

If the Omnipod 5 App for iPhone needs to be updated, you'll receive a notification to **Update Now**.

It is important that you use TestFlight to do the update. **Avoid uninstalling and reinstalling the app.**

Uninstalling the app will result in the loss of your settings, and you will have to complete the first-time setup again!

**For additional assistance,
contact Customer Care at 1-800-591-3455 Option 1.**