


# Linking to LibreView and LibreLinkUp




In order for you to share your FreeStyle Libre sensor data with LibreLinkUp, you first need to connect your PodderCentral® account to your LibreView account. You can link and unlink your accounts at any time.

To link and unlink your LibreView accounts in PodderCentral  
visit [poddercentral.com](https://poddercentral.com)


## To Link your Accounts:

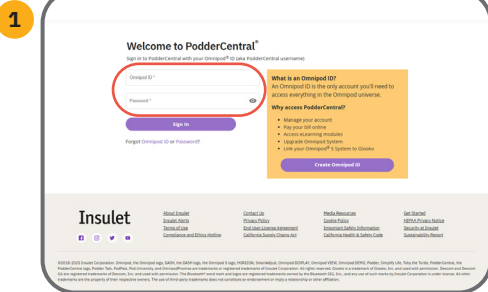


Sign in to your PodderCentral account.

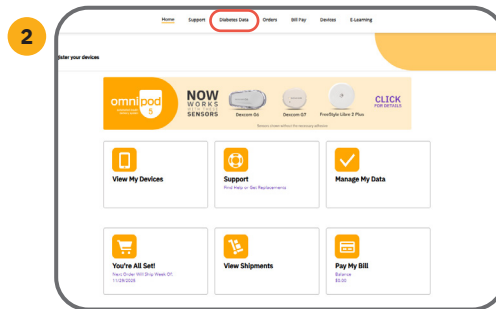


Click the 'Diabetes Data' tab.

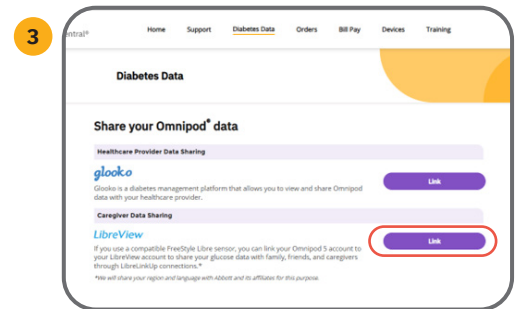


Click 'Link' next to LibreView.

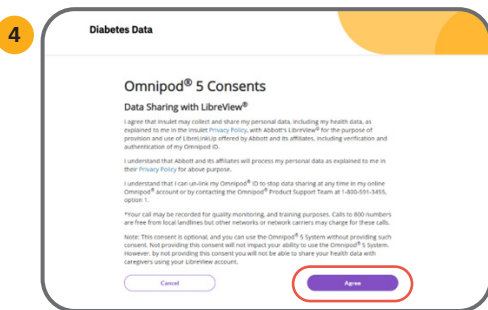


Agree to the Data Sharing Consents.

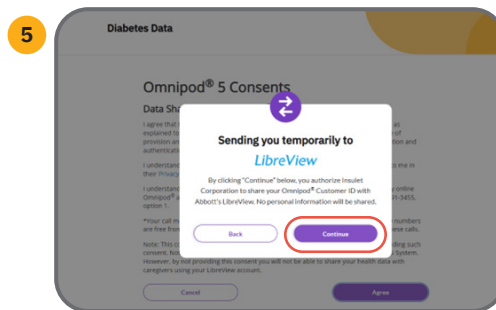

You will then be directed to the LibreView website.

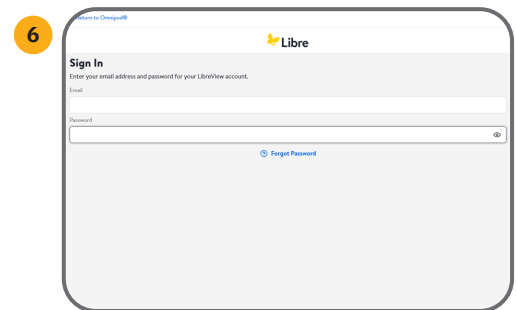

Log in to your Libre account or create a new one.

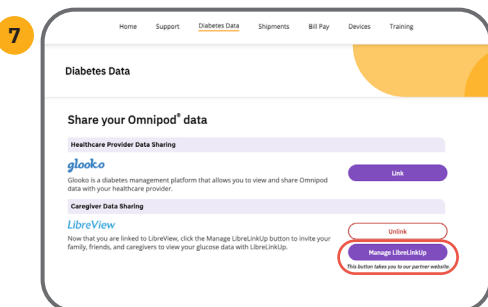


After logging in, you will be brought back to PodderCentral. You can now click 'Manage LibreLinkUp' to add followers.


# Linking to LibreView and LibreLinkUp




## To Unlink your Accounts:

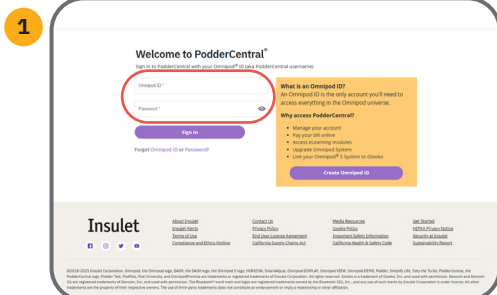


Sign in to your PodderCentral account.

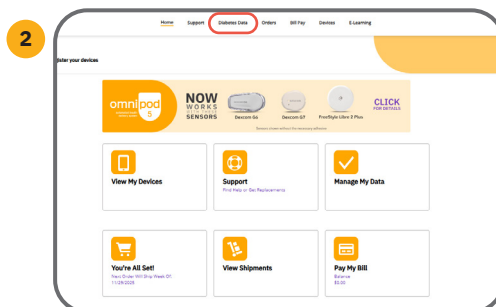

Click the **'Diabetes Data'** tab.

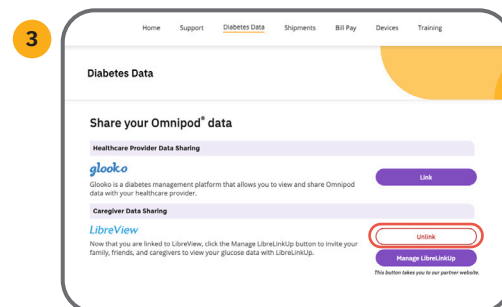

Click **'Unlink'** next to LibreView.

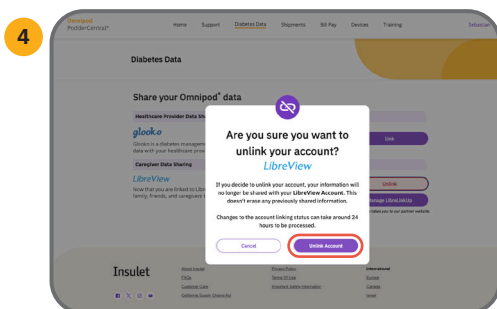

Confirm that you would like to unlink your accounts.

**Questions?** Sign in to PodderCentral and chat online with a live agent or contact Customer Care at **1-800-591-3455**, option 1.

For full FreeStyle Libre 2 Plus sensor safety information, please visit <https://www.freestyle.abbott/us-en/safety-information.html>

©2026 Insulet Corporation. Omnipod, the Omnipod logo, Simplify Life and PodderCentral are trademarks or registered trademarks of Insulet Corporation. All rights reserved. The sensor housing, FreeStyle, Libre, and related brand marks are marks of Abbott and used with permission. All other trademarks are the property of their respective owners. The use of third-party trademarks does not constitute an endorsement or imply a relationship or other affiliation. INS-DMS-10-2025-00001 V1.0