



Omnipod® 5 & Glooko® Key information


AN OMNIPOD® 5 LEAFLET FOR
PEOPLE LIVING WITH DIABETES



LINK your Omnipod 5 Automated Insulin Delivery System data with Glooko

If you have missed linking your Omnipod 5 and Glooko, please follow these four simple steps:

1. Log into your online Omnipod account at www.omnipod.com, using your Omnipod ID
2. Go to the **My Account** tab and then select “**Account Linking**” from the drop-down menu
3. Select “**Sign Up**” or “**Log In**” with Glooko
4. Proceed to Glooko and complete the process to link your account

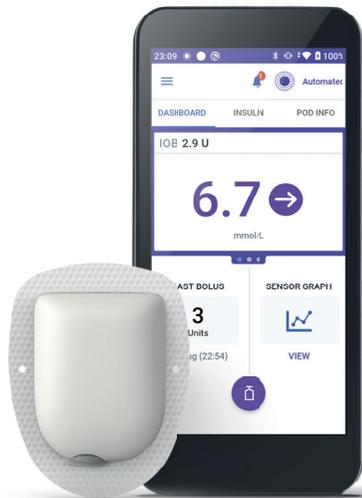

When complete, your status will update to “**Linked**” and data will automatically upload into your personal Glooko account*. You can always unlink your account between Omnipod and Glooko to stop data sharing.

NOTE: All screen images are examples, for instructional purposes only.

*Must be connected to WiFi or cellular data, and Omnipod 5 users must have WiFi or cellular data access to receive updates in Glooko.

Choose to **SHARE** your Omnipod 5 Automated Insulin Delivery System data with your healthcare provider



Entering a ProConnect code in Glooko is the only way for your healthcare provider to see your Omnipod 5 data on their computer.

**ProConnect Code:
Provided by your
healthcare practitioner**


If you have missed linking your Glooko account to your Healthcare Professional, please follow these five simple steps:

1. Log into your Glooko account at my.glooko.com
2. Select “**Settings**” from the drop-down menu (top right on the home screen)
3. Scroll down to Account and select “**+ Add New Code**”
4. Select “**Continue**”
5. Enter ProConnect Code (listed below) then select “**Submit**”

NOTE: All screen images are examples, for instructional purposes only.

Using the GLOOKO MOBILE APP

*You have the choice to use the
Glooko mobile app or the online
Glooko platform my.glooko.com*

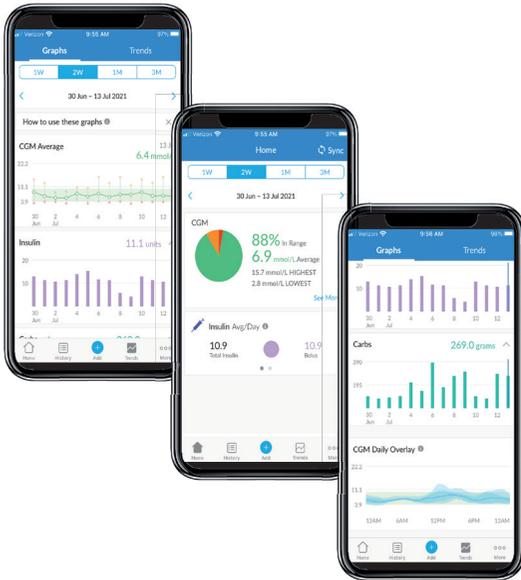

NOTE: All screen images are examples, for instructional purposes only.

For iOS



Download the app and log in

- From the App Store search for and install the “Glooko mobile app”
- Log in or create an account by filling out the requested information


View your data

Once your data is uploaded to Glooko, you can view your trends and graphs on your iPhone or iPad

Choose to share data with your healthcare provider

- Click the **More** tab,
- Click **Share Reports** to email or fax the reports

For Android

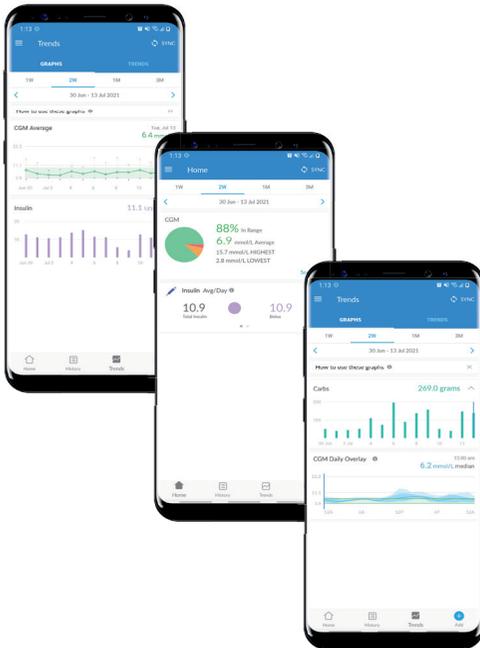

Download the app and log in

- From the Google Play™ Store search for and install the “Glooko mobile app”
- Log in or create an account by filling out the requested information

Sync and view your data

- Tap Sync from the Home Screen, and select Omnipod 5 Automated Insulin Delivery System
- Follow the prompts to complete the sync

Choose to share data with your healthcare provider

Click **Share Reports** to email or fax the reports.

NOTE: All screen images are examples, for instructional purposes only.

Create, store or print a **PDF REPORT**

You can create and view reports at **my.glooko.com**.

1. Log In to your account

2. To create a report:

- Click on your **Profile**
- Click **Create PDF Report** (top right)
- Choose desired reports and time frame and click **Create PDF**

3. To save/print a report:

Once the report is created, either save it to your desktop or a desired location, or print directly.

Create PDF Close

Jane Doe Preferred PDF Setting
DOB: 17/08/09 No Profile Available
Diabetes: Type 1 Manage Favourites

Time: 2 weeks 10/11/2021 - 23/11/2021

Summary 2 pages Logbook 2 pages Overview 1 page Daily Overview 14 pages Week View 1 page
Overlay 1 page Calendar 2 pages Insights 1 page Devices 14 pages

Estimated report length: 20

Type comment here! Comments will appear in the Summary report section.

Print mode: Black and White Colour

Save selection as Favourite Profile Create PDF

NOTE: All screen images are examples, for instructional purposes only.

TIP! Save Customised Report Sets:


- Click **Settings**
- Scroll down and click **New Favourite**
- Select desired reports, and name the report set (*i.e. Dr. Smith reports*)



Great reports to get started with are:


Summary reports, Weekly, Daily

Snapshot of **key statistics** and **aggregated information** to identify positives and challenges to support your care.

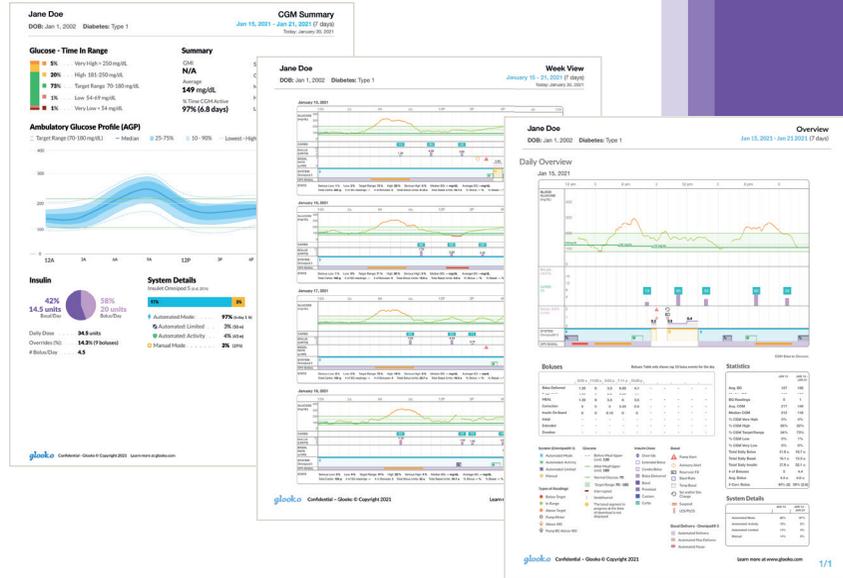

The Device Settings Report

Your most recent device settings are available in this report which is a great resource for you and your care team.

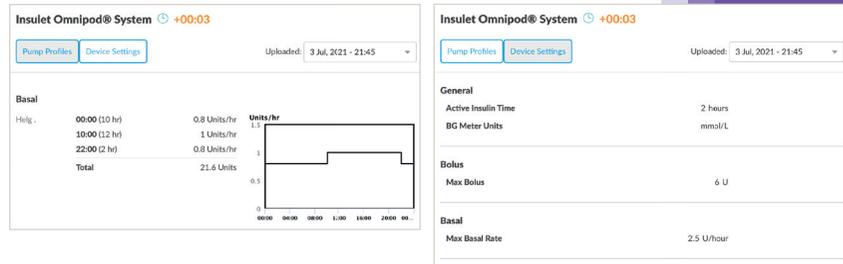

NOTE: All screen images are examples, for instructional purposes only.



If you have any questions or concerns regarding Omnipod 5 please contact the Omnipod team on:

1800954075*

Select Option 2.

If you are experiencing any issues with Glooko, please contact Glooko Support:

1300 851 056*

**www.glooko.com
help@glooko.com**

Always read the label and follow directions for use.

*Your call may be monitored and recorded for quality monitoring purposes. Calls to 1800 numbers are free from local landlines, but other networks may charge for these calls.

The Omnipod 5 Automated Insulin Delivery System is indicated for use by individuals with Type 1 diabetes mellitus in persons 2 years of age and older. The Omnipod 5 System is intended for single patient, home use and requires supervision of the user's healthcare provider.

Refer to the Omnipod® 5 Automated Insulin Delivery System User Guide and www.omnipod.com/safety for complete safety information including indications, contraindications, warnings, cautions, and instructions.

Warning: DO NOT start to use the Omnipod 5 System or change settings without adequate training and guidance from a healthcare provider. Initiating and adjusting settings incorrectly can result in over-delivery or under-delivery of insulin, which could lead to hypoglycaemia or hyperglycaemia.

©2025 Insulet Corporation. Insulet, Omnipod, the Omnipod logo, Podder, Simplify Life and SmartAdjust are trademarks or registered trademarks of Insulet Corporation. All rights reserved. Glooko is a trademark of Glooko, Inc. and used with permission. All other trademarks are the property of their respective owners. The use of third party trademarks does not constitute an endorsement or imply a relationship or other affiliation. Insulet Australia Pty Ltd, Level 16, Tower 2, Darling Park, 201 Sussex Street, Sydney, NSW, 2000. 1800 954 075.

INS-OHS-02-2025-00223 V1.0

