

OMNIPOD® 5 AUTOMATED INSULIN DELIVERY SYSTEM POD START CHECKLIST



Confidential: Protected Health Information

Trainings Performed: *(check all that apply)*

___/___/___
Assessment

___/___/___
Insulin start

Training Format: In-Person Virtual

Patient Name *(print)*: _____ DOB: _____

RESOURCES & SUPPORT

- Omnipod 24/7 Customer Care 1-800-591-3455
- User Guide
- Reviewed Customer's Bill of Rights & Responsibilities
- Reviewed indications, contraindications, and safety precautions

SYSTEM FIRST TIME SETUP

Omnipod® 5 App setup

- Choose a controller (compatible smartphone vs. provided Controller)
 - Connectivity (auto-uploading of data and updates)
- Omnipod ID
- Controller specific settings (Ex: Notification, Battery charging & optimization)
- Basic settings – Personalized lock screen/time/time zone/date/date format
- Initial insulin settings from Pump Therapy Order Form

Pod

- Fill port/adhesive/needle cap/pink slide insert/waterproof IP28/storage guidelines
- Communication process/distance

Compatible CGM *(as indicated in User Guide)*

- CGM in use
 - Yes
 - No, directed to CGM manufacturer training resources
- Communication/site placement

POD ACTIVATION/DEACTIVATION

- Activate Pod
 - Room temperature insulin
 - Fill syringe – min/max amounts
 - DO NOT prefill Pod
 - Site selection/rotation & prep
 - Pod & CGM in direct line of sight
 - Automated cannula insertion – check infusion site/viewing window & pink slide insert
- When to change Pod and removal
- Steps to deactivate

CONNECT TO CGM

- Review use of CGM app
- Pair/connect CGM to Pod
- Connection process

AUTOMATED MODE

- Active with CGM values
 - Yes No, why: _____

Patient/Guardian Signature: _____ Date: _____

Trainer Name *(print)*: _____ Trainer Signature: _____ Date: _____

OMNIPOD® 5 APP OVERVIEW

Home Screen

- Status Bar, Menu Icon, Notification/Alarms, System Mode Indicator (Automated or Manual)
- Tabs
 - Dashboard – IOB, CGM value and trend
 - Insulin
 - Pod Info – View Pod Details
- Last Bolus
- Bolus Icon
- CGM Graph

Menu Icon Overview

- Switch Mode
- Pause
- History Details & Notifications
- Settings – Basal Programs, Bolus

System States

- Automated Mode: Limited (missing CGM values, sensor warm-up)
- No Pod Communication

Advanced Features

- Additional education prior to use (i.e. Activity feature, extended bolus, temp basal, additional basal programs)

Customize Screen & Alerts

- Pod Expiration alert
- Low Pod Insulin alert

ADVISORY & HAZARD ALARMS

- Advisory alarms – response required
- Hazard alarm – urgent attention required

SAFETY

- Hypoglycemia
- Hyperglycemia, ketones & DKA
- Sick day management

KEY INSULIN DELIVERY ACTIONS *(User demonstrates)*

- Switch Mode
- Bolus delivery
- Change Pod
- Troubleshoot BG/CGM values out of range