

Omnipod® 5 System for iPhone Testflight Instructions



Thank you for taking part in the limited market release for the new Omnipod 5 App for iPhone. At present, the Omnipod 5 App for iPhone is only available to a select group of people. That's why the app is not yet in the Apple App store. To download it, you need to use a unique method, which involves the TestFlight app.



What is TestFlight?

Think of TestFlight as an early access version of the Apple App Store. It's a platform for downloading apps that are not yet publicly available, and it was created by Apple for this purpose.


Note: While TestFlight will work on iOS 14.0 and above, the **Omnipod 5 App requires iOS 17**. Please update your phone to iOS 17 prior to downloading the Omnipod 5 App for iPhone.

Downloading TestFlight




For the next steps, you must use the **device** you plan to use with the Omnipod 5 App for iPhone with!


Note: The Omnipod 5 App requires iOS 17!

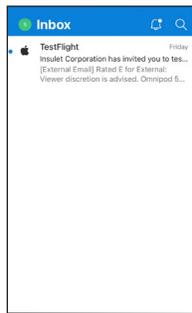

You'll receive a personalized TestFlight invitation via email.

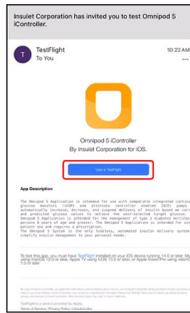


In the email, tap **View in TestFlight**. Your device's browser opens.

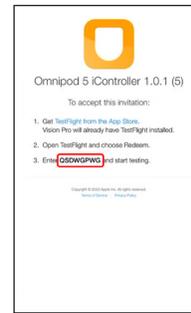


Write down the Redeem Code. You'll need to enter it later.




Tap **Get TestFlight from the App Store**.

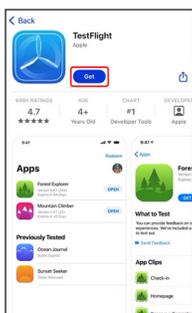


You'll be redirected to the Apple App Store. Tap the **download icon**.

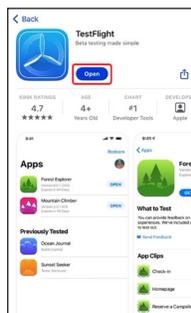

Once TestFlight has finished downloading, tap **Open**.

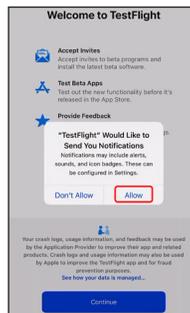

You'll be prompted to allow notifications. We recommend enabling them. Tap **Allow**.

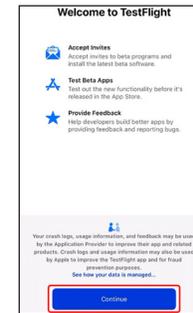


Carefully read TestFlight's terms and conditions. You must accept them to use the Omnipod 5 App. Tap **Continue**.

Omnipod® 5 System for iPhone Testflight Instructions




Redeeming the invitation and installing the Omnipod 5 App for iPhone

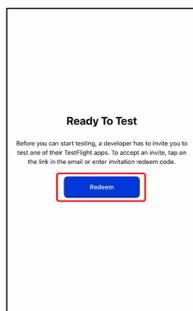


After you've accepted Testflight's Terms and Conditions, you will see this screen.

Tap **Redeem**.

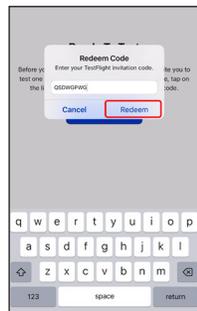


Enter the **Redeem Code** you wrote down earlier.

Tap **Redeem**.

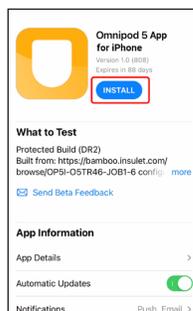

Tap **INSTALL** to download the Omnipod 5 App for iPhone.

Note: The Omnipod 5 App for iPhone requires iOS 17.

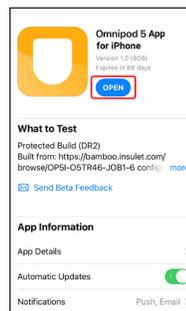

Once the Omnipod 5 App for iPhone has finished installing, tap **OPEN**.



If prompted to allow Bluetooth, tap **OK**.

Then tap **Next**.

Updating the Omnipod 5 App for iPhone during limited market release



If the Omnipod 5 App for iPhone needs to be updated, you'll receive a notification to **Update Now**.

Tap **Update Now**.

Note: It is important that you use TestFlight to do the update. **Avoid uninstalling and reinstalling the app.** Uninstalling the app will result in the loss of your settings, and you will have to complete first-time setup again!

**For additional assistance,
contact Product Support at 1-800-591-3455 Option 1.**