



Technical User Guide

Omnipod DASH® Handbook



PDM Model: PDM-INT1-D001-MG

Serial Number _____

Omnipod DASH® Insulin Management System Start Date _____

Healthcare Provider

Name

Address

Phone

Email

Nurse/Educator

Name

Address

Phone

Email

Health Insurance

Name

Address

Phone

Policy Number

Pharmacy

Name




Address

Phone




Email






Australia

 omnipodAUS@insulet.com
 1800954074
 +61-272084352




België / Belgique / Belgien

 omnipod-BE@insulet.com
 0800 17682
 +32 289 62 983




Canada

 omnipodcanada@insulet.ca
 1-855-POD-INFO
(1-855-763-4636)
 +1 513-562-6446




Danmark

 omnipod-DK@insulet.com
 80 25 36 09
 +45 70 28 10 24




Deutschland

 omnipod-DE@insulet.com
 0800 182 1629
 +49 8995 467389




Ellada / Ελλάδα

 omnipod-GR@insulet.com
 00800 128 542
 +30-217-000-9999




France

 omnipod-FR@insulet.com
 0800 918442
 +33 1 85 65 37 47




Hrvatska

 omnipod-HR@insulet.com
 0800 806 259
 +385 91 2358040




Ísland

 sykursyki@fastus.is
 +354 580 3900
 (+354) 580 3900


Israel / إسرائيل / ישראל

 omnipod-IL@insulet.com
 1 809 216 160
 +972 369 00300


Italia

 omnipod-IT@insulet.com
 800 587 270
 +39 0524 587 334

Kingdom of Saudi Arabia / المملكة

ةيدوعسلا ةيدوعسلا
 omnipod-SA@insulet.com
 00966558491266

Kuwait / الكويت


 omnipod@sultan.com.kw


 +965 97297678

 +965 66134426

Lëtzebuerg / Luxemburg / Luxembourg

 omnipod-LU@insulet.com

 800 24368

 +352 27871306

Nederland


 omnipod-NL@insulet.com

 0800 022 9512

 +31 20 798 9337

Norge / Noreg

 omnipod-NO@insulet.com


 800 22 612

 +47 22 20 60 00

Österreich


 omnipod-AT@insulet.com

 0800 281 248


 +43 72 088 3504


Qaṭar / قطر


 Customer@aldanahmedical.com

 + 974 5039 5048

Schweiz / Suisse / Svizzera

 omnipod-CH@insulet.com

 0800 89 76 18

 +41 44 585 9019

Suomi / Finland

 omnipod-FIN@insulet.com

 0800 91 2942

 +358 985 653 300

Sverige

 omnipod-SE@insulet.com

 020 033 6809

 +46 8 601 24 40

Türkiye

 omnipod-TR@insulet.com

 00800 142 030 007

 +90 850 480 53 00

United Arab of Emirates /

دولة الامارات العربية المتحدة

 omnipod-AE@insulet.com

 800 018 7092

United Kingdom

 omnipod-GB@insulet.com

 0800 011 6132

 +44 20 3887 1709

© 2019-2024 Insulet Corporation. Insulet, Omnipod, the Omnipod logo, DASH, the DASH logo and Podder are trademarks or registered trademarks of Insulet Corporation in the United States of America and other various jurisdictions. All rights reserved. The Bluetooth® word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by Insulet Corporation is under licence. All other trademarks are the property of their respective owners. The use of third-party trademarks does not constitute an endorsement or imply a relationship or other affiliation. Patent information at www.insulet.com/patents.

Contents

Introduction	xi
About This Technical User Guide.....	xi
Indications and Contraindications.....	xii
General Warnings	xii
Working with Your Healthcare Provider	xiv
Safety Features	xv

Getting Started

1 Your Omnipod DASH® Insulin Management System	1
Welcome	1
The Pod and PDM.....	2
Navigating Screens and Entering Information	4
Touchscreen Basics	4
Entering Numbers and Text	5
Selecting, Adding and Deleting Items	7
Navigation Icons and Navigation Shorthand	7
The Status Bar	9
The PDM's Lock and PIN Screens	10
The PDM's Home screen	11
Dashboard tab.....	12
Basal tab / Temp Basal tab.....	13
Pod Info tab.....	14
Last BG section	15
Last bolus section	16
Bolus button.....	16
The Home Page's Menu	17
PDM Messages to You.....	19
Alarms	19
Notifications.....	19
Confirmation messages.....	19
2 Initial PDM Set-up	21
Preparing for Your Training.....	21
General PDM Settings.....	22
Turning on and personalising the PDM.....	23
Enter a PIN	24

Set the time zone and time25

Set the date.....26

Basal Insulin Settings.....26

Maximum Basal Rate26

Create a Basal Programme27

Temp basal configuration29

Blood Glucose Settings30

Upper and lower limits for BG Goal Range.....30

Bolus Calculator Settings.....30

Turn the Bolus Calculator on or off.....31

Target BG and Correct Above Values.....31

Minimum BG for Calcs.....32

Insulin to carb ratio33

Correction Factor33

Reverse Correction34

Duration of Insulin Action.....34

Other Bolus Settings35

Maximum Bolus35

Extended bolus setting.....35

PDM Set-up Complete35

Technical User Guide

3 Changing Your Pod.....37

Beginning the Pod Change Process37

First Time Activating a Pod.....38

Deactivate an Active Pod38

No Active Pod39

Preliminary Pod Change Steps.....40

Fill the Syringe with Insulin.....41

Fill, Pair, and Apply the Pod42

Fill the Pod with insulin.....42

Pair the PDM and Pod43

Prepare the Pod site44

Remove the Pod's needle cap.....46

Apply the Pod47

Begin insulin delivery.....47

Check the Infusion Site.....48

Avoid Infusion-Site Infections.....49

More Information about Pod Use.....50

4 Entering Blood Glucose Readings51

About Blood Glucose Testing51

Entering Your Blood Glucose Reading51

Tagging Your Blood Glucose Reading	53
How Blood Glucose Readings are Displayed	54
5 Delivering a Bolus of Insulin	57
Bolusing with the Bolus Calculator	57
Enter your meal information	58
Enter your blood glucose reading	59
Delivering an immediate or extended bolus	60
Manually-calculated Bolus	63
Tracking the Progress of a Bolus	65
Altering a Bolus in Progress	66
6 Modifying Basal Insulin Delivery	69
Using Temporary Basal Rates	69
Activate a temp basal	69
Activate a temp basal preset	71
Cancel a temp basal	71
Switching to a Different Basal Programme	72
Suspending and Resuming Insulin Delivery	73
7 Manage Programmes and Presets	75
Basal Programmes	75
Create a new Basal Programme	75
Review all Basal Programmes	76
Edit or rename a Basal Programme	77
Delete a Basal Programme	78
Temp basal presets	79
Create a new temp basal preset	79
Edit or rename a temp basal preset	80
Delete a temp basal preset	80
Bolus presets	81
Create a new bolus preset	81
Edit or rename a bolus preset	82
Delete a bolus preset	82
8 Browsing Your PDM Records	83
Notifications & Alarms Screens	83
Overview of the Insulin and BG History Screens	84
Date Selection	85
Summary Section	86
Details Section	88
Blood glucose details	89
Bolus details	89
Carbs details	91
Basal rate details	91

Pod details92

Insulin suspended and resumed details92

Time change details92

Date change details92

9 Adjusting Settings93

 PDM Device Settings.....93

 Network connectivity93

 Screen display94

 Lock screen94

 Date and time, and language95

 Diagnostics97

 Pod Sites Setting98

 Reminder Settings98

 Pod expiry99

 Low reservoir level.....99

 Pod auto-off99

 Check BG after Bolus reminders..... 100

 Missed bolus reminders..... 100

 Confidence reminders 102

 Programme reminders 102

 Custom reminders..... 102

 Blood Glucose Settings - BG GoalRange..... 104

 Basal and Temp Basal Settings..... 104

 Maximum Basal Rate 104

 Temp basal 105

 Bolus Delivery Settings..... 105

 Maximum Bolus 105

 Extended bolus..... 106

 Bolus calculator settings..... 106

Reference

10 Alarms, Notifications, and Communication Errors 109

 Overview 109

 Responding to Alarms 110

 Responding to Notifications 110

 Priority and Predictability of Alarms and Notifications 112

 Sounds and Vibrations 112

 Hazard Alarm List 114

 Advisory Alarm List..... 116

 Notifications List 117

 Informational Signals List 119

Communication Errors.....	120
Error when cancelling a bolus.....	120
Error when sending insulin instructions to the Pod	121
Error when activating a Pod.....	122
Error when deactivating a Pod.....	122
Silencing an Unresolved Alarm	123
Pod alarm.....	123
PDM alarm.....	123
11 Taking Care of Your PDM and Pod	125
Pod and Insulin Care	125
Pod and insulin storage	125
Pods and the environment.....	125
Cleaning your Pod.....	126
PDM Care.....	127
PDM storage.....	127
PDM and the environment.....	127
Cleaning your PDM.....	128
If you drop the PDM	129
PDM Battery Care	129
Safe use of the PDM battery.....	129
PDM battery messages you may see	131
Charging the PDM battery	131
Inserting the PDM battery during first time setup	132
12 Understanding PDM and Pod Function.....	135
PDM and Pod Interactions	135
Pod actions controlled by the PDM	135
What the Pod can do between PDM instructions	137
Auto-off.....	138
Basal Insulin Delivery.....	138
Basal Programmes.....	139
Temporary basal rates.....	140
Methods to temporarily stop insulin delivery.....	143
Immediate and Extended Boluses.....	144
About Manually-calculated Boluses	144
The Bolus Calculator	145
Bolus Calculator boluses	145
When the Bolus Calculator does not work.....	146
Factors used in the Bolus Calculator's calculations	146
Insulin on board (IOB)	149
Bolus calculator equations	151
Bolus Calculator rules.....	152
Overview of the Bolus Calculator CALCULATIONS screen.....	153
Bolus Calculator examples	155

Contents

Calculations for History Summaries	160
Blood glucose summaries.....	160
Insulin delivery summaries.....	161
13 Living with Diabetes.....	163
Daily Activities.....	163
Infusion site checks.....	163
Check your blood glucose frequently.....	164
Prepare for Emergencies.....	165
Travelling and Holidays	165
Keep supplies accessible	166
Plan for changing time zones	166
Airports and flying.....	166
Avoid Lows, Highs and DKA	168
General precautions.....	168
Hypoglycaemia (low blood glucose).....	168
Hyperglycaemia (high blood glucose).....	171
Diabetic ketoacidosis (DKA).....	174
Handling Special Situations.....	175
Sick days	175
Exercising, playing sports or working hard	176
X-rays, MRIs and CT scans	176
Surgery or hospitalisation.....	176
Appendix	177
Troubleshooting PDM Start-up.....	177
Summary of Settings and Options.....	178
Pod Specifications.....	179
PDM Specifications	181
Protection from Over-infusion or Under-infusion	182
Omnipod DASH® System Label Symbols.....	183
Medical Device Regulation	184
EU General Data Protection Regulation (GDPR).....	184
Device Complaints	185
EU Authorised Representative details	185
Omnipod DASH® System Notice Concerning Interference	185
Electromagnetic Compatibility	186
Warranty for the Omnipod DASH® Insulin Management System Personal Diabetes Manager and Pods (all countries except for Canada and the United States)	188
Glossary	193
Index.....	199

Introduction

About This Technical User Guide

Caution: This *Technical User Guide* is intended for use only with Personal Diabetes Manager (PDM) model PDM-INT1-D001-MG. To learn which version of the PDM you have, turn it over. If you see 'PDM-INT1-D001-MG' on the back of the PDM, this is the correct *Technical User Guide*. If you do not see it, call Customer Care.

Note: Screen images shown in this *Technical User Guide* are examples only and are not suggestions for user settings. Always consult with your healthcare provider to determine the appropriate settings for you.

Healthcare and treatment are complex subjects requiring the services of qualified healthcare providers. This *Technical User Guide* is informational only and not intended as medical or healthcare advice or recommendations to be used for diagnosis, treatment or for any other individual needs. This *Technical User Guide* is not a substitute for medical or healthcare advice, recommendations and/or services from a qualified healthcare provider. This *Technical User Guide* may not be relied upon in any way in connection with your personal healthcare, related decisions and treatment. All such decisions and treatment should be discussed with a qualified healthcare provider who is familiar with your individual needs.

This *Technical User Guide* is periodically updated. Visit omnipod.com to see the most recent version and to find other helpful information. To access this *Technical User Guide* in other languages, see omnipod.com.

Term	Meaning
Warning	Alerts you to the possibility of injury, death or other serious adverse reactions associated with the use or misuse of the device.
Caution	Alerts you to the possibility of a problem with the device associated with its use or misuse. Such problems include device malfunctions, device failure, damage to the device or damage to other property.
Note	Provides helpful information.
Tip	Offers a suggestion for successful use of the device.

Indications and Contraindications

Indications for use

The Omnipod DASH® Insulin Management System is intended for subcutaneous delivery of insulin at set and variable rates for the management of diabetes mellitus in persons requiring insulin.

The intended clinical benefit of the Omnipod DASH System is improved glycaemic control in people with diabetes requiring insulin.

Contraindications

Insulin pump therapy is NOT recommended for people who are:

- Unable to monitor blood glucose levels as recommended by their healthcare provider
- Unable to maintain contact with their healthcare provider
- Unable to use the Omnipod DASH® System according to instructions

General Warnings

Warnings:

Rapid-acting U-100 insulin: The Omnipod DASH® System is designed to use rapid-acting U-100 insulin. The following U-100 rapid-acting insulin analogues have been tested and found to be safe for use in the Pod: NovoLog®/NovoRapid® (insulin aspart), Fiasp® (insulin aspart), Trurapi®/Insulin aspart Sanofi® (insulin aspart), Kirsty® (insulin aspart), Humalog® (insulin lispro), Lyumjev® (insulin lispro-aabc), Admelog®/Insulin lispro Sanofi® (insulin lispro) and Apidra® (insulin glulisine). NovoLog/NovoRapid, Fiasp, Trurapi/Insulin aspart Sanofi, Kirsty, Humalog, Lyumjev and Admelog/Insulin lispro Sanofi are compatible with the Omnipod DASH® System for use up to 72 hours (3 days). Apidra is compatible for use up to 48 hours (2 days). Before using a different insulin with the Omnipod DASH® System, check the insulin medication label and consult your healthcare provider. Refer to the insulin labelling and follow your healthcare provider's directions for how often to replace the Pod. Fiasp and Lyumjev have a faster initial absorption than other rapid-acting U-100 insulins; always consult with your healthcare provider and refer to the insulin labelling prior to use.

Read all the instructions provided in this *Technical User Guide* before using the Omnipod DASH® System. Monitor your blood glucose with the guidance of your healthcare provider. Undetected hyperglycaemia or hypoglycaemia can develop without proper monitoring.

Warnings:

Not recommended for individuals with hearing loss. Always verify your ability to hear Pod/PDM alarms and notifications.

If you are unable to use the Omnipod DASH® System according to the instructions, you may be putting your health and safety at risk. Talk with your healthcare provider if you have questions or concerns about using the Omnipod DASH® System properly.

After use, parts of the device are considered biohazardous and can potentially transmit infectious diseases.

Do not attempt to alter, modify or disassemble any part of the Omnipod DASH® System. Doing so may put your health and safety at risk.

If you are having symptoms that are not consistent with your blood glucose test results and you have followed all the instructions described in this *Technical User Guide*, contact your healthcare provider.

The Omnipod DASH® System should NOT be used at low atmospheric pressure (below 700 hPa). You could encounter such low atmospheric pressures at high elevations, such as when mountain climbing or living at elevations above 3,000 metres (10,000 feet).

The Omnipod DASH® System should NOT be used in oxygen-rich environments (greater than 25% oxygen) or at high atmospheric pressure (above 1,060 hPa), both of which can be found in a hyperbaric chamber. Hyperbaric, or high pressure, chambers are sometimes used to promote healing of diabetic ulcers or to treat carbon monoxide poisoning, certain bone and tissue infections and decompression sickness.

Working with Your Healthcare Provider

Insulin pump therapy requires substantial caregiver involvement, especially with children. Work with your healthcare provider to establish diabetes management guidelines and settings that best fit your or your child's needs. These may include:

Insulin to carbohydrate (IC) ratio: Number of grams of carbohydrate covered by one unit of insulin. For example, if your insulin to carbohydrate ratio is 1 to 15, then you need to deliver one unit of insulin to cover every 15 grams of carbohydrate you eat.

Correction (or Sensitivity) factor: How much one unit of insulin lowers your blood glucose. For example, if your correction factor is 50, one unit of insulin lowers your blood glucose by 50 mg/dL.

Target Blood Glucose (Target BG): The blood glucose level you want to achieve. For example, you may want to keep your blood glucose close to 100 mg/dL.

Duration of insulin action: The length of time that insulin remains active and available in your body after a correction or meal bolus.

Your healthcare provider is a valuable resource. You will rely on him or her for critical information about your Omnipod DASH® System, especially during the first few weeks and months. When you have questions about diabetes management after starting on the Omnipod DASH® System, do not hesitate to talk with your healthcare provider.

For technical questions about your Omnipod DASH® System set-up or operation, or to order Omnipod DASH® System products and supplies, you can call Customer Care 24 hours a day, 7 days a week. See the first page of this *Technical User Guide* for your Customer Care contact information.

Your healthcare provider will give you all the tools and training you need to be successful with the Omnipod DASH® System. As you become actively involved in your diabetes management, you can enjoy the control and flexibility that is possible with the Omnipod DASH® System.

Being actively involved means to frequently monitor your blood glucose levels, to learn how to operate your Omnipod DASH® System, to practise proper techniques and to visit your healthcare provider regularly.

Safety Features

The Omnipod DASH® System's safety features include:

Automatic priming, safety checks and insertion

When you activate a new Pod, the Omnipod DASH® System automatically primes and performs safety checks on the Pod, then inserts and primes the cannula (a small, thin tube). These safety checks take only a few seconds.

The Omnipod DASH® System also performs safety checks on the Personal Diabetes Manager (PDM) whenever the PDM is awake. If it detects any problems in the PDM or the Pod – or in the communication between them – it informs you with vibrations or beeps and on-screen messages.

Alarms and notifications

For your safety, the Omnipod DASH® System provides a range of alarms and notifications to tell you that your attention is needed or to warn you of hazardous situations. For a description of the PDM's alarms and notifications, see 'Alarms, Notifications, and Communication Errors' on page 109.

Note: Brief alarm and notification messages appear on the Lock screen. You must unlock the screen and enter your personal identification number (PIN) to see the full alarm messages.

Note: Wake up your PDM periodically to confirm that there are no notification or alarm messages that require a response.

Warning: Do NOT attempt to use the Omnipod DASH® System before you receive training. Inadequate training could put your health and safety at risk.

This page is intentionally left blank.

CHAPTER 1

Your Omnipod DASH® Insulin Management System

Welcome

The Omnipod DASH® Insulin Management System is a continuous insulin delivery system that provides all the proven benefits of continuous subcutaneous insulin infusion (CSII) therapy.

Features of the Omnipod DASH® System include:

No tubing: There is no tubing connecting the Pod to the Personal Diabetes Manager (PDM). You can wear the Pod under your clothes and carry the PDM separately. You can swim wearing the Pod and leave the PDM on dry land. The Pod is waterproof to 7.6 metres (25 feet) for up to 60 minutes (IP28).

Bolus Calculator: If your blood glucose is high or if you are planning to eat, the PDM's Bolus Calculator can suggest a bolus dose based on your individual settings.

Glucose readings: You can manually enter blood glucose readings from any BG meter into the PDM. Alternatively, you can enter glucose readings from a CGM indicated for dosing insulin.

Record-keeping: The PDM displays up to 90 days of information, including basal rates, bolus doses, carbohydrates, alarms and blood glucose entries.

The *Getting Started* section of this book gives an overview of the basic operation and set-up of the Omnipod DASH® System. The *Technical User Guide* section gives step-by-step instructions for using the Omnipod DASH® System. The *Reference* section explains how the Omnipod DASH® System works.

Terms and conventions used in this *Technical User Guide*

Screen	Area of the PDM that displays menus, instructions and messages.
Menu	List of options on the screen.
Icon	An image on the screen that indicates a menu option or that provides information.
Button	Physical buttons on the PDM, such as the Power button. Also, areas of the screen that you can tap to perform an action.
Field	The area of a screen where you enter information.

1 Omnipod DASH® Insulin Management System

The Pod and PDM


The Omnipod DASH® System is made up of two primary parts: The Pod, which delivers insulin to your body, and the Personal Diabetes Manager (PDM), which lets you control the Pod.

The Pod

The Pod is a lightweight device that you fill with insulin and wear directly on your body. The Pod receives insulin delivery instructions from the PDM. The Pod then delivers insulin into your body through a small, flexible tube called a cannula.

The Pod is applied to your skin with an adhesive, similar to an adhesive bandage.

The Pod that is used with the Omnipod DASH® System has a needle cap.

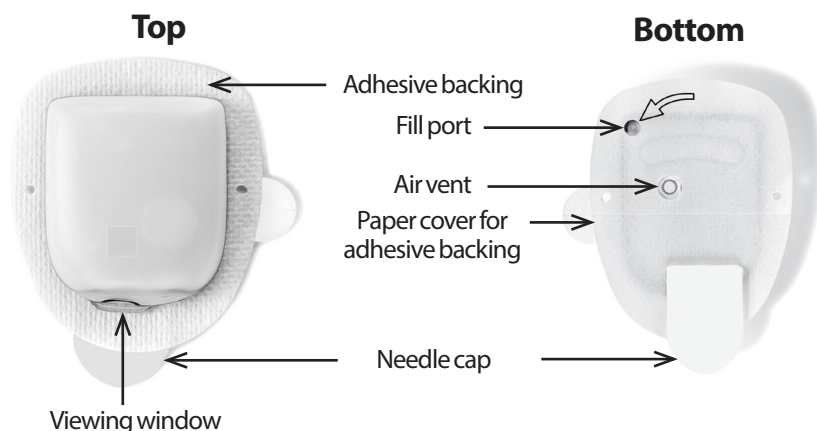

The Personal Diabetes Manager (PDM)

The PDM is a handheld device that controls and monitors the Pod's operations using wireless technology.



Navigating Screens and Entering Information



This section explains how to use the touchscreen, how to enter numbers or text into the PDM, and how this *Technical User Guide* describes moving between PDM screens.

Touchscreen Basics

The PDM displays messages and options for you on its touchscreen. You communicate with the PDM by tapping or swiping your finger on the touchscreen.

Tapping and swiping

The basic instructions for interacting with the touchscreen are explained here.

	Tap	Touch the screen, then lift your finger up.
	Swipe	<p>Touch a starting point and move your finger up, down, left or right.</p> <p>Note: Scrolling and swiping are related actions. When you swipe up, the screen display scrolls up to show items that are currently off-screen.</p>

Note: A screen protector may decrease the touchscreen sensitivity.


Screen time-out and brightness

The PDM screen turns black, called timing-out, after a period of inactivity. To control the screen time-out and brightness settings, see 'Screen display' on page 94. The PDM screen dims 6-10 seconds before it times out. If the screen dims, touch the screen briefly to prevent the screen from timing out.

Entering Numbers and Text

Tapping fields to enter data


Fields allow you to specify values to be used by the PDM. Tapping an editable field (which is either framed by a box or underlined) opens a keypad, number pad or scroll wheel. Tap the letters or numbers, or scroll the wheel to enter data for use in that field.

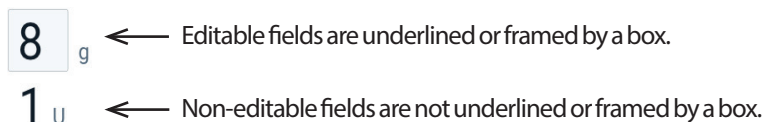

Tapping blue and underlined text brings up a screen where you can view an explanation or enter data.

Using a keypad


Tapping in an editable field brings up a keypad. To bring up a different keypad, tap the ?123, ABC or =\< in the lower left of a keypad.

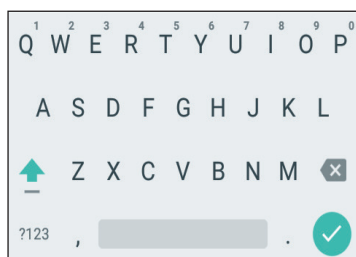

Tap a character to add it to the text field.

Touch a letter in the top row for a second to display other options such as a number or an accented letter such as é.


Tap the up arrow (↑) to toggle between uppercase and lowercase letters. Tap the up arrow twice in rapid succession to type in ALL CAPITALS mode. Tap the up arrow one more time to exit ALL CAPITALS mode.

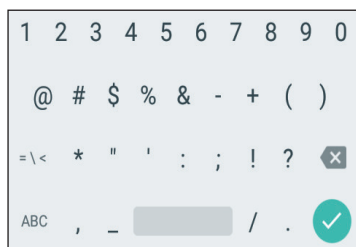

Tap the spacebar icon () to add a space between characters.

Tap the backspace icon (X) to delete the most recent entry from the entry field.


Tap the green tick symbol (✓) to close the keypad when you have finished entering your text.

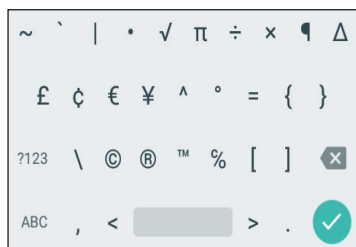


Note: The PDM considers CAPITAL and lowercase letters to be the same. In other words, the PDM considers 'myWeekend' and 'myweekend' to be the same name.

1 Omnipod DASH® Insulin Management System

Using a number pad

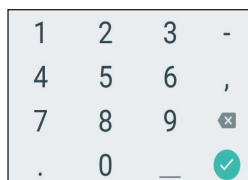

Tap a number to add it to the entry field. Tap the period (.) to add a decimal point. Tap the backspace icon () to delete the most recent entry from the entry field. The comma (,), dash (-), and underscore () are not used.

Tap the green tick symbol () to accept the entered number. You can also tap outside the number pad to accept the entered number.

Using a scroll wheel




Place your finger on the scroll wheel. Move your finger up to select a smaller number and down to select a larger number. The faster you move your finger, the faster the wheel will scroll.


When your desired selection is shown in the centre of the wheel, select the value by tapping it or by tapping outside the scroll wheel.

Using a slider

Use sliders to select a value from a scale. Place your finger on the small open circle and move your finger until your desired value is displayed. Depending on the on-screen graphic, you may move your finger vertically or in a circle to move the slider. Move your finger up or clockwise for a larger number and down or anti-clockwise for a smaller number. Lift your finger when the desired value is displayed.

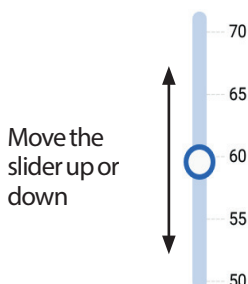

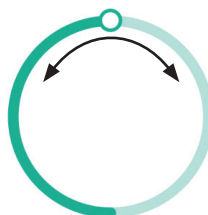

Move the slider around the circle








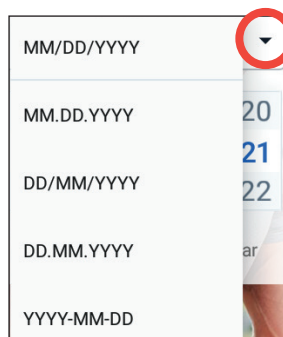
Selecting, Adding and Deleting Items

Toggles

-  Tap a toggle to change the selection from one side to the other.
-  Toggles allow you to choose between two options or to turn a feature on or off. The toggle is on the right side and blue when a feature is on, and on the left and grey when a feature is off.



Tick boxes, radio buttons and drop-down lists

-  Tick boxes are squares. Tap a tick box to tick or untick it. You can tick more than one tick box in a list.
-  
-  Radio buttons are circles. Tap a radio button to select it. A dot appears inside the selected radio button. You can select only one radio button at a time.
-  To deselect a radio button, tap the radio button next to a different option.



Drop-down lists are indicated by a down or up arrow next to an item. The currently selected setting appears to the left of the drop-down arrow. To select a different option, tap the down arrow to show the drop-down list of options. To replace the currently selected option at the top, tap the desired option from the drop-down list.

Tap the up arrow to hide the list of options.

-  A plus symbol in a circle indicates that you can add an item to a list. Tap the plus symbol to add the item to the list.
-  A red x in a circle indicates that you can remove an item from a list. To remove the item, tap the red x.

Navigation Icons and Navigation Shorthand

The PDM has buttons below the screen that can be used for navigation. In addition, some screen icons can be used for navigation.

Back arrow and back button

Many screens display a 'back' arrow icon in the upper left corner. Tapping the back arrow returns to the previous screen.



Tapping the left-hand button below the PDM screen generally also returns to the previous screen.



1 Omnipod DASH® Insulin Management System

Note: The middle and right-hand buttons below the PDM screen have no function.

Options icon



The Options icon (⋮) appears on the right side of certain lists. Tapping the Options icon brings up a list of options relevant to the item on that row.

Navigation shorthand

The *Technical User Guide* uses the '>' symbol to indicate navigating from one screen to another. For example, the following notation:

- Menu icon (☰) > Pod > CHANGE POD

tells you to:

1. Tap the Menu icon (☰) in the upper left of the Home screen.
2. Tap Pod to open the Pod screen.
3. Tap CHANGE POD.

- Menu icon (☰) > History: Insulin & BG History

tells you to:

1. Tap the Menu icon (☰) in the upper left of the Home screen.
2. Expand the History entry, if it is collapsed, by tapping anywhere in the row with the word 'History'.

When expanded, selections for the Notifications & Alarms history screen and the Insulin & BG History screen appear.

3. Tap Insulin & BG History.

- Settings icon (⚙️) > Reminders > Pod Expiry


tells you to:

1. Tap the Settings icon (⚙️) in the upper right of the Home screen.
2. Tap Reminders.
3. Tap Pod Expiry.

The Status Bar

Most screens have a status bar that shows you:

- The Pod status, including how much insulin is left in an active Pod
- Whether a temp basal rate is running
- Whether the PDM will vibrate for selected notifications
- The Bluetooth® icon
- The status of the PDM's battery
- The current time

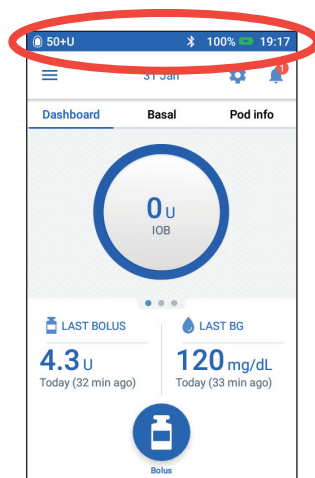

Status bar icon definitions:



More than 50 units in Pod



Between 50 and 26 units in Pod



Between 25 and 5 units in Pod



Less than 5 units in Pod



PDM unable to communicate with Pod; units in Pod shown as '-' when unable to communicate



Flight mode on



Vibration mode on



Bluetooth® setting on



Temp basal rate active



No active Pod

The PDM's battery icons, during normal operation and while charging, are:



100% to 31% charge



30% to 16% charge



15% to 0% charge

For charging instructions, see 'Charge the battery' on page 21.

1 Omnipod DASH® Insulin Management System


The PDM's Lock and PIN Screens

After you set up your PDM, the Lock and PIN screens appear whenever you wake up your PDM. The Lock and PIN screens help confirm that you are using the correct PDM.

The Lock screen displays:

- Your selected background image
- Your customised message
- The amount of insulin on board, if the Bolus Calculator is on
- Today's date
- Any alarm or notification messages

Warning: Always identify the PDM as yours before using it. Using someone else's PDM can result in incorrect insulin delivery for both of you.

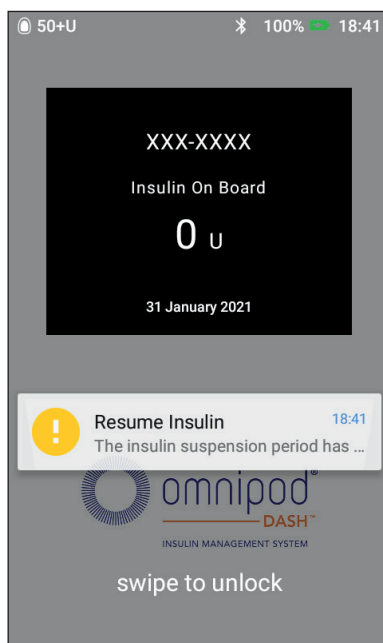

Unlock your PDM

In the remainder of this *Technical User Guide*, instructions to 'wake up' or 'unlock' the PDM mean to do the following:

1. Press the Power button briefly. See the PDM image on page 3 for the location of the Power button.
2. Unlock the Lock screen by either swiping left to right or by swiping up from the bottom. The PIN screen appears.
3. Enter your 4-digit PIN.
4. Tap the tick symbol. The Home screen or your most recent screen appears.

Note: If a hazard alarm or an advisory alarm is occurring, the alarm message appears.

Lock your PDM

To lock your PDM when you have finished using it:

1. Press the Power button briefly. This locks the PDM by putting it to sleep.
2. Store your PDM in a safe, accessible location.

Caution: Only press the Power button briefly. If the PDM asks if you would like to 'Power off', tap outside the message to cancel the instruction. Once you begin using your PDM, do not turn the power off. The PDM can sound an alarm only when powered on.

Forgotten your PIN?


You can use the last four digits of the PDM's serial number as an alternative PIN:

1. Find the serial number next to the **SN** icon on the back of your PDM.
2. Note the last four digits of the serial number.
3. Enter those four digits on the PIN screen and tap the tick symbol.

If you have problems with your PIN, call Customer Care.

The PDM's Home screen

The Home screen provides information about recent boluses, blood glucose values, the active basal rate and the active Pod. It also provides access to the PDM functions, alarm information and settings. The Home screen also has a Bolus button to give quick access to the bolus screen.

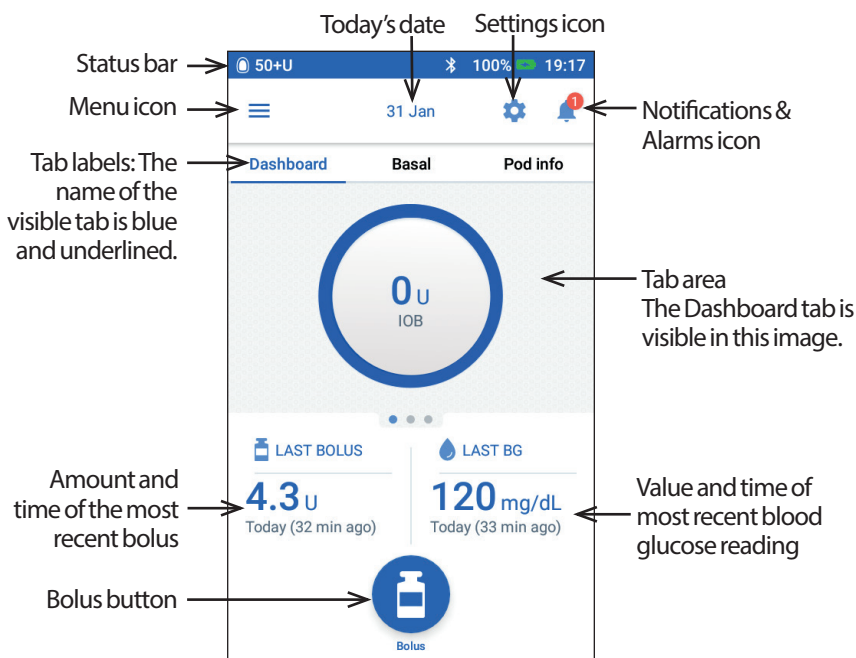

1 Omnipod DASH® Insulin Management System

The tab area of the Home screen can display bolus information on the Dashboard, basal information or Pod information. To change which information is displayed:

- Tap Dashboard, Basal or Pod Info to display each tab.
- Swipe right or swipe left in the middle section to move between the tabs.


Note: If an immediate bolus is running, a bolus progress bar is shown on the Home screen, and the Bolus button and three tabs are not visible (see 'Immediate bolus progress' on page 65). A button to cancel the bolus is visible.

Dashboard tab

The Dashboard tab is available if the Bolus Calculator is on and is usually not available if the Bolus Calculator is off.

When the Bolus Calculator is on


When the Bolus Calculator is on, the Dashboard tab displays:

- IOB (insulin on board), when IOB is available.
- IOB is not available, when the Bolus Calculator is disabled. Tap the  icon for a list of reasons for the Bolus Calculator to be disabled.

A progress bar appears when a bolus is in progress (see 'Tracking the Progress of a Bolus' on page 65).

When the Bolus Calculator is off

When the Bolus Calculator is off, there are usually only two tabs: Basal and Pod Info. The Dashboard tab displays only when an extended bolus is running (see 'Tracking the Progress of a Bolus' on page 65).

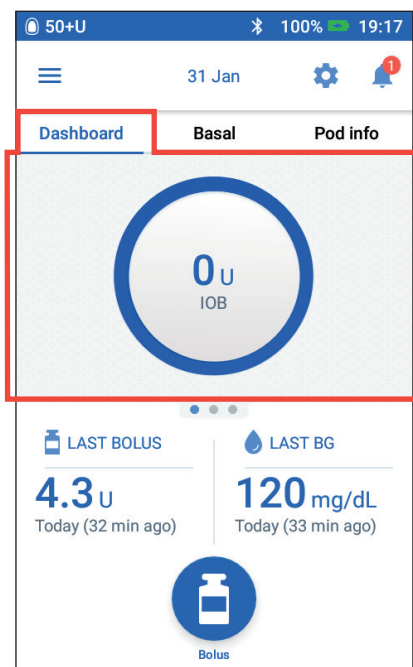

Basal tab / Temp Basal tab

This tab is labelled Basal or Temp Basal, depending on whether a Basal Programme or temp basal is active.

Basal Programme

By default, the Basal tab shows the name and graph of the active Basal Programme.

A label below the Basal Programme's name indicates whether the listed Basal Programme is:

- **Active** – This programme is running on the active Pod.
- **Suspended** – This program will resume when you resume insulin delivery.
- **Current** – There is no active Pod. This programme will be sent to your next Pod during activation.

When there is an active Basal Programme, a green line marks the current time. The numbers above the graph indicate the basal rate for each time segment.


The total daily amount of basal insulin is shown beneath the graph.

This total is the amount of insulin that the active Basal Programme is scheduled to deliver in a 24-hour period. This total does not account for temp basals or boluses. To see the total amount of insulin that your Pod has actually delivered, including temp basals and boluses, see 'Overview of the Insulin and BG History Screens' on page 84.

Tap **VIEW** to see details about other Basal Programmes (see page 75).

Temp basal


If a temp basal is running, the Basal tab is labelled Temp Basal and is highlighted in green. It displays the temp basal graph and allows you to cancel the temp basal.

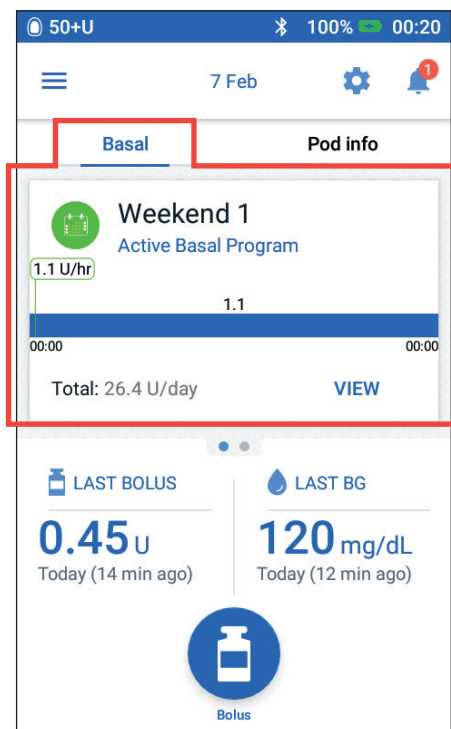

1 Omnipod DASH® Insulin Management System

Pod Info tab

The Pod Info tab shows whether there is an active Pod and, if so, how much insulin is left in the Pod and when the Pod will expire.

If the PDM cannot communicate with an active Pod, the Pod Info tab says 'No Pod Communication'. Tap the  icon for a list of reasons why the PDM may not be able to communicate with the Pod.

Note: Even when there is no communication, the Pod continues delivering insulin according to the instructions it was given. See 'What the Pod can do between PDM instructions' on page 137 for details.


Tip: Bringing the PDM and the active Pod within 1.5 metres (5 feet) of one another may quickly restore their communication.



If there is no active Pod, the Pod Info tab says 'No active Pod'. To set up a new Pod, see 'Changing Your Pod' on page 37.

If there is an active Pod, tap VIEW POD DETAILS to bring up a screen that lets you deactivate or change your Pod.


The VIEW POD DETAILS screen shows:

- Amount of insulin in the Pod.
- Time of the last successful communication between the PDM and Pod.
- Date and time of the Pod's expiry.
- Any active reminders.
- A CHANGE POD button.


Tip: You can also access this screen by tapping the Menu icon () > Pod.

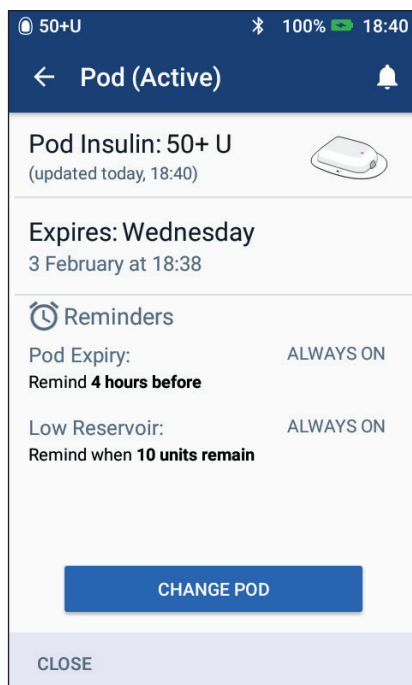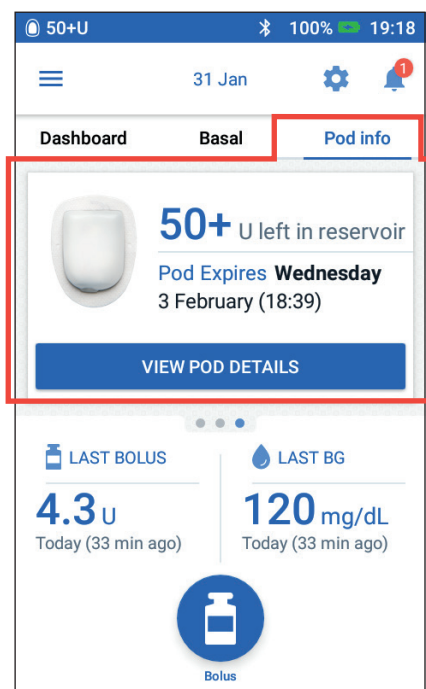

When your Pod is going to expire soon, a yellow Change Pod Soon banner appears on the Pod Info tab. The yellow banner appears 12 hours before Pod expiry or at the time of your Pod expiry reminder, whichever is earlier.

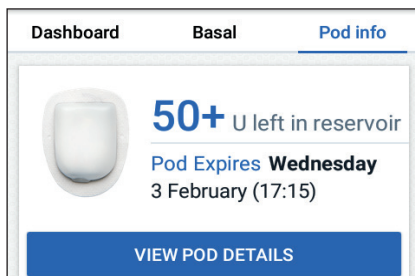


At six hours before Pod expiry, a red Change Pod banner appears on the Pod Info tab.

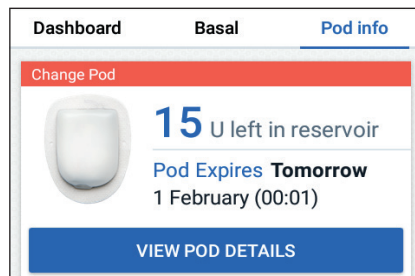


When less than five units of insulin remain in the Pod, a red Low Reservoir banner appears on the Pod Info tab. If the Pod is going to expire soon and there are less than five units of insulin in the Pod, the red Low Reservoir banner is displayed on the Pod Info tab.

Note: If you change the date or time, the Pod expiry date and time will be adjusted to match the new date and time.

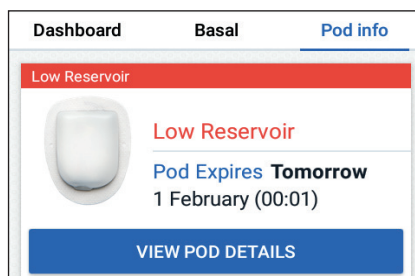

Last BG section

The lower right side of the Home screen displays the value and time of your most recent blood glucose reading.

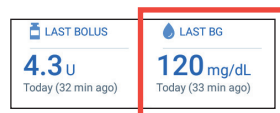

1 Omnipod DASH® Insulin Management System

Last bolus section

The lower left side of the Home screen displays bolus information.


Between boluses

When a bolus is not being delivered, the section is labelled LAST BOLUS and shows the amount and time of the most recent bolus.

During a bolus when the Bolus Calculator is on

When a bolus is being delivered and the Bolus Calculator is on, the last bolus information is replaced by an estimate of the insulin on board (IOB):

- During an immediate bolus, the IOB estimate is updated every ten seconds.
- During an extended bolus, the IOB estimate includes IOB from previous boluses, an estimate of the amount already delivered from the ongoing bolus and the amount projected to be delivered within the time period defined by your duration of insulin action setting.

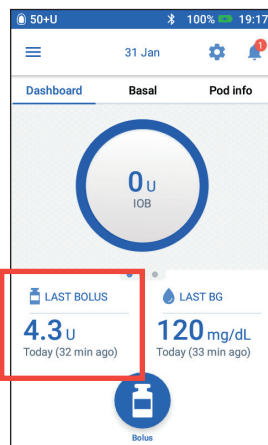

During a bolus when the Bolus Calculator is off

When a bolus is being delivered and the Bolus Calculator is off, the lower left of the screen shows the amount of the preceding bolus (not the ongoing bolus).

Bolus information if the PDM cannot communicate with the Pod

If the Pod is out of range of the PDM and the PDM cannot confirm the recent bolus amount, an estimated bolus amount is shown. Once the Pod is in range again and the bolus delivery is confirmed, the confirmed bolus amount is shown.

Estimated and unconfirmed bolus amounts

The PDM estimates bolus amounts during an ongoing bolus and when the Pod is out of range. A grey icon (ⓘ) marks estimated bolus amounts. A yellow icon (⚠) marks unconfirmed bolus amounts (see 'When the Pod has not confirmed a bolus delivery' on page 91).

Bolus button

The Bolus button at the bottom of the Home screen provides access to the bolus screen. The Bolus button does not appear while an immediate bolus is being delivered, or when the Bolus Calculator is off and there is no active Pod.



The Home Page's Menu

The Menu on the Home page lets you access most of the PDM's functions. To display the Menu:

- Tap the Menu icon (☰) in the upper left corner of the Home screen.
- Place your finger on the far left of the PDM and swipe right across the screen. Swipe left to hide the Menu.

Tap an option on the Menu to bring up the related screen.

Tip: *The Menu extends beyond the bottom of the screen. Swipe up or down to see all parts of the Menu.*

Frequent tasks

The first section of the Menu gives access to frequent tasks: activating a temp basal, changing your Pod, entering a blood glucose reading and suspending or resuming insulin.

Special situations:











- Set Temp Basal is greyed-out if there is no active Pod, insulin is suspended or a temp basal is already running.
- Set Temp Basal does not appear if the temp basal setting is turned off.
- Suspend Insulin changes to Resume Insulin if insulin delivery is suspended.
- Suspend Insulin is greyed-out if there is no active Pod.

Manage Programmes & Presets

The next section allows you to create or activate Basal Programmes, temp basal presets and bolus presets (see 'Manage Programmes and Presets' on page 75).

Special situations:

- Temp Basal Presets is only visible if the temp basal setting is on.
- Bolus Presets is only visible if the Bolus Calculator setting is off.



	Set Temp Basal
	Pod
	Enter BG
	Suspend Insulin
MANAGE PROGRAM & PRESETS	
	Basal Programs
	Temp Basal Presets
	Bolus Presets
	History ^
	Notifications & Alarms
	Insulin & BG History
	Settings ^
	PDM Device
	Reminders
	Blood Glucose
	Basal & Temp Basal
	Bolus
	About

1 Omnipod DASH® Insulin Management System

History menu

Tap the down arrow (∨) next to History to reveal the sub-menu:

- Notifications & Alarms history (see page 83)
- Insulin & BG History (see page 84)

Tip: Tap the bell icon ( or ) at the top right of many screens to open the Notifications & Alarms history screen.

Settings menu

Tap the down arrow (∨) next to Settings to reveal the sub-menu:

- PDM Device
- Pod Sites
- Reminders
- Blood Glucose
- Basal & Temp Basal
- Bolus

Tip: Tap the Settings icon () at the top of the Home screen to open the Settings portion of the Menu.

These settings allow you to change the time and date, create or remove reminders, turn the Bolus Calculator on or off, and much more. For more information, see 'Adjusting Settings' on page 93.

About screen

The About screen displays details about your PDM, such as the software revision number, the PDM serial number, the Pod version number, the time of the most recent PDM-Pod communication, and other device and legal information.

Note: You can also find the PDM's serial number next to the  icon on the back of your PDM.

PDM Messages to You

The PDM can deliver alarm messages, notifications and confirmation messages.

Alarms

Hazard alarms and advisory alarms require your immediate attention (see 'Alarms, Notifications, and Communication Errors' on page 109). If you ignore an alarm, you could develop hypoglycaemia or hyperglycaemia.

Hazard alarms (⚠️) are accompanied by a continuous vibration or tone and interrupt anything else the PDM is doing except another hazard alarm. For example, a hazard alarm is issued if the Pod runs out of insulin.

Advisory alarms (⚠️) alert you to some aspect of the PDM or Pod that needs your attention in the near future. For example, if the level of insulin in your Pod is getting low, the PDM issues an advisory alarm.

Notifications


Notifications (🕒) remind you about actions you may want to perform (see 'Notifications List' on page 117). For example, you can set a reminder to tell you to measure your blood glucose after a specified amount of time.

Confirmation messages

The PDM confirms that an instruction was successful by displaying a green banner with a confirmation message. The confirmation message disappears after several seconds.

Tip: *Swipe to the right on the message to dismiss it sooner.*

If an instruction is not successful, then the PDM displays a communication error message (see 'Communication Errors' on page 120).

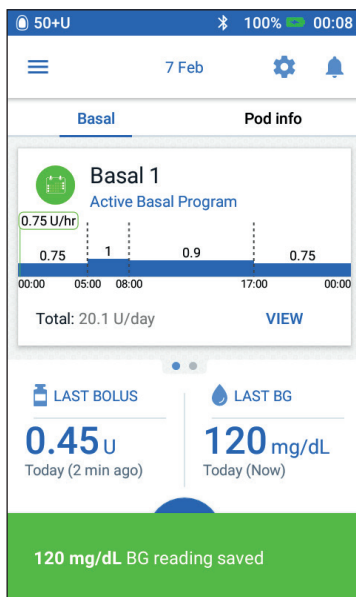

This page intentionally left blank.

CHAPTER 2

Initial PDM Set-up

This chapter guides you through setting up your first PDM or a replacement PDM.

If you are setting up a replacement PDM, go to 'Charge the battery' within the following section.

Preparing for Your Training

If you are a first-time Omnipod DASH® System user, you need to meet with your nurse/educator to set up your Personal Diabetes Manager (PDM) and first Pod.

If you have used an Omnipod® Insulin Management System in the past and are upgrading to the Omnipod DASH® System, review the online training module before setting up your new system.

To get a head start on learning about the Omnipod DASH® System, review this *Technical User Guide*, including the 'Introduction' on page xi, 'Your Omnipod DASH® Insulin Management System' on page 1 and 'Taking Care of Your PDM and Pod' on page 125.

Warnings:

Do NOT use the Omnipod DASH® System before you have been trained. Inadequate training could put your health and safety at risk.

Follow your healthcare provider's instructions for initialising the PDM. Improper set-up could put your health and safety at risk.

Note: Keep an emergency kit with you at all times to quickly respond to a diabetes emergency (see 'Prepare for Emergencies' on page 165).

Charge the battery

It is important to charge the PDM's battery before setting up your Omnipod DASH® System:

1. Install the rechargeable battery in the PDM, if necessary (see 'Inserting the PDM battery during first time setup' on page 132).
2. Assemble the charger by plugging the charger cable's USB Type A plug into the charger's wall adapter.

2 Initial PDM Set-up

Caution: Use ONLY the Insulet-provided micro-USB charger to charge your PDM. Using unapproved chargers can cause the battery to explode or damage the PDM and may void the warranty.

3. Plug the charger into a wall socket. Select an outlet that you can access easily and unplug the charger from easily when necessary.
4. Plug the charger cable's USB Micro-B plug into the PDM's Micro-B USB port.
5. Charge the PDM until the battery level icon shows 100% charge (see 'The Status Bar' on page 9).
6. Disconnect the charger from the PDM and the wall socket.

Caution: Use ONLY the rechargeable battery that came with your PDM. Contact Customer Care if you have questions.


Items needed when you meet with your nurse/educator

- Your PDM
- Two Pods
- *Omnipod DASH Instructions for Use*
- A blood glucose meter
- Test strips and a lancing device (available from many pharmacies)
- Vial of rapid-acting U-100 insulin
- Alcohol prep swabs
- Instructions from your healthcare provider with PDM settings tailored to your needs. These settings include Basal Programme, IC Ratio, Correction Factor, Target BG values and Duration of Insulin Action.

General PDM Settings

Although PDM set-up is easy, if you are a first-time user, your nurse/educator must guide you through the set-up process. You can adjust these settings later as needed.

Tip: Use the pages at the end of this *Technical User Guide* to write down all of your settings. This list will be helpful if you ever need to reset or replace your PDM.

Note: Tapping the back arrow () returns you to the previous screen. However tapping the CANCEL button in any of these set-up steps takes you to the first screen of each section and erases any entries in that section. A pop-up screen warns you that you could lose these entries.

Turning on and personalising the PDM

1. Turn on the PDM by pressing and holding the Power button on the right side until the Omnipod® logo appears. The PDM runs through a series of checks.

Caution: Do not turn off the PDM power. The PDM must be ON in order to sound an alarm. If you press the Power button for too long, the PDM will display a menu with a Power Off option. Do NOT tap Power Off. Tap outside the menu to dismiss it and to keep the PDM on.

Note: You can press the Power button briefly to toggle the PDM between the wake and sleep modes. Omnipod DASH® System alarms will also wake a PDM from sleep mode.

2. When a screen with the Omnipod DASH® logo appears, use your finger to swipe from left to right across the screen.

Note: If you see an unexpected screen instead of the Omnipod DASH® logo screen, see 'Troubleshooting PDM Start-up' on page 177.

3. Tap a language to select your preferred language for the PDM screens. Then tap the arrow (>).
4. Tap Select One to display the list of countries. Select your country, then tap CONTINUE.
5. Review the terms and conditions, including the End User Licence Agreement (EULA), Warranty and Legal Notices, as follows:
 - a. Tap the EULA, Warranty and Legal Notices link to read the Omnipod DASH® System's legal notices. Then tap AGREE.
 - b. Tap the box to add a tick symbol.
 - c. Tap CONTINUE.

Note: If you are under 18, your parent or guardian must accept for you.

6. Tap BEGIN SET-UP.
7. Tap letters on the keypad to enter a personalised screen message (see 'Selecting, Adding and Deleting Items' on page 7). Then tap the tick symbol.

Note: You must enter at least one character.

The screenshot shows a screen titled "Personalise Your PDM". Below the title is the heading "Lock Screen Message". There is a large rectangular text input field with a vertical cursor. Below the input field, a note states: "This message displays on your PDM when it is locked. (e.g. Alexandra's PDM. If found call 555-555-5555)". At the bottom of the screen is a QWERTY keyboard with a numeric row (1-0), letters (Q-Z), a spacebar, and a checkmark button. The input field currently contains a single vertical line, indicating the start of a new message.

2 Initial PDM Set-up

Tip: Including a contact phone number in the message could help in recovering a misplaced PDM.

8. Tap CONTINUE.

A screen appears showing several background images.

9. Swipe right or left to see more images. Tap on your preferred image, then tap CONTINUE.

Note: When you wake up your PDM, you will see your personalised screen message and background image. Always confirm that the PDM is yours before using it.


Enter a PIN

To protect against unintentional screen touches, you must create a 4-digit personal identification number or PIN.

1. Choose four numbers to be your PIN. You will use this PIN number every time you wake up your PDM. You may want to record the PIN in a safe place.

Tip: To keep the PIN visible, tap the eye icon located to the right of the PIN entry fields. To hide the number, tap the eye icon again.

2. On the number pad, tap each number of your PIN. Then tap the tick symbol.
 3. Enter the same four numbers again to confirm your PIN. Tap the tick symbol.
- If the second PIN entry does not match the first, you must repeat the above steps.

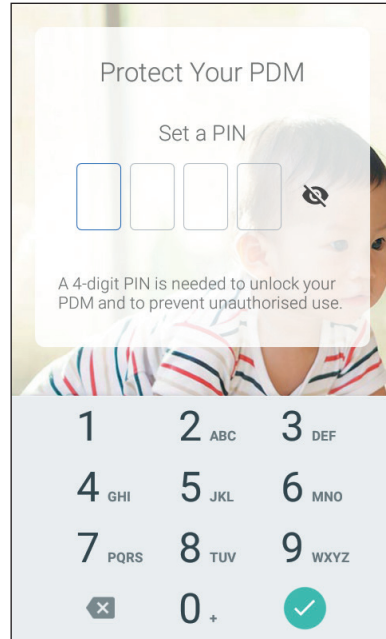


Set the time zone and time

To set the current time zone and time:

1. Tap **Select One** to show the list of time zones.
2. Tap your time zone. You may need to swipe up or down to find your time zone.
3. Tap **CONTINUE**.
4. Tap the 12 hr / 24 hr toggle to choose your preferred time display. For example, six hours before midnight is:
 - 18:00 in 24 hr display
 - 6:00 PM in 12-hr display
5. Place your finger on the Hour scroll wheel. Slide your finger up or down to scroll the wheel up or down. Stop scrolling when the correct hour appears in the centre of the wheel.
6. Scroll the Minute scroll wheel up or down until the correct minute appears in the centre of the scroll wheel.
7. If using 12-hr time, scroll the right-most field and select AM or PM.
8. Review the time and tap **CONTINUE**.

Caution: Check that you have set the time correctly. The time setting affects the operation of several Omnipod DASH® System features and can impact your insulin delivery.

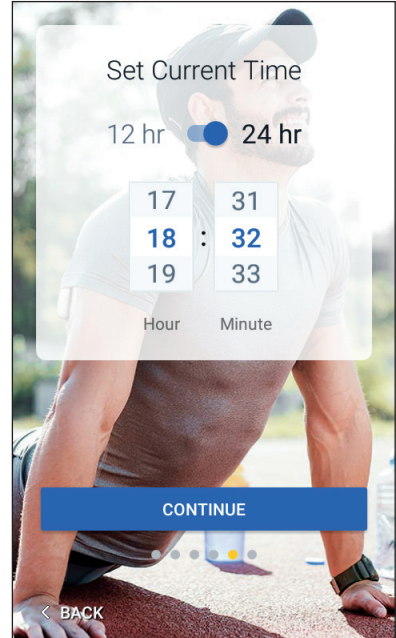


2 Initial PDM Set-up

Set the date

On the Set Current Date screen:

1. Tap DD/MM/YYYY to display a list of date formats. Tap the desired date format.
2. Set the Day, Month and Year fields by scrolling their respective wheels one at a time.
3. Review the date and date format, then tap CONTINUE.
4. On the Confirm Time & Date screen, review the time zone, time and date. If correct, tap CONTINUE.

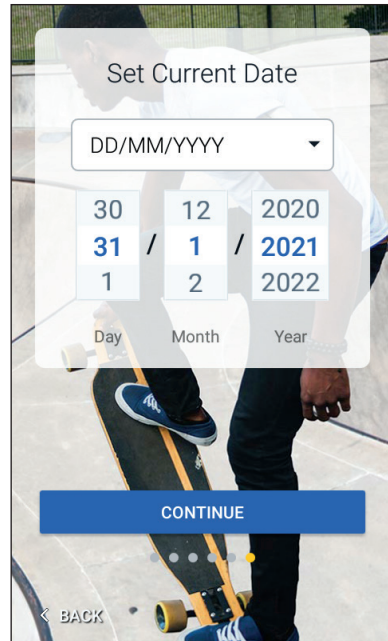

Basal Insulin Settings

Maximum Basal Rate


The maximum basal rate sets the upper limit of any basal insulin rate that you can use.

1. Tap SET UP PROFILE.
2. Review the description of basal insulin, then tap the arrow (>).
3. Tap the Max Basal Rate field.
4. Scroll to your desired maximum basal rate. When the correct number is in the centre of the scroll wheel, tap the number to select it.

Tip: Alternatively, tapping outside of the scroll wheel selects the value in the centre of the scroll wheel.

5. Tap NEXT.

You can adjust your maximum basal rate later, if your needs change.

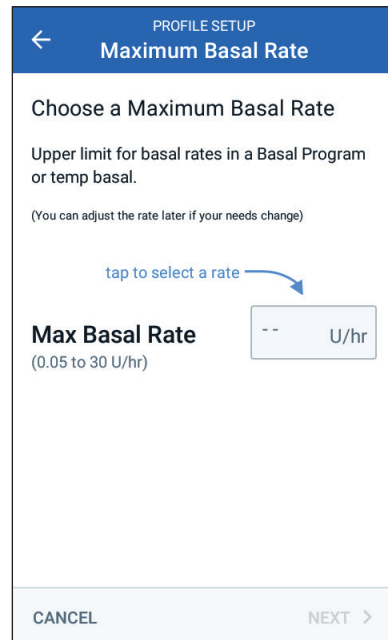

Create a Basal Programme

The next step is to define a Basal Programme. For a description of basal rates, basal segments, and Basal Programmes, see 'Basal Insulin Delivery' on page 138.

Name and tag the Basal Programme

1. Review the description of Basal Programmes, then tap NEXT.
2. The default name for the Basal Programme is Basal 1. To change the name, tap the Programme Name field and enter the new name (see 'Selecting, Adding and Deleting Items' on page 7) and then tap the tick symbol.
3. Optional: To add a visual icon to your Basal Programme, tap one of the tags. Tap the tag a second time to deselect it.
4. Tap NEXT.

Define the first segment

1. The Start Time for the first segment is always midnight. Tap the End Time field and scroll to select the desired end time.
2. Tap the Basal Rate field and scroll to select the basal rate for the segment.

PROFILE SETUP
Basal Program

Basal 1: Segment 1 Graph: U/hr

00:00 00:00

Start Time End Time

00:00 - --

Night

Basal Rate -- U/hr

(up to 3 U/hr)

CANCEL NEXT >

PROFILE SETUP
Basal Program

Basal 1: Segment 1 Graph: U/hr

0.75

00:00 05:00 00:00

Start Time End Time

00:00 - 05:00

Night Night

Basal Rate 0.75 U/hr

(up to 3 U/hr)

CANCEL NEXT >

The Maximum Basal Rate that you entered earlier is displayed under the Basal Rate text. You cannot enter a basal rate greater than this number.

2 Initial PDM Set-up

Note: The two vertical blue lines on the graph near the top of the screen show the start and end time for the basal segment. The selected basal rate for the segment is shown between the two vertical lines.

- 3. Check the values of your start and end times and the basal rate. Then tap NEXT.

Define additional segments

If the Basal Programme does not cover 24 hours, you must add additional segments:

- 1. Tap the End Time field and select the duration of the next segment.
- 2. Tap the Basal Rate field and select the basal rate for the segment.
- 3. Review the segment details and the graph.
- 4. Tap NEXT to move to the next segment.
- 5. Repeat the preceding steps as needed until your final segment ends at midnight.


After specifying a basal rate for the final segment, your Basal Programme is complete.

Review the Basal Programme

The next screen summarises the start and end times, duration and basal rate for each segment of the Basal Programme.

- 1. Tap CONTINUE to review your Basal Programme.
- 2. Check that the graph and the individual segment values are correct.
The total daily amount of basal insulin to be delivered by this Basal Programme is listed below the graph.

- 3. To change an end time or basal rate for a segment:
 - a. Tap the row containing the segment you would like to change.
 - b. Tap the End Time field, and enter the new end time for the segment.
 - c. Tap the Basal Rate field and enter the desired basal rate.
 - d. Tap NEXT.
 - e. Then define the end time and basal rate for any following segments, as needed.

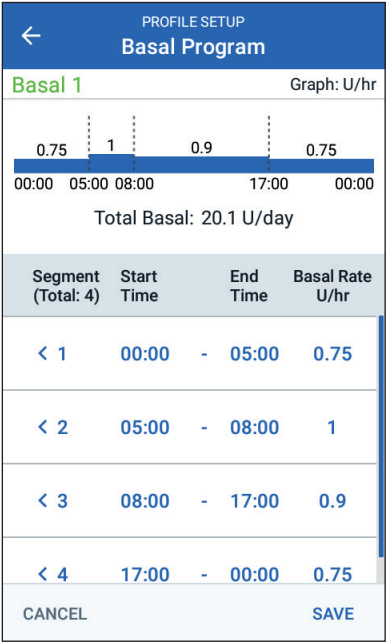

4. To add a new segment:
 - a. Tap the row containing the start time of the new segment.
 - b. Tap the End Time field and enter the start time of the new segment as the end time of this segment.
 - c. Change the basal rate, if necessary.
 - d. Tap NEXT.
 - e. Then define the end time and basal rate for any following segments, as needed.
5. To delete a segment:
 - a. Note the end time of the segment you want to delete.
 - b. Tap the segment that precedes the segment you want to delete.
 - c. Tap the End Time field and enter the end time of the segment you want to delete. This 'overwrites' the segment you want to delete.
 - d. Tap NEXT.
 - e. Then define the end time and basal rate for any following segments, as needed.
6. When the Basal Programme is correct, tap SAVE.

Note: If the basal rate for a segment is 0 U/hr, the PDM displays a message calling this to your attention. Tap OK if the 0 U/hr rate is correct. Otherwise, tap CANCEL and edit the segment with the 0 U/hr rate.

Note: To create additional Basal Programmes after set-up is complete, see page 75.

Temp basal configuration

For a description of temporary basal rates, also called temp basals, see page 140.

1. If you want the ability to use temp basals, tap the toggle to the 'on' position. The toggle is in the 'on' position when it is in the right-hand position and is blue.

If you turn temp basals on, percentages are used by default. To specify temp basals as a flat rate (U/hr), see 'Temp basal' on page 105.
2. Tap NEXT.

2 Initial PDM Set-up

Blood Glucose Settings

The blood glucose settings define what blood glucose readings are considered in range.

Upper and lower limits for BG Goal Range

You can set the upper and lower limits of your BG Goal Range by using the number pad or the slider:

1. Review the description of blood glucose levels, then tap the arrow (>).
2. To define the lower limit, tap the Lower Limit field and type the number using the number pad.

Tip: Alternatively, you can touch the lower (red) circle with your finger and slide it up or down.

3. To define the upper limit, tap the Upper Limit field or slide the upper (yellow) circle.
4. Review your upper and lower limits, and tap NEXT to save them.

← PROFILE SETUP
BG Goal Range

Set upper and lower BG limits

These limits are used in BG history displays and in BG reading statistics.

Upper Limit 140 mg/dL

Lower Limit 70 mg/dL

CANCEL NEXT >

Bolus Calculator Settings

The Bolus Calculator suggests boluses based on your current blood glucose value and any food you are about to eat (see 'The Bolus Calculator' on page 145).

Turn the Bolus Calculator on or off

1. Review the description of insulin boluses, then tap the arrow (>).
2. Toggle the Bolus Calculator setting on or off:
 - Select the On position (blue and to the right) to have the Bolus Calculator suggest boluses.
 - Select the Off position (grey and to the left) to turn off the Bolus Calculator. You will have to calculate your own bolus amounts.
3. Tap NEXT.
4. If you turned the Bolus Calculator off, skip to 'Other Bolus Settings' on page 35.

Target BG and Correct Above Values

The Bolus Calculator aims to bring your blood glucose to the Target BG value if your current blood glucose level is above the Correct Above value. You can establish up to eight different blood glucose targets for different times of day.

Define the segments

Setting values for each segment follows the same pattern that you used to create your Basal Programme:

1. Review the description of Target BG and Correct Above, then tap NEXT.
2. Tap the End Time field and enter an end time for the segment.
3. Tap the Target BG field and specify the Target BG for that segment.
4. Tap the Correct Above field and specify the correct above value for that segment.
5. Tap NEXT.
6. Repeat the above steps as needed until you have specified values for the segment that ends at midnight.

The screenshot shows a mobile app interface for setting blood glucose targets. At the top, a blue header bar contains a back arrow, the text 'PROFILE SETUP', and the title 'Target BG & Correct Above'. Below the header, 'Segment 1' is highlighted in green. The main area has two rows of settings. The first row is for 'Night' and includes a 'Start Time' field set to '00:00' and an 'End Time' field with a minus sign and three dashes. The second row includes a 'Target BG' field with a range '(70 to 200 mg/dL)' and a 'Correct Above' field with a range '(Target BG to 200 mg/dL)'. Both fields have input boxes showing '--' followed by 'mg/dL'. At the bottom, there are two buttons: 'CANCEL' on the left and 'NEXT >' on the right.

2 Initial PDM Set-up

Review the Target BG and Correct Above values

- 1. Review the segments for the full 24-hour profile.
- 2. To change any of the entries:
 - a. Tap the row containing the entry to be changed and enter the corrected value.
 - b. Review and correct any remaining segments as needed.
- 3. When the segments and values are correct, tap **SAVE**.

←

PROFILE SETUP

Target BG & Correct Above

Review the values entered for each time segment.

Start Time	End Time	Target BG mg/dL	Correct Above mg/dL
< 00:00	- 18:00	100	110
< 18:00	- 00:00	105	115

CANCEL

SAVE

Minimum BG for Calcs

If your blood glucose level is below the Minimum BG for Calcs that you specify, the Bolus Calculator is disabled (see 'The Bolus Calculator' on page 145).

- 1. To enter your Minimum Glucose for calcs, tap the Min BG for Calc field and enter the number on the number pad.
Tip: Alternatively, touch the blue circle and slide it to the desired number.
- 2. Tap **NEXT**.

←

PROFILE SETUP

Min BG for Calcs

Set Min BG for Calcs

If your blood glucose is below your Minimum BG for Calculations (Min BG for Calcs), the Bolus Calculator is disabled so that it does not suggest a bolus.

Min BG for Calcs

(50 to 70 mg/dL)

70 mg/dL

70

65

60

55

50

CANCEL

NEXT >

Insulin to carb ratio

Your insulin to carbohydrate ratio, or 'IC ratio', defines how many carbohydrates are covered by one unit of insulin. The Bolus Calculator uses the IC ratio to calculate the meal portion of a suggested bolus. You can create up to eight IC ratio segments per day.

1. Review the description of IC ratio, then tap NEXT.
2. Enter the end time.
3. Enter the IC ratio value in the 1 Unit of Insulin Covers field.
4. Tap NEXT.
5. Add additional segments as necessary until the final end time is midnight.
6. Review your 24-hour IC ratio segments. Tap any entry to correct it.
7. When the segments and values are correct, tap SAVE.

PROFILE SETUP
Insulin to Carb Ratio

Segment 1

Start Time

00:00

End Time

--

Night

1 Unit of Insulin Covers

-- g

(1 to 150 g carbs)

CANCEL

NEXT >

Correction Factor

Your correction factor defines how much one unit of insulin lowers your blood glucose level. The Bolus Calculator uses the correction factor to calculate the correction portion of a suggested bolus. You can create up to eight correction factor segments per day.

1. Review the description of correction factor, then tap NEXT.
2. Enter the end time.
3. Enter the correction factor in the 1 Unit of Insulin Decreases BG by field.
4. Tap NEXT.
5. Add additional segments as necessary until the final end time is midnight.

PROFILE SETUP
Correction Factor

Segment 1

Start Time

00:00

End Time

--

Night

1 Unit of Insulin Decreases BG by

-- mg/dL

(1 to 400 mg/dL)

CANCEL

NEXT >

2 Initial PDM Set-up

6. Review your correction factor segments. Tap any entry to correct it.
7. When the segments and values are correct, tap **SAVE**.

Reverse Correction

There may be times when you are about to eat and your blood glucose level is below your Target BG level. If reverse correction is on, the Bolus Calculator decreases a suggested meal bolus to help counteract your low blood glucose.

1. Toggle the Reverse Correction button on or off:
 - Select the On position if you want the Bolus Calculator to reduce the suggested meal bolus when your blood glucose is below your Target BG.
 - Select the Off position if you do not want the Bolus Calculator to adjust the suggested meal bolus based on a low blood glucose value.
2. Tap **NEXT**.

Duration of Insulin Action

The duration of insulin action is the length of time that insulin stays active in your body. The Bolus Calculator uses this setting to determine how much insulin remains in your body from previous boluses (called insulin on board or IOB).

1. Tap the Duration of Insulin Action field and use the scroll wheel to specify your duration of insulin action.
2. Tap **NEXT**.

Other Bolus Settings

The remaining bolus settings are used whether the Bolus Calculator is on or off.

Maximum Bolus

The PDM will not let you request a bolus above the maximum bolus setting. You will see a message if the Bolus Calculator calculates a bolus that is above this amount.

1. Tap the Max Bolus field and use the number pad to specify your maximum bolus. Tap the tick symbol to close the number pad.
2. Tap NEXT.

Extended bolus setting

Extending a bolus means that a meal bolus can be delivered over a prolonged period of time. For a description of the extended bolus feature, see page 144.

1. Toggle the Extended Bolus button to turn the extended bolus feature on or off.
2. Tap COMPLETE.
The PDM restarts.

PDM Set-up Complete

Congratulations! Your PDM is now ready for use. When you are ready to activate your first Pod, go to 'First Time Activating a Pod' on page 38.

Tip: *To adjust the PDM screen time-out or screen brightness, see 'Screen display' on page 94.*

Note: Confidence reminders and programme reminders are turned on by default. These reminders cause the PDM or Pod to beep at the beginning and end of boluses and temp basals, and also once an hour during a long-lasting bolus or temp basal. For more information, see 'Confidence reminders' and 'Programme reminders' on page 102.

This page is intentionally left blank.

CHAPTER 3

Changing Your Pod

Beginning the Pod Change Process

Changing your Pod consists of deactivating your current Pod and activating a new Pod.

Your Pod should be changed at least once every 48 to 72 hours (2 to 3 days) or after delivering 200 units of insulin. Consult with your healthcare provider and refer to the insulin labelling to determine if you should change your Pod more often.

Warnings:

Do NOT use a Pod if you are sensitive to or have allergies to acrylic adhesives, or have fragile or easily damaged skin.

Because the Pod uses only rapid-acting U-100 insulin, you are at increased risk for developing hyperglycaemia if insulin delivery is interrupted. Severe hyperglycaemia can quickly lead to diabetic ketoacidosis (DKA). DKA can cause symptoms such as abdominal pain, nausea, vomiting, breathing difficulties, shock, coma or death. If insulin delivery is interrupted for any reason, you may need to replace the missing insulin. Ask your healthcare provider for instructions for handling interrupted insulin delivery, which may include an injection of rapid-acting insulin.

The Pod and its accessories, including the needle cap, contain small parts that may be dangerous if swallowed. Be careful to keep these small parts away from young children.

The steps for deactivating or activating a Pod depend on whether you have an active Pod or not:

- If this is your first time activating a Pod, go to 'First Time Activating a Pod' on page 38.
- If you currently have an active Pod, go to 'Deactivate an Active Pod' on page 38.
- If you have already deactivated your most recent Pod, go to 'No Active Pod' on page 39.

3 Changing Your Pod

First Time Activating a Pod

Your nurse/educator will help you set up your first Pod.

After completing the PDM set-up flow, the Home screen displays 'No Active Pod'.

1. Tap SET UP NEW POD.
2. Go to 'Preliminary Pod Change Steps' on page 40.

Deactivate an Active Pod

To deactivate and remove an active Pod:

1. Navigate to the Pod change screen:
Home > Pod Info tab > VIEW POD DETAILS

or


Menu icon (☰) > Pod

2. Tap CHANGE POD, then tap DEACTIVATE POD.

If a temp basal or extended bolus was running, it will be cancelled now.

If you see a communication error message, see 'Error when deactivating a Pod' on page 122.

Warning: Do NOT apply a new Pod until you have deactivated and removed the old Pod. A Pod that has not been deactivated properly can continue to deliver insulin as programmed, putting you at risk of over-infusion and possible hypoglycaemia.

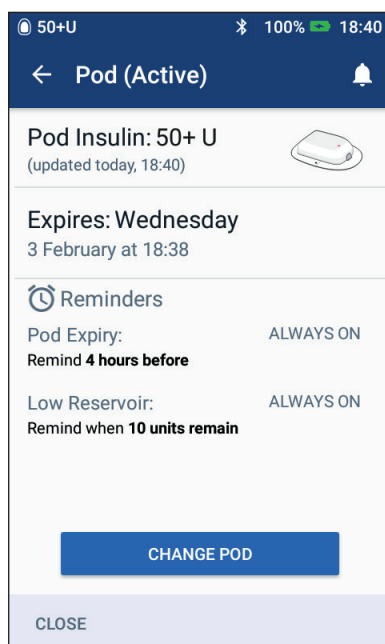

3. When the green banner tells you to remove your Pod, remove the deactivated Pod from your body:

- a. Gently lift the edges of the adhesive tape from your skin and remove the entire Pod.

Tip: Remove the Pod slowly to help avoid possible skin irritation.




- b. Use soap and water to remove any adhesive that remains on the skin, or if necessary, use an adhesive remover.
 - c. Check the infusion site for signs of infection (see 'Avoid Infusion-Site Infections' on page 49).
 - d. Dispose of the used Pod according to local waste disposal regulations.
4. To activate a new Pod, tap SET UP NEW POD.
 5. Go to 'Preliminary Pod Change Steps' on page 40.

No Active Pod

If you have no active Pod and are ready to activate a new Pod:

1. Navigate to the Pod change screen:
Home > Pod Info tab > SET UP NEW POD
or
Menu icon (☰) > Pod > SET UP NEW POD
2. Go to 'Preliminary Pod Change Steps' on page 40.

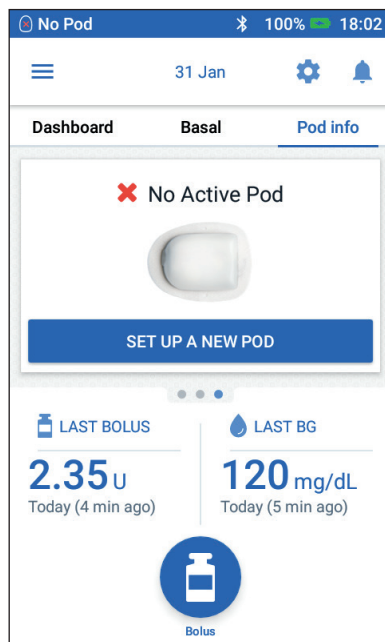

Preliminary Pod Change Steps

Warnings:

NEVER use insulin that is cloudy; it may be old or inactive. Always follow the insulin manufacturer's instructions for use. Failure to use rapid-acting U-100 insulin, or using insulin that has expired or is inactive, could put your health at risk.

Do NOT apply or use a Pod if the sterile packaging is open or damaged, or if the Pod has been dropped after removal from the package, as this may increase the risk of infection. Pods are sterile unless the packaging has been opened or damaged.

Do NOT apply or use a Pod that is damaged in any way. A damaged Pod may not deliver insulin accurately or otherwise work properly.

Do NOT use a Pod if it is past the expiry date on the package.

To minimise the possibility of site infection, do NOT apply a Pod without first using the aseptic technique. This means to:


- Wash your hands.
- Clean the insulin vial with an alcohol prep swab.
- Clean the infusion site with soap and water or an alcohol prep swab.
- Keep sterile materials away from any possible germs.

1. Gather the necessary supplies:
 - A vial of rapid-acting U-100 insulin cleared for use in the Omnipod DASH® System
 - An unopened DASH® Pod (look for the Omnipod DASH® logo on the Pod tray lid)
 - Alcohol prep swabs
 - Your Omnipod DASH® PDM
2. Wash your hands before starting and keep them clean throughout the Pod change process.
3. Check the insulin for signs of deterioration according to the manufacturer's instructions for use.
4. Check the Pod's packaging for damage. If undamaged, open it and inspect the Pod for signs of damage.
5. If the insulin or Pod is below 10°C (50°F), allow it to warm up to room temperature before proceeding.

Fill the Syringe with Insulin


The next step is to fill the syringe that came with the Pod (the 'fill syringe') with insulin

1. Use an alcohol prep swab to clean the top of the insulin vial.
2. Securely twist the fill needle onto the fill syringe.
3. Pull outwards to remove the protective cap from the needle. Save the cap; you will need it later.
4. Determine how much insulin you will put into the Pod. For example, to use the Pod for 72 hours, determine how much insulin you will use over the next 72 hours. Your healthcare provider can help you determine the correct amount.

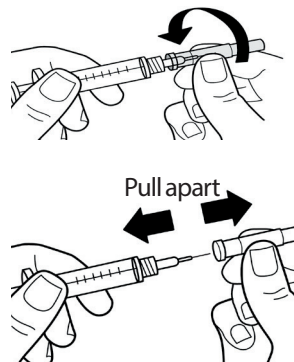

Note: The Pod requires a minimum of 85 units of rapid-acting U-100 insulin to begin operation. The Pod can deliver up to 200 units of rapid-acting U-100 insulin.

5. Draw air into the fill syringe up to the amount of insulin you want.
6. Insert the needle into the insulin vial and inject the air. Injecting air makes it easier to withdraw insulin from the vial.
7. Turn the vial of rapid-acting U-100 insulin and the fill syringe upside down. Pull down on the plunger to withdraw the desired amount of insulin from the vial into the fill syringe.
 - Fill the syringe at least to the MIN (minimum) fill line.
 - To fill the Pod with enough insulin to deliver 200 units, pull the plunger down until it stops. This will be below the 200 mark.
8. With the needle still in the vial, flick the side of the syringe with your fingertip to dislodge any air bubbles so they collect at the top of the syringe. Then push in the plunger to expel any air bubbles out of the syringe and into the insulin vial. Pull down on the plunger again, if necessary, to refill the fill syringe to the desired amount of insulin.

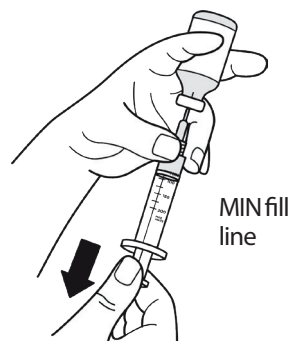

Warning: Make sure there are no air bubbles or pockets of air in the fill syringe before filling a Pod with insulin. Air transferred from the fill syringe into the Pod may result in interrupted insulin delivery.

9. Remove the needle from the vial.

3 Changing Your Pod

Fill, Pair, and Apply the Pod

Warning: Before filling a Pod, ensure that no other Pods are being activated within 1.5 m (5 ft) of your PDM.

Caution: Be sure to insert the fill syringe into the fill port and not any other location on the Pod. Do not insert the fill syringe more than once into the fill port. Only use the fill syringe and needle that came with your Pod. The fill syringe is intended for single use only and should only be used with the Omnipod DASH® System.

Fill the Pod with insulin

To fill the Pod with insulin (screen step 1):


1. Locate the arrow on the underside of the Pod. The arrow points to the insulin fill port.

Tip: You can leave the Pod in its tray during filling and pairing.

2. Insert the fill syringe straight down – not at an angle – into the fill port.
3. Depress the fill syringe plunger to transfer the insulin into the Pod.
4. Listen for two beeps from the Pod during the filling process. Be sure to completely empty the fill syringe, even after hearing the two beeps.

Note: The Pod must contain a minimum of 85 units of insulin in order to function. The Pod beeps twice after it has been filled with 85 units of insulin. If you have filled the Pod with more than 85 units and still do not hear the two beeps, please call Customer Care.

Note: After filling the Pod, continue to the next step immediately. If two hours pass before pairing the filled Pod to the PDM, the Pod becomes unusable.

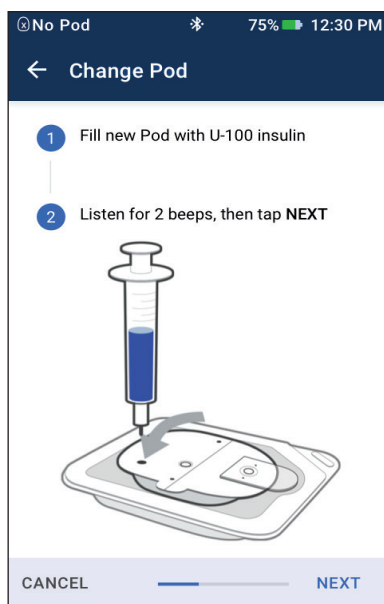

Warnings:

NEVER use a Pod if you feel resistance when you depress the plunger. This condition can result in interrupted insulin delivery.


NEVER inject air into the fill port. Doing so may result in unintended or interrupted insulin delivery.

5. Remove the needle from the insulin fill port. The port is self-sealing; insulin will not leak after the needle is removed.
6. Place the protective cap back on the syringe needle and remove the needle from the fill syringe.

Pair the PDM and Pod

To pair the PDM and Pod (screen step **2**):

1. Place the PDM next to the Pod so they are touching. The Pod should be in its plastic tray during this process.
2. Tap NEXT.
 - If more than one filled DASH Pod is in range, the PDM informs you of this. Move at least 4.5 m (15 ft) away from any other DASH Pod and tap TRY AGAIN.
 - If only one Pod is in range, the Omnipod DASH® System performs a series of safety checks and primes the Pod.
3. Listen for the tone from the PDM that indicates that the Pod is paired to the PDM and is ready to be applied.

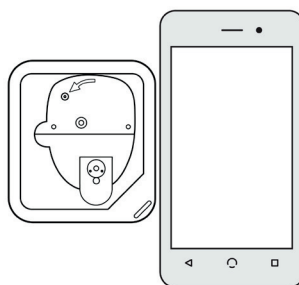

Note: After pairing, the PDM should always be able to communicate with a Pod that is up to 1.5 m (5 ft) away. Depending on the location, the PDM may be able to communicate with a Pod that is as much as 15 metres (50 feet) away.

Note: After pairing, the Pod beeps every 5 minutes until you apply it. If you do not apply it within 60 minutes after pairing, the Pod becomes unusable.

If you see a communication error message when you attempt to pair your Pod, see 'Communication Errors' on page 120.

3 Changing Your Pod

Prepare the Pod site

Select the Pod infusion site (screen step 3):



Guidelines for Pod site selection

Discuss suitable Pod placement sites with your healthcare provider using the following guidelines:



- Ideal sites have a layer of fatty tissue.
- Ideal sites offer easy access and viewing.
- The site should be at least 2.5 cm (1 inch) away from the previous site to avoid skin irritation.
- The site should be at least 5 cm (two inches) away from your navel.
- Avoid sites where belts, waistbands or tight clothing may rub against or dislodge the Pod.
- Avoid sites where the Pod will be affected by folds of skin.
- Avoid placing the Pod over a mole, tattoo or scar, where insulin absorption may be reduced.
- Avoid areas of the skin with an active infection.

Pod site map (optional)

The Pod site map is an optional feature that helps you track your current and recent Pod site locations. This option only appears if the Pod Sites setting is turned on (see 'Pod Sites Setting' on page 98).

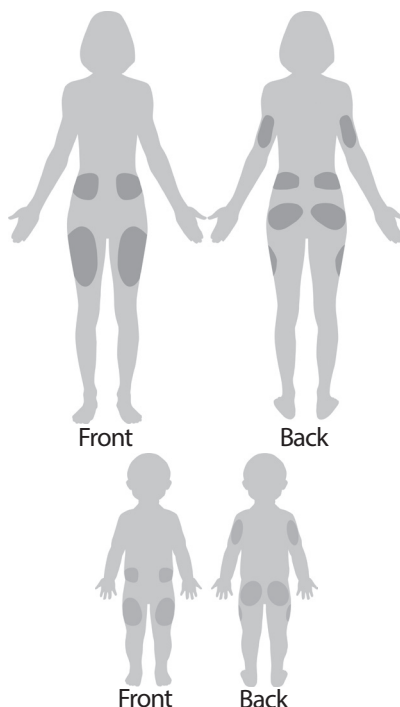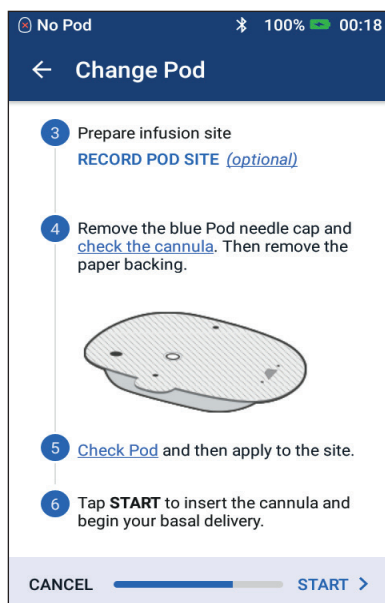


1. Tap RECORD POD SITE to bring up the Record Pod Site screen.
2. Tap the FRONT or BACK tab to select an area of your body for your Pod. To help you avoid recent Pod sites, the screen indicates the two most recent dates that each site was selected.
3. Tap a circle to indicate the location on your body where you will place your new Pod. A blue dot appears inside the selected circle. Tap again to deselect the location.

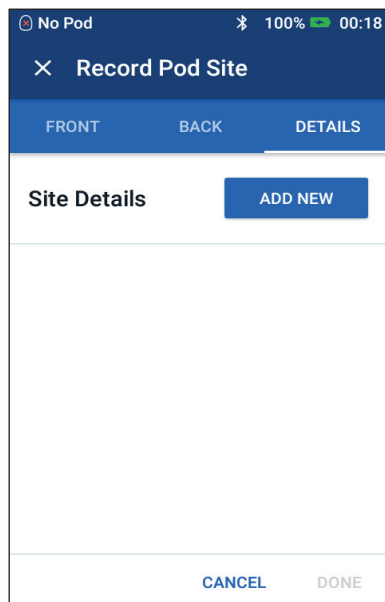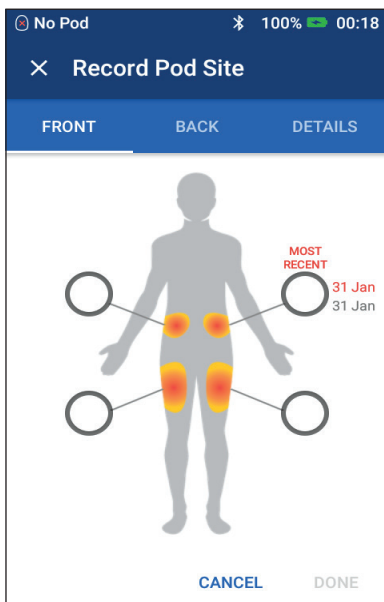

4. Tap the DETAILS tab to add a detail about the placement of this Pod. For example, you could add a detail that says 'Facing up' or 'Facing down' to describe the Pod's orientation.
 - a. To add a new detail, tap ADD NEW and type in the new detail. Tap ADD when done. The new detail is added to the list.

Note: The PDM considers uppercase letters and lowercase letters to be identical. For example, you can only use one of the following details: 'Facing up', 'FACING UP' or 'facing up', because the PDM considers these to be identical entries.
 - b. Select a detail for the new Pod by tapping the circle next to that detail. You may only add one detail for each Pod. Tap again to deselect the detail.

Note: To delete a site detail, tap the  next to the detail.
5. Tap DONE when finished to return to the Change Pod screen.

3 Changing Your Pod

Prepare the infusion site

To reduce the risk of infection at the infusion site:

1. Wash your hands with soap and water.
2. Wash your selected infusion site with soap and water.
Note: Antibacterial soap may irritate skin, especially at the infusion site. Ask your healthcare provider how to treat any skin irritation.
3. Dry the infusion site with a clean towel.
4. Use an alcohol prep swab to disinfect the infusion site. Start at the centre of the site and gently rub outwards in a circular motion.
5. Let the infusion site air-dry thoroughly. Do not blow on the site to dry it.

Remove the Pod's needle cap

Remove the Pod's needle cap (screen step 4):


1. Turn the Pod so the needle cap is up and facing you.
2. Place your thumb on the bottom (flat edge) of the needle cap and pull the cap upwards. The cap snaps off. Throw the cap away.

When you remove the needle cap, a drop of insulin may be visible at the end of the cannula or in the well.

3. If any of the following apply, tap CANCEL and then dispose of the Pod and begin again with a new Pod:
 - The Pod is accidentally dropped, as this may compromise sterility.
 - The Pod or its adhesive pad is wet, dirty or damaged.
 - The cannula extends beyond the adhesive backing when the needle cap is removed.

Warning: Verify that the cannula does not extend beyond the adhesive backing once the Pod's needle cap is removed.

4. Using the pull tabs, remove the white paper backing covering the adhesive pad. Be careful not to remove the adhesive pad itself. Do not allow the adhesive to fold back on itself.

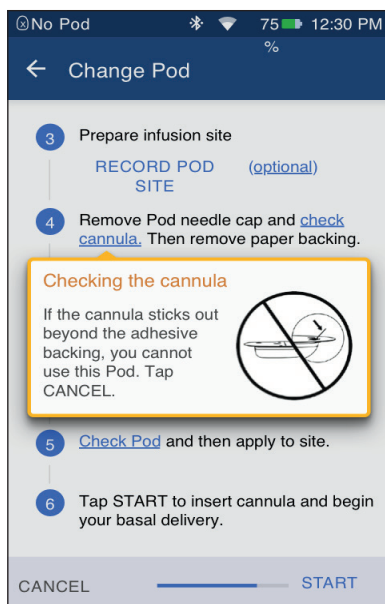


Apply the Pod

Inspect and apply the Pod (screen step 5):

1. Examine the Pod. Tap CANCEL and dispose of the Pod if the adhesive pad is folded, torn or damaged, and begin again with a new Pod.
2. Orientate the Pod so it is:
 - Horizontal or diagonal on your abdomen, hip, lower back or buttocks.
 - Up and down or at a slight angle on your upper arm or thigh.
3. Apply the Pod to the selected infusion site, pressing down firmly to secure the Pod to your skin.

The adhesive is designed for single use. Once a Pod is placed on your body, you cannot move that Pod to another infusion site.

Note: The Pod's adhesive keeps it securely in place for up to three days. However, if necessary, several products are available to enhance adhesion. Ask your healthcare provider about these products. Avoid getting any lotion, creams, sprays or oils near the infusion site as these products may loosen the adhesive.

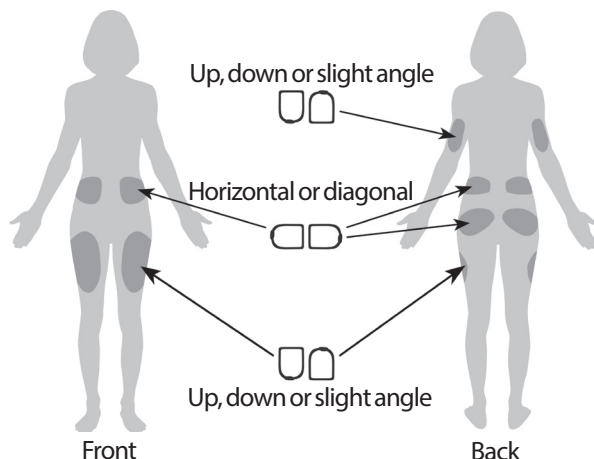

Begin insulin delivery

Begin insulin delivery (screen step 6):

Warning: If you are applying a Pod in a place that does not have a lot of fatty tissue, squeeze the skin around the Pod throughout the next step. Occlusions may result if you do not use this technique for lean areas.

1. If you applied the Pod to a lean area, squeeze the skin around the Pod.
2. Tap START.



3 Changing Your Pod

3. Confirm that the Pod is securely attached to your body, then tap **CONFIRM**.
4. If you are squeezing your skin, stop squeezing when the PDM asks if the cannula is properly inserted.

Once the cannula is inserted, the Pod automatically fills the cannula with insulin. The Pod then begins delivering the basal rate of insulin according to the active Basal Programme.


The cannula can only be inserted once with each Pod.

Caution: Be sure to check the alarm function at every Pod change (see 'Check alarms' on page 97).

Check the Infusion Site

Following insertion of the cannula, check the Pod and infusion site:

1. Look through the viewing window on the edge of the Pod to verify that the cannula is inserted into the skin. The cannula is tinted light blue.
2. Verify that there is a pink colour in the area indicated in the figure. This is an additional check that the cannula was inserted.
3. Verify that there is no wetness or scent of insulin at the infusion site. The presence of either may indicate that the cannula has dislodged.

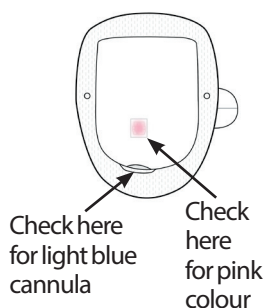

Warning: Check the infusion site after insertion to ensure that the cannula was properly inserted. If the cannula is not properly inserted, hyperglycaemia may result.

If you see blood in your cannula, check your blood glucose more frequently to ensure that insulin delivery has not been affected. If you experience unexpected elevated blood glucose levels, change your Pod.

4. If the cannula is not properly inserted, tap **NO**. Then tap **DEACTIVATE POD**. Restart the process with a new Pod.
5. If the cannula is properly inserted, tap **YES**.
A green banner notifies you that the Pod set-up is complete. The screen shows details about the Pod and a list of reminders.
6. Review the list of active reminders, then tap **CLOSE**.

Warning: Never inject insulin (or anything else) into the fill port while the Pod is on your body. Doing so may result in unintended or interrupted insulin delivery.

Avoid Infusion-Site Infections

Check the infusion site at least once a day:

- Be aware of signs of infection, including pain, swelling, redness, discharge or heat at the infusion site. If you suspect an infection, immediately remove the Pod and apply a new Pod in a different location. Then contact your healthcare provider.
- If you observe any problems with the Pod, replace it with a new Pod.

Warnings:

Check often to make sure that the Pod and soft cannula are securely attached and in place. A loose or dislodged cannula may interrupt insulin delivery. Verify that there is no wetness or scent of insulin, which may indicate that the cannula has been dislodged.

If an infusion site shows signs of infection:



- Immediately remove the Pod and apply a new Pod at a different infusion site.
- Contact your healthcare provider. Treat the infection according to instructions from your healthcare provider.

3 Changing Your Pod

More Information about Pod Use

Tip: *Develop a routine so you can change your Pod at a convenient time. If you know of an upcoming event that could interfere with changing your Pod, you can change your Pod early to avoid a disruption in insulin delivery.*

For additional information on using your Pods as effectively as possible, see the following sections:

- For care of your Pod, see 'Pod and Insulin Care' on page 125.
- To learn about the Pod alarms, see 'Alarms, Notifications, and Communication Errors' on page 109.
- If a Pod alarm is sounding, first attempt to silence it with your PDM. If that is not successful, you can manually turn off the Pod alarm (see 'Silencing an Unresolved Alarm' on page 123).
- To understand the Pod's informational and notification beeps, including which beeps are optional, see 'Notifications List' on page 117 and 'Informational Signals List' on page 119.
- To understand how to handle situations where the PDM cannot communicate with your Pod, see 'Communication Errors' on page 120.
- If the Home: Pod Info tab says 'No Pod Communication':
 - To find the last time the PDM successfully communicated with the Pod, navigate to: Menu icon () > Pod.
 - If you are unable to restore communication with the Pod and want to change to a new Pod, navigate to: Menu icon () > Pod > CHANGE POD.
- For a description of the how the PDM communicates with the Pod, see 'PDM and Pod Interactions' on page 135.

CHAPTER 4

Entering Blood Glucose Readings

About Blood Glucose Testing

Warning: Follow the guidance of your healthcare provider for proper blood glucose monitoring.

You may want to test for low blood glucose when:


- You feel symptoms such as weakness, sweating, nervousness, headache, irritability or confusion.
- You have delayed a meal after taking insulin.
- Your healthcare provider advises you to do so.

Entering Your Blood Glucose Reading

Note: Only enter blood glucose readings; do not enter control solution readings. Any readings you enter are averaged into your blood glucose history records.

To enter your blood glucose reading:

1. Check your blood glucose following the BG meter's instructions for use.
2. Go to the Enter BG screen on your PDM:
Menu icon (☰) > Enter BG
3. Manually enter or edit a blood glucose value as follows:
 - a. Tap the box inside the circle.
 - b. Use the number pad to enter your blood glucose reading.
 - c. Tap the tick symbol to close the number pad.

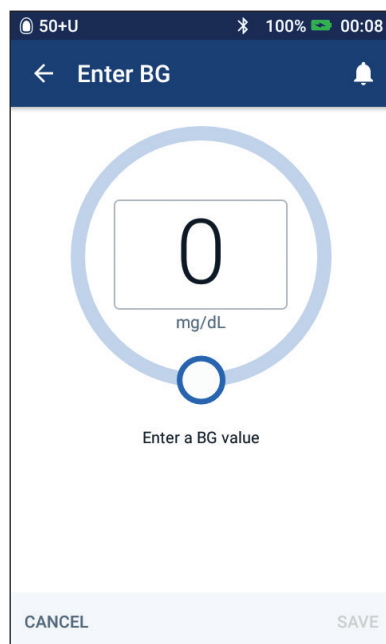

4 Blood Glucose Readings

Note: Alternatively, you can enter a blood glucose reading using the circular slider (see 'Using a slider' on page 6). When using the slider, '+' and '-' buttons appear briefly. Tap these buttons to make small adjustments to the blood glucose number.

Note: When you enter a blood glucose reading above 600 mg/dL, the PDM stores it as 'HI'. When you enter a blood glucose reading below 20 mg/dL, the PDM stores it as 'LO'.

For the meaning of the screen's colours and messages, see 'How Blood Glucose Readings are Displayed' on page 54.

4. Optional: To add one or more informational tags to the blood glucose reading shown in the circle, see 'Tagging Your Blood Glucose Reading' on page 53.
5. After the blood glucose reading and any tags are entered, do one of the following:
 - Tap **ADD TO CALCULATOR** to save any tags and enter the blood glucose reading into the Bolus Calculator. Then go to 'Bolusing with the Bolus Calculator' on page 57. If the Bolus Calculator is off or disabled, or if insulin is suspended, **ADD TO CALCULATOR** does not appear.
 - Tap **SAVE** to save the blood glucose reading and any tags in the history records. If you accessed this screen from the Bolus Calculator, **SAVE** does not appear.
 - Tap **CANCEL** then **YES** to exit the screen without saving the blood glucose reading or tags.

The PDM records the current time as the time of the blood glucose reading.


Tagging Your Blood Glucose Reading

You can add informational tags to your blood glucose reading for future reference. For example, you may tag the blood glucose reading as pre-meal.

To add a tag to the current blood glucose reading:

1. Tap ADD TAGS.
2. Tap a tag to select it. You can add up to four tags.
3. To remove a tag, tap the tag again to deselect it.

You can add or remove tags for LO and HI blood glucose readings just as you would for any other reading.

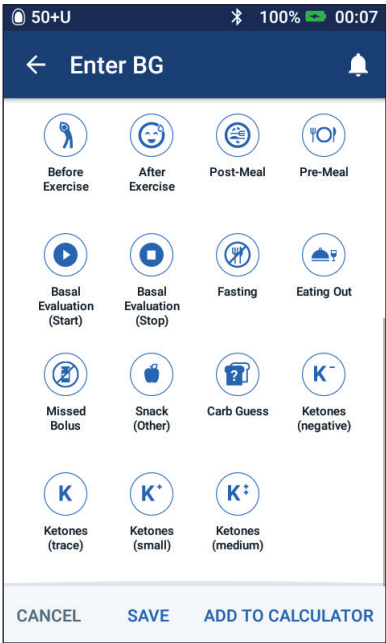


4 Blood Glucose Readings

How Blood Glucose Readings are Displayed

The PDM displays the blood glucose reading inside a coloured circle. The circle is:

- Yellow if your blood glucose is above your BG Goal Range.
- Green if your blood glucose is within your BG Goal Range.
- Red if your blood glucose is below your BG Goal Range.

To change your BG Goal Range, see 'Blood Glucose Settings - BG GoalRange' on page 104.

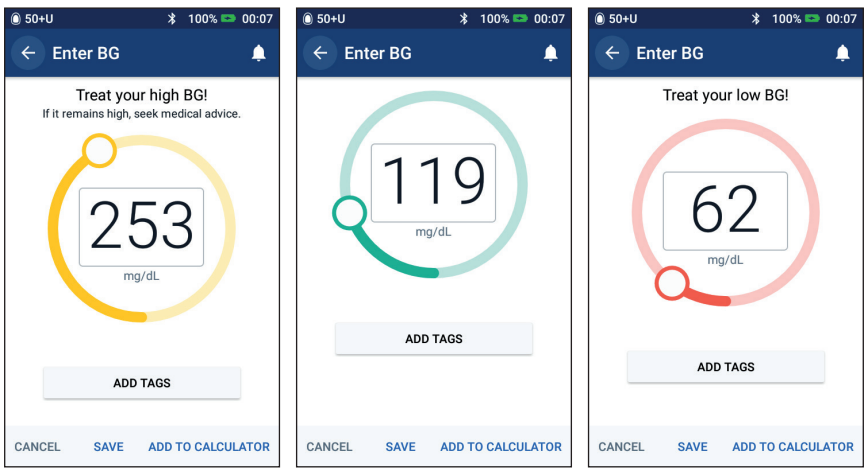

The PDM adds a warning message for high and low blood glucose readings.

Blood glucose reading	Screen display	Screen warning message
Above 600 mg/dL or HI	HI	Treat your high BG! If it remains high, seek medical advice.
251-600 mg/dL	<BG reading>	Treat your high BG! If it remains high, seek medical advice.
70-250 mg/dL	<BG reading>	
20-69 mg/dL	<BG reading>	Treat your low BG!
0-19 mg/dL or LO	LO	Treat your low BG!

If the blood glucose is HI or above 600 mg/dL, the PDM records 'HI' in the history. This indicates severe hyperglycaemia (high blood glucose). If the blood glucose reading is LO or below 20 mg/dL, the PDM records 'LO' in the history. This indicates severe hypoglycaemia (low blood glucose). HI and LO readings are saved in your blood glucose history, but are not used in averages.

Warnings:

Blood glucose readings below 70 mg/dL may indicate hypoglycaemia (low blood glucose). Blood glucose readings above 250 mg/dL may indicate hyperglycaemia (high blood glucose). Follow your healthcare provider's suggestions for treatment.

If you get a 'Treat your low BG!' message and feel symptoms such as weakness, sweating, nervousness, headache, irritability or confusion, follow your healthcare provider's recommendation to treat hypoglycaemia.

If you get a 'Treat your high BG! If it remains high, seek medical advice' reading message and feel symptoms such as fatigue, thirst, excess urination or blurry vision, follow your healthcare provider's recommendation to treat hyperglycaemia.

'LO' or 'HI' blood glucose readings can indicate potentially serious conditions requiring immediate medical attention. If left untreated, these situations can quickly lead to diabetic ketoacidosis (DKA), shock, coma or death.

This page is intentionally left blank.

CHAPTER 5

Delivering a Bolus of Insulin

A bolus of insulin is used to lower a high blood glucose level and to cover the carbohydrates in a meal. Bolus insulin is delivered in addition to insulin from a Basal Programme or temp basal. For more information about boluses, see page 144.

Warning: When using the extended bolus function, check your blood glucose levels more frequently to avoid hypoglycaemia or hyperglycaemia.

Caution: Always measure your blood glucose prior to delivering a bolus.

If the Bolus Calculator is set to 'off' or is temporarily disabled, go to 'Manually-calculated Bolus' on page 63.

Bolusing with the Bolus Calculator

The Bolus Calculator calculates a suggested bolus amount of insulin to correct an elevated blood glucose level (a correction bolus) and/or to cover carbohydrates in a meal (a meal bolus). For details about how the Bolus Calculator works, see page 145.

If you enter a blood glucose value but not carbs, the Bolus Calculator calculates a correction bolus only. If you enter carbs but not a blood glucose value, the Bolus Calculator calculates a meal bolus only. If you enter both a blood glucose value and carbs, the Bolus Calculator uses both factors to calculate a suggested bolus.

To change your personal settings used by the Bolus Calculator, or to turn the Bolus Calculator on or off, see 'Bolus calculator settings' on page 106.

5 Delivering a Bolus of Insulin

Enter your meal information

To enter the carbohydrates, or 'carbs', for your meal:

1. On the Home screen, tap the Bolus button ().

Tip: If you do not want to use the Bolus Calculator and already know the amount of the bolus you want to deliver, tap the Total Bolus field to bring up the number pad. Enter the bolus amount and tap the tick symbol. Then go to 'Delivering an immediate or extended bolus' on page 60.

2. To enter carbs, tap the Enter Carbs field. Enter the grams of carbs and tap the tick symbol.

Note: Consult your healthcare provider about how to calculate the grams of carbs.


3. Review the suggested meal bolus, which is shown next to the grams of carbs.
4. Optionally, tap CALCULATIONS to see the details of the bolus calculations.

Note: If you have entered your blood glucose reading, the meal bolus may have been adjusted for:

- Insulin on board (IOB) from a previous correction bolus.
- Reverse correction, if this feature is turned on and your blood glucose is below your Target BG.

For more information, see 'Bolus Calculator rules' on page 152.

Note: If you have not entered your blood glucose reading, the Bolus Calculator does not adjust the meal bolus for IOB.

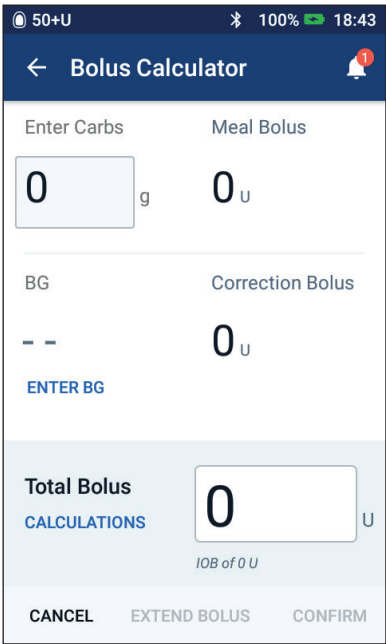

The screenshot shows the 'Bolus Calculator' app interface. At the top, the status bar displays '50+U', '100%' battery, and '18:43'. The app title 'Bolus Calculator' is in the header. Below the header, there are two input fields: 'Enter Carbs' with a value of '0' and unit 'g', and 'Meal Bolus' with a value of '0' and unit 'U'. Below these, there are two more fields: 'BG' with a value of '--' and 'Correction Bolus' with a value of '0' and unit 'U'. A blue button labeled 'ENTER BG' is positioned below the BG field. At the bottom, there is a section for 'Total Bolus' with a value of '0' and unit 'U', and a blue button labeled 'CALCULATIONS'. Below this, the text 'IOB of 0 U' is displayed. At the very bottom, there are three buttons: 'CANCEL', 'EXTEND BOLUS', and 'CONFIRM'.

Enter your blood glucose reading

Note: While the Bolus Calculator can generate a suggested bolus dose based only on the carbohydrates in a meal, you should always enter a recent blood glucose reading for added safety and accuracy. This allows the Bolus Calculator to adjust the suggested bolus to account for your current blood glucose level.

To enter a blood glucose reading:

1. Tap ENTER BG.

If the PDM has a blood glucose reading from within the last 10 minutes, that value automatically appears in the BG field. If you want the Bolus Calculator to use that value, skip the next step.

2. Tap the box in the circle and enter the blood glucose reading. Alternatively, slide the indicator along the circle to enter the blood glucose reading.

While the Bolus Calculator will use any blood glucose value you enter manually, you should only enter a blood glucose value that has been taken within the last 10 minutes.

For the meaning of the colours and messages displayed on the Enter BG screen, see 'How Blood Glucose Readings are Displayed' on page 54.

3. Optional: To tag the blood glucose reading, tap ADD TAGS and tap a tag. Tap again to deselect a tag. You can add up to four tags.
4. Tap ADD TO CALCULATOR. The Bolus Calculator screen reappears.
5. Review the suggested correction bolus, which is shown next to the blood glucose value. The correction bolus has been adjusted for any insulin on board (IOB) (see 'Bolus Calculator rules' on page 152).

5 Delivering a Bolus of Insulin

Delivering an immediate or extended bolus

The Total Bolus field shows the proposed bolus. The amount of any IOB adjustment appears below the Total Bolus field.

To review and deliver the bolus:

- 1. Review the suggested bolus. To adjust it, tap the Total Bolus field and enter a revised bolus.

50+U100%18:44

←

Bolus Calculator

Total Carbs

Meal Bolus

60g

4U

BG (18:44)

Correction Bolus

120mg/dL

0.3U

ENTER BG

Total Bolus

4.3U

CALCULATIONS

Adjusted for IOB of 0 U

CANCEL

EXTEND BOLUS

CONFIRM

- 2. To review the Bolus Calculator's calculations, tap CALCULATIONS. You may need to swipe up or down to see all of the calculations. Tap CLOSE when done (see 'Bolus calculator equations' on page 151 for details).

50+U100%18:44

←

Bolus Calculation

Correction Bolus

0.3U

BG = 120, Target BG = 105

Correction Factor = 50

$(120 - 105) / 50 \approx 0.3 \text{ U}$

Meal IOB adjustment

Meal IOB = 0 U

$0.3 \text{ U} - 0 \text{ U} = 0.3 \text{ U}$

Correction IOB adjustment

Correction IOB = 0 U

$0.3 \text{ U} - 0 \text{ U} \approx 0.3 \text{ U}$

Meal Bolus

4U

Carbs = 60 g, IC Ratio = 15 g/U

$60 / 15 \approx 4 \text{ U}$

Correction IOB adjustment

CLOSE

3. To deliver the entire bolus immediately, tap CONFIRM and skip to step 5.
4. To extend some or all of a meal bolus:
 - a. Tap EXTEND BOLUS.

Note: The EXTEND BOLUS option is available when there is a meal bolus and the Extended Bolus setting is enabled.

- b. Tap the Now field and type in the percentage of the bolus to be delivered immediately. Alternatively, tap the Extended field and enter the percentage to be extended.

The number of units to be delivered now and over the extended period appear below the percentage (%).

Note: You can only extend the meal portion of the bolus. A correction portion of the bolus, if any, is always delivered immediately.

- c. Tap the Duration field and enter the duration for the extended portion of the bolus.
 - d. Tap CONFIRM.

Extended Bolus

Now	Extended
40 %	60 %
1.6 U	2.4 U

Duration
(0.5 to 8 hrs)

0.5 hrs

Total Bolus 4.3 U

Meal Bolus 4 U

Correction Bolus 0.3 U

CANCEL CONFIRM

5 Delivering a Bolus of Insulin

5. Review the bolus details on the Confirm Bolus screen.

Immediate bolus: The screen for an immediate bolus shows the carbs and blood glucose values used for the bolus calculation and also the total bolus.

50+U

100%

19:21

Confirm Bolus

Carbs

60 g

← Grams of carbs

BG (19:20)

120 mg/dL

← Blood glucose value

Total Bolus

3.7 U

← The total bolus includes any IOB adjustments and any adjustments you made to the calculated bolus.

CALCULATIONS

Adjusted for IOB of 4 U

← Adjusted for IOB of x U means that the Bolus Calculator took IOB into account (see page 135–page 159).

START

CREATE BG REMINDER

CANCEL

Extended bolus: In addition to the information on the immediate bolus screen, the extended bolus screen also shows how much of the bolus will be delivered immediately and how much will be extended.

50+U

100%

18:44

Confirm Bolus

Carbs

60 g

BG (18:44)

120 mg/dL

Bolus Now

40 %

← % of meal bolus to be given immediately

Meal: 1.6 U

Correction: 0.3 U

Bolus Extended

60 % (0.5 hrs)

← % of meal bolus to be given over the extended period

Meal: 2.4 U

Total Bolus

4.3 U

CALCULATIONS

Adjusted for IOB of 0 U

START

CREATE BG REMINDER

CANCEL

The entire correction bolus is delivered immediately.

Only the meal bolus can be extended.

6. Optional: Set up a reminder to check your blood glucose:
 - a. Tap CREATE BG REMINDER.
 - b. Tap the Check BG in field and enter the time for the reminder.
 - c. Tap SAVE.

Note: The CREATE BG REMINDER button appears only if the 'Check BG after Bolus' reminder setting is turned on. To turn this setting on or off, see 'Check BG after Bolus reminders' on page 100.

7. Tap START to begin the bolus.


The Home screen tracks the delivery of an immediate or extended bolus (see 'Tracking the Progress of a Bolus' on page 65).

Manually-calculated Bolus


A manually-calculated bolus is a bolus that you calculate yourself. If the Bolus Calculator is set to 'off' or is temporarily disabled, you must enter a manually-calculated bolus.

A bolus cannot be greater than your maximum bolus setting (see 'Maximum Bolus' on page 105).

To deliver a manually-calculated bolus:

1. On the Home screen, tap the Bolus button ().
The Bolus screen appears.
2. Enter the bolus amount using one of the following methods:

- Tap the Total Bolus field and enter the bolus amount. Then tap the tick symbol.
- Tap SELECT FROM PRESETS and select the desired preset from the list. This option is only available if the Bolus Calculator is turned off. To create or edit a bolus preset, see 'Bolus presets' on page 81.

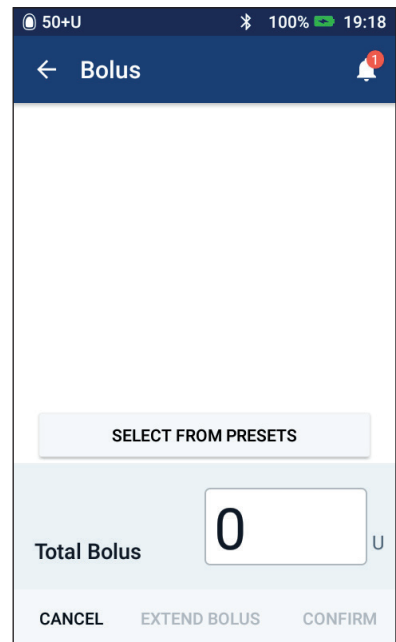

5 Delivering a Bolus of Insulin

- 3. To deliver the entire bolus immediately, tap CONFIRM. Then skip to step 5.
- 4. To extend the bolus:
 - a. Tap EXTEND BOLUS.

Note: The EXTEND BOLUS option is only available if extended boluses are enabled (see page 106).

- b. Tap the Now field and type in the percentage of the bolus to be delivered now. Alternatively, tap the Extended field and enter the percentage to be extended.

The number of units to be delivered now and over the extended period appear below the percentage (%).

Note: You can extend the entire bolus.

- c. Tap the Duration field and enter the duration for the extended portion of the bolus.
 - d. Tap CONFIRM.

- 5. Review the bolus details on the Confirm Bolus screen.
- 6. Optional: Set up a reminder to check your blood glucose as follows:
 - a. Tap CREATE BG REMINDER.
 - b. Tap the Check BG in field and enter the time for the reminder.
 - c. Tap SAVE.

Note: The CREATE BG REMINDER button appears only if the 'Check BG after Bolus' reminder setting is turned on. To turn this setting on or off, see 'Check BG after Bolus reminders' on page 100.

- 7. Review the bolus details, then tap START to begin the bolus.

The bolus amount and details about the bolus preset, if a preset was used, are stored in your history records. The Home screen tracks the delivery of an immediate or extended bolus (see "Tracking the Progress of a Bolus" on page 65).

The screenshot shows the 'Extended Bolus' screen. At the top, the status bar shows '50+U', '100%' battery, and '19:18'. The screen title is 'Extended Bolus'. There are two columns: 'Now' and 'Extended'. Under 'Now', there is a text input field with '40' and a '%' symbol, with '2.4 U' below it. Under 'Extended', there is a text input field with '60' and a '%' symbol, with '3.6 U' below it. Below these is a 'Duration' field with '0.5 hrs' and a note '(0.5 to 8 hrs)'. At the bottom, there is a 'Total Bolus' section showing '6 U'. At the very bottom, there are 'CANCEL' and 'CONFIRM' buttons.

Tracking the Progress of a Bolus

During a bolus, the Home screen displays a progress bar.

Immediate bolus progress


During an immediate bolus, the Home screen displays a Delivering Bolus message along with a progress bar and details.

If the Bolus Calculator is on, an estimate of the IOB is displayed in the lower left of the screen.

If the Bolus Calculator is off (not shown), the amount of the last completed bolus is displayed in the lower left of the screen.

You cannot use your PDM during an immediate bolus.

To cancel or replace a bolus, see 'Altering a Bolus in Progress' on page 66.

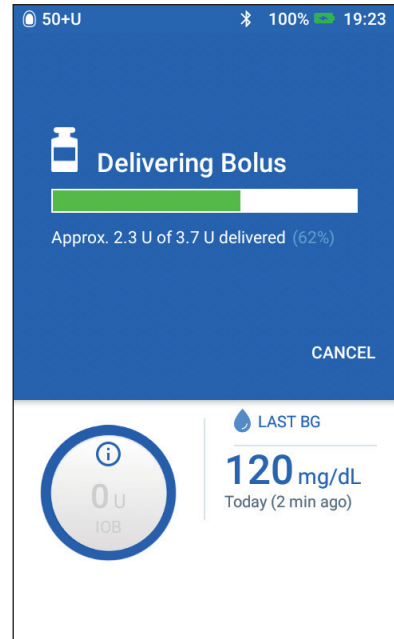

Extended bolus progress

During an extended bolus, the Home screen's Dashboard tab displays a Delivering Extended Bolus message along with a progress bar and other details.


Note: If the Bolus Calculator is off, the Dashboard tab only appears when an extended bolus is in progress.

If the Bolus Calculator is on, the displayed IOB is updated with estimated amounts from the ongoing bolus.

If the Bolus Calculator is off (not shown), the amount of the last completed bolus is displayed in the lower left of the screen.

You can use your PDM for most actions during an extended bolus.

Unless you cancel the bolus, the Pod finishes delivering a bolus whether or not it is in range of the PDM. To cancel or replace a bolus, see 'Altering a Bolus in Progress' in the next section.

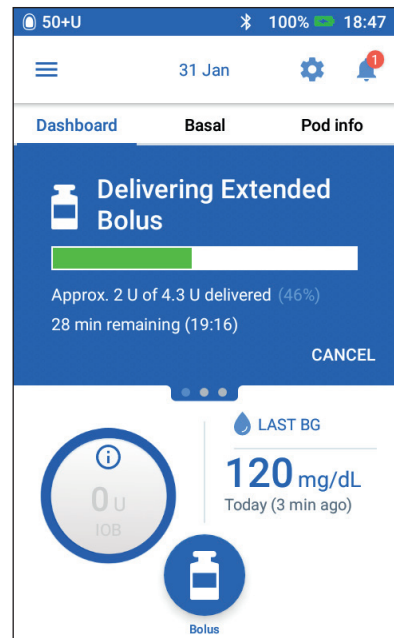

5 Delivering a Bolus of Insulin

Altering a Bolus in Progress

When an immediate bolus is in progress, you must cancel it before performing any other action.

During an extended bolus, you can:


- Cancel the bolus.
- Deliver a new immediate bolus and continue the extended bolus in progress.
- Replace the bolus in progress with another bolus.

If you get a communication error message when canceling a bolus, see 'Error when cancelling a bolus' on page 120.

Cancel a bolus

To cancel an immediate or extended bolus:

1. On the Home screen (immediate bolus) or the Home screen's Dashboard tab (extended bolus), tap **CANCEL**.
2. Tap **YES** to confirm cancelling the bolus. The Pod beeps to confirm that the bolus is cancelled.

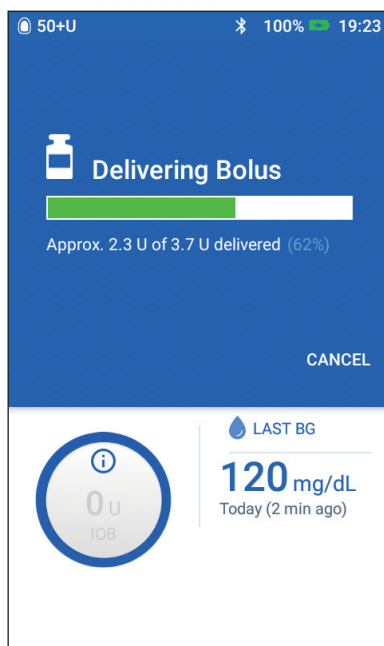

Deliver a new bolus during an extended bolus


To deliver an immediate bolus while an extended bolus is in progress:

1. On the Home screen, tap the Bolus button.
2. Enter the carbs and blood glucose information. Then tap **CONFIRM**.
3. Tap **START**.

The new bolus is delivered, and the extended bolus continues.

Replace an extended bolus

To replace an extended bolus in progress with a new extended bolus, you must cancel the ongoing bolus first.

1. On the Home screen, tap the Bolus button ().
2. Enter the amount of the new bolus (see 'Bolusing with the Bolus Calculator' on page 57 or 'Manually-calculated Bolus' on page 63).
3. Tap **EXTEND BOLUS**. A screen appears telling you that an extended bolus is already active.

4. Tap **CANCEL & CREATE NEW** to cancel the active bolus. The PDM tells you how much of the bolus was undelivered.
5. Select how to handle the undelivered bolus amount:
 - If you want to add the undelivered bolus amount to the new extended bolus, tap **ADD TO TOTAL BOLUS**. Verify that the new total bolus is correct. Then tap **EXTEND BOLUS**.

Note: The Bolus Calculator considers this added amount to be a user adjustment to the new bolus. This added amount will be delivered over the same duration as the new extended bolus.
 - If you do not want to add the undelivered bolus amount to the new extended bolus, tap **NO**.
6. Return to the extended bolus flow:
 - If the Bolus Calculator is on, go to step 4 on page 61.
 - If the Bolus Calculator is off, go to step 4 on page 64.

This page is intentionally left blank.

CHAPTER 6

Modifying Basal Insulin Delivery

This chapter describes how to modify your basal insulin delivery by using temporary basal rates or by changing to a different Basal Programme (see 'Temporary basal rates' on page 140).

To create a new Basal Programme, see page 75. To edit an existing Basal Programme, see page 77.

Using Temporary Basal Rates

Use a temporary basal rate, or 'temp basal', to handle a temporary change in your routine. For example, a temp basal can be used when you are exercising or when you are ill. When a temp basal ends, the PDM automatically reverts to delivering the scheduled Basal Programme.

You cannot activate or cancel a temp basal during an immediate bolus, but you can activate or cancel a temp basal while an extended bolus is running.

To create a temp basal preset for a frequently used temp basal, see 'Create a new temp basal preset' on page 79.

To activate a temp basal preset, see 'Activate a temp basal preset' on page 71.

Tip: By default, the PDM or Pod sounds a tone at the beginning and end of a temp basal and every 60 minutes while a temp basal is running. To turn these on or off, see 'Informational Signals List' on page 119.

Activate a temp basal

You can only activate a temp basal if the Temp Basal setting is on. To turn on the Temp Basal setting, see 'Temp basal' on page 105.

To define and activate a temp basal:

1. Navigate to: Menu icon () > Set Temp Basal.

The screen shows a graph of the active Basal Programme.


Note: To activate a temp basal preset, tap SELECT FROM PRESETS and go to step of 'Activate a temp basal preset' on page 71 .

6 Modifying Basal Insulin Delivery

2. Tap the Basal Rate field and enter the desired change in the basal rate:

- If temp basals are configured as a percentage (%) change, scroll the wheel up or down to define the per cent change to the active Basal Programme. An up arrow indicates increasing the basal rate above that of the active Basal Programme. A down arrow indicates decreasing the basal rate below that of the active Basal Programme.
- If temp basals are configured as a flat rate (U/hr), scroll the wheel to select the basal rate for the entire temp basal period.

To change whether temp basals are configured as percentage (%) or U/hr, see 'Temp basal' on page 105.

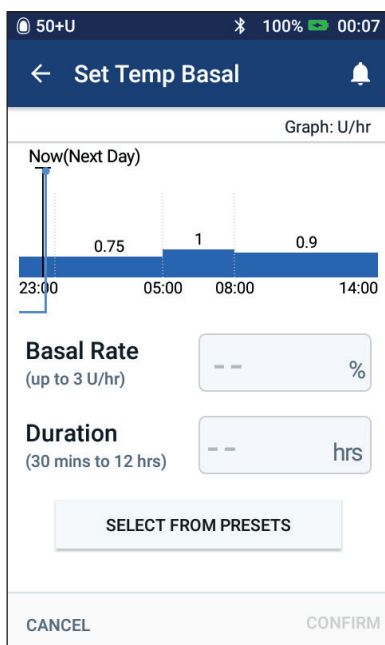

Note: The scroll wheel will not scroll above your maximum basal rate. To adjust your maximum basal rate, see 'Maximum Basal Rate' on page 104.

Tip: You can turn off insulin delivery for the duration of the temp basal by setting a decrease of 100% or setting the temp basal to 0 U/hr. For more information, see 'Temp basal limitations' on page 142 and 'Methods to temporarily stop insulin delivery' on page 143.

3. Tap the Duration field and enter the temp basal duration (between 30 minutes and 12 hours).

Examine the temp basal graph at the top of the screen. The proposed temp basal is superimposed on the active Basal Programme.

- The lighter blue shaded area shows the proposed temp basal rate for each segment.
- If you set a decrease, the active Basal Programme is shown as a dotted line.

4. Tap CONFIRM to continue.
5. Review the temp basal details. If corrections are needed, tap on the row that you would like to change. Then enter your corrections and confirm them.

Note: If necessary, the PDM limits the basal rate of any segment that would exceed your maximum basal rate setting. This is indicated by a red line on the graph and the word 'Max'.

- To activate the temp basal, tap **ACTIVATE**. Then tap **ACTIVATE** again.

After activation, the Home screen's Basal tab, renamed Temp Basal, indicates that the temp basal is running and how much time remains. At the end of the temp basal time period, the Pod automatically resumes the active Basal Programme.

Activate a temp basal preset

A temp basal preset stores the details of a temp basal that you use frequently. To create or modify a temp basal preset, see 'Temp basal presets' on page 79.


To activate a temp basal preset:

- Navigate to:
Menu icon (☰) > Temp Basal Presets
- Swipe to scroll up or down, and tap the temp basal preset that you want to activate.
- To change the duration of the temp basal, tap **EDIT**.
- To activate the temp basal, tap **ACTIVATE**, and then tap **ACTIVATE** again.

Cancel a temp basal

A temp basal stops automatically at the end of its time period and the last active Basal Programme resumes. To cancel a temp basal before the end of its time period:

- Navigate to the Home screen's Temp Basal tab.
- Tap **CANCEL**.
- Tap **YES** to confirm cancellation. The PDM cancels the temp basal and restarts the active Basal Programme.

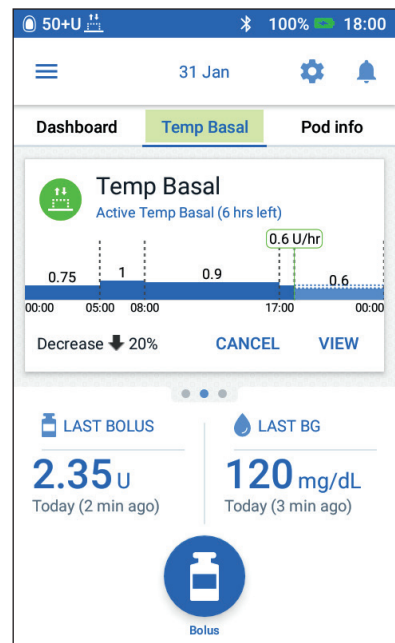



6 Modifying Basal Insulin Delivery

Switching to a Different Basal Programme

Different days can have different routines. The PDM lets you create different Basal Programmes for your different routines. For example, you may use one Basal Programme on weekdays and a different one at the weekend.

To create, edit or delete a Basal Programme, see 'Basal Programmes' on page 75.

To switch to a different Basal Programme:

1. Navigate to: Menu icon () > Basal Programmes.
A list of Basal Programmes appears with the current Basal Programme at the top.
2. Select a different Basal Programme in one of the following ways:
 - To see a graph of an inactive Basal Programme prior to activating it, tap on the name of that Basal Programme. Then tap **ACTIVATE**.
***Tip:** Double-tap the graph to see an expanded view of the Basal Programme. Swipe horizontally to view basal rates for later or earlier times.*
 - Tap the Options icon () to the right of an inactive Basal Programme, then tap **Activate**.
3. Tap **ACTIVATE** again to replace the current Basal Programme with the newly selected one.

Note: You must cancel a running temp basal prior to switching to a different Basal Programme (see 'Cancel a temp basal' on page 71). You can, however, start or cancel a temp basal while an extended bolus is running.

Note: If one or more segments are set to deliver 0 U/hr, the PDM notifies you that insulin delivery will stop during those segments.

Suspending and Resuming Insulin Delivery

Sometimes you may need to stop insulin delivery briefly. For example, you must suspend insulin delivery prior to editing an active Basal Programme or resetting the time or date. The Omnipod DASH® System lets you suspend all insulin delivery for up to two hours.

For the difference between stopping insulin delivery using the suspend feature or the temp basal feature, see 'Methods to temporarily stop insulin delivery' on page 143.

Suspend insulin delivery

To suspend insulin delivery:


1. Navigate to: Menu icon (☰) > Suspend Insulin.
2. Tap the Suspend Insulin for field. Scroll the scroll wheel to specify the duration of insulin suspension. The suspension can last 30 minutes, 1 hour, 1.5 hours or 2 hours.
3. Tap SUSPEND INSULIN.
4. Tap YES to confirm that you want to stop all insulin delivery.

The basal insulin delivery is suspended.

The Home screen displays a yellow banner stating that 'Insulin delivery is suspended'.

Note: The Pod beeps every 15 minutes throughout the suspension period. At the end of the suspension period, insulin delivery does not automatically resume. The Pod and PDM notify you every minute for three minutes and repeat this notification every 15 minutes until you have resumed insulin delivery.

Note: Temp basals or extended bolus are automatically cancelled when you suspend insulin delivery.

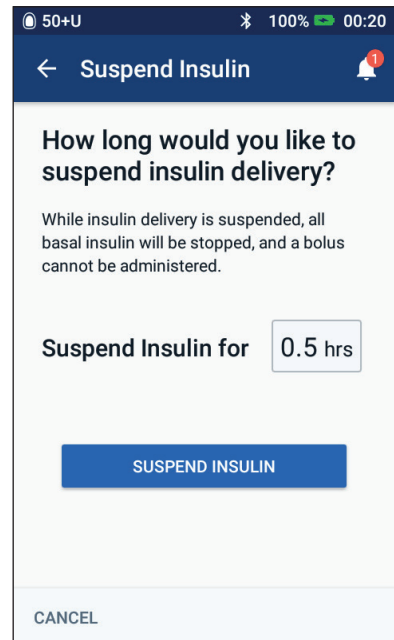

6 Modifying Basal Insulin Delivery


Resume insulin delivery before the suspension period ends

1. Navigate to:
Menu icon (☰) > Resume Insulin
or

If the Bolus Calculator is on, tap the large circle showing the IOB on the Home screen's Dashboard tab.

2. Tap RESUME INSULIN to confirm restarting the Basal Programme scheduled for the current time.

The PDM beeps to confirm that insulin delivery has resumed.

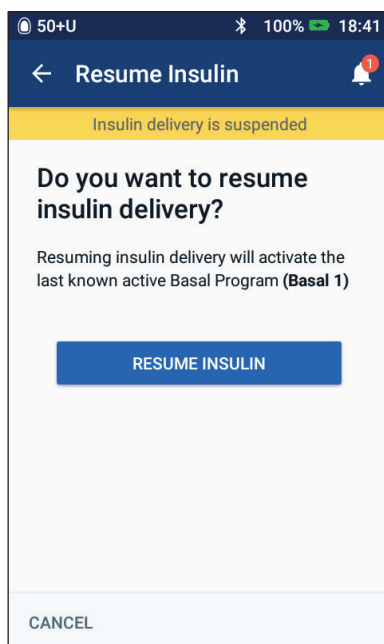


Resume insulin delivery after the suspension period ends

1. Wake up your PDM, if necessary.
2. Tap RESUME INSULIN to resume insulin delivery.

The PDM activates the Basal Programme that is scheduled for the current time and beeps to alert you that insulin delivery has resumed.

If you do not resume insulin delivery immediately, this screen reappears and the PDM and Pod beep every 15 minutes until insulin delivery is resumed.

Warning: Insulin delivery does not automatically resume at the end of the suspension period. You must tap RESUME INSULIN to resume insulin delivery. If you do not resume insulin delivery, you could develop hyperglycaemia.

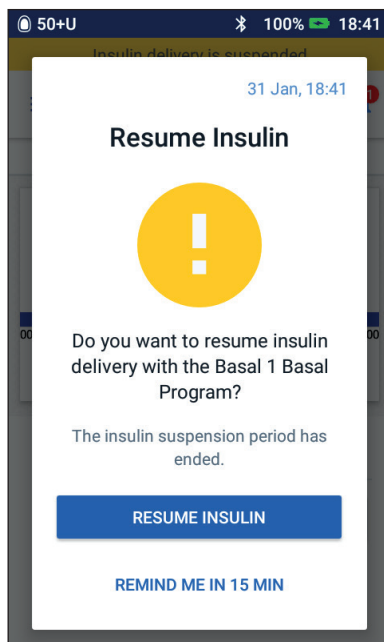

CHAPTER 7

Manage Programmes and Presets

This chapter describes how to create, edit and delete Basal Programmes, temp basal presets and bolus presets. Once set up, you can activate these quickly at a later time.


Basal Programmes

You can create up to 12 Basal Programmes. Each Basal Programme can contain from one to 24 basal rates.

Caution: Check with your healthcare provider before adjusting these settings.

Create a new Basal Programme

To create a new Basal Programme:

1. Navigate to the Create Basal Programme screen:
Menu icon () > Basal Programmes
2. Tap CREATE NEW.
Note: If you already have 12 Basal Programmes, CREATE NEW does not appear. If necessary, you can delete an existing Basal Programme.
3. Optional: Tap the Programme Name field and enter a descriptive name for your new Basal Programme. Tap the tick symbol to save the name. The default name for the Basal Programme is 'Basal' followed by a number.
4. Optional: Tap a tag to appear next to the name of the new Basal Programme. Tap a second time to deselect the tag. If you do not select a tag, a default tag is used.
5. Tap NEXT.


7 Manage Programmes and Presets

6. Define the basal segments one at a time. For each segment:
 - a. Tap the End Time field and select the end time for the segment.
 - b. Tap the Basal Rate field and select the desired basal rate for the segment.
 - c. Tap NEXT.
 - d. Repeat these steps until your final segment ends at midnight.
7. When your Basal Programme covers 24 hours, tap CONTINUE.
8. A screen appears showing the full Basal Programme as a graph and as a list. Review the Basal Programme to be sure all the values are correct.

Tip: Double-tap the graph to see an expanded view of the Basal Programme. Swipe horizontally to view basal rates for later or earlier times.

To edit any of the segments in the Basal Programme, see steps 7 - 9 under 'Edit or rename a Basal Programme' on page 77.

9. Tap SAVE to store the new Basal Programme.
10. If you have an active Pod and you want to use the new Basal Programme now, tap ACTIVATE to start using the new Basal Programme. If you do not want to use the new Basal Programme now, tap NOT NOW.

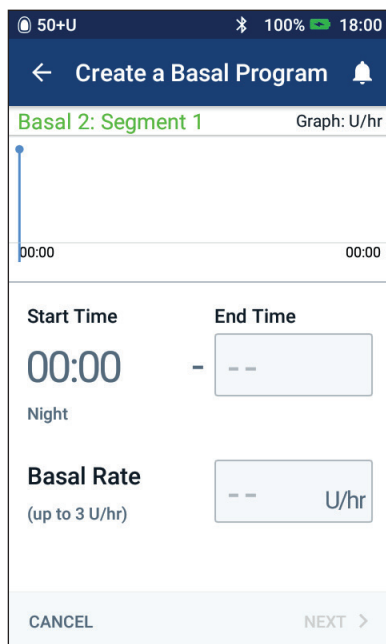

Review all Basal Programmes

1. Navigate to the list of Basal Programmes:
Menu icon (☰) > Basal Programmes
2. A graph of the current Basal Programme is shown at the top of the screen (see 'Basal tab / Temp Basal tab' on page 13 for details about the graph).
3. Swipe to scroll up or down if the list of Basal Programmes is long.
4. Tap on the name of a non-active Basal Programme to see its graph and basal rates. Tap outside the graph to close that graph.

Edit or rename a Basal Programme

To edit or rename a Basal Programme:

1. Write a list of all of the revised basal segments to guide you through re-entering the values for each segment.

Tip: You can write this list on the pages at the end of this Technical User Guide.

2. Navigate to the list of Basal Programmes:


Menu icon (☰) > Basal Programmes

3. Swiping up or down as necessary, select the Basal Programme to edit:

- To edit the active Basal Programme, tap EDIT beneath the graph of the active programme. Then tap SUSPEND INSULIN.

- To edit an inactive Basal Programme, tap the Options icon (⋮) next to the Basal Programme you would like to edit. Then tap Edit.

4. To rename the Basal Programme, tap the Programme Name field and enter the new name.
5. To change the tag, tap a different tag. Tap again to deselect a tag.
6. Tap NEXT.
7. To change an end time or basal rate for a segment:
 - a. Tap the row containing the segment you would like to change.
 - b. Tap the End Time field, and enter the new end time for the segment.
 - c. Tap the Basal Rate field and enter the desired basal rate.
 - d. Tap NEXT.
 - e. Then define the end time and basal rate for any following segments, as needed.

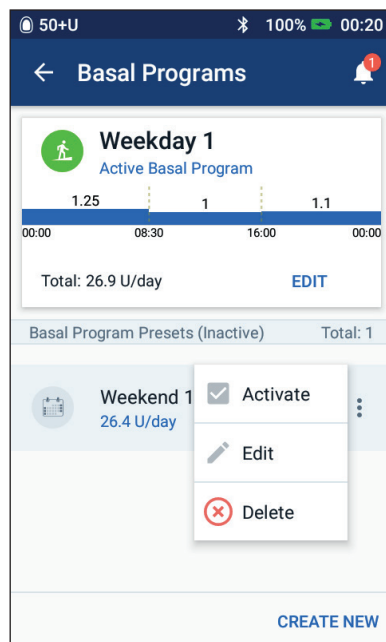



7 Manage Programmes and Presets

8. To add a new segment:
 - a. Tap the row containing the start time of the new segment.
 - b. Tap the End Time field and enter the start time of the new segment as the end time of this segment.
 - c. Change the basal rate, if necessary.
 - d. Tap NEXT.
 - e. Then define the end time and basal rate for any following segments, as needed.
9. To delete a segment:
 - a. Note the end time of the segment you want to delete.
 - b. Tap the segment that precedes the segment you want to delete.
 - c. Tap the End Time field and enter the end time of the segment you want to delete. This 'overwrites' the segment you want to delete.
 - d. Tap NEXT.
 - e. Then define the end time and basal rate for any following segments, as needed.
10. When the Basal Programme is correct, tap SAVE.
11. To activate the newly edited Basal Programme:
 - If you edited the current Basal Programme, tap RESUME INSULIN.
 - If you edited an inactive Basal Programme and you have an active Pod, tap ACTIVATE.
12. If you do not want to activate this Basal Programme, tap NOT NOW.

Delete a Basal Programme

You can only delete a Basal Programme that is not running; you cannot delete an active or suspended Basal Programme. Delete a Basal Programme:


1. Navigate to the list of Basal Programmes:
Menu icon () > Basal Programmes
2. Tap the Options icon () next to the Basal Programme you want to delete.
3. Tap Delete.
4. Tap DELETE to confirm deletion of the Basal Programme.


Temp basal presets

If there is a temp basal that you use often, you can create a 'temp basal preset' for rapid activation in the future. To activate a temp basal preset, see page 71. To turn on or off the ability to activate temp basals, or to change between specifying the temp basal as a percentage or in U/hr, see page 105. You can create up to 12 temp basal presets.

Create a new temp basal preset

To create a new temp basal preset:

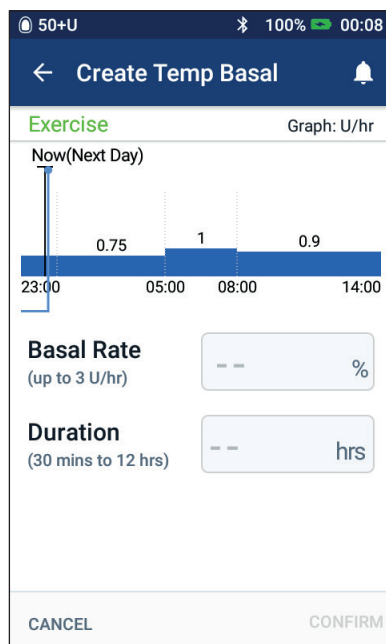

1. Navigate to the Temp Basal Presets screen:
Menu icon () > Temp Basal Presets
2. Tap CREATE NEW.
Note: If you already have 12 temp basal presets, CREATE NEW does not appear. If necessary, you can delete an existing preset.
3. Optional: Tap the Preset Name field and enter a descriptive name for your preset. The default name for the preset is 'Temp Basal Presets' followed by a number.
4. Optional: Tap a tag to appear next to the name of the new preset. Tap a second time to deselect the tag. If you do not select a tag, a default tag is used.
5. Tap NEXT.
6. Tap the Basal Rate field and scroll to the desired value.
7. Tap the Duration field and enter a duration for the preset. The temporary change to the Basal Programme is shown on the graph.
Note: You will be able to modify the duration of a temp basal preset when you activate it.
8. Tap CONFIRM.
9. Tap SAVE to save this as a temp basal preset.
10. If you would like to start this temp basal now, tap ACTIVATE. Otherwise, tap NOT NOW. This choice does not appear if a temp basal is already running.



7 Manage Programmes and Presets

Edit or rename a temp basal preset



Note: You cannot edit a temp basal preset that is actively running.

1. Navigate to the Temp Basal Presets screen:
Menu icon () > Temp Basal Presets
2. Swipe up or down as needed to find the preset that you want to edit. Then tap the Options icon () and tap Edit.
3. To rename the preset, tap the Preset Name field and enter the new name.
4. To change the tag, tap a different tag. Tap again to deselect a tag.
5. Tap NEXT.
6. Change the basal rate and duration as desired.
7. Tap CONFIRM.
8. Tap SAVE to save your changes.
9. If you would like to start this temp basal now, tap ACTIVATE. Otherwise, tap NOT NOW. This choice does not appear if a temp basal is already running.

Delete a temp basal preset

Note: You cannot delete a temp basal preset that is running.

To delete a temp basal preset:

1. Navigate to the Temp Basal Presets screen:
Menu icon () > Temp Basal Presets
2. Tap the Options icon () next to the preset you want to delete.
3. Tap Delete.
4. Tap DELETE to confirm deletion of the preset.

Bolus presets

Use bolus presets to store commonly used bolus amounts for quick retrieval in the future. Bolus presets are only available if the Bolus Calculator is off.


A bolus preset stores the units of insulin in a bolus. You can choose to extend some or all of the bolus preset when you activate it.

You can set up to seven bolus presets. A bolus preset cannot exceed your maximum bolus.

Create a new bolus preset

To create a new bolus preset:

1. Navigate to the Bolus Presets screen:

On the Home screen, tap the Bolus button ()

or

Menu icon () > Bolus Presets

2. Tap SELECT FROM PRESET.

3. Tap CREATE NEW.

Note: If you already have seven bolus presets, CREATE NEW does not appear. If necessary, you can delete an existing preset.


4. Optional: Tap the Preset Name field and enter a descriptive name for your preset. The default name for the preset is 'Bolus Preset' followed by a number.

5. Optional: Tap a tag to appear next to the name of the new preset. Tap a second time to deselect the tag. If you do not select a tag, a default tag is used.

6. Tap NEXT.

7. Tap the Total Bolus field and enter the units of insulin for the bolus preset.

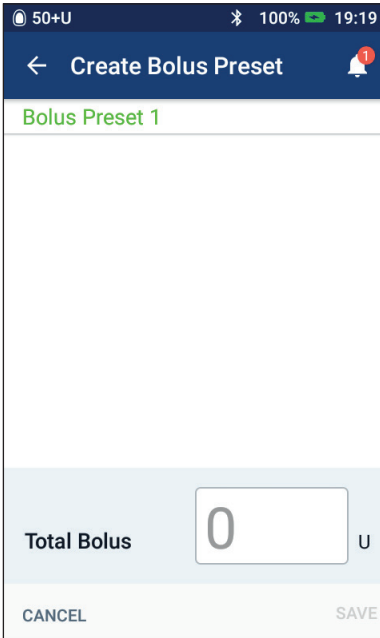

8. Tap SAVE to save the new bolus preset.



7 Manage Programmes and Presets



Edit or rename a bolus preset

To edit or rename a bolus preset:

1. Navigate to the Bolus Presets screen:
Menu icon () > Bolus Presets
2. Tap the Options icon () next to the name of the preset you want to edit.
3. Tap Edit.
4. To rename the preset, tap the Preset Name field and enter the new name.
5. To change the tag, tap a different tag. Tap again to deselect a tag.
6. Tap NEXT.
7. To change the bolus amount, tap the Total Bolus field and enter the units of insulin.
8. Tap SAVE to save your changes.

Delete a bolus preset

To delete a bolus preset:

1. Navigate to the Bolus Presets screen:
Menu icon () > Bolus Presets
2. Tap the Options icon () next to the name of the preset you want to delete.
3. Tap Delete.
4. Tap DELETE to confirm deletion of the preset.

CHAPTER 8

Browsing Your PDM Records

Your PDM records are displayed on:

- Notifications & Alarms screens
- Insulin & BG History screens, which show insulin, blood glucose, carbohydrate and Pod events

The PDM can store 90 days of history records. Once the memory is full, new records begin to replace the oldest records. You can browse but not edit the information in your records.

Note: The data stored in the PDM is preserved if the PDM battery runs out. You may need to reset the date and time, but PDM records are unaffected.

Notifications & Alarms Screens

To review past alarms and notifications:


1. Tap the bell icon (🔔) found in the upper right of many screens.

If you have new notifications, the number of new notifications appears in a red circle next to the bell icon (🔔⁴).

Note: Alternatively, navigate to: Menu icon (☰) > History: Notifications & Alarms.

Messages from today are displayed first, followed by messages from previous days.

2. Swipe up to see additional messages.
3. To display a specific date range, tap FILTER BY DATE.
 - a. Tap the starting date on the calendar. Tap the '<' arrow to view an earlier month.
 - b. Tap END at the top of the screen.

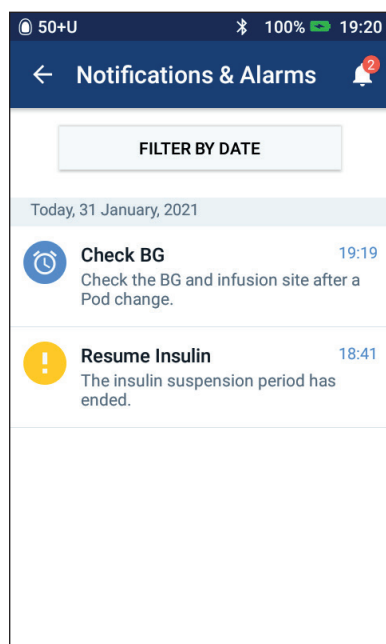

8 Browsing Your PDM Records

- c. Tap the ending date for the time range.
 - d. Tap OK.
4. When finished, tap the back arrow (←) in the upper left of the screen.
- New notifications have a blue icon (🔔). When a notification has been displayed on this screen and you have exited by tapping the back arrow (←), the icon will be grey (🔕) the next time you view this screen.

For a description of the alarms and notifications, see page 109.

Overview of the Insulin and BG History Screens

To access your insulin and blood glucose records, navigate to:

Menu icon (≡) > History: Insulin & BG History

You have the option of viewing records for one day or for multiple days.

One day

Multiple days

50+U

100%

19:23

History

1 Day

🔔

<

Today, 31 January

Current BG Goal Range: 80 to 130 mg/dL

112

mg/dL

100%

0%

0%

Average BG

BGs in Range

BGs Above

BGs Below

4

120

mg/dL

90

mg/dL

Total BGs

Highest BG

Lowest BG

Insulin & Carbs

7.15

U

7%

0.5 U

93%

6.65 U

120

g

Total Insulin

Basal Insulin

Bolus Insulin

Total Carbs

50+U

100%

00:06

History

7 Days

🔔

<

1 - 7 February

Current BG Goal Range: 80 to 130 mg/dL

131

mg/dL

37%

57%

7%

Average BG

BGs in Range

BGs Above

BGs Below

4

30

369

mg/dL

64

mg/dL

Avg Readings per Day

Total Number of BG Readings

Highest BG

Lowest BG

Insulin & Carbs

1.34

U

4%

0.06 U

96%

1.29 U

184.4

g

Average Total Insulin

Average Basal Insulin

Average Bolus Insulin

Average Total Carbs

History

1 Day

🔔

<

Today, 31 January

Current BG Goal Range: 80 to 130 mg/dL

112

mg/dL

100%

0%

0%

Average BG

BGs in Range

BGs Above

BGs Below

4

120

mg/dL

90

mg/dL

Total BGs

Highest BG

Lowest BG

Insulin & Carbs

7.15

U

7%

0.5 U

93%

6.65 U

120

g

Total Insulin

Basal Insulin

Bolus Insulin

Total Carbs

History

7 Days

🔔

<

1 - 7 February

Current BG Goal Range: 80 to 130 mg/dL

131

mg/dL

37%

57%

7%

Average BG

BGs in Range

BGs Above

BGs Below

4

30

369

mg/dL

64

mg/dL

Avg Readings per Day

Total Number of BG Readings

Highest BG

Lowest BG

Insulin & Carbs

1.34

U

4%

0.06 U

96%

1.29 U

184.4

g

Average Total Insulin

Average Basal Insulin

Average Bolus Insulin

Average Total Carbs

Details section

Swipe up to see the Details section

19:21

🔔

120

📄

2.35

📄

60

✓

19:20

🔔

120

📄

✓

19:20

🔔

118

📄

✓

19:16

📄

Extended Bolus delivered: 2.4 U over 0.5 hrs

19:16

📄

Extended Bolus started:

BG mg/dL

Bolus U

Carbs g

19:21

🔔

120

📄

2.35

📄

60

✓

19:20

🔔

120

📄

✓

19:20

🔔

118

📄

✓

19:16

📄

Extended Bolus delivered: 2.4 U over 0.5 hrs

19:16

📄

Extended Bolus started:

BGs in Range %

Total Insulin U

Total Carbs g

7 Feb

🔔

75%

📄

1.45

📄

176

6 Feb

🔔

50%

📄

1.50

📄

202

5 Feb

🔔

100%

📄

1.55

📄

220

4 Feb

🔔

0%

📄

1.30

📄

170

3 Feb

🔔

20%

📄

1.10

📄

186

84

Date Selection

When you first access the History screens, information from the current day is displayed. You can view records from a different day or a range of dates.



One day

To view records from today or a recent day:

1. Navigate to: Menu icon (☰) > History : Insulin & BG History.
Today's history records are displayed.
2. To view a recent date, tap the previous date arrow (<).
3. To move forward in time, tap the next date arrow (>). This arrow is not visible when today's records are displayed.

Range of days

To view records from a range of dates, or to quickly find a day from weeks or months ago:

1. Navigate to: Menu icon (☰) > History : Insulin & BG History.
2. Tap the date range selector.
3. Choose 1, 7, 14, 30, 60 or 90 days to select the number of days to display. Select a larger number to display older data.

The multi-day screen appears showing the selected range of dates.

4. To see details for a single day in the date range:
 - a. Swipe up to see the details section of the multi-day screen.
 - b. Tap the desired date to view its one-day history screen.

The one-day History screen for that date appears.

8 Browsing Your PDM Records

Summary Section

The upper section of the History screens summarises the records for the day or date range.

Summary item	Description
Row 1 (see page 84)	
Current BG Goal Range	Lower limit and upper limit of your BG Goal Range.
Average BG	The average of all blood glucose readings. LO and HI blood glucose readings are excluded.
BG in Range	Percentage of blood glucose readings within your BG Goal Range.
BGs Above	Percentage of blood glucose readings above the upper limit of your BG Goal Range.
BGs Below	Percentage of blood glucose readings below the lower limit of your BG Goal Range.
Row 2 (see page 84)	
Avg Readings per Day	Average number of blood glucose readings per day. Note: Appears in multiple-day view only.
Total Number of BG Readings	Total number of blood glucose readings in the day (or date range).
Highest BG	Highest blood glucose reading in the day (or date range).
Lowest BG	Lowest blood glucose reading in the day (or date range).

Summary item	Description
Row 3 (see page 84)	
Total Insulin	Total insulin (basal + bolus) delivered for the day or average total insulin delivered for the date range.
Average	
Total Insulin	
Basal Insulin	Amount of basal insulin delivered for the day or average delivered for the date range. This includes insulin delivered by the Basal Programme, adjusted for temp basals, periods of insulin suspension, and Pod deactivation. Shown as a percentage of the total (or average total insulin). Also shown in units.
Average	
Basal Insulin	
Bolus Insulin	Amount of insulin delivered via boluses for the day or average for the date range. This includes boluses calculated by the Bolus Calculator and any manually-calculated boluses. Shown as a percentage of the total (or average total insulin). Also shown in units. If you cancel a bolus before it has completed, only the amount actually delivered is included.
Average	
Bolus Insulin	
Total Carbs	Sum of all meal carbohydrates (in grams) entered into the Bolus Calculator for the day or average for the date range.
Average	
Total Carbs	

Note: If the PDM has not received confirmation from the Pod that the basal, bolus or total insulin delivery has finished, the summary circles include a grey or yellow exclamation mark similar to the ones shown in 'When the Pod has not confirmed a bolus delivery' on page 91.

Note: Percentages may not add to 100 due to rounding.

For the summary calculations, see page 160.

8 Browsing Your PDM Records

Details Section

Swipe on the history screens to see the details section.

One-day view

The details section of the one-day History screen shows individual records listed by time of day.

BG, Bolus and Carbs column headings

Today, 31 January

	BG mg/dL	Bolus U	Carbs g	
19:21	120	2.35	60	▼
19:20	120			▼
19:20	118			▼
19:16	Extended Bolus delivered: 2.4 U over 0.5 hrs			
18:46	Extended Bolus started: 2.4 U over 0.5 hrs			
18:44	120	4.3	60	▼
18:44	120			▼

Tap a down arrow to display more details. Tap again to hide the details.

Banners indicate PDM or Pod events that continue until a change is signalled on a new banner.

Swipe to see more records.

Multiple-day view

The details section of the multiple-day History screens list the individual days in the selected range. The columns show the percentage (%) of blood glucose readings within the BG Goal Range, the total daily insulin delivered and the total daily grams of carbs.




Tap a row to view its one-day History screen.

	BGs in Range %	Total Insulin U	Total Carbs g	
7 Feb	75%	1.45	176	
6 Feb	50%	1.50	202	
5 Feb	100%	1.55	220	

Swipe to see more records.

Blood glucose details



The icon displayed with a blood glucose entry indicates whether the blood glucose reading was in range.

Relation to BG Goal Range	BG entry icon
Above BG Goal Range	
Within BG Goal Range	
Below BG Goal Range	

Tapping a row with a blood glucose reading expands the row to show any applied tags.

Bolus details

The icon displayed with a bolus entry indicates whether the Bolus Calculator was used:

- The insulin bottle icon () indicates that the Bolus Calculator was enabled.
- The syringe icon () indicates the Bolus Calculator was disabled or off.

Immediate and extended boluses

The bolus amount listed next to the bolus icon is the sum of an immediate bolus plus any extended portion of the bolus. If you cancelled an immediate or extended bolus, the amount listed is the amount that was actually delivered.

8 Browsing Your PDM Records

Tapping a row with a bolus entry brings up additional details about the bolus, including:

- Whether the bolus was calculated by the Bolus Calculator or was a manually-calculated bolus.
- The name of the bolus preset, if used.
- The **VIEW BOLUS CALCULATIONS** button if the Bolus Calculator was used. Tapping this button brings up a screen showing calculation details and whether you made a manual adjustment to the calculated bolus.

You may need to swipe up or down to see all of the calculations. Tap **CLOSE** when done (see 'Bolus calculator equations' on page 151 for details).

- If you cancelled an immediate bolus, the amount originally scheduled for delivery.
- For an extended bolus, the amount delivered now and extended. Also the percentage (%) of the meal bolus delivered now and extended.
- If a bolus is ongoing, unconfirmed or lost, how much of the bolus has been confirmed.

The screenshot shows the 'Bolus Calculation' screen. At the top, it says '50+U', '100%', and '18:44'. Below the title bar, there's a back arrow and a notification icon. The screen is divided into two main sections: 'Correction Bolus' and 'Meal Bolus'. The 'Correction Bolus' section shows a value of '0.3 u' and includes calculations: 'BG = 120, Target BG = 105', 'Correction Factor = 50', and '(120-105) / 50 ≈ 0.3 U'. It also shows 'Meal IOB adjustment' with 'Meal IOB = 0 U' and '0.3 U - 0 U = 0.3 U'. The 'Meal Bolus' section shows a value of '4 u' and includes calculations: 'Carbs = 60 g, IC Ratio = 15 g/U' and '60 / 15 ≈ 4 U'. At the bottom, there's a 'CLOSE' button.

Extended bolus events



A bolus icon and banner indicate an extended bolus event:

- An **Extended bolus started** banner marks the time when the immediate bolus has finished and the extended bolus begins. In addition to the start time of the bolus, the banner lists the number of units extended and the duration of the extension.
- An **Extended bolus completed** banner marks the end of the extended bolus.
- An **Extended bolus cancelled** banner marks the cancellation of an extended bolus and states the amount of the bolus that was actually delivered.

The screenshot shows a banner for an 'Extended Bolus started' event. The banner is blue with white text. It says '18:46' on the left, followed by a bolus icon and 'Extended Bolus started: 2.4 U over 0.5 hrs'. Below the banner, there's a row of icons and numbers: '18:44', a green circle with '120', a blue circle with a bolus icon and '4.3', and a grey circle with a trash can icon and '60'. An arrow points from the text 'Extended amount only' to the '2.4 U' in the banner. Another arrow points from the text 'Immediate and extended amount' to the '4.3' in the row below.

When the Pod has not confirmed a bolus delivery

After you confirm the amount of a bolus that you want delivered, a bolus instruction is sent to your Pod. When the Pod completes delivery, it sends a confirmation to the PDM that the bolus was delivered.



Before the PDM receives confirmation from the Pod that the bolus has been delivered, the PDM estimates the amount delivered. During this time, the History screens use a grey exclamation icon to indicate that the bolus is estimated.



In most cases, once a PDM and Pod are back in range, the Pod confirms the bolus delivery. However, in rare cases, the Pod is unable to confirm bolus delivery due to a communication error. If you tap the DISCARD POD option in this situation, the History screens use a yellow exclamation mark to mark the bolus as 'unconfirmed'.

If a Pod is discarded with an unconfirmed bolus, the basal and total insulin amounts for that day are also marked as unconfirmed with a yellow exclamation mark. The listed bolus amount includes the amount that was scheduled for delivery up until the Pod was discarded.

Carbs details

A carbs icon () is displayed next to carbohydrate entries used by the Bolus Calculator.

Basal rate details

Basal Programmes



A Basal Programme icon and banner indicate the activation of a Basal Programme and the reactivation of a Basal Programme at the end of a temp basal or insulin suspension.

Temp basals



A temp basal icon and banner indicate the start, end or cancellation of a temp basal.

If a temp basal was defined as a percentage (%) of the active Basal Programme, the banner displays the per cent increase or decrease, as well as the duration. If a temp basal was defined as a flat basal rate (U/hr), the banner displays the temp basal rate and the duration.

If a temp basal preset was used, the banner displays the name of the preset.

Tapping a Temp basal started banner displays a graph and a list of basal rates associated with each time segment.

8 Browsing Your PDM Records

If a temp basal was cancelled, the Temp basal started banner contains the initially scheduled duration and the Temp basal cancelled banner contains the actual duration.

Basal rate at midnight

The first entry for each day is a banner displaying the status of the basal insulin delivery at midnight. If a Basal Programme or temp basal was carried over from the day before, the banner indicates that this is a continued programme. If insulin was suspended at midnight, the banner states this.

Pod details



A Pod icon and banner mark the activation and deactivation (or discarding) of each Pod. Tapping a Pod banner displays the Pod's lot number and sequence number.

Insulin suspended and resumed details



An Insulin Delivery Suspended icon and banner indicate the time when an insulin suspension began.



An Insulin Delivery Resumed icon and banner indicate the time when insulin delivery was restarted after a suspension.

Time change details

A Time Change banner appears if you change the time setting. A Time Zone Changed banner appears if you change the time zone.

The records below the banner show the old time and the records above the banner show the new time.

Date change details

A DATE CHANGED banner appears if you change the date setting.

When the date is changed, two History screens are created for the current day. One screen shows the old date with the history records from midnight up to the time of the date change. The second screen shows the new date with the history records from the time of the date change up to the following midnight.

CHAPTER 9

Adjusting Settings

As your needs change, you may want to adjust various settings on your PDM.

Many settings use toggles to turn settings on or off. When the button is blue and on the right side, the setting is on. When the button is grey and on the left, the setting is off. Tapping anywhere on the button toggles it between the left and right sides.



You can see an on-screen description of some settings by tapping the name of the setting. If a description is available, the screen expands to show the information. Tap the setting name again to hide the description.

Note: Some settings have default values, but all user settings can be changed.

PDM Device Settings

The PDM device settings allow you to control network connectivity, your personal identification settings, date and time, and other general device settings.

Network connectivity

Bluetooth® wireless technology is always on whether the PDM is awake or asleep. Flight mode does not turn the Bluetooth® setting off.

Flight mode

Toggling flight mode on or off does not affect the Bluetooth® setting, which remains on in either case.

To turn flight mode on or off:

1. Press and hold the Power button until a screen appears with a Flight Mode option.
2. Tap Flight Mode to toggle between on and off.

9 Adjusting Settings

Screen display

Use the screen display settings to control the screen time-out and brightness.

Screen time-out

To preserve battery power, the PDM screen turns black if you have not used it for the specified time interval. To change the time interval:

1. Navigate to: Settings icon (⚙️) > PDM Device > Screen Time-Out.
2. Tap a screen time-out interval to select it.

Tip: *A shorter time-out interval extends the battery charge.*

3. Tap SAVE.

Screen brightness

To adjust the brightness of the PDM screen:

1. Navigate to: Settings icon (⚙️) > PDM Device > Brightness.
2. Place your finger on the blue dot of the slider. Slide your finger to the right to increase the screen brightness. Slide your finger to the left to decrease the screen brightness.

Tip: *Lower screen brightness extends the battery charge.*

Lock screen

You can edit your Lock screen message, background image and your PIN. The lock screen message, image, and PIN help ensure that you are using the correct PDM.

Lock screen message

To change your Lock screen message:

1. Navigate to: Settings icon (⚙️) > PDM Device > Message.
2. Tap the Lock screen message field and enter the message you want the PDM to display when you turn it on.
3. Tap SAVE.

Lock screen background

To change the background image on the Lock screen:

1. Navigate to: Settings icon (⚙️) > PDM Device > Background.
2. Tap the background image you want to use.
3. Tap SAVE.

PIN

To change your personal identification number or PIN:

1. Navigate to: Settings icon () > PDM Device > PIN.
2. Enter your current PIN.
3. Enter the new four-digit PIN.

Tip: Tap the eye icon to display or hide the PIN numbers.

4. Tap the tick symbol to accept the PIN.
5. Re-enter the new PIN, and then tap the tick symbol.

Date and time, and language


You may need to reset the date and time to adjust for daylight saving time or changing time zones. The PDM does not automatically adjust time, date or time zone. If you are travelling to a different time zone, see 'Plan for changing time zones' on page 166.

If you do change the date or time, the change is noted on the Home and History screens. The Pod expiry time on the Pod Info tab is adjusted to reflect the new date and time.

You can also change the language used on the PDM screens.


Date and date format

To change the date:

1. Navigate to: Settings icon () > PDM Device > Date.
2. To change the date format, tap the upper box with the date format and select the desired date format.
3. To change the date:
 - a. If you have an active Pod, tap SUSPEND INSULIN and tap YES.
 - b. Scroll the Day, Month or Year fields to the desired values.
4. Tap SAVE, then tap CONFIRM.
5. If you suspended insulin, tap YES to resume insulin delivery.

Time zone

To change the time zone:

1. Navigate to: Settings icon () > PDM Device > Time Zone.
2. If you have an active Pod, tap SUSPEND INSULIN and tap YES.
3. Tap the time zone field and select the desired time zone from the drop-down list.

9 Adjusting Settings

4. Tap **SAVE**, then tap **CONFIRM**.


The PDM changes the time to match the new time zone.

5. If you suspended insulin, tap **YES** to resume insulin delivery.

Time and time format


Caution: Be careful to set the time correctly. The time setting affects the operation of several Omnipod DASH® System features.

To change the time:

1. Navigate to: **Settings icon** () > **PDM Device** > **Time**.
2. To change the time format, tap the toggle to choose between 24-hr format and 12-hr format (AM/PM).
3. To change the time:
 - a. If you have an active Pod, tap **SUSPEND INSULIN** and tap **YES**.
 - b. Scroll the Hour and Minute fields to the desired values. If using 12-hr format, select between AM and PM.
4. Tap **SAVE**, then tap **CONFIRM**.
5. If you suspended insulin, tap **YES** to resume insulin delivery.


Country

To change the PDM's country setting:

1. Navigate to: **Settings icon** () > **PDM Device** > **Country**.
2. Tap the country name to display the list.
3. Select the country you want.
4. Tap **SAVE**.

Language

To change the language used on PDM screens:

1. Navigate to: **Settings icon** () > **PDM Device** > **Language**.
2. Tap the current language and select the desired language from the drop-down list.
3. Tap **SAVE**.
4. Tap **RESTART PDM**.

Note: Restarting the PDM will not deactivate your Pod. The Pod will continue to deliver insulin according to the active Basal Programme's schedule and will deliver any extended bolus in progress.

Diagnostics

Check alarms

To verify that your PDM and Pod's alarms and vibration functions are working properly, you can test them as follows:

1. Navigate to: Settings icon (⚙️) > PDM Device > Check Alarms.
2. If you have an active Pod, tap SUSPEND INSULIN and tap YES.
3. Tap CHECK ALARMS to initiate the alarm check.
4. Listen and feel: The PDM beeps and vibrates three times. Then, if you are wearing a Pod, the Pod beeps several times and sounds the alarm tone for several seconds.
5. If the Pod alarms did not work properly, tap NO. Then either tap CHECK AGAIN to retry testing the alarms, or tap DONE and activate a new Pod.
6. If the PDM alarms did not work properly, tap NO. Then either tap CHECK AGAIN to retry testing the alarms or call Customer Care.
7. If the beeps and vibrations worked properly, tap YES. If you suspended insulin to check the alarms, tap YES to resume insulin delivery.

Warning: If the PDM fails to beep, call Customer Care immediately. If an activated Pod fails to beep, change the Pod immediately. Continuing to use the Omnipod DASH® System in these situations may put your health and safety at risk.

Reset PDM

Reset PDM is a rarely used function that erases many of your personal settings.

Caution: Resetting the PDM deletes your Basal Programmes, temp basal presets, bolus presets and all Bolus Calculator settings. Before using this feature, check with your healthcare provider and be sure you have a written record of your current information so that you can reprogramme your PDM. You will also need to activate a new Pod after resetting your PDM.

Caution: Resetting the PDM resets your IOB to zero; however, the Bolus Calculator is not disabled.


Note: The Reset PDM option does not delete history records.

9 Adjusting Settings

To reset your PDM:

1. Prepare a list of your personalised settings so you can re-enter them after you reset your PDM. Consult with your healthcare provider to ensure that the settings are appropriate for you.

Tip: *Record your current settings on the pages at the end of this Technical User Guide.*

2. Navigate to: Settings icon () > PDM Device > Reset PDM.
3. If you have an active Pod, tap DEACTIVATE POD, then tap DEACTIVATE POD again.
4. Tap RESET PDM and then tap RESET PDM again.
This deletes your current user settings.
5. Follow the PDM set-up directions in 'Initial PDM Set-up' on page 21 to re-enter your personalised information or call Customer Care for assistance.

Pod Sites Setting

The Pod Sites setting controls whether a diagram of the body is available when you activate a new Pod. You can use this body diagram during Pod activation to review where you placed your recent Pods and to mark where you place the new Pod.

To turn the Pod Sites setting on or off:

1. Navigate to: Settings icon () > Pod Sites.
2. Tap the toggle to turn the Pod Sites setting on or off. If you want the ability to record your Pod site on the diagram, turn Pod Sites on. If you do not want to use this diagram, turn it off.
3. Tap SAVE.

Reminder Settings

Reminders and notifications bring items related to PDM and Pod function to your attention (see 'Notifications List' on page 117 and 'Informational Signals List' on page 119).

Pod expiry

The Pod Expiry notification tells you when the Pod is nearing its expiry so you can plan to change your Pod at a convenient time. You can set this notification to appear from 1 to 24 hours before the Pod Expiry advisory alarm. At the selected time, the Pod and PDM vibrate or beep and the PDM displays a message.

To set the timing of the Pod Expiry notification:

1. Navigate to: Settings icon (⚙️) > Reminders > Pod Expiry.
2. Tap the Pod Expiry field and select how long before your Pod expires that you would like to be notified.
3. Tap SAVE.

Low reservoir level

The Pod and PDM sound an advisory alarm when the insulin level in your Pod reaches the Low Reservoir setting. This setting can range from 10 to 50 units.

To set the insulin level for the Low Reservoir advisory alarm:

1. Navigate to: Settings icon (⚙️) > Reminders > Low Reservoir Level.
2. Tap the Low Reservoir Level field and select the level of Pod insulin at which you would like to be notified.
3. Tap SAVE.

Pod auto-off

Consult your healthcare provider prior to changing the Auto-Off setting. This feature requires active participation on your part. See 'Auto-off' on page 138 for a description of how this feature works.

To enable or disable auto-off:

1. Navigate to: Settings icon (⚙️) > Reminders > Pod Auto-off.
2. Tap the Pod Auto-Off toggle to enable or disable the auto-off feature.
3. If auto-off is enabled, tap the Inactivity Timer field and select the length of time for the countdown timer. This setting can range from 1 to 24 hours.

Example: If you choose 10 hours, you must wake up your PDM and unlock it at least once every 10 hours, day and night, to prevent the Auto-Off alarm.

4. Tap SAVE.

9 Adjusting Settings

Warning: You must use the PDM within 15 minutes of the onset of the Auto-Off advisory alarm. If you do not, the PDM and Pod sound a hazard alarm and your Pod stops delivering insulin.

Check BG after Bolus reminders

Turn on Check BG After Bolus reminders if you want a reminder to check your blood glucose after you deliver a bolus. If BG reminders are on, you can define the timing of the BG reminder at the time that you give a bolus.

To enable BG reminders:

1. Navigate to: Settings icon (⚙️) > Reminders.
2. Tap the Check BG After Bolus toggle to enable or disable the Check BG After Bolus reminders.

Missed bolus reminders

If the Omnipod DASH® System does not deliver a meal bolus or a manually-calculated bolus during the missed bolus time period, the PDM vibrates or beeps and displays a reminder. You can set time intervals from 30 minutes to 4 hours. You can set up to six missed bolus reminders per day.

Note: To add, edit or delete missed bolus reminders, the missed bolus reminders toggle must be set to on.

Enable or disable all missed bolus reminders

To enable or disable the ability to use missed bolus reminders:

1. Navigate to: Settings icon (⚙️) > Reminders.
2. Tap the Missed Bolus toggle to enable or disable all missed bolus reminders.

If you disable these reminders, the PDM saves any previously set reminders for later use.


Enable or disable a single missed bolus reminder

To turn an individual missed bolus reminder on or off:

1. Navigate to: Settings icon (⚙️) > Reminders > Missed bolus.
2. Tap the toggle next to an individual missed bolus reminder to turn it on or off.


Add a new missed bolus reminder

To add a new missed bolus reminder:

1. Navigate to: Settings icon () > Reminders > Missed bolus.
2. Tap Add Reminder.
The Add Reminder button does not appear if missed bolus reminders are disabled.
3. Tap the toggle to select Single Reminder or Recurring Reminder.
 - For single reminders, tap the Reminder date field and select the date for the reminder.
 - For recurring reminders, specify the days of the week for the reminder by tapping the boxes next to the desired days. A selected box has a tick symbol in it.
4. Tap the No bolus between field and select the start of the missed bolus time interval.
5. Tap the next field and select the end of the time interval.
6. Tap SAVE.


Edit a missed bolus reminder

To edit a missed bolus reminder:

1. Navigate to: Settings icon () > Reminders > Missed bolus.
2. Tap the name of the reminder you would like to edit.
3. Make the desired changes to the frequency, day(s), or interval start or end time.
4. Tap SAVE.

Delete a missed bolus reminder

To delete a missed bolus reminder:

1. Navigate to: Settings icon () > Reminders > Missed bolus.
2. Tap the name of the reminder you would like to delete.
3. Tap DELETE.
4. Tap YES to confirm deletion.

9 Adjusting Settings

Confidence reminders

When confidence reminders are on, you will hear a tone at the start and end of a bolus, extended bolus, or temp basal:

- The PDM beeps at the start.
- The Pod beeps at the end.

Confidence reminders are especially useful when you are getting familiar with your PDM and Pod. To turn confidence reminders on or off:

1. Navigate to: Settings icon (⚙️) > Reminders.
2. Tap the Confidence Reminders toggle to turn confidence reminders on or off.

Note: You cannot turn off beeps that occur at the start of a temp basal set to deliver no (zero) insulin.

Programme reminders

When programme reminders are on, the Pod beeps every 60 minutes while a temp basal or extended bolus is in progress. To turn programme reminders on or off:

1. Navigate to: Settings icon (⚙️) > Reminders.
2. Scroll down and tap the Programme Reminders toggle to turn programme reminders on or off.

Note: You cannot turn off beeps that occur during a temp basal set to deliver no (zero) insulin.

Custom reminders

A custom reminder consists of a name and a time of day. You can set a one-off custom reminder or have the reminder repeat daily. Custom reminders repeat every 15 minutes until acknowledged. You can set up to 4 custom reminders.

Enable or disable all custom reminders

To enable or disable the ability to display custom reminders:

1. Navigate to: Settings icon (⚙️) > Reminders.
2. Scroll down and tap the Custom Reminders toggle to enable or disable all custom reminders.

If you disable these reminders, the PDM saves any previously set reminders for later use.

Note: To add, edit or delete custom reminders, the custom reminders toggle must be set to on.

Edit or turn on or off an individual custom reminder

If custom reminders are enabled, you can edit or turn on or turn off an individual custom reminder:

1. Navigate to: Settings icon (⚙️) > Reminders > Custom Reminders.
2. Tap the name of a custom reminder.
3. Tap the toggle next to Custom Reminder to turn this custom reminder on or off.
4. Make any desired changes to the frequency, day(s), time, name and text.
5. Tap SAVE.

Add a new custom reminder

To add a new custom reminder:

1. Navigate to: Settings icon (⚙️) > Reminders > Custom Reminders.
2. Tap Add Reminder.
3. Tap the toggle to select Single Reminder or Recurring Reminder.
 - For single reminders, tap the Reminder date field and select the date for the reminder.
 - For recurring reminders, specify the days of the week by tapping the boxes next to the days you want. A selected box has a tick symbol in it.
4. Tap the Reminder time field and select the time for the reminder. The PDM will vibrate or beep one minute after this reminder time.
5. Tap the Reminder Name field and enter a descriptive name for the reminder. The name can have up to 32 characters.
6. Tap the Reminder Text field and enter a message to be displayed on the screen at the reminder time. The message can have up to 64 characters.
7. Tap SAVE.

Delete a custom reminder

To delete a custom reminder:

1. Navigate to: Settings icon (⚙️) > Reminders > Custom Reminders.
2. Tap the name of the custom reminder that you would like to delete.
3. Tap DELETE.
4. Tap YES to confirm deletion.

9 Adjusting Settings

Blood Glucose Settings - BG GoalRange

The blood glucose settings allow you to change the BG Goal Range.

The goal of using the Omnipod DASH® System is to keep your blood glucose within a certain range. This is called your BG Goal Range. You define this range by setting the upper and lower limits of your BG Goal Range. The history graphs and the Enter BG screens use the BG Goal Range to determine which blood glucose readings are within your goal and which are above or below your goal.

Note: The Bolus Calculator does not use the BG Goal Range values to calculate a bolus.

Caution: Check with your healthcare provider before adjusting these settings.

To set the upper and lower limit of your BG Goal Range:

1. Navigate to: Settings icon (⚙️) > Blood Glucose > BG Goal Range.
2. Set the limits of the BG Goal Range:
 - a. Tap the Upper Limit field and enter the desired value. You can also touch the upper (yellow) circle and slide it up or down to change the upper limit.
 - b. Tap the Lower Limit field and enter the desired value. You can also touch the lower (red) circle and slide it up or down to change the lower limit.
3. Tap SAVE.

Basal and Temp Basal Settings

The following sections describe how to change settings that control basal insulin delivery: the maximum basal rate and temp basal settings.

Maximum Basal Rate

The Maximum Basal Rate defines an upper limit for any basal rate used in your Basal Programmes and temp basals. Consult your healthcare provider before changing this setting.

To change your Maximum Basal Rate:

1. Navigate to: Settings icon (⚙️) > Basal & Temp Basal > Max Basal Rate.
2. Tap the Max Basal Rate field and enter the new value for your Maximum Basal Rate.
3. Tap SAVE.

Note: You cannot set a maximum basal rate that is lower than the highest basal rate of an existing Basal Programme, temp basal preset or currently running temp basal.

Temp basal

To turn on or off the ability to set temp basals:

1. Navigate to: Settings icon (⚙️) > Basal & Temp Basal.
2. To enable or disable the ability to set temporary basal rates (temp basals), tap the toggle on or off.
3. To change between using per cent (%) or flat-rate (U/hr) temp basals:
 - a. Tap Temp Basal.
 - b. Select the desired method for setting a temp basal:
 - Tap Per cent (%) to modify the active Basal Programme by a set percentage increase or decrease.
 - Tap Flat Rate (U/hr) to replace the active Basal Programme with a fixed basal rate for the specified duration.
 - c. Tap SAVE.

Bolus Delivery Settings

These settings allow you to change your Maximum Bolus, extended bolus and Bolus Calculator settings.

Maximum Bolus

The Maximum Bolus defines the upper limit for a bolus. The Bolus Calculator informs you if it calculates a bolus that is over this amount. The largest allowed value of the Maximum Bolus is 30 units.

Caution: Check with your healthcare provider before adjusting this setting.

To change your Maximum Bolus:

1. Navigate to: Settings icon (⚙️) > Bolus > Maximum Bolus.
2. Tap the Max Bolus field and enter the new Maximum Bolus value.
3. Tap SAVE.

9 Adjusting Settings

Extended bolus

An extended bolus is delivered over a prolonged period of time. Only the meal portion of a bolus can be extended. A correction bolus cannot be extended.

To turn off the extended bolus feature or to change your extended bolus configuration:

1. Navigate to: Settings icon (⚙️) > Bolus.
2. Tap the toggle next to Extended Bolus to turn on or off the ability to extend a bolus.

Bolus calculator settings

This section describes how to adjust the settings used by the Bolus Calculator to calculate meal and correction boluses.

Warning: The Bolus Calculator displays a suggested bolus dose based on the personalised settings you have programmed into the PDM. Check with your healthcare provider before adjusting your Bolus Calculator settings. Giving too much insulin can cause hypoglycaemia.

Turn the Bolus Calculator on or off

To turn the Bolus Calculator on or off:

1. Navigate to: Settings icon (⚙️) > Bolus.
2. Tap the toggle on the Bolus Calculator row to turn it on or off.
3. If this is the first time you have turned the Bolus Calculator on, follow the screen's instructions or see 'Target BG and Correct Above Values' on page 31. The following sections tell you how to edit these settings.

Target BG and Correct Above

When calculating a correction bolus, the Bolus Calculator aims to bring your blood glucose to your Target BG value. However, the Bolus Calculator only calculates a correction bolus if your blood glucose is above your Correct Above setting. To edit Target BG or Correct Above values:

1. Write a list of all of the desired Target BG and Correct Above segments to guide you through re-entering the values for each segment.
Tip: *You can write this list on the pages at the end of this Technical User Guide.*
2. Navigate to: Settings icon (⚙️) > Bolus > Target BG & Correct Above.
3. Tap NEXT.

4. To edit a segment, tap the row containing the segment you want to edit.
 - a. Tap the End Time field to enter a new end time.
 - b. Tap the Target BG field to enter a new Target BG value.
 - c. Tap the Correct Above field to enter a new Correct Above value.
 - d. Tap NEXT.
5. Repeat the previous step as needed for the remaining segments.
6. After confirming that all segments are correct, tap SAVE.

Note: You can add and delete segments by editing the existing segments.

Minimum BG for Calcs

Minimum BG for Calcs is a threshold that you set. If your blood glucose reading is below your minimum BG for Calcs, the Bolus Calculator is disabled and does not calculate a bolus. To edit this value:

1. Navigate to: Settings icon (⚙️) > Bolus > Minimum BG for Calc.
2. Tap the Min BG for Calc field and enter the desired value. Alternatively, touch the blue circle and slide your finger up or down the slider to select the minimum BG for Calcs value.
3. Tap SAVE.

Insulin to Carb ratio (IC ratio)

The insulin to carbohydrate ratio (IC Ratio) defines how many grams of carbohydrates are covered by one unit of insulin. The Bolus Calculator uses your IC ratio to calculate a meal bolus when you are going to eat. To edit this value:

1. Write a list of all of the desired IC ratio segments to guide you through re-entering the values for each segment.

Tip: You can write this list on the pages at the end of this Technical User Guide.

2. Navigate to: Settings icon (⚙️) > Bolus > Insulin to Carb Ratio.
3. Tap NEXT.
4. To edit a segment, tap the row containing the segment you want to edit.
 - a. Tap the End Time field to enter a new end time.
 - b. Tap the 1 Unit of Insulin Covers field and enter a new IC Ratio.
 - c. Tap NEXT.
5. Repeat the previous step as needed for the remaining segments.
6. After confirming that all segments are correct, tap SAVE.

Note: You can add and delete segments by editing the existing segments.


9 Adjusting Settings

Correction Factor

The Bolus Calculator uses your Correction Factor to calculate a correction bolus when your blood glucose value is above your Correct Above setting (see 'The Bolus Calculator' on page 145). To edit this value:

1. Write a list of all of the desired Correction Factor segments to guide you through re-entering the values for each segment.

Tip: *You can write this list on the pages at the end of this Technical User Guide.*

2. Navigate to: Settings icon () > Bolus > Correction Factor.
3. Tap NEXT.
4. To edit a segment, tap the row containing the segment you want to edit.
 - a. Tap the EndTime field to enter a new end time.
 - b. Tap the 1 Unit of Insulin decreases BG by field and enter a new Correction Factor.
 - c. Tap NEXT.
5. Repeat the previous step as needed for the remaining segments.
6. After confirming that all segments are correct, tap SAVE.

Note: You can add and delete segments by editing the existing segments.

Reverse Correction


The Reverse Correction setting determines how the Bolus Calculator handles meal boluses when your blood glucose value is below your Target BG (see 'Reverse Correction' on page 147 for more details).

To turn reverse correction on or off:

1. Navigate to: Settings icon () > Bolus.
2. Tap the toggle on the Reverse Correction line to turn it on or off.

Duration of Insulin Action

The Bolus Calculator uses your duration of insulin action setting to calculate the amount of insulin on board (IOB) from a previous bolus. To edit this value:

1. Navigate to: Settings icon () > Bolus > Duration of Insulin Action.
2. Tap the Duration of Insulin Action field and enter the new value.
3. Tap SAVE.

CHAPTER 10

Alarms, Notifications, and Communication Errors




Overview

Warnings:

Respond to hazard alarms as soon as possible. Pod hazard alarms indicate that insulin delivery has stopped. Failure to respond to a hazard alarm can result in hyperglycaemia.

If you need to return the PDM for replacement, contact your healthcare provider for instructions about using injections to ensure appropriate insulin delivery.



The Omnipod DASH® System generates the following types of alarms and messages:

-  **Hazard alarms** are high-priority alarms that indicate a serious problem has occurred and you should remove your Pod.
-  **Advisory alarms** are lower-priority alarms that indicate that a situation exists that requires your attention.
-  **Notifications**, also called reminders, remind you about an action that you may want to perform.
- **Informational signals** inform you that the Pod is following your insulin delivery instructions. No action is required.
- **Communication error** messages display when the PDM is unable to communicate with the Pod.

For how to view the alarms and notifications history, see page 83.

Responding to Alarms

To respond to a hazard alarm or advisory alarm:

1. Wake up your PDM. The Lock screen shows an alarm message along with the hazard alarm () icon or advisory alarm () icon.
2. Unlock your PDM. After you enter your PIN, a full-screen alarm message appears.

Note: If you are using your PDM when an alarm occurs, the alarm message appears as a full-screen message.

3. Follow the on-screen instructions or see the individual alarm details starting on page 114.

Note: You can use your PDM even if you do not address the cause of an advisory alarm immediately. However, you must acknowledge a hazard alarm before you can use your PDM for anything else.

Tip: If you follow the PDM's instructions and are still not able to silence a hazard alarm, see 'Silencing an Unresolved Alarm' on page 123.

Note: If a temp basal or extended bolus is running when a Pod hazard alarm occurs, the PDM informs you that it was cancelled.


Responding to Notifications

Notifications remind you about actions that you may want to perform.

Finding out about notifications

To alert you to a notification, the Pod sounds a tone and the PDM either sounds a tone or vibrates (see 'Sounds and Vibrations' on page 112). When you hear a sound or feel a vibration, check your PDM for a message.

Note: Programme reminders, confidence reminders and some informational signals do not have an accompanying message.

If your PDM is asleep when you hear or feel a notification, wake up the PDM. The Lock screen shows the notification icon () and the notification message.


- If there are multiple messages, the most recent message is shown at the top of the list.
- If there are more messages than can be displayed on the Lock screen, a banner beneath the messages shows the number of additional messages.

If you are using your PDM when a notification is triggered, the notification message appears at the top of the screen. To remove the message from the screen:





- Do nothing. The message disappears after several seconds and is saved as a new message.
- Swipe up to remove the message immediately and save it as a new message.
- Swipe right to remove the message immediately and save it as an acknowledged message.


Acknowledging notifications

Note: Waking up and using your PDM does not automatically acknowledge, or silence, notifications.

All new notifications are included in the Notifications & Alarms count () in the red circle in the upper right of most screens.

To acknowledge the notification:

1. Wake up your PDM.
2. Tap the bell icon () to bring up the Notifications & Alarms screen.
3. Read any messages with blue notification icons (). Scroll down the screen, if necessary, to see any additional notifications with blue icons ().
4. Tap the back arrow () in the upper left of the screen to mark the notifications as acknowledged.







Note: Putting the PDM to sleep by pressing the Power button does not mark the notifications as acknowledged. You must tap the back arrow () to acknowledge the notification.

Priority and Predictability of Alarms and Notifications


Hazard alarms take priority over advisory alarms. In general, only one hazard alarm can occur at a time. In the unlikely event that a Pod and PDM hazard alarm occur simultaneously, the PDM hazard alarm is displayed first.

Advisory alarms take priority over notifications. If multiple advisory alarms occur, the PDM displays the most recent message first.

The following advisory alarms always precede a hazard alarm:

Advisory alarm		Hazard alarm
 Pod Expired	→	 Pod Expired
 Low Reservoir	→	 Empty Reservoir
 Auto-off Alert	→	 Auto-Off

You can prevent these hazard alarms by responding to their advisory alarm in a timely manner.

Note: As an extra alert, a Pod Expiry () notification precedes the Pod Expired advisory alarm.

Sounds and Vibrations

The Omnipod DASH® System uses sounds and vibrations to attract your attention to an alarm or notification.

Tip: To test the sounds and vibrations, see 'Check alarms' on page 97.

Pod alarms

The Pod sounds an alarm tone when it detects a problem that can affect insulin delivery.

- Pod hazard alarm tones are continuous tones broken up periodically by a set of beeps.
- Pod advisory alarms and notifications tones are intermittent beeps, which repeat periodically until you acknowledge them.

PDM alarms

The PDM sounds an alarm tone in response to an internal problem or when it detects that a Pod alarm is occurring.

- PDM hazard alarms begin with a continuous vibration and, after approximately 1.5 minutes, add a continuous audible tone.



For safety reasons, PDM alarms always progress to an audible alarm even if the Sound/vibrate mode is set to vibrate.

- PDM advisory alarms are intermittent tones, which repeat periodically until you acknowledge them.
- PDM notifications have intermittent vibrations or tones, which repeat periodically until you acknowledge them.

See 'Timing of alarms originating in the Pod' on page 136 for more information.

The Sound/vibrate button


The PDM's Sound/vibrate button, located on the upper right edge of the PDM, controls whether the following reminders use vibrations or audible tones: Check BG after bolus, Missed bolus, No Pod, and Custom reminders.

- To turn vibrations on, press and hold the lower end of the Sound/vibrate button until the vibrate icon () appears on the indicator.
- To turn sound on, press the upper end of the Sound/vibrate button until the bell icon () appears on the indicator.
- To increase or decrease the volume, press the upper or lower end of the Sound/vibrate button to move the round indicator. Move the round indicator to the left to decrease the volume. Move it to the right to increase the volume.

Hazard Alarm List

Hazard alarms make you aware of serious situations. Always respond to a hazard alarm immediately. Some alarm messages give you a unique number called a reference number. Give that number to Customer Care when you call about that alarm.

The individual hazard alarms are summarised here.

 **Pod Expired** – the Pod has reached the end of its operating life and insulin delivery has stopped. Both the Pod and PDM sound the alarm.


Message: Pod Expired. Insulin delivery stopped. Change Pod now.

What to do: Tap OK, DEACTIVATE POD NOW. Change your Pod. Check your blood glucose.

 **Empty Reservoir** – the Pod's insulin reservoir is empty and insulin delivery has stopped. Both the Pod and PDM sound the alarm.

Message: Empty Reservoir. Insulin delivery stopped. Change Pod now.

What to do: Tap OK, DEACTIVATE POD NOW. Change your Pod. Check your blood glucose.

 **Auto-Off** – the Pod has stopped delivering insulin because you did not respond to the Auto-off advisory alarm. Both the Pod and PDM sound the alarm. To change the auto-off setting, see 'Pod auto-off' on page 99. For more information about how auto-off works, see 'Auto-off' on page 138.

Message: Auto-Off. Insulin delivery stopped. Your PDM and Pod have not communicated in the last <amount of time>. To resume insulin delivery, change your Pod.

What to do: Tap OK, DEACTIVATE POD NOW. Change your Pod. Check your blood glucose.

 **Occlusion** – The Pod's cannula is blocked, which has stopped insulin delivery. Both the Pod and PDM sound the alarm.

Message: Occlusion Detected. Insulin delivery stopped. Change Pod now. Check your BG.

What to do: Tap OK, DEACTIVATE POD NOW. Change your Pod. Check your blood glucose.

 **Pod Error** – the Pod detects an unexpected error. Both the Pod and PDM sound the alarm.


Message: Pod Error. Insulin delivery stopped. Change Pod now.

What to do: Tap OK, DEACTIVATE POD NOW. Change your Pod. Check your blood glucose.

 **Call Customer Care** – An unexpected error is detected in the Pod or PDM. The Pod, PDM or both may sound the alarm.

Message: Call Customer Care. Remove Pod now. Call Customer Care.
Ref: <reference number>.

What to do: Tap OK to silence the alarm. Remove your Pod. Call Customer Care immediately. Check your blood glucose.

 **System Error** – An unexpected error is detected in the Pod or PDM. The Pod, PDM or both may sound the alarm.

Message: System Error. Remove Pod now. Call Customer Care.
Ref: <reference number>.

What to do: Tap OK to silence the alarm. Remove your Pod. Call Customer Care immediately. Check your blood glucose.

 **PDM Error** – An unexpected error is detected in the PDM. The PDM sounds the alarm.

Message: PDM Error. Remove Pod now. Call Customer Care.
Ref: <reference number>.

What to do: Tap OK to silence the alarm. Remove your Pod. Call Customer Care immediately. Check your blood glucose.

 **PDM Error**

Message: PDM Error. Tap 'OK' to reset the clock.

What to do: Tap OK. Reset the clock. Change your Pod. Check your blood glucose.

 **Pod deactivation required**

Message: Pod deactivation required. Tap 'OK' to deactivate.

What to do: Tap OK, DEACTIVATE POD NOW to deactivate your Pod. Remove your Pod.

 **PDM Memory Corruption**

Message: PDM Memory Corruption. Remove Pod now. Call Customer Care. Tap OK to reset the PDM and delete all user settings.

What to do: Tap OK to reset your PDM. This deletes all of your user settings, but does not delete your history records. The PDM guides you through re-entering your user settings, and the date and time, if necessary. Consult your healthcare provider if you do not know your user settings.


Advisory Alarm List

Advisory alarms inform you of a situation that needs your attention in the near future.

Warnings:

Three advisory alarms (Pod Expired, Low Reservoir, and Auto-Off) become hazard alarms and result in a stoppage of insulin delivery if ignored. Be sure to respond to all advisory alarms when they occur.


The individual advisory alarms are summarised here.

 **Pod Expired** – Your Pod will stop delivering insulin soon. This escalates to the Pod Expired hazard alarm if ignored. Both the Pod and PDM may sound the alarm.

Vibration/tone: Periodically, until acknowledged.

Message: Pod Expired. Change Pod now.


What to do: Tap OK. Change your Pod.

 **Low Reservoir** – The volume of insulin in the Pod reservoir is below the specified value. This escalates to the Empty Reservoir hazard alarm if ignored. To change this value, see 'Low reservoir level' on page 99. Both the Pod and PDM may sound the alarm.

Vibration/tone: Once a minute for 3 minutes. Repeats every 60 minutes until acknowledged.

Message: Low Reservoir. <x> U insulin or less remain in Pod. Change Pod soon.

What to do: Tap OK. Change your Pod.

 **Auto-Off** – You have not used your PDM in the auto-off countdown period that you specified. The Pod will stop delivering insulin soon if you do not respond to this alarm. To enable or disable the auto-off feature or to change the countdown period, see 'Pod auto-off' on page 99. Both the Pod and PDM sound the alarm.

Vibration/tone: Once a minute for 15 minutes.

Message: Auto-Off alert. Your PDM and Pod have not communicated in the last <amount of time specified by you>. Tap OK to trigger a communication between your PDM and Pod.

What to do: Tap OK to reset the auto-off countdown timer to the beginning of the countdown period.

! Resume Insulin – the time period that you specified for insulin suspension has passed. If you do not resume insulin delivery, you could develop hyperglycaemia. Both the Pod and PDM sound the alarm.

Vibration/tone: Once a minute for 3 minutes. Repeats every 15 minutes until insulin delivery is resumed.

Message on Lock screen: Resume Insulin. The insulin suspension period has ended.

Full-screen message: Do you want to resume insulin delivery with the <name> Basal Programme? The insulin suspension period has ended.

What to do: Tap RESUME INSULIN to restart the scheduled Basal Programme or tap REMIND ME IN 15 MIN to keep insulin delivery suspended. This alarm recurs every 15 minutes until you resume insulin delivery.

! Low PDM Battery—The PDM battery charge has 15% or less remaining.

Vibration/tone: None. To preserve the battery, the PDM is silent.

Message: Low PDM Battery. PDM battery is getting low. Recharge battery soon.

What to do: Tap OK. Recharge your PDM battery as soon as possible.

Caution: There is no hazard alarm when the battery completely runs out. Plug in the charger as soon as possible after seeing the low battery message.

Note: The battery icon in the status bar of most screens displays the remaining battery charge. When the charge is completely gone, the PDM powers off. You must recharge the PDM to use it again. You can use your PDM while it is charging.


Notifications List

Notifications, also called reminders, remind you about various actions you may want to perform. Some are generated automatically and others have settings that you control (see 'Adjusting Settings' on page 93).

Tip: Use the Sound/vibrate button on the right-side of the PDM to control whether the PDM's notification sounds a tone or vibrates (see 'The Sound/vibrate button' on page 113).

10 Alarms, Notifications, Communication Errors

The notifications are:

 **Pod Expiry** – Tells you how much time is left before the Pod Expiry advisory alarm. To change the timing of this notification, see 'Pod expiry' on page 99. Both the Pod and PDM sound the notification.

Vibration/tone: A set of three beeps. Repeats periodically until acknowledged.

Message: Pod Expiry. Pod expires at <time, date>.

What to do: Acknowledge the message (see 'Acknowledging notifications' on page 111). Change your Pod.

 **No Active Pod** – The PDM reminds you to activate a new Pod to begin basal insulin delivery.

Vibration/tone: Every 15 minutes.

Message: No Active Pod. Activate a Pod to start insulin delivery.


What to do: Unlock the PDM. Activate a new Pod.

 **Check BG After Bolus** – The PDM reminds you to check your blood glucose after a bolus. To edit these reminders, see 'Check BG after Bolus reminders' on page 100.

Vibration/tone: Every 5 minutes until acknowledged.

Message: Reminder: Check BG. <x> hours have passed since your bolus.


What to do: Acknowledge the message (see 'Acknowledging notifications' on page 111). Check your blood glucose.

 **Missed bolus** – The PDM reminds you that you have not delivered a bolus within the time period you have specified. To edit these reminders, see 'Missed bolus reminders' on page 100.

Vibration/tone: Every 15 minutes until acknowledged.

Message: Missed Bolus. Meal bolus not delivered between <start time> - <end time>.

What to do: Acknowledge the message (see 'Acknowledging notifications' on page 111). Consider your meal schedule.

 **Custom Reminder** – The PDM shows a message of your choice. To create or edit these reminders, see 'Custom reminders' on page 102.

Vibration/tone: Every 15 minutes until acknowledged.

Message: <Your reminder text>.

What to do: Acknowledge the message (see 'Acknowledging notifications' on page 111).

Informational Signals List

The Pod and PDM can provide informational tones or vibrations to let you know that normal PDM and Pod events are occurring as expected. You do not need to do anything in response to these signals. The PDM does not show an explanatory message.

Tip: Use the Sound/vibrate button on the right-side of the PDM to control whether some of these signals sound an audible tone or vibrate (see 'The Sound/vibrate button' on page 113).

Confidence reminders – These tones or vibrations let you know that your temp basals and boluses are working as expected. These reminders are on by default. To turn them on or off, see 'Confidence reminders' on page 102.

PDM tones/vibrations: At the start of a temp basal, bolus or extended bolus.

Pod beeps: Once at the end of a temp basal, bolus or extended bolus.

Programme reminders – These beeps remind you that you have a temp basal or extended bolus running. These reminders are on by default. To turn them on or off, see 'Programme reminders' on page 102.

Pod beeps: Once every 60 minutes while a temp basal or extended bolus is running.

Pod activation – These tones or vibrations occur at various times during the Pod activation process to indicate progress. You cannot turn these off.

Pod beeps: Twice when it has been filled with the minimum amount of insulin needed for activation (see page 41).

PDM tones/vibrations: When the Pod and PDM are successfully paired.

Pod beeps: Beginning 10 minutes after the Pod is filled with insulin, the Pod beeps every five minutes until insulin delivery has started.

Pod deactivation – PDM sounds a tone or vibrates: Twice when the Pod is successfully deactivated.

Basal Programme changes – These tones or vibrations inform you of changes to your Basal Programme. You cannot turn these off.

PDM tones/vibrations: When a Basal Programme is activated, edited, suspended or resumed.

Pod beeps: Once every 15 minutes while insulin is suspended.

Cancelling temp basals and boluses – Informs you that the temp basal or bolus has been successfully cancelled.

Pod beeps: Once when you cancel a temp basal, bolus, or extended bolus. You cannot turn these beeps off.

Communication Errors

When your PDM sends an instruction to your Pod, the communication usually succeeds quickly. If an error in communication occurs, the PDM displays the No Pod communication icon on the status bar (see 'The Status Bar' on page 9).

Communication can fail if:

- The PDM is, or has temporarily been, too far from the Pod – for successful communication, the PDM and Pod should be side by side during activation and within 1.5 metres (5 feet) after activation. Any insulin command will initiate a connection between the PDM and Pod.
- Communication is disrupted by outside interference—See the 'Omnipod DASH® System Notice Concerning Interference' on page 185.

Warnings:

If you are unable to deactivate a Pod, it continues to pump insulin. Be sure to remove the old Pod before activating a new Pod. Giving too much insulin can cause hypoglycaemia.

If your PDM is damaged or not working as expected, call Customer Care for assistance. Be sure to check your blood glucose frequently. Remove your Pod and contact your healthcare provider for treatment guidelines.

Error when cancelling a bolus

If you are trying to cancel a bolus when a communication error occurs, the PDM offers these options:

- **CANCEL:** Select this option to stop attempting to cancel the bolus. The Pod continues to deliver the bolus.
Note: If the 'cancel bolus' instruction has already been sent, this CANCEL option is not available.
- **TRY AGAIN:** Move to a new location, then select this option to tell the PDM to continue attempting to communicate with the Pod.
- **DEACTIVATE POD:** Select this option and follow the PDM's instructions for replacing your Pod.

Error when sending insulin instructions to the Pod

A communication error may occur when the PDM attempts to send the following insulin delivery instructions to the Pod – suspending or resuming insulin, activating a new Basal Programme, starting or cancelling a temp basal, or starting a bolus.

If a communication error occurs when the PDM attempts to send an insulin delivery instruction, the PDM offers you different options. The options offered depend on whether the PDM has sent the Pod the instruction and hasn't received confirmation that it was carried out, or whether the PDM hasn't sent the instruction.

If the PDM has sent the Pod the instruction and hasn't received confirmation that it was carried out, the PDM offers these options:

- **TRY AGAIN:** Move to a new location, then select this option to recheck for confirmation that the instruction was carried out.
- **DEACTIVATE POD:** Select this option and follow the PDM's instructions for replacing your Pod.

If the PDM has *not* sent the Pod the instruction, the PDM tells you to move to a new location and tap **TRY AGAIN** to reattempt communication. After you tap **TRY AGAIN**, if the next communication attempt fails, the PDM offers these options:

- **CANCEL:** Select this option to cancel sending the instruction. In this case, the Pod continues with its prior insulin delivery mode. You can try to send the instruction later.
- **TRY AGAIN:** Move to a new location, then select this option to tell the PDM to reattempt to send the instruction to the Pod.
- **DEACTIVATE POD:** Select this option and follow the PDM's instructions for replacing your Pod.

Note: At any point, if the PDM receives confirmation from the Pod that the instruction was successfully received, a green banner appears informing you that the instruction has been carried out.

10 Alarms, Notifications, Communication Errors

Error when activating a Pod

If a communication error occurs during Pod activation, the PDM offers these options:

- **DISCARD POD:** Select this option to stop attempting to use this Pod.
- **TRY AGAIN:** Select this option to attempt to re-establish communication.

Error when deactivating a Pod

If a communication error occurs during Pod deactivation, the PDM offers these options:

- **DISCARD POD:** Select this option to tell your PDM to unpair from that Pod. The PDM instructs you to remove your Pod and tap **CONTINUE**.
- **TRY AGAIN:** Select this option to attempt to re-establish communication.

Note: After selecting the discard option, you can prevent future alarms from the discarded Pod by following the instructions in 'Silencing an Unresolved Alarm' on page 123.

Note: If there is an unconfirmed bolus when you discard a Pod, the PDM does not know how much of the bolus was delivered. Therefore, the PDM temporarily disables the Bolus Calculator. If you tap the Bolus button while the Bolus Calculator is disabled, the PDM displays a message that says 'Bolus calculator temporarily disabled'. You can deliver a manually-calculated bolus when the Bolus Calculator is disabled.


Silencing an Unresolved Alarm

You can usually silence Pod or PDM alarms by tapping a button on the alarm screen. If the alarm continues, follow the directions in this section.

Pod alarm

To permanently silence a Pod alarm:

1. If the Pod is on your body, remove it.
2. Peel back a little bit of the adhesive pad from the bottom of the Pod at the square end (see figure).
3. Locate the alarm deactivation port to the right of the gold circle. The alarm deactivation port can be felt with a fingernail or paper clip as a soft plastic.
4. Firmly press a paper clip or similar item straight down into the alarm deactivation port. If an alarm is sounding, the alarm stops. You need to apply enough force to break a thin layer of plastic.

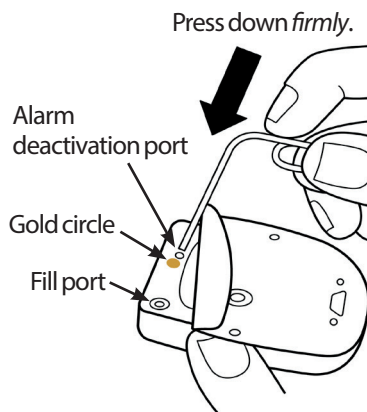

PDM alarm

If a PDM alarm is not silenced when you tap the alarm screen's button:

1. Press and hold the Power button, then tap Power off.
2. Press and hold the Power button to turn the PDM back on.

This action silences the PDM's alarm. Your history records and personal settings are preserved. However, you may have to reset the time and date and change your Pod. After you reset the time, the Bolus Calculator is temporarily disabled. A PDM message tells you when the Bolus Calculator can be used again. You can deliver a manually-calculated bolus when the Bolus Calculator is disabled.

This page is intentionally left blank.

CHAPTER 11

Taking Care of Your PDM and Pod

The Omnipod DASH® Insulin Management System has no user-serviceable parts. If you require assistance operating or maintaining the Omnipod DASH® System, call Customer Care.

Warnings:

Store all Omnipod DASH® System products and supplies, including unopened Pods, in a cool, dry place. Products or supplies that have been exposed to extreme temperatures may not function properly.

Caution: Avoid storing Omnipod DASH System components and supplies in a place accessible to children, pets, or pests. Unintended access may result in damage to system parts or impact their sterility.

Pod and Insulin Care

This section describes proper care of your Pod. To order all Omnipod DASH® System products and supplies, call Customer Care.

Pod and insulin storage

Extreme heat or cold can damage pods and cause them to lose insulin delivery accuracy or otherwise malfunction. If Pods are exposed to extreme temperatures, inspect them carefully before use.

It is especially important to store your insulin in a well-controlled environment. Inspect insulin before using it; never use insulin that looks cloudy or discoloured. Insulin that is cloudy or discoloured may be old, contaminated or inactive. Check the insulin manufacturer's instructions for use and the insulin's expiry date.

Pods and the environment

Avoid extreme temperatures

The Pod's operating temperature has been tested and found to operate safely between 5°C and 40°C (between 41°F and 104°F). Under normal circumstances, your body temperature keeps the Pod within a range of 23°C and 37°C (73°F and 98.6°F).

11 Taking Care of Your PDM and Pod

Caution: Never use a hairdryer or hot air to dry the Pod. Extreme heat can damage the electronics.

Warning: Do NOT expose a Pod to direct sunlight for long periods of time. Remove your Pod prior to using hot tubs, whirlpools or saunas. These conditions could expose the Pod to extreme temperatures and may also affect the insulin inside the Pod.

If you remove your Pod to avoid exposing it to extreme temperatures, remember to check your blood glucose levels frequently. Check with your healthcare provider for guidelines if you will not use a Pod for extended periods.

Water and your Pod

The Pod is waterproof up to a depth of 7.6 metres (25 feet) for up to 60 minutes (IP28). After swimming or similar exposure to water, rinse off the Pod with clean water and gently dry it with a towel.

Warning: Do NOT expose your Pod to water at depths greater than 7.6 metres (25 feet) or for longer than 60 minutes.

Cleaning your Pod

Pods are waterproof. If you need to clean a Pod, gently wash it with a clean, damp cloth, or you can use mild soap and water.

Caution: DO NOT use sprays, strong detergents or solvents on or near your Pod. The use of spray sunscreen, DEET-containing insect repellent spray, personal care sprays and other aerosols, detergents and strong chemicals on the Pod can irritate the infusion site or damage the Pod, increasing the risk that the Pod housing will crack. Pod damage could result in external fluids seeping into the Pod, which can impact the ability of the Pod to function properly. This could result in over-delivery or under-delivery of insulin, which can lead to hypoglycaemia or hyperglycaemia.

Caution: Hold the Pod securely and take care while cleaning it, so the cannula does not kink and the Pod does not detach from your skin.

PDM Care

This section describes proper care of your PDM.

Warning: If your PDM is damaged or not working as it should, please call Customer Care for assistance. Be sure to check your blood glucose frequently. Remove your Pod and contact your healthcare provider for treatment guidelines to follow.

PDM storage

When you are not using your PDM, store it in a convenient, nearby location that is cool and dry.

If the PDM is not safe from children or others who may press the buttons by accident, do not share your PDM's PIN number with anyone. Also, store the PDM in a safe place.

Long-term storage of the PDM

If you are not going to use your PDM for an extended period of time, allow your battery to reach approximately 50% charge. Then press and hold the Power button to turn the PDM off.

Caution: Do not turn your PDM off for more than six months at a time.

PDM and the environment

Avoid extreme temperatures

Extreme operating temperatures can affect the PDM's battery and interfere with Omnipod DASH® System operation. Avoid using the PDM in temperatures below 5°C (41°F) or above 40°C (104°F). Avoid charging the PDM in temperatures below 5°C (41°F) or above 40°C (104°F).

Caution: Do not store, charge or leave the PDM where it may be exposed to extreme temperatures, such as inside a car. Extreme heat or cold can cause the PDM to malfunction. See the PDM Specifications for appropriate operating, charging and storage temperature ranges. For specific operating temperatures, see the PDM Specifications in these Instructions for Use.

Water and your PDM

The PDM is not waterproof. Do not place it in water or leave it near water where it can accidentally fall in. If it gets wet, contact Customer Care for a replacement PDM.

Caution: Never use a hairdryer or hot air to dry the PDM. Extreme heat can damage the electronics.

11 Taking Care of Your PDM and Pod

Caution: The PDM is not waterproof. Do NOT place it in or near water.

Electrical interference

The PDM is designed to withstand normal radio interference and electromagnetic fields, including airport security and mobile phones. However, as with all wireless communication technology, certain operating conditions can interrupt communication. For example, electric appliances such as microwave ovens and electric machinery located in manufacturing environments may cause interference. In most cases, interruptions are easy to resolve by moving to a new location.

USB cable

When you use a USB cable to upload your PDM records to another device or to charge your PDM, disconnect the USB cable as soon as you are finished.

Caution: Only connect a USB cable to your PDM when charging the battery or transferring data to a computer or another device. Never connect a USB cable to the PDM for any other reason.

Note: You can use the PDM while it is charging or transferring data.

Caution: When you connect a USB cable to the PDM, only use a cable that is less than or equal to 1.2 metres (4 feet) in length.

Caution: To avoid the risk of strangulation, keep your cables away from young children and pets.

Cleaning your PDM

Always keep your PDM USB port free of debris and liquids. Dirt, dust and liquids can impair the functionality of your PDM or damage it.

Caution: Do not use solvents to clean your PDM. Do not immerse your PDM in water.

To clean your PDM:

1. Press the Power button briefly to put your PDM to sleep.
2. Wipe the outer surface of the PDM with a damp, lint-free cloth.
3. Dry the outer surface with a dry, lint-free cloth.

Caution: While cleaning, do NOT allow debris or liquid to get into the USB port, speaker, earphone jack socket, Sound/vibrate button, or Power button.

Every time you clean your PDM, examine the entire PDM for discolouration, cracks or separations. Also check for deteriorating performance, such as illegible messages, button malfunction or repeated communication failures. If you notice any signs of deterioration, stop using the PDM. Call Customer Care if you have questions or for information about PDM replacement.

If you drop the PDM

Shock or a severe impact can damage your PDM. If you drop the PDM or if it is otherwise subjected to a severe impact:

1. Inspect the outside of the PDM for visible signs of damage.
2. Press and hold the Power button to see whether the PDM turns on and the Lock screen appears.

Caution: Do not use the PDM if it appears damaged or is not working as it should. Do not use the PDM if the PDM screen is broken.

PDM Battery Care

The PDM uses a rechargeable lithium ion battery. If there is a problem with your battery or charger, contact Customer Care for information about replacements.

Safe use of the PDM battery

Warnings:

Do not expose your battery to high heat. Do not puncture, crush or apply pressure to your battery or the back of the PDM. If you see that the back of the PDM is curved or will not stay in place, contact Customer Care. Failure to follow these instructions could result in an explosion, fire, electric shock, damage to the PDM or battery, or battery leakage.

Do not incinerate a battery. Dispose of an old battery in accordance with local waste disposal regulations.

Do not remove the battery from the PDM after first-time insertion of the battery. If you see any battery performance issues, contact Customer Care.

To safely use the rechargeable battery:

- To prolong battery life, store and charge it in a cool, dry place out of direct sunlight. Avoid leaving or charging the battery where temperature extremes can permanently damage the battery, such as inside a car.
- Your PDM may become warm after prolonged use or when exposed to high temperatures. If your PDM or battery becomes hot to the touch, unplug the USB cable if it is plugged in and avoid prolonged skin contact. Place your PDM on a metal surface and contact Customer Care.
- Do not connect the battery poles with pieces of metal, such as keys or jewellery. Doing so may short-circuit the battery and cause injuries or burns.

11 Taking Care of Your PDM and Pod

- Do not expose the battery or its charger to liquids, including water, rain or snow, as this can cause malfunction. If the battery or charger is exposed to liquid, contact Customer Care.
- Do not allow anyone, including children and pets, to put the battery in their mouth. Doing so may result in damage or explosion.
- Do not place the PDM or battery on or in heating devices, such as microwaves, ovens or radiators. The battery may explode if overheated.
- Do not drop the battery. If the battery is dropped, contact Customer Care.
- Only use an Insulet-approved battery, charger and cable to charge your PDM. Using unapproved batteries, chargers or cables can cause the battery to explode or damage the PDM, and may void the warranty.
- If the battery is damaged so that fluid leaks from the battery, do not allow the leaked fluid to make direct contact with your skin or eyes. If this happens, immediately flush your skin or eyes with clean water and consult a doctor.
- If the battery is deformed or hot to the touch before you insert it into the PDM for the first time, do not use this battery in your PDM. Use of this battery may lead to battery leakage, fire or explosion. Place the battery on a metal surface and leave it outside, if possible. Contact Customer Care for a new battery.
- If the battery deforms, changes colour or overheats while charging, during use or in storage, do not handle the battery and do not continue to use it. Continued use may lead to battery leakage, fire or explosion. Place the PDM on a metal surface and leave it outside, if possible. Contact Customer Care for further instructions.
- Inspect your PDM battery charger before each use. If the adapter for the charger falls in water or is cracked, do not use it.

Contact Customer Care if you notice the following:

- Charging taking too long
- Battery discharging too quickly
- PDM too hot to touch
- Swelling of the PDM and/or battery
- Smell of gas coming from PDM and/or battery
- Wall charger getting too hot

PDM battery messages you may see

Battery message	What you should do
Incompatible Charger	Disconnect charger. Connect provided PDM charger and cable.
Battery Too Warm	Disconnect charger if connected. Allow the PDM to cool down.
Battery Too Cold	The PDM is too cold to be charged. Allow the PDM to reach room temperature.
Charging Error	Disconnect the charger. Allow the PDM to reach room temperature.

Charging the PDM battery

Caution: Use ONLY the Insulet-provided micro-USB charger to charge your PDM. Using unapproved chargers can cause the battery to explode or damage the PDM and may void the warranty.

After a full charge, the battery should have adequate power to operate at least two days under normal use.

A PDM message alerts you when the battery charge is low. The battery icon in the status bar tracks the remaining charge in the battery (see 'The Status Bar' on page 9). To charge the battery, see 'Charge the battery' on page 21.

You can charge your battery many times, but all batteries have a limited lifespan. If you notice a significant deterioration in the duration of the PDM's battery charge, contact Customer Care.

Note: Charging times can vary depending on the surrounding temperature and the remaining battery level.

Tip: *Develop a routine to charge the PDM battery at the same time every day. Do not wait for the low PDM battery message.*

Warnings:

If the battery power becomes critically low, the PDM turns itself off to preserve the data in memory. At this point, you cannot use the PDM until you have plugged in the charger.

Note: If the PDM battery is critically low and the PDM has turned off, your Pod continues to deliver basal insulin according to the active Basal Programme or temp basal. If you do not charge your PDM battery, this insulin delivery continues until the Pod expires.

Note: The history records stay in memory for 60 days even if the battery power is critically low.

11 Taking Care of Your PDM and Pod



Inserting the PDM battery during first time setup

Caution: Use ONLY the rechargeable battery that came with your PDM. Contact Customer Care if you have questions.

Caution: Do not insert the PDM battery while the PDM is connected to external power (e.g. a wall socket via the USB cable and charger).


To insert the PDM battery:

1. Ensure the USB cable is disconnected from the PDM.
2. If your PDM is on, turn it off by pressing the Power button, tapping Power off and then tapping OK.
3. If the white pull tab is still on the PDM, remove the back cover using the pull tab as follows:
 - a. Hold the PDM with the back cover facing you.
 - b. Place two or more fingers of one hand between the pull tab and the PDM.
 - c. Brace the thumb of that hand against the back of the PDM.
 - d. Pull back on the pull tab and lift off the cover.
 - e. Skip to step.
4. If the white pull tab is no longer on the PDM, remove the back cover as follows:
 - a. Hold the PDM sideways and locate the notch in the PDM case. The notch is located in the corner below the Power button.
 - b. Holding the PDM firmly, place a fingernail into the notch and pull the front cover away from the back cover.
 - c. Slide your finger along the opening to completely separate the front from the back.
 - d. Lift off the back cover.
5. Orientate the new battery so the three gold tabs on the battery line up with the three gold pins in the battery compartment. The '-' and '+' signs on the battery label should face you.

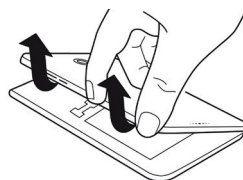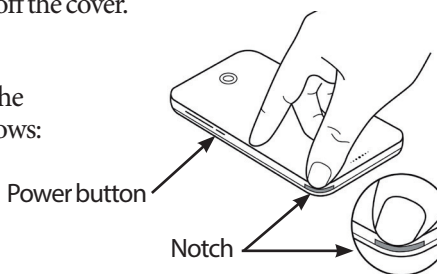

Caution: Do not remove the label from the battery.

Caution: Be careful not to damage any of the small metal parts inside the battery compartment.

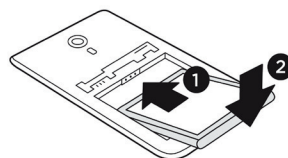

6. Slide the battery into place. Gently push the bottom of the battery to fit it into place.
7. To replace the back cover of the PDM:
 - a. Line up the cover's edge along the side with the Sound/vibrate and Power buttons. Press down on this side first.
 - b. Press down along the back cover until the cover has clicked into place.

The PDM is now ready to use. If this is a new PDM, see 'Power button' on page 132 for instructions for entering your settings.

Note: Do not attempt to replace the battery after first-time insertion. If you are experiencing battery or charging issues, contact Customer Care.

This page intentionally left blank.

CHAPTER 12

Understanding PDM and Pod Function

This chapter explains how the PDM and Pod interact, features such as basal and bolus insulin delivery, and how the Bolus Calculator works.

PDM and Pod Interactions

This section describes how the PDM controls the Pod, and what the Pod can do without input from the PDM.

Pod actions controlled by the PDM

The Pod requires input from the PDM in order to:

- Become activated or deactivated
- Change to a different Basal Programme
- Start a bolus or temp basal
- Cancel a bolus or temp basal
- Suspend or resume insulin delivery

Pod activation

Pods are dormant until they are activated by a PDM. Once a PDM activates a Pod, that PDM and Pod are paired together, which means that they can only communicate with each other even if there are other Pods or PDMs within range. During Pod activation, the PDM transfers the active Basal Programme details to the Pod.

The PDM's communication range is automatically reduced during activation. Therefore, the PDM should touch the Pod (in or out of its tray) during Pod activation. As an added precaution, do not activate your Pod in the same area as someone else who is activating a Pod.

How close do the PDM and Pod need to be after activation?

After activation, the PDM should always be able to communicate with a Pod that is up to 1.5 m (5 ft) away. Depending on the surroundings, the PDM may be able to communicate with a Pod that is as much as 15 metres (50 feet) away. To ensure that you can deliver a bolus, change the basal rate or receive status updates from the Pod, make sure the PDM is within 1.5 metres (5 feet) of the Pod.

12 Understanding PDM and Pod Function

When you are not actively using the PDM, store it in a nearby location, such as a shirt pocket or in a drawer, briefcase or handbag.

Status checks: How the PDM checks Pod function

The PDM periodically queries the Pod about its status, called a 'status check'. The PDM performs a status check:

- When you wake up the PDM.
- When you navigate to the Pod Info tab.
- Periodically, whether the PDM is awake or asleep. These 'sleeping status checks' occur every five minutes.

Note: Following a sleeping status check, the PDM only wakes up and sounds an audible alarm if the Pod has a hazard alarm. If the Pod has an advisory alarm or a notification, the PDM remains asleep and silent.

During a status check, the PDM collects information from the Pod about bolus deliveries, the amount of remaining insulin and any alarm situations. The PDM displays this information as follows:

- The Home screen displays information about the most recent bolus.
- The Pod icon on the status bar shows how much insulin remains in the Pod's reservoir (see "The Status Bar" on page 9).
- The Lock screen and a full-screen message display information about Pod alarms.

Timing of alarms originating in the Pod

If the Pod is sounding a hazard alarm, the Pod broadcasts a signal to the PDM.

- If the PDM is in range and awake, within 25 seconds of the Pod's initial alarm sound, the PDM also sounds an alarm and displays the alarm message.
- If the PDM is in range but asleep, the Pod cannot wake it up. The PDM finds out about the Pod alarm when it does a 'sleeping status check.' In this case, there could be a delay of up to five minutes and 25 seconds between when the Pod sounds the alarm and when the PDM sounds the alarm.
- If the PDM is out of range of the Pod, the PDM cannot receive any communication from the Pod. Therefore, if you hear a Pod alarm or notification, bring the PDM in range of the Pod and wake the PDM up. Within 25 seconds, the PDM sounds the alarm and displays the alarm message.

When the PDM cannot communicate with the Pod

When the PDM attempts to communicate with an active Pod that is in range, communication usually occurs quickly.

The PDM cannot communicate with the Pod when:

- The PDM is, or has temporarily been, too far from the Pod; for example, when you attend a meeting, leaving your PDM back at your desk.
- The PDM's battery has run down.
- There is too much outside interference (see 'Omnipod DASH® System Notice Concerning Interference' on page 185).

For information about handling PDM-Pod communication problems, see 'Communication Errors' on page 120.

Pod deactivation

Deactivation unpairs a PDM and Pod from each other. Deactivation:

- Stops the current Pod's insulin delivery.
- Permanently silences any alarms from that Pod.
- Frees the PDM to activate a new Pod.

Note: Deactivation does not occur automatically when the Pod expires or runs out of insulin. In these situations, you must still use the PDM to deactivate the current Pod before the PDM can activate a new Pod.

Discarding a Pod

When the PDM cannot resolve a communication error, the PDM is unable to deactivate the Pod. In this situation, the PDM asks if you want to 'discard' the Pod. 'Discarding' unpairs the PDM from that Pod but does not stop the Pod's insulin delivery. Therefore, if you tell the PDM to 'discard' a Pod, be sure to remove and dispose of the old Pod before activating a new Pod. To prevent the 'discarded' Pod from sounding an alarm at a later time, follow the instructions for silencing a Pod alarm on page 122. Otherwise, if a discarded Pod sounds an alarm, the alarm stops after 15 hours.

What the Pod can do between PDM instructions

Once activated, the Pod can perform the following without input from the PDM:

- Deliver insulin according to the active Basal Programme's schedule. The Pod has a built-in clock that allows it to track the timing of the Basal Programme's segments.
- Stop delivery of a temp basal at the scheduled time and resume delivery of the active Basal Programme.
- Once a bolus or an extended bolus is started, complete the delivery of the bolus.
- Track the amount of insulin remaining inside the Pod.

12 Understanding PDM and Pod Function

- Track the amount of time until Pod expiry. Stop delivering insulin once the Pod expires.
- Perform self-checks to verify that insulin delivery is occurring as expected. Sound an alarm if it is not.
- Send alarm and notification messages to the PDM, if it is in range.
- Stop insulin delivery if the auto-off feature is enabled and you have not used the PDM within the specified period of time.

Auto-off

The optional auto-off feature is useful if you are prone to hypoglycaemia unawareness. When you enable the auto-off feature, you define a length of time for an auto-off countdown timer. The Pod and PDM will sound an alarm if you do not use your PDM within that amount of time. To change the auto-off setting, see 'Pod auto-off' on page 99.

Resetting the auto-off countdown timer

Any action you take that causes the PDM to communicate with the Pod resets the auto-off countdown timer to the beginning. Therefore, if auto-off is enabled, make sure the PDM is in range of the Pod, then wake up your PDM. This resets the timer and prevents the alarm from sounding.

Auto-off alarms

If you have not used your PDM within the amount of time specified by the auto-off countdown timer, the Pod and PDM sound an advisory alarm every minute for 15 minutes. The PDM also displays an on-screen message.

If you do not use your PDM within 15 minutes of the onset of the Pod's auto-off advisory alarm, the Pod stops delivering insulin and both the PDM and Pod sound a hazard alarm. Tap OK to silence the alarm and deactivate your Pod.

Basal Insulin Delivery

Even without eating, our bodies need a small, constant supply of insulin for normal daily living, which is referred to as 'basal' insulin. In people without diabetes, the pancreas continuously delivers this basal insulin. For people using the Omnipod DASH® System, the Pod mimics the pancreas of a person without diabetes by delivering basal insulin at the rate that you programme into the PDM.

Approximately 50% of a person's total daily insulin dose typically comes from basal insulin delivery; the remaining 50% typically comes from bolus doses.


This section describes the Omnipod DASH® System's two modes for delivering continuous basal insulin: Basal Programmes and temp basals.

Basal Programmes

A basal rate is the number of units of insulin delivered per hour.

A basal segment defines the time of day during which a given basal rate is delivered.

A collection of basal segments covering a midnight-to-midnight period is called a 'Basal Program'. In other words, a Basal Programme describes the rate of insulin delivery throughout an entire 24-hour period.

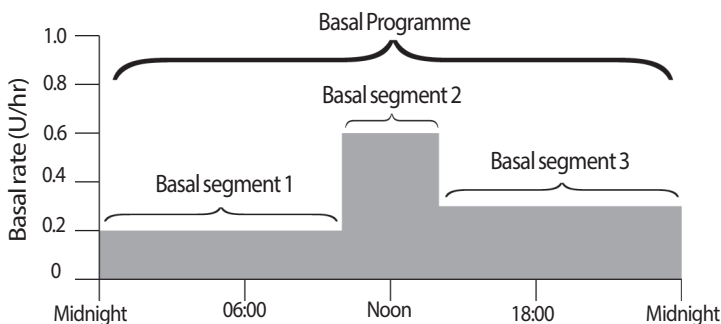


Insulin needs vary throughout the day. Therefore, most people set their basal rates to deliver more or less insulin at certain times of day. For example, you could deliver a lower rate of insulin during the night and a higher rate during the day. This figure shows a Basal Programme with three basal segments.

To create the Basal Programme shown in the preceding figure, the following basal segments are programmed into the PDM:

Segment	Basal rate	
1: Midnight–10:00	0.20 U/hr	Between midnight and 10:00, the Pod delivers 0.20 units of insulin per hour.
2: 10:00 –14:00	0.60 U/hr	Between 10:00 and 14:00, the Pod delivers 0.60 units of insulin per hour.
3: 14:00–midnight	0.30 U/hr	Between 14:00 and midnight, the Pod delivers 0.30 units of insulin per hour.

12 Understanding PDM and Pod Function

You may have different routines on different days of the week; for example your weekend routine may differ from your weekday routine. To handle these predictable changes in your routine, you can create up to 12 different Basal Programmes (see 'Basal Programmes' on page 75). This figure shows two possible Basal Programmes, one for weekdays and the other for weekends.

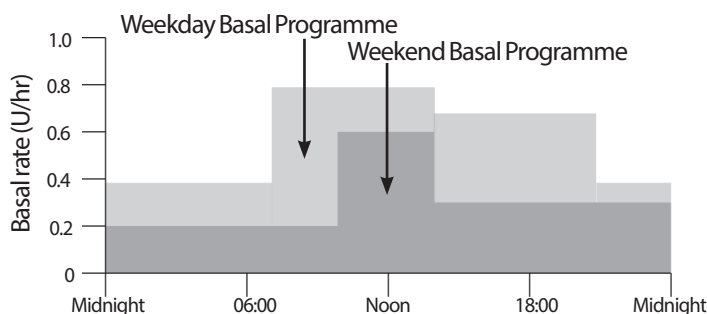

Temporary basal rates

The ability to set temporary basal rates, also called 'temp basals', is an important feature of the Omnipod DASH® System. A temp basal lets you override the currently running Basal Programme by setting a different basal rate for a predetermined period of time.

For example, if you are going cross-country skiing for several hours, you could set a temp basal to lower your basal rate during and after your exercise (see 'Using Temporary Basal Rates' on page 69).


Temp basals can last from 30 minutes to 12 hours. At the end of the specified time, the Pod automatically reverts to the active Basal Programme.

Temp basal settings: Units per hour (U/hr) or per cent (%)

Temp basals can be set using percentage (%) or units per hour (U/hr).

Setting temp basals to units per hour (U/hr) means that the Pod delivers insulin at a flat rate for the duration of the temp basal. In other words, the details of the currently scheduled Basal Programme are ignored during these temp basals.

Setting temp basals to per cent (%) means that insulin delivery follows the pattern defined by the currently scheduled Basal Programme, but increases or decreases the insulin delivery by the specified percentage. For example, a 50% increase raises the Basal Programme's insulin delivery by 50%, while a 50% decrease lowers the Basal Programme's insulin delivery by 50%.

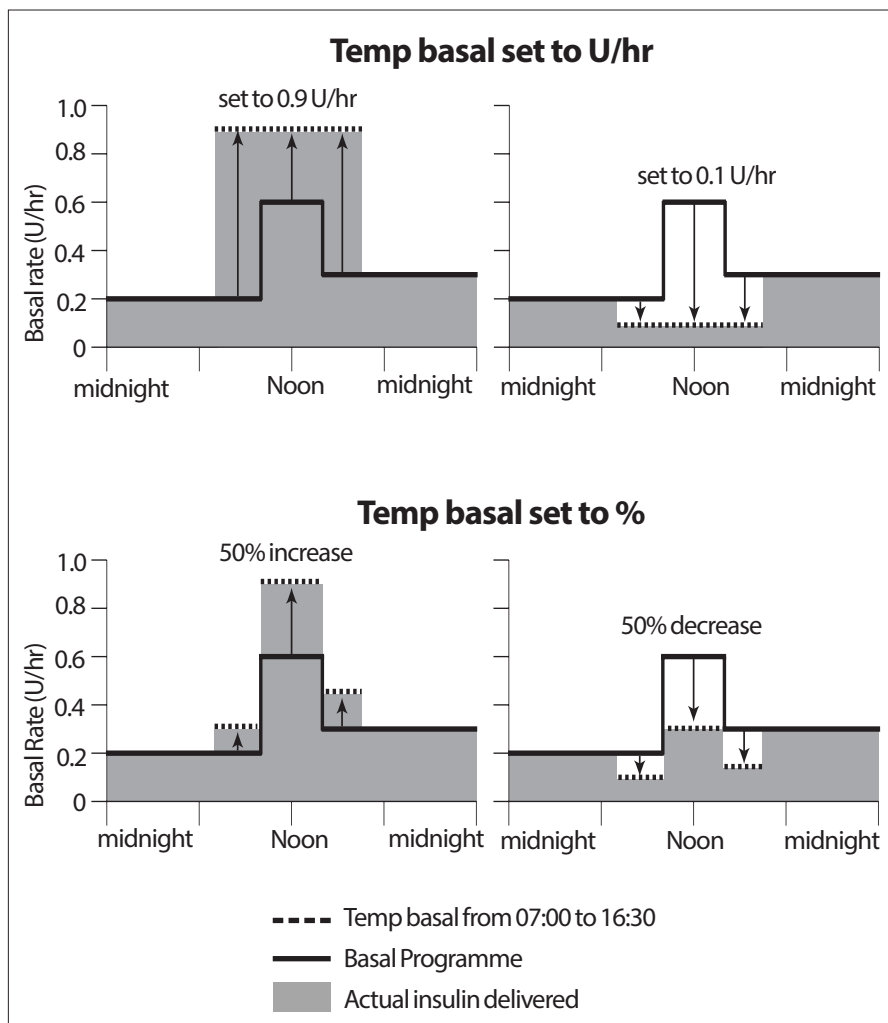

The calculations for the 50% increase temp basal in the preceding figure are:

Segment boundaries*	Basal rate of Basal Programme (U/hr)	50% increase (U/hr)	Resulting temp basal rate: (U/hr)
Midnight–7:00	0.20		
7:00–10:00	0.20	$0.20 \times 50\% = 0.10$	$0.20 + 0.10 = 0.30$
10:00–14:00	0.60	$0.60 \times 50\% = 0.30$	$0.60 + 0.30 = 0.90$
14:00–16:30	0.30	$0.30 \times 50\% = 0.15$	$0.30 + 0.15 = 0.45$
16:30 – midnight	0.30		

* Segments are defined by the currently scheduled Basal Programme.

12 Understanding PDM and Pod Function

Temp basal limitations

Prohibited temp basals: You cannot set a temp basal of 0%, as there would be no change from the active Basal Programme.

Maximum temp basal:

- When using per cent (%), you can set the temp basal up to 95% more than your active Basal Programme's rate with the following exception: You cannot set a temp basal that would go above your maximum basal rate during any time segment covered by the temp basal duration.
- When using a flat rate (U/hr), you cannot set a temp basal above your maximum basal rate.

Temp basals that turn off basal insulin delivery: When using per cent (%), if you set a decrease that results in a flow of less than 0.05 U/hr for a segment, the PDM informs you that you will receive 0 U/hr of insulin for one or more segments.

If the temp basal is long enough, you will eventually receive some insulin. This is because the Pod delivers insulin in 0.05 U pulses. For example, if the flow rate for a basal segment is 0.10 U/hr and you create a temp basal with a 60% decrease for:

- One hour, the resulting flow rate of 0.04 U/hr results in no insulin being delivered for the one-hour duration of the temp basal.
- Two hours, the resulting flow rate of 0.04 U/hr results in a delivery of 0 U insulin in the first hour and 0.05 U insulin in the second hour.

Tip: *Using a temp basal to turn off basal insulin delivery is useful if you want your Basal Programme to automatically resume when the temp basal ends (see 'Methods to temporarily stop insulin delivery' on page 143).*



Temp basal presets

Some temporary changes in your daily routine are easy to predict, and you may know from experience how they affect your insulin needs. For example, you might join a summer football league or attend an exercise class. For women, your monthly hormonal change can affect blood glucose in a predictable manner.


To handle predictable, short-term changes, you can define temp basal presets (see 'Temp basal presets' on page 79). Once stored, a temp basal preset can be activated quickly at a later time (see 'Activate a temp basal preset' on page 71).

Methods to temporarily stop insulin delivery

There may be times when you want to stop all insulin delivery, or at least all basal insulin delivery, for a period of time. If you do not want to deactivate your current Pod, you can request a temporary halt of insulin delivery as follows:

- Suspend insulin delivery:
Menu icon () > Suspend Insulin.
- Set a temp basal to turn off insulin delivery:
Menu icon () > Set Temp Basal. Then select 100% decrease or 0 U/hr.

The following table compares these options for pausing insulin delivery.


	Suspend insulin	Temp basal of 0 U/hr
Effect on basal and bolus insulin delivery	No basal delivery No bolus delivery	No basal delivery boluses allowed
Minimum duration for stopping insulin	30 min	30 min
Maximum duration for stopping insulin	2 hrs	12 hrs
Insulin delivery resumes automatically	No	Yes
Screen display at end of specified duration	'Resume insulin. The insulin suspension period has ended.'	Middle tab of Home screen titled 'Basal' (not 'Temp Basal')
Beeps while insulin is stopped	Every 15 min	At beginning and every 60 min
Beeps at end of a specified duration	Every 15 min until you tap Resume	One beep, then insulin resumes automatically
Must be used when	Editing an active Basal Programme Changing the time or date Testing alarm and vibrate feature	Use is never required
How to cancel	Menu icon () > Resume Insulin	Home: Temp Basal tab > CANCEL

12 Understanding PDM and Pod Function

Immediate and Extended Boluses

A bolus is an extra dose of insulin that is delivered in addition to the continuous basal rate of insulin delivery. Use boluses to bring down high blood glucose levels and to cover the carbohydrates (carbs) in a meal.

You have the option of delivering the entire bolus at once. This is referred to as an 'immediate bolus' or, simply, a 'bolus'. Alternatively, you can spread out the delivery of all or part of a meal bolus so that it is delivered steadily over a specified period of time. This is referred to as an 'extended bolus'.

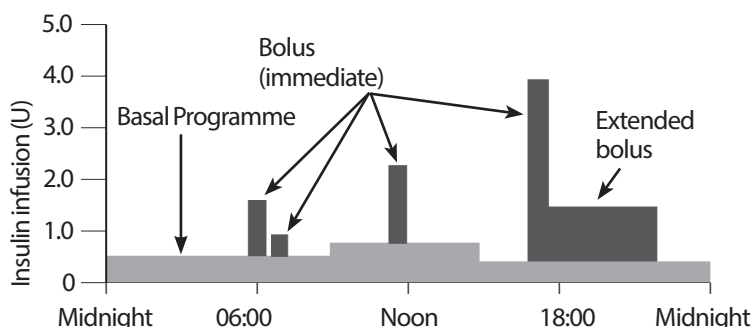

You may want to extend a bolus if your meal contains high-fat or high-protein foods. These foods slow down digestion and therefore slow down the post-meal rise in your blood glucose.

About Manually-calculated Boluses

A manually-calculated bolus is a bolus that you have calculated without the help of the Bolus Calculator. You can use manually-calculated boluses when the Bolus Calculator is set to off or if the Bolus Calculator is disabled (see 'Maximum Bolus' on page 145). Consult your healthcare provider for instructions about how to calculate a bolus.

You can extend some or all of a manually-calculated bolus.

If there is a bolus amount that you deliver frequently, you can create bolus presets (see 'Bolus presets' on page 81) that can be activated quickly at a later time.

Note: You can only use bolus presets if the Bolus Calculator is off.

The Bolus Calculator

The PDM's Bolus Calculator can do a lot of the work of calculating a bolus for you. The Bolus Calculator uses your personal settings and also takes into account any insulin that remains from recent boluses (referred to as insulin on board or IOB).

Bolus Calculator boluses

When calculating a bolus, the Bolus Calculator considers a bolus to be made up of the following two components:

- **Correction bolus:** Used to lower blood glucose when it gets too high.
- **Meal bolus:** Used to cover carbs in a meal.

Extended boluses

When using the Bolus Calculator, you can extend some or all of a meal bolus, but a correction bolus cannot be extended. A correction bolus is always delivered first. In the following example, three units of insulin are extended:

Total bolus = 5 units (1 unit correction bolus + 4 units meal bolus)

→ Deliver now = 2 units (1 unit correction + 1 unit meal bolus)

→ Extend = 3 units (3 units meal bolus)

Maximum Bolus

The PDM does not allow you to enter a bolus that is above your maximum bolus setting. If the Bolus Calculator calculates a bolus amount greater than your maximum bolus, you will only be able to deliver up to the maximum bolus amount. To adjust it, tap the Total Bolus field and enter a revised bolus.

Controlling the bolus amount

The Bolus Calculator is a useful tool, but you have the ultimate control over the amount of a bolus to be delivered. After the Bolus Calculator suggests a bolus amount, you can confirm the suggested bolus or increase or decrease it.

When the Bolus Calculator does not work

The Bolus Calculator does not work when it is disabled or when it is set to off. You control whether the Bolus Calculator is turned on or off, but the PDM controls when it is disabled.

If the Bolus Calculator is set to off, the PDM does not track IOB and does not suggest a bolus.

If the Bolus Calculator is set to on, the PDM may disable it in a few situations. Being disabled means that the Bolus Calculator is temporarily unable to calculate a suggested bolus.

Conditions that disable the Bolus Calculator:	The Bolus Calculator is disabled until:	While the Bolus Calculator is disabled:
Your blood glucose reading is below your Minimum BG for Calcs setting.	Ten minutes pass. or A new blood glucose reading is above your Minimum BG for Calcs setting.	IOB is displayed on the Home screen.
Your blood glucose reading is 'HI'.	Ten minutes pass. or A new blood glucose reading is lower than 'HI'.	IOB is displayed on the Home screen.
There is an unconfirmed bolus when you discard a Pod.	A complete duration of insulin action period passes.	IOB is not displayed on the Home screen.
There is an internal clock reset.	8.5 hours pass.	IOB is not displayed on the Home screen.

Factors used in the Bolus Calculator's calculations

- The Bolus Calculator accounts for the following when it calculates a bolus:
- Your current blood glucose level, Target BG, Correct Above threshold and Correction Factor
 - The carbs you are about to eat and your IC ratio
 - Your duration of insulin action and insulin on board (IOB)
 - Your Minimum BG for Calcs
 - Reverse correction, if it is enabled

Target BG

When calculating a correction bolus, the Bolus Calculator aims to bring your blood glucose down to the Target BG.

Correct Above threshold

The Bolus Calculator only suggests a correction bolus if your blood glucose reading is above your correct above setting. This feature can prevent corrections to blood glucose values that are only slightly higher than your Target BG.

Insulin on board

Insulin on board (IOB) is the amount of insulin still active in your body from earlier boluses. IOB from previous correction boluses is referred to as correction IOB. IOB from previous meal boluses is referred to as meal IOB.

When calculating a new bolus, the Bolus Calculator may reduce the suggested bolus based on the IOB.

The Duration of Insulin Action setting represents the amount of time that insulin remains 'on board' or 'active' in your body.

Note: The Bolus Calculator only subtracts IOB from a suggested bolus if your current blood glucose is known. You should always test your blood glucose prior to delivering a bolus.

Duration of insulin action

The Bolus Calculator uses your duration of insulin action setting to calculate the insulin on board from prior boluses.

Minimum BG for Calcs

The Bolus Calculator does not suggest a bolus if your blood glucose reading is below your Minimum BG for Calcs level.

Reverse Correction


If the Reverse Correction setting is turned on and your blood glucose level is below your Target BG, the Bolus Calculator uses the calculated negative correction bolus to reduce the meal bolus. This allows part of the meal to be used to raise the low blood glucose level towards the Target BG.

If the Reverse Correction setting is turned off, the Bolus Calculator suggests the full meal bolus even if your blood glucose level is below your Target BG.

12 Understanding PDM and Pod Function


Boundaries of the Bolus Calculator suggestions

The following figure shows the boundaries between the types of calculations performed by the Bolus Calculator. For example, the Bolus Calculator suggests a meal bolus, but not a correction bolus, if your blood glucose reading is between your Target BG and your Correct Above settings. If your blood glucose is above the range of your BG meter or is above 600 mg/dL, the reading is recorded as 'HI' and the Bolus Calculator cannot calculate a bolus.

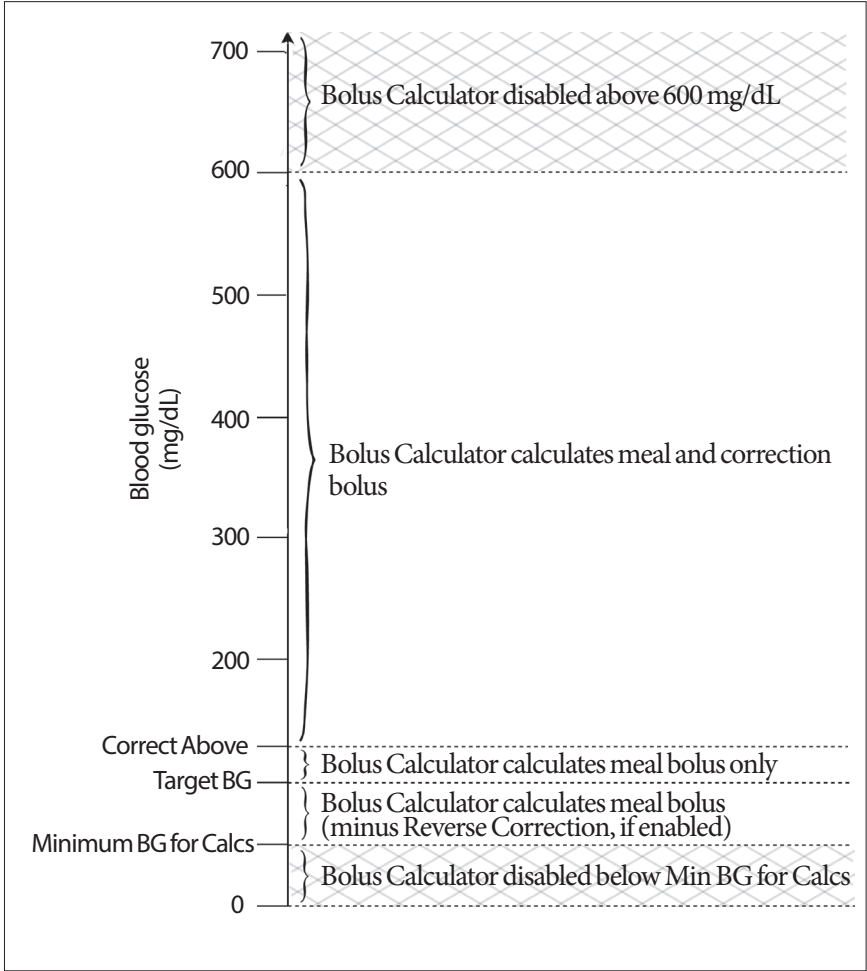

Insulin on board (IOB)

After a bolus is delivered, the amount of insulin that is active in the body decreases over several hours. The Bolus Calculator approximates this decrease in insulin by calculating an 'insulin on board' (IOB) value. The calculated IOB value decreases over time and reaches zero at the time set by the duration of insulin action value. For example, if your duration of insulin action is 4 hours and an 8 unit bolus is given at 18:00, the IOB amount decreases steadily over time as shown in this graph.

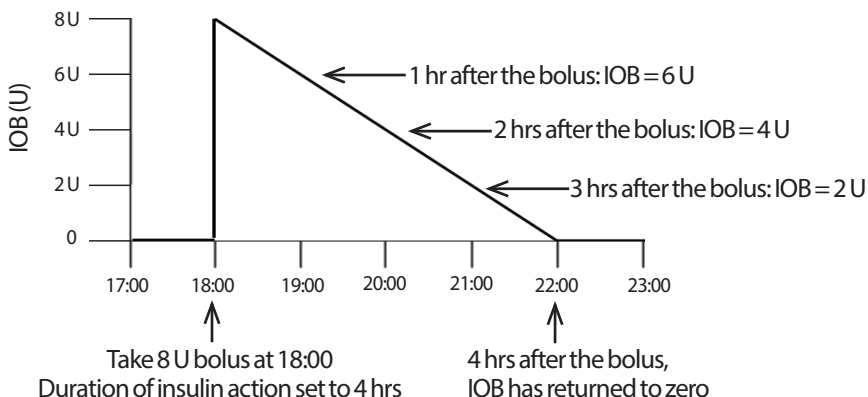


Types of IOB: meal IOB, correction IOB or both

A bolus, and the IOB from that bolus, can be entirely meal, entirely correction or both. The following graphs show how the IOB from a 4 U total bolus decreases over time with a duration of insulin action of 4 hours.


Entirely meal

A meal bolus is calculated when you are eating a meal and your blood glucose is less than your correct above value. The IOB from this bolus is entirely meal IOB.

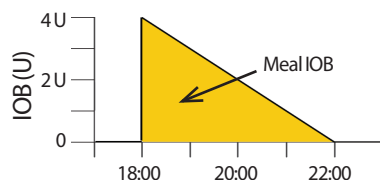


Entirely correction

A correction bolus is calculated when you are not eating and your blood glucose is above your correct above value. The IOB from this bolus is entirely correction IOB.

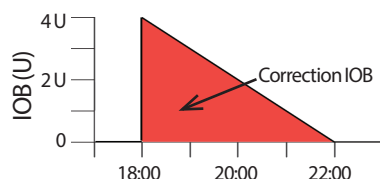

Both meal and correction

When you are eating and your blood glucose is above your correct above value, the total bolus includes a meal bolus and a correction

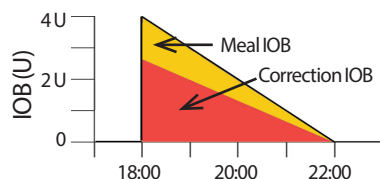

12 Understanding PDM and Pod Function

bolus component. Likewise, the IOB from this bolus has a meal IOB and a correction IOB component.

Insulin on board (IOB) calculations

$$\frac{\text{Duration of insulin action} - \text{time since previous bolus}}{\text{Duration of insulin action}} \times \text{previous bolus}$$

IOB from a previous correction bolus is called a 'correction IOB'.

IOB from a previous meal bolus is called a 'meal IOB'.

Correction IOB example

Duration of insulin action: 3 hours

Time since previous correction bolus: 1 hour

Previous correction bolus: 3 U

$$\frac{3 \text{ hours} - 1 \text{ hour}}{3 \text{ hours}} \times 3 \text{ U} = 2 \text{ U correction IOB}$$

In other words, one hour after your previous correction bolus, your body has used up only 1 unit from the correction bolus. The remaining 2 units of insulin are still in your body working to correct your blood glucose level. Accounting for correction IOB in Bolus Calculations prevents the delivery of too much insulin when correcting a high BG or eating a meal.

Meal IOB example

Duration of insulin action: 3 hours

Time since previous meal bolus: 2 hours

Previous meal bolus: 4.5 U

$$\frac{3 \text{ hours} - 2 \text{ hours}}{3 \text{ hours}} \times 4.5 \text{ U} = 1.5 \text{ U meal IOB}$$

In other words, two hours after your previous meal bolus, your body has used up 3 units from the meal bolus. The remaining 1.5 units of insulin are still in your body working to cover your meal.

Meal IOB is only used to reduce a correction bolus; it does not reduce a new meal bolus. The correction bolus is only reduced until it becomes 0 units.

Bolus calculator equations

The Bolus Calculator first calculates a preliminary correction and meal bolus. It then adjusts these preliminary values for IOB, if applicable. The final suggested bolus is equal to the sum of the resulting correction bolus and meal bolus.

$$\text{Preliminary correction bolus} = \frac{\text{Current BG} - \text{Target BG}}{\text{Correction Factor}}$$

Example: Current BG: 200 mg/dL, Target BG: 100 mg/dL
Correction Factor (CF): 50

$$\frac{200 \text{ mg/dL} - 100 \text{ mg/dL}}{50} = 2 \text{ U prelim. correction bolus}$$

$$\text{Preliminary meal bolus} = \frac{\text{Carb intake}}{\text{Insulin to Carb (IC) ratio}}$$

Example: Carb intake: 45 grams of carb, IC ratio: 15

$$\frac{45}{15} = 3 \text{ U prelim. meal bolus}$$

$$\text{Final correction bolus} = (\text{prelim. correction bolus} - \text{meal IOB}) - \text{correction IOB}$$

The meal IOB is subtracted first. If the preliminary correction bolus is still above zero, then the correction IOB is subtracted

Note: A correction bolus is never reduced below 0 U.

$$\text{Final meal bolus} = \text{prelim. meal bolus} - \text{remaining correction IOB}$$

Meal IOB is never subtracted from a meal bolus. Only a remaining correction IOB is subtracted from a meal bolus (see 'Remaining correction IOB' on page 153).

Note: A meal bolus is never reduced below 0 U.

$$\text{Calculated bolus} = \text{final correction bolus} + \text{final meal bolus}$$

Reverse correction bolus calculation: If the Reverse Correction feature is turned on and if your current blood glucose is below your Target BG but above your minimum BG for Calcs, the Bolus Calculator subtracts a correction amount from the preliminary meal bolus.

$$\text{Meal bolus with reverse correction} = \text{Reverse correction} + \text{prelim meal bolus}$$

Example: Current BG: 75 mg/dL, Target BG: 100 mg/dL
Correction Factor: 50, Preliminary meal bolus: 1.5 U

$$\frac{75 \text{ mg/dL} - 100 \text{ mg/dL}}{50} = -0.5 \text{ U Reverse Correction}$$

$$-0.5 \text{ U (reverse correction)} + 1.5 \text{ U (prelim meal bolus)} = 1.0 \text{ U meal bolus}$$

A reverse correction is only applied to the meal bolus. In this example, the meal bolus is reduced by 0.5 units.

Bolus Calculator rules

The Bolus Calculator applies the following rules to the suggested bolus doses:


- Suggested bolus doses are rounded down to the nearest 0.05 U.
- If the total of the suggested bolus calculation (correction bolus plus meal bolus) is less than zero, the suggested bolus dose is 0.00 U.
- The Bolus Calculator does not suggest a bolus dose if your current blood glucose reading is below your minimum BG for Calcs.
- The Bolus Calculator suggests a correction bolus only when your blood glucose reading is above your correct above threshold.
- Meal IOB, which is insulin on board from a previous meal bolus, is subtracted first from the current correction bolus, if any, until the correction bolus is zero. However, any remaining meal IOB is *never* subtracted from the current meal bolus.
- Correction IOB, which is insulin on board from a previous correction bolus, is then subtracted from the current correction bolus, if any remains, until the correction bolus is zero. At that point, any remaining correction IOB is subtracted from the current meal bolus.
- The calculated insulin on board is rounded up to the nearest 0.05 U.
- A reverse correction only occurs if the meal bolus is greater than 0.00 U.
- If your blood glucose value is below your Target BG value and Reverse Correction is on, the Reverse Correction bolus is subtracted from the suggested meal bolus. If reverse correction is off, the Bolus Calculator does not subtract insulin from the suggested meal bolus dose.

If you manually adjust the suggested bolus, the following rules apply:

- If you increase a suggested bolus, the increase is applied to the meal bolus unless there is no meal bolus, in which case it is applied to the correction bolus.
- If you decrease a suggested bolus, the decrease is applied to the meal bolus until it reaches zero; then it is applied to the correction bolus. If there is no meal bolus, the decrease is applied to the correction bolus.

Overview of the Bolus Calculator CALCULATIONS screen

The Bolus Calculator's screen can be accessed by tapping VIEW BOLUS CALCULATIONS from the Insulin & BG history screen or by tapping CALCULATIONS from the Bolus Calculator screen.

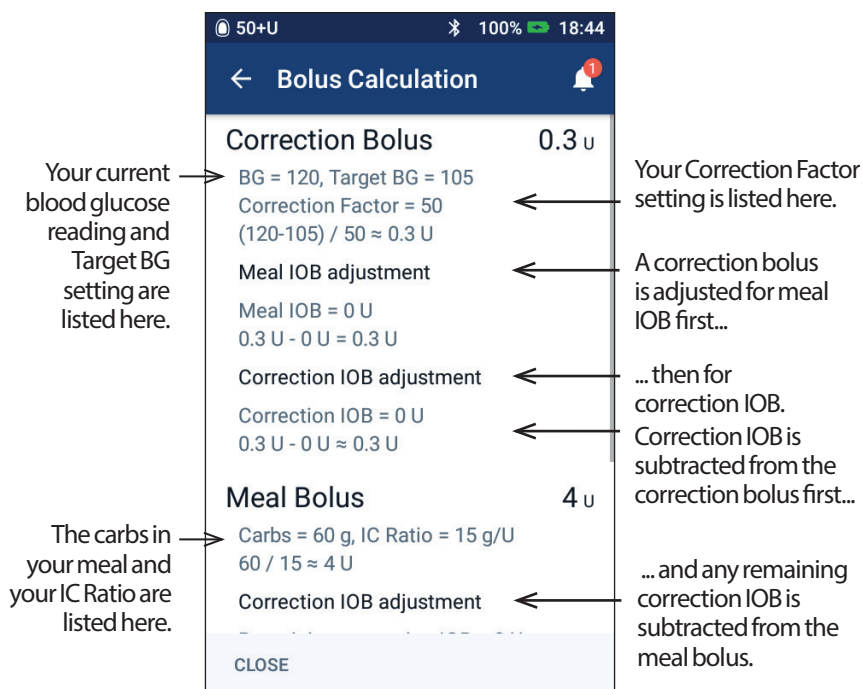

You need to scroll down to see the total bolus and any adjustments you may have made to the calculated bolus.

The blood glucose units on the Calculations screen are mg/dL.

Note: IOB adjustments are only made if you have entered a blood glucose reading.

Remaining correction IOB

A calculated bolus amount can never be less than zero. If a correction IOB is greater than the preliminary correction bolus, subtracting only part of the correction IOB from the preliminary bolus brings the correction bolus to zero.


The 'remaining correction IOB' is the amount of the correction IOB that was not needed to bring the preliminary correction bolus to zero. This remaining amount is then used to reduce the meal bolus, if there is a meal bolus.

12 Understanding PDM and Pod Function

The preliminary and final boluses in the CALCULATIONS screen

The Bolus Calculation screen shows the calculations for a preliminary bolus first (correction and meal bolus), and then shows the IOB adjustments that produce the final correction bolus and final meal bolus. See 'Bolus calculator equations' on page 151 for a step-by-step description of these calculations.

The preliminary correction bolus calculations and where the preliminary correction boluses appear on the Calculations screen, as well as which numbers represent the final correction bolus, are identified here:

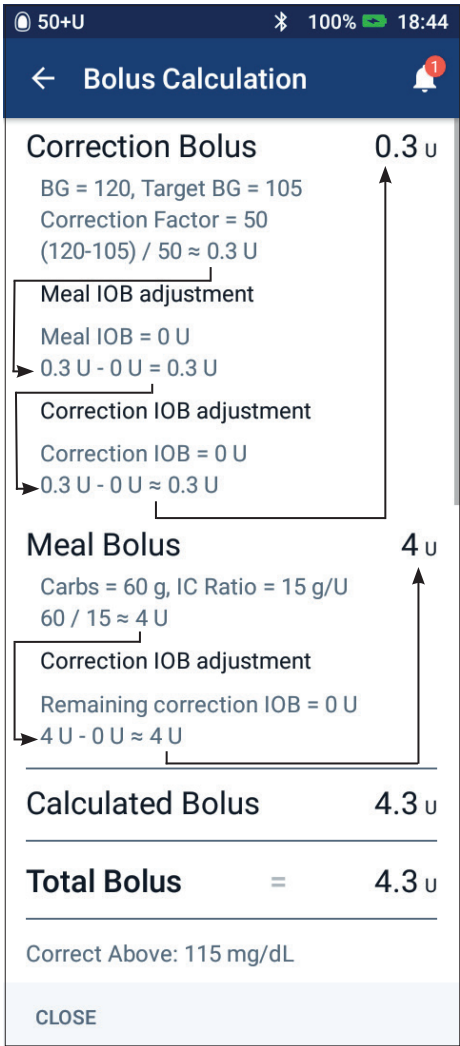

Final correction bolus
(rounded down to the
nearest 0.05 U)

Final meal bolus

The following pages show examples of the Calculations screen with various blood glucose, meal and IOB scenarios.

Bolus Calculator examples

For an explanation of IOB, see page 149. For an explanation of the correction and meal bolus equations, see page 151.

Example 1: Eating 45 g carbs, BG of 150 mg/dL is above target, no IOB

Duration of insulin action = 4 hours. You have not delivered a bolus in the last 4 hours, so there is no IOB (no meal IOB, no correction IOB).

Correction Bolus BG = 150, Target BG = 100 Correction factor = 50 $(150 - 100) / 50 = 1 \text{ U}$ Meal IOB adjustment Meal IOB = 0 U $1 \text{ U} - 0 \text{ U} = 1 \text{ U}$ Correction IOB adjustment Correction IOB = 0 U $1 \text{ U} - 0 \text{ U} = 1 \text{ U}$	1 U	← Your blood glucose is 50 mg/dL over your target. With your correction factor at 50 and no IOB adjustment, you will need a 1 U correction bolus.
		← No IOB, so no IOB adjustment.
		← No IOB, so no IOB adjustment.
Meal Bolus Carbs = 45 g, IC ratio = 15 g/U $45 / 15 = 3 \text{ U}$ Correction IOB adjustment Remaining correction IOB = 0 U $3 \text{ U} - 0 \text{ U} = 3 \text{ U}$	3 U	← You are eating 45 g carbs. With your IC ratio at 15 and no IOB adjustment, you will need a 3 U meal bolus.
		← No IOB, so no IOB adjustment.
Calculated Bolus Your Adjustment	4 U 0 U	← The calculated bolus is the sum of your correction bolus and meal bolus. ← Any adjustment you make to the calculated bolus is shown here. No adjustment was made.
Total Bolus	4 U	← The total bolus is the sum of the calculated bolus and any adjustment made by you.

12 Understanding PDM and Pod Function

Example 2: Not eating, BG of 150 mg/dL is above target, 1 U meal IOB, 1 U correction IOB

Duration of insulin action = 4 hours.

Three hours ago, you delivered an 8 U bolus (4 U meal bolus, 4 U correction bolus) because you were eating and had a high blood glucose reading.

Now, three hours after the 8 U bolus, there are 2 U of IOB left (1 U meal IOB, 1 U correction IOB).

Correction Bolus	0 U	
BG = 150, Target BG = 100		
Correction factor = 50		
$(150 - 100) / 50 = 1 \text{ U}$	←	Your blood glucose is 50 mg/dL over your target. A correction factor of 50 gives a 1 U preliminary correction bolus.
Meal IOB adjustment		
Meal IOB = 1 U		
$1 \text{ U} - 1 \text{ U} = 0 \text{ U}$	←	However, this preliminary correction bolus is reduced to zero by the 1 U meal IOB adjustment.
Correction IOB adjustment		
Correction IOB = 1 U		
N/A: Correction bolus is $\leq 0 \text{ U}$	←	Because the preliminary correction bolus has been reduced to zero, the correction IOB is not subtracted.
Meal Bolus	0 U	← You are not eating, so your meal bolus is 0 U.
Carbs = 0 g, IC ratio = 15 g/U		
$0 / 15 = 0 \text{ U}$		
Correction IOB adjustment		
Remaining correction IOB = 1 U	←	None of the correction IOB was subtracted from the correction bolus, so the remaining correction IOB is 1 U. However, the meal bolus is already zero.
$0 \text{ U} - 1 \text{ U} = 0 \text{ U}$		
Calculated Bolus	0 U	← Although your blood glucose is above target, the meal IOB reduces the total bolus to 0 U.
Your Adjustment	0 U	
Total Bolus	0 U	

Example 3: Eating 45 g carbs, BG of 100 mg/dL is at target, 1 U meal IOB

Duration of insulin action = 4 hours.

Three hours ago, you delivered a 4 U bolus (4 U meal bolus) because you were eating. There was no correction bolus because your blood glucose was at your Target BG.

Now, three hours later and just before your next meal, there is 1 U of IOB from the previous meal, which is a 1 U meal IOB. There is no correction IOB because the earlier bolus had no correction component.

Correction Bolus		0 U	← Blood glucose is at target, so the correction bolus is zero.
BG = 100, Target BG = 100			
Correction factor = 50			
$(100 - 100) / 50 = 0 \text{ U}$			
Meal IOB adjustment			
Meal IOB = 1 U			
N/A: Correction bolus is $\leq 0 \text{ U}$			← The correction bolus is zero, so the meal IOB is not subtracted.
Correction IOB adjustment			
Correction IOB = 0 U			
N/A: Correction bolus is $\leq 0 \text{ U}$			← No IOB adjustment.
Meal Bolus		3 U	
Carbs = 45 g, IC ratio = 15 g/U			
$45 / 15 = 3 \text{ U}$			← You are eating 45 g carbs. With your IC ratio at 15 and no correction IOB adjustment, the meal bolus is 3 U.
Correction IOB adjustment			
Remaining correction IOB = 0 U			
$3 \text{ U} - 0 \text{ U} = 3 \text{ U}$			← No IOB adjustment is made. There is no correction IOB, and meal IOB is never subtracted from a meal bolus.
Calculated Bolus		3 U	← Even though there was a 1 U meal IOB, it did not reduce the calculated bolus.
Your Adjustment		0 U	
Total Bolus		3 U	

12 Understanding PDM and Pod Function

Example 4: Eating 60 g carbs, BG of 150 mg/dL is above target, 1 U correction IOB

Duration of insulin action = 2 hours.

One hour ago, you delivered a 2 U bolus to correct a high blood glucose. Because you were not eating at that time and did not enter a carbohydrate value into the Bolus Calculator, this was a 2 U correction bolus.

Now, one hour later, you are about to eat. There is a 1 U correction IOB from the earlier bolus, and no meal IOB because you did not eat at the time of your last bolus.

Correction Bolus	0 U	
BG = 150, Target BG = 100		
Correction factor = 50		
$(150 - 100) / 50 = 1 \text{ U}$		← Your blood glucose is 50 mg/dL over your target. A correction factor of 50 gives a 1 U preliminary correction bolus.
Meal IOB adjustment		
Meal IOB = 0 U		
$1 \text{ U} - 0 \text{ U} = 1 \text{ U}$		← No meal IOB, so no meal IOB adjustment.
Correction IOB adjustment		
Correction IOB = 1 U		
$1 \text{ U} - 1 \text{ U} = 0 \text{ U}$		← The 1 U correction IOB reduces the correction bolus to zero.
Meal Bolus	4 U	
Carbs = 60 g, IC ratio = 15 g/U		
$60 / 15 = 4 \text{ U}$		← You are eating 60 g carbs. An IC ratio of 15 gives a 4 U preliminary meal bolus.
Correction IOB adjustment		
Remaining correction IOB = 0 U		
$4 \text{ U} - 0 \text{ U} = 4 \text{ U}$		← The entire correction IOB was used to reduce the correction bolus to zero, so no correction IOB remains. The meal bolus remains at 4 U.
Calculated Bolus	4 U	← The calculated bolus is entirely a meal bolus. Even though your blood glucose is above target, the correction IOB reduced the correction bolus to zero.
Your Adjustment	0 U	
Total Bolus	4 U	

Example 5: Reverse correction on, eating 45 g carbs, BG of 75 mg/dL is below target, no IOB

Duration of insulin action = 2 hours. You have not given yourself a bolus in the past 2 hours, so there is no IOB (no meal IOB, no correction IOB).

Correction Bolus BG = 75, Target BG = 100 Correction factor = 50 $(75 - 100) / 50 = -0.5 \text{ U}$ (Reverse correction ON)	-0.5 U	← Your blood glucose is below your target, so a reverse correction is calculated.
Meal IOB adjustment Meal IOB = 0 U N/A: Correction bolus is $\leq 0 \text{ U}$		← No change in the reverse correction bolus for two reasons: IOB is zero, and IOB is never subtracted from a reverse correction.
Correction IOB adjustment Correction IOB = 0 U N/A: Correction bolus is $\leq 0 \text{ U}$		← No change in the reverse correction bolus for two reasons: IOB is zero, and IOB is never subtracted from a reverse correction.
Meal Bolus Carbs = 45 g, IC ratio = 15 g/U $45 / 15 = 3 \text{ U}$	3 U	← You are eating 45 g carbs. With your IC ratio at 15 and no correction IOB adjustment, the meal bolus is 3 U.
Correction IOB adjustment Remaining correction IOB = 0 U $3 \text{ U} - 0 \text{ U} = 3 \text{ U}$		← No IOB, so no IOB adjustment.
Calculated Bolus	2.50 U	← The negative reverse correction bolus reduces the meal bolus.
Your Adjustment	0 U	With reverse correction set to On and a blood glucose below your target, the meal bolus is reduced to allow some of the meal carbs to increase your blood glucose towards your target.
Total Bolus	2.50 U	

Calculations for History Summaries

This section lists the calculations for the summary data shown on the one-day and multiple-day history screens.

Blood glucose summaries

Blood glucose summaries are calculated using all readings entered, including any HI and LO readings, unless otherwise noted. The multi-day calculations include all days in the time period.

Summary item	Calculation
Row 1 (see page 84)	
Average BG	$= \frac{\text{Sum of all BG readings}}{\text{Total number of BG readings}}$ <p>Note: The sum and the total do not include HI or LO blood glucose readings.</p>
BG in Range	$= \frac{\text{Number of BG readings within BG Goal Range}}{\text{Total number of BG readings}} \times 100$
BGs Above	$= \frac{\text{Number of BG readings above BG Goal Range upper limit}}{\text{Total number of BG readings}} \times 100$
BGs Below	$= \frac{\text{Number of BG readings below BG Goal Range lower limit}}{\text{Total number of BG readings}} \times 100$
Row 2 (see page 84)	
Avg Readings per Day	$= \frac{\text{Number of BG readings}}{\text{Number of days}}$ <p>Note: This appears in the multiple day view only.</p>
Number of BG Readings	$= \text{Total number of BG readings in the day (or date range)}$
Highest BG	$= \text{Highest BG reading in the day (or date range)}$
Lowest BG	$= \text{Lowest BG reading in the day (or date range)}$

Insulin delivery summaries

Bolus insulin calculations include Bolus Calculator boluses and manually-calculated boluses. If you cancel an immediate or extended bolus before it completes, only the amount actually delivered is included in the calculation.

Basal insulin calculations include insulin delivered according to the active Basal Programme adjusted for periods when a temp basal was running, insulin was suspended or there was no active Pod.

When the PDM has not received confirmation from the Pod about actual insulin delivery, the insulin delivery calculations are estimates based on the scheduled insulin delivery (see page 91).

For calculations of insulin delivery over multiple days, the number of days in the time period does not include days in which no insulin (either basal or bolus) was delivered.

For calculations of total carbohydrates over multiple days, the number of days in the time period does not include days in which no carbohydrates were entered into the Bolus Calculator.

Summary item	Calculation
Row 3 (see page 84)	
Total Insulin	= Sum of basal and bolus insulin delivered
Average Total Insulin	= $\frac{\text{Sum of basal and bolus insulin delivered}}{\text{Number of days}}$
Basal Insulin	= Amount of basal insulin delivered
Average Basal Insulin	= $\frac{\text{Amount of basal insulin delivered}}{\text{Number of days}}$
Bolus Insulin	= Amount of bolus insulin delivered
Average Bolus Insulin	= $\frac{\text{Amount of bolus insulin delivered}}{\text{Number of days}}$
Total Carbs	= Total grams of carbs entered into the Bolus Calculator
Average Total Carbs	= $\frac{\text{Total grams of carbs entered into the Bolus Calculator}}{\text{Number of days}}$

This page is intentionally left blank.

CHAPTER 13

Living with Diabetes

Warning: If you are unable to use the Omnipod DASH® System according to the instructions, you may be putting your health and safety at risk. Talk with your healthcare provider if you have concerns about using the Omnipod DASH® System.

Before deciding on the Omnipod DASH® System, you and your healthcare provider discussed the benefits of the Omnipod DASH® System, as well as the responsibilities that come with insulin pump therapy. Remember: safe use begins and ends with you. If you have questions or doubts about being able to safely use the Omnipod DASH® System at any time, consult your healthcare provider immediately.

Daily Activities

To ensure proper Omnipod DASH® System operation and your continued good health, check your infusion site, your PDM messages and your blood glucose frequently.

Infusion site checks

At least once a day, use the Pod's viewing window to inspect the infusion site. Check the site for:

- Leakage or scent of insulin, which may indicate that the cannula has dislodged
- Signs of infection, such as pain, swelling, redness, discharge or heat

Warnings:

If an infusion site shows signs of infection:

- Immediately remove the Pod and apply a new Pod at a different infusion site.
- Contact your healthcare provider. Treat the infection according to instructions from your healthcare provider.

If you see blood in your cannula, check your blood glucose more frequently to ensure that insulin delivery has not been affected. If you experience unexpected elevated blood glucose levels, change your Pod.

Tip: Consider making infusion-site checks part of your daily routine, like showering or brushing your teeth.

Check your blood glucose frequently

When you routinely check your blood glucose level, you can identify and treat high or low blood glucose before it becomes a problem.

Check your blood glucose:

- At least 4 to 6 times a day: when you wake up, before every meal and before going to bed
- Whenever you feel nauseous or sick
- Before driving a car
- Whenever your blood glucose has been running unusually high or low, or if you suspect that your blood glucose is high or low
- Before, during and after exercise
- As directed by your healthcare provider

Prepare for Emergencies

Tip: Ask your healthcare provider to help you develop plans for handling emergency situations, including what to do if you cannot reach your healthcare provider.

Warning: ALWAYS keep an emergency kit with you to quickly respond to any diabetes emergency or in the case that your Omnipod DASH System stops working. Always carry supplies to perform a Pod change should you need to replace your Pod at any time.

Prepare an emergency kit to keep with you at all times. The kit should include:

- Several new, sealed Pods
- A vial of rapid-acting U-100 insulin (see 'General Warnings' on page xii for insulins cleared for use in the Omnipod DASH® System)
- Syringes or pens for injecting insulin
- Blood glucose test strips
- Blood glucose meter
- Ketone test strips
- Lancing device and lancets
- Glucose tablets or another fast-acting source of carbohydrate
- Alcohol prep swabs
- Instructions from your healthcare provider about how much insulin to inject if delivery from the Pod is interrupted
- A signed letter from your healthcare provider explaining that you need to carry insulin supplies and the Omnipod DASH® System
- Phone numbers for your healthcare provider and/or doctor in case of an emergency
- Glucagon kit and written instructions for giving an injection if you are unconscious (see 'Avoid Lows, Highs and DKA' on page 168)

Travelling and Holidays

It is important that you check your blood glucose more frequently while you are travelling. Changes in time zones, activity levels and meal times can all affect your blood glucose levels.

Proper preparation is important when travelling. The following sections will help you prepare for your travels.

Keep supplies accessible

On aeroplanes, trains and buses, keep these items with you, rather than checking them in:

- Personal Diabetes Manager (PDM)
- Extra Pods
- An emergency kit
- Vials of insulin (cargo area temperatures may affect insulin)
- A signed letter from your healthcare provider explaining that you need to carry insulin supplies and the Omnipod DASH® System
- Prescriptions for all medications
- Medications and supplies with their original prescription label

Note: Generic medications may be easier to find than brand names outside your country.

- Snacks and hypoglycaemia treatment, in case food is not available
- Bottled water (especially on planes) to prevent dehydration
- The name and phone number of your doctor and of a doctor at your final destination

Note: Keep your emergency kit with you during trips or holidays (see "Prepare for Emergencies" on page 165). As it may be difficult or impossible to get insulin or supplies in an unfamiliar place, take more supplies than you think you'll need.

Tip: When you travel outside the country or for long periods of time, be sure to take extra Pod supplies. Prior to departure, call Customer Care to enquire about additional Omnipod DASH® System supplies for your trip.

Plan for changing time zones

If you are planning a holiday or business trip to a different time zone, you may need to adjust your Basal Programmes. For changes of just a few hours, basal rate adjustments are minor and easy to calculate. For long-distance travel, however, figuring out the correct Basal Programme can be more challenging. Your healthcare provider can help with these adjustments.

Airports and flying

Before travelling by plane, familiarise yourself with the airport's security procedures and prepare your diabetes supplies for the security process and flight.

Airport security

Prepare for your travel:

- Airport security checks and screening procedures may change, so review the airport website for travel updates before your trip.
- Arrive at the airport 2-3 hours before your flight.
- Have your insulin management supplies easily accessible to ensure that airport security checks run smoothly.

The airport may offer the option of requesting a visual inspection of your medical supplies rather than putting them through the X-ray. You must request this before the screening process begins. Your medical supplies should be in a separate bag when you approach the security officer.

To prevent contamination or damage to your supplies, you should be asked at the security checkpoint to display, handle and repack your own supplies during the visual inspection process. Any medication and/or associated supplies that cannot be cleared visually must be submitted for X-ray screening.

If you are concerned about going through the walk-through metal detector, notify the security officer that you're wearing an insulin pump. You should advise the security officer that the insulin pump cannot be removed because it is inserted with a catheter (tubing) under the skin.

Check your airport's website if you have any further questions or concerns.

Note: Pods and PDMs can safely pass through airport X-ray machines (see the 'Omnipod DASH® System Notice Concerning Interference' on page 185).

Flying and flight mode

Warning: The atmospheric pressure in an aeroplane cabin can change during flight, which may affect the Pod's insulin delivery. Check your blood glucose frequently while flying. If needed, follow your healthcare provider's treatment instructions.

The PDM controls the Pod using Bluetooth® wireless technology. Before flying, check your airline's policy regarding the use of Personal Medical Electronic Devices that communicate using Bluetooth® technology.

If use of a Personal Electronic Device using Bluetooth® technology is allowed, set your PDM to flight mode while on the aeroplane (see 'Flight mode' on page 93). The Bluetooth® setting remains enabled in the PDM's airplane mode so you can communicate with your Pod.

Note: The Omnipod DASH® System is safe to use at atmospheric pressures typically found in aeroplane cabins during flight. The Omnipod DASH® System can be used at atmospheric pressures as low as 700 hPa, which is lower than the typical pressure in aeroplane cabins.

Avoid Lows, Highs and DKA

Act promptly at the first sign of hypoglycaemia, hyperglycaemia or diabetic ketoacidosis. The easiest and most reliable way to avoid these conditions is to check your blood glucose often.

General precautions

- Keep careful records and discuss changes and adjustments with your healthcare provider.
- Tell your healthcare provider if you have extreme highs or lows, or if highs or lows are occurring more often than usual.
- If you have technical problems with your Omnipod DASH® System and cannot resolve them, call Customer Care immediately.

Hypoglycaemia (low blood glucose)

Hypoglycaemia can occur even when a Pod is working properly. Never ignore the signs of low blood glucose, no matter how mild. If left untreated, severe hypoglycaemia can cause seizures or lead to unconsciousness. If you suspect that your blood glucose level is low, check your blood glucose level to confirm.

Symptoms of hypoglycaemia (low blood glucose)

Never ignore the following symptoms, as they could be signs of hypoglycaemia:

- Shakiness
- Fatigue
- Unexplained sweating
- Cold, clammy skin
- Weakness
- Blurred vision or a headache
- Sudden hunger
- Rapid heart rate
- Confusion
- Tingling in the lips or tongue
- Anxiety
- Irritability

Tip: Hypoglycaemia unawareness is a condition in which you do not realise when your blood glucose level is low. If you are prone to hypoglycaemia unawareness, you may want to use the PDM's blood glucose reminder and check your blood glucose more frequently (see 'Check BG after Bolus reminders' on page 100).

Tip: Make sure that your blood glucose is at least 100 mg/dL before driving or working with dangerous machinery or equipment. Hypoglycaemia may cause you to lose control of a car or dangerous equipment. Also, when you focus intently on a task, you may miss the symptoms of hypoglycaemia.

Tip: Even if you cannot check your blood glucose, do NOT wait to treat symptoms of hypoglycaemia, especially if you are alone. Waiting to treat symptoms could lead to severe hypoglycaemia, which can quickly lead to shock, coma or death.

Tip: Teach people you trust (like family members and close friends) how to give a glucagon injection. You will need to rely on them to give it to you if you have severe hypoglycaemia and become unconscious. Include a copy of the glucagon instructions in your emergency kit and periodically review the procedure with family and friends.

To avoid hypoglycaemia (low blood glucose)

- Work with your healthcare provider to establish individualised BG Targets and guidelines.
- Keep a fast-acting carbohydrate with you at all times to respond quickly to low blood glucose. Examples of fast-acting carbs are glucose tablets, hard sweets or juice.
- Teach your friends, family members and colleagues to recognise the signs of hypoglycaemia, so they can help if you develop hypoglycaemia unawareness or a severe adverse reaction.
- Keep a glucagon injection kit with your emergency supplies. Teach friends and family members how to give a glucagon injection in case you have severe hypoglycaemia and become unconscious.

Periodically check the expiry date of your glucagon kit to make sure it has not expired.

Note: Always carry medical identification (such as an emergency wallet card) and wear an emergency medical necklace or bracelet such as the Medic Alert tag.

Again, **frequent blood glucose checks are the key to avoiding potential problems.** Detecting low blood glucose early lets you treat it before it becomes a problem.

Check with your healthcare provider for guidance in any and all areas listed above.

To treat hypoglycaemia (low blood glucose)

Any time your blood glucose is low, treat it immediately according to your healthcare provider's instructions. Check your blood glucose every 15 minutes while you are treating to make sure you don't overtreat the condition and cause blood glucose levels to rise too high. Contact your healthcare provider as needed for guidance.

13 Living with Diabetes

Possible causes of hypoglycaemia	Suggested action
Incorrect Basal Programme	<p>Confirm that the correct Basal Programme is active.</p> <p>Confirm that the PDM time is set correctly.</p> <p>Consult your healthcare provider about adjusting your Basal Programmes or using a temp basal.</p>
Incorrect bolus timing or bolus too large	<p>Take bolus with food.</p> <p>Check blood glucose before giving a meal bolus. If necessary, adjust the bolus.</p> <p>Check the bolus size and timing.</p> <p>Do not overcorrect for post-meal glucose levels.</p> <p>Check carb intake.</p> <p>Consult your healthcare provider for guidance.</p>
Incorrect Target BG level or incorrect Correction Factor or incorrect IC Ratio	<p>Consult your healthcare provider about refining these settings as needed.</p>
Prone to severe hypoglycaemia or hypoglycaemia unawareness	<p>Consult your healthcare provider about hypoglycaemia unawareness and about raising Target BG levels.</p>
Unplanned physical activity	<p>Consult your healthcare provider about using a temp basal (temporary basal rate).</p>
Prolonged or intense exercise	<p>Adjust insulin delivery as instructed by your healthcare provider.</p> <p>Check blood glucose before, during and after activity and treat as necessary.</p> <p>Note: Effects of exercise can last several hours – even a full day – after activity ends.</p> <p>Consult your healthcare provider about adjusting your Basal Programmes or using a temp basal.</p>
Low carbohydrate intake prior to activity	<p>Check blood glucose before activity.</p> <p>Consult your healthcare provider for guidance.</p>

Possible causes of hypoglycaemia	Suggested action
Alcohol consumption	Check blood glucose frequently, especially before going to bed. Consult your healthcare provider for guidance.

Hyperglycaemia (high blood glucose)

Pods use rapid-acting insulin, so you have no long-acting insulin in your body. If an occlusion (interruption of insulin delivery from the Pod) occurs, your blood glucose can rise rapidly.

Warning: An occlusion may result from a blockage, Pod malfunction or from using old or inactive insulin (see 'Occlusion detection' on page 182). If insulin delivery is interrupted by an occlusion, check your blood glucose level and follow the treatment guidelines established by your healthcare provider. Hyperglycaemia could result if appropriate actions are not taken.

Tip: *Hyperglycaemia symptoms can be confusing. Always check your blood glucose before you treat for hyperglycaemia.*

Symptoms of hyperglycaemia (high blood glucose)

Never ignore the following symptoms, as they could be a sign of hyperglycaemia:

- Fatigue
- Frequent urination, especially during the night
- Unusual thirst or hunger
- Unexplained weight loss
- Blurred vision
- Slow healing of cuts or sores

To avoid hyperglycaemia (high blood glucose)

Check your blood glucose:

- At least 4–6 times a day (when you wake up, before each meal and before going to bed)
- If you feel nauseous or sick
- Before driving a car
- Whenever your blood glucose has been running unusually high or low
- If you suspect that your blood glucose is high or low

13 Living with Diabetes

- Before, during and after exercise
- As directed by your healthcare provider

To treat hyperglycaemia (high blood glucose)

Always check your blood glucose levels frequently while treating hyperglycaemia. You don't want to overtreat the condition and cause your blood glucose level to drop too far.

1. Check your blood glucose level. The result will help you to find out how much insulin is needed to return your blood glucose to your blood glucose goal.
2. If your blood glucose is 250 mg/dL or above, check for ketones. If ketones are present, follow your healthcare provider's guidelines.
3. If ketones are not present, take a correction bolus as prescribed by your healthcare provider.
4. Check blood glucose again after 2 hours.
5. If blood glucose levels have not decreased, do both of the following:
 - Take a second bolus by injection, using a sterile syringe. Ask your healthcare provider whether to inject the same amount of insulin as in step 3.
 - Replace the Pod. Use a new vial of insulin to fill the new Pod. Then contact your healthcare provider for guidance.
6. If you feel nauseated at any point, check for ketones and contact your healthcare provider immediately (see 'Diabetic ketoacidosis (DKA)' on page 174).

Warning: If you need emergency attention, ask a friend or family member to take you to the Accident and Emergency department or call an ambulance. Do NOT drive yourself.

7. Investigate possible causes for hyperglycaemia to avoid similar problems in the future (see the following table).

Possible causes of hyperglycaemia	Suggested action
Expired insulin or insulin exposed to extreme temperatures	Deactivate and remove the used Pod. Apply a new Pod filled from a new vial of insulin.
Infusion site in or near a scar or mole	Deactivate and remove the used Pod. Apply a new Pod in a different location.

Possible causes of hyperglycaemia	Suggested action
Infected infusion site	Deactivate and remove the used Pod. Apply a new Pod in a different location and consult your healthcare provider.
Dislodged cannula	Deactivate and remove the used Pod. Apply a new Pod in a different location. Note: Avoid sites near a waistband, belt or other areas where friction may dislodge the cannula.
Empty Pod	Deactivate and remove the used Pod. Apply a new Pod in a different location.
Incorrect Basal Programme	Confirm that the correct Basal Programme is active. Confirm that the PDM time is set correctly. Consult your healthcare provider about adjusting your Basal Programmes or using a temp basal.
Incorrect bolus timing or bolus too small	Check carb intake. Take bolus with food. Check blood glucose before giving a meal bolus. If necessary, adjust bolus. Consult your healthcare provider for guidance.
High-protein or high-fat meal	Calculate protein/fat intake and account for it in your bolus timing and bolus type. Consult your healthcare provider about using the extended bolus option.
Less activity than normal	Consult your healthcare provider about adjusting your Basal Programmes or using a temp basal.
Blood glucose value greater than 250 mg/dL (with ketones present) before exercise	Do not exercise when ketones are present. Note: Blood glucose increases with exercise when ketones are present. Consult your healthcare provider for guidance.
Infection or illness or medication change	See 'Sick days' on page 175. Consult your healthcare provider about sick day guidelines and about medication changes.

Possible causes of hyperglycaemia	Suggested action
Weight loss or gain or menstrual cycle or pregnancy	Consult your healthcare provider for guidance.

Diabetic ketoacidosis (DKA)

Pods use rapid-acting insulin, so you have no long-acting insulin in your body. If insulin delivery from the Pod is interrupted (an occlusion), your blood glucose can rise rapidly and lead to diabetic ketoacidosis (DKA). DKA is a serious – but preventable – emergency that can occur if you ignore high blood glucose levels.

Warnings:

If left untreated, DKA can cause breathing difficulties, shock, coma and eventually death.

If you need emergency attention, ask a friend or family member to take you to the Accident and Emergency department or call an ambulance. Do not drive yourself.

Symptoms of DKA

- Nausea and vomiting
- Abdominal pain
- Dehydration
- Fruity-smelling breath
- Dry skin or tongue
- Drowsiness
- Rapid pulse
- Laboured breathing

The symptoms of DKA are much like those of the flu. Before assuming you have the flu, check your blood glucose and check for ketones to rule out DKA.

To avoid DKA

The easiest and most reliable way to avoid DKA is by checking your blood glucose at least 4–6 times a day. Routine checks allow you to identify and treat high blood glucose before DKA develops.

To treat DKA

- Once you have begun treatment for high blood glucose, check for ketones. Check for ketones whenever your blood glucose is 250 mg/dL or above.
- If ketones are negative or trace, continue treating for high blood glucose.
- If ketones are positive and you are feeling nauseous or ill, immediately contact your healthcare provider for guidance.
- If ketones are positive but you are not feeling nauseous or ill, replace the Pod, using a new vial of insulin.
- Check blood glucose again after 2 hours. If blood glucose level has not declined, immediately contact your healthcare provider for guidance.

Handling Special Situations

Sick days

Any physical stress can cause your blood glucose to rise, and illness is a physical stress. Your healthcare provider can help you make a plan for sick days. The following are only general guidelines.

When you are ill, check your blood glucose more often to avoid DKA. The symptoms of DKA are much like those of the flu. Before assuming you have the flu, check your blood glucose to rule out DKA (see 'Diabetic ketoacidosis (DKA)' on page 174).

To handle sick days:

- Treat the underlying illness to promote faster recovery.
- Eat as normally as you can.
- Adjust bolus doses, if necessary, to match changes in meals and snacks.
- Always continue your basal insulin, even if you are unable to eat. Contact your healthcare provider for suggested basal rate adjustments during sick days.
- Check your blood glucose every 2 hours and keep careful records of results.
- Check for ketones when your blood glucose is 250 mg/dL or higher.
- Follow your healthcare provider's guidelines for taking additional insulin on sick days.
- Drink plenty of fluids to avoid dehydration.
- Contact your healthcare provider if symptoms persist.

Exercising, playing sports or working hard

Check your blood glucose levels before, during and after exercising, playing sports or doing unusually hard physical labour.

The Pod's adhesive keeps it securely in place for up to 3 days. However, if necessary, several products are available to enhance adhesion. Ask your healthcare provider about these products.

Avoid getting body lotion, creams or oils near the infusion site; these products may loosen the adhesive.

For some contact sports, if the Pod is in a location where it is likely to be knocked off, consider removing the Pod and placing a new one in a more protected location.

Be sure to check your blood glucose levels before removing the Pod and after applying a new one. Pods are designed for single use. Do not attempt to reapply a Pod that has been removed.

Tip: *If possible, plan removal times to coincide with a scheduled Pod replacement.*

If you will need to remove the Pod for more than one hour, ask your healthcare provider to recommend appropriate guidelines.

X-rays, MRIs and CT scans

The Pod and PDM can tolerate common electromagnetic and electrostatic fields, including airport security and mobile phones.

Warning: The Pod and PDM may be affected by strong radiation or magnetic fields. Before having an X-ray, MRI or CT scan (or any similar test or procedure), remove and dispose of your Pod and place your PDM outside the treatment area. Check with your healthcare provider on Pod removal guidelines.

Surgery or hospitalisation

For scheduled surgeries or hospitalisation, you should tell the doctor/surgeon or hospital staff about your Pod. It may be necessary to remove it for certain procedures or treatments. Remember to replace the basal insulin that was missed while the Pod was removed. Your healthcare provider can help you prepare for these situations.

Appendix

Troubleshooting PDM Start-up

While unlikely to happen with normal use, certain button press combinations during PDM start-up can cause the PDM screen to display Boot Mode or Safe Mode.

Even when the PDM is in Boot Mode, the Pod continues delivering insulin according to the instructions it was given. Safe Mode does not affect PDM or Pod functionality.

Return to normal PDM operation as follows:

Boot Mode

If your PDM screen displays 'Select Boot Mode', then do the following:

1. Press the Sound/Vibrate UP (VOLUME_UP) button as needed to move the arrow (<=>) until it points to **[Recovery Mode]**. Then, press the Sound/Vibrate DOWN (VOLUME_DOWN) button to select **[Recovery Mode]**.

Caution: Do not select the [Fastboot Mode] because it will stop the PDM from responding. If the PDM stops responding contact Customer Care.

2. From the Android Recovery screen, press the Sound/Vibrate (VOLUME_UP/VOLUME_DOWN) button to highlight **Reboot system now**. Then press the Power button to select it.

The PDM restarts.

Safe Mode

If your PDM screen displays 'Safe Mode' in the lower left corner, then restart the PDM and remove the 'Safe Mode' text from the screen as follows:

1. Press and hold the Power Button, then tap Power Off.
2. Tap OK to confirm.
3. Press and hold the Power button to turn the PDM back on.

Summary of Settings and Options

The options for the various Omnipod DASH® Insulin Management System settings are:

Time	12-hour or 24-hour clock.
Time zones	GMT-11:00 to GMT+13:00.
Date	MM.DD.YYYY DD/MM/YYYY DD.MM.YYYY YYYY-MM-DD
Screen time-out	30, 60, 120 seconds. Default is 30 seconds.
PIN	4 digits from 0 to 9.
Maximum Basal Rate	0.05 to 30 U/hr. Default is 3.00 U/hr.
Basal rate	Units/hr. Range: 0 U/hr to Maximum Basal Rate in 0.05 U/hr increments.
Basal Programmes	Maximum of 12.
Basal rate segments	24 per Basal Programme.
Temp basal	%, units/hr or Off. Default is Off. Duration: 30 min to 12 hrs in 30-min increments.
Temp basal (set to %)	Range: 100% decrease (0 U/hr) to 95% increase from current basal rate in 5% increments. Cannot exceed the Maximum Basal Rate.
Temp basal (set to U/hr)	Range: 0 U/hr to Maximum Basal Rate in 0.05 U/hr increments.
Temp basal presets	Maximum of 12.
BG Goal Range for blood glucose history	Lower and upper limits: 70 to 200 mg/dL in 1 mg/dL increments.
BG reminder	On or Off. Default is Off. Maximum of 4 active at one time. Reminder can occur between 30 min to 4 hrs after bolus is started. Set in 30-minute increments.
Custom reminder	Maximum of 4. Set to Daily, Once only, Off.
Bolus Calculator	On or Off. Default is On.
Target BG value	Maximum of 8 segments; 70 to 200 mg/dL in 1 mg/dL increments.
Correct Above threshold	Maximum of 8 segments; Target BG to 200 mg/dL in 1 mg/dL increments.
Minimum BG for Calcs	50 to 70 mg/dL in 1 mg/dL increments. Default is 70 mg/dL.
Insulin to carb (IC) ratio	Maximum of 8 segments; 1 to 150 g carb/U in 0.1 g carb/U increments.

Correction (sensitivity) factor	Maximum of 8 segments; 1 to 400 mg/dL in 1 mg/dL increments. Default is 50 mg/dL.
Reverse Correction	On or Off. Default is On.
Duration of insulin action	2 to 6 hours in 30-minute increments. Default is 4 hours.
Maximum bolus size	0.05 to 30 U.
Extended bolus	%, Units or Off. Default is Off. 30 minutes to 8 hours in 30-minute increments.
Bolus preset	Maximum of 7. Cannot exceed the Maximum Bolus.
Suspend	30 minutes to 2 hours.
Low reservoir volume advisory	10 to 50 units in 1-unit increments. Default is 10.0 U.
Pod expiry notification	1 to 24 hours in 1-hour increments. Default is 4 hours.
Auto-off timer	Off, or 1 to 24 hours in 1-hour increments. Default is Off.
History screen display	Rolling 90-day period.
Language	Multiple languages. Default is English.

Pod Specifications

Size: 3.9 cm wide x 5.2 cm long x 1.45 cm high (1.53' x 2.05' x 0.57')

Weight (without insulin): 26 grams (0.92 oz)

Operating temperature range: Pod operating environment of 5°C to 40°C (41°F to 104°F).

Note: The Pod temperature equilibrates from 23°C to 37°C (73°F to 98.6°F) when worn on the body.

Start-up temperature: above 10°C (50°F)

Storage temperature range: 0°C to 30°C (32°F to 86°F)

Warm-up time (0°C to 20°C [32°F to 68°F]): 7 minutes

Cool-down time: No time is required for cool-down from maximum storage temperature (30°C) to operating temperature.

Reservoir volume (deliverable): 200 units

Cannula insertion depth: 4-7 mm (0.16-0.28')

Depth of insulin infusion: ≥ 4 mm (0.16')

Waterproof rating: IP28 (7.6 m (25 ft) for up to 60 minutes)

Insulin concentration: U-100

Alarm type: Audible. Output: ≥ 45 db(A) at 1 metre

Sterilising agent: sterilised using ethylene oxide

Operating relative humidity range: 20 to 85%, non-condensing

Storage relative humidity range: 20 to 85%, non-condensing

Operating atmospheric pressure: 700 hPA to 1,060 hPA

Storage atmospheric pressure: 700 hPA to 1,060 hPA

Non-pyrogenic: fluid pathway only

Type BF applied part: Protection from electrical shock

Maximum infusion pressure: 35 psi

Maximum volume infused under single fault conditions: 0.5 U

Flow Capability:

Prime rate: 0.05 units per second

Basal: Programmable by the user in 0.05 U increments of up to 30.0 U per hour

Bolus rate: 1.5 units per minute. Dose range from 0.05 to 30.0 units


Delivery accuracy (tested per IEC 60601-2-24):

Basal: $\pm 5\%$ at rates ≥ 0.05 U/hr

Bolus: $\pm 5\%$ for amounts ≥ 1.0 unit

± 0.05 units for amounts < 1.0 unit

Accuracy test results: The following graph shows the flow accuracy of the Pod against given time periods. The measurements were made using a Pod with a basal rate of $0.5 \mu\text{l/h}$ (which delivers 0.05 U/h of U-100 insulin) at a high operating temperature. The overall mean percentage flow error was 1.40%.

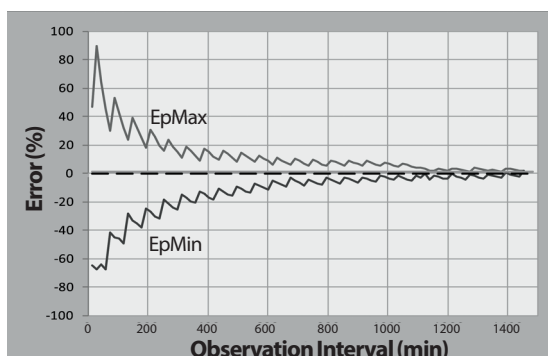

PDM Specifications

Size: 6.4 cm wide x 12.2 cm long x 1.0 cm high (2.52' x 4.79' x 0.39')

Weight: 106 grams (3.74 oz)

Screen active area: 10.2 cm diagonal $\pm 5\%$ (4.0' $\pm 5\%$)

Operating temperature range: 5°C to 40°C (41°F to 104°F)

Charging temperature range: 5°C to 40°C (41°F to 104°F)

Storage temperature range: 0°C to 30°C (32°F to 86°F)

Operating relative humidity range: 20 to 90%, non-condensing

Storage relative humidity range: 20 to 90%, non-condensing

Operating atmospheric pressure: 700 hPa to 1,060 hPa

Storage atmospheric pressure: 700 hPa to 1,060 hPa

Communication distance: The PDM and Pod should be

- At start-up: Adjacent and touching, either in or out of the tray, to ensure proper communication during priming.
- During normal operation: Within 1.5 m (5 ft) of each other. Depending on the location, the communication distance may handle separations of up to 15 metres (50 feet) away.

Waterproof rating: IP22 when used with the outer case (avoid liquid)

Note: The IP22 rating applies ONLY when your PDM is used with the provided outer case (gel skin). The risk of water ingress into the PDM is greater without the outer case. Contact Customer Care about purchasing additional outer cases, if needed.

Alarm type: Audible. Output: ≥ 45 db(A) at 1 metre

Notification type: Audible and vibratory

PDM Service Life: 5 years

Battery life: Full charge covers approximately 1.5 days of typical use after 2 years of typical use.

Battery charger operating line voltage: 100 to 240 VAC, 50/60 Hz

Use only the Insulet-provided charger to charge your PDM. Using unapproved chargers can cause the battery to explode or damage the PDM, and may void the warranty.

Battery charger service life: 10,000 operating hours

Protection from Over-infusion or Under-infusion

The Pod software monitors the infusion rate. If an error that would result in over- or under-infusion is detected and cannot be corrected, insulin delivery stops and an alarm sounds.

Occlusion detection

An occlusion is a blockage or interruption in insulin delivery from the Pod. If the Omnipod DASH® System detects an occlusion, it sounds a hazard alarm and prompts you to deactivate and change your Pod.

An occlusion hazard alarm sounds when an average of 3 units to 5 units of missed insulin occurs. The following table depicts occlusion detection for three different situations when using U-100 insulin. For example, if the Pod’s cannula becomes occluded when delivering a 5 U bolus, 35 minutes may pass before the Pod sounds a hazard alarm.

	Time between occlusion and Pod alarm	
	Typical time	Maximum time
5.00 U bolus	33 minutes	35 minutes
1.00 U/hr basal	3.0 hr	5.5 hr
0.05 U/hr basal	51 hr	80 hr (Pod expiry)


















If an occlusion spontaneously clears up, a volume of insulin could be released. That volume would not exceed the volume of the programmed insulin intended for delivery.

If an occlusion is detected during an immediate bolus, the Pod sounds a hazard alarm at the conclusion of the immediate bolus.

Warning: At very low basal flow rates, checking your blood glucose frequently may give you an early indication of an occlusion. Occlusions can result in hyperglycaemia.

Omnipod DASH® System Label Symbols

Symbol	Meaning	Symbol	Meaning
	Single use only		MR unsafe
	Consult accompanying documents		Do not use if package is damaged
	Sterilised using ethylene oxide		Type BF applied part
	Date of manufacture		Manufacturer
	Batch code		Keep dry
	Use by date		Storage temperature, Operational temperature
	Reference number		Serial number
	Storage relative humidity, Operational relative humidity		Storage atmospheric pressure, Operational atmospheric pressure
	Submersible: Waterproof to 7.6 meters (25 feet) for up to 60 minutes		Non-pyrogenic fluid path
	Avoid liquid		Authorised Representative in the European Community
	Do not dispose with household waste		RoHS-compliant
	DASH® PDM		Marking of conformity
	Omnipod® carry case		PDM skin
	User Guide/ Technical User Guide		Pod
	Charging cable		Charging adapter

	PDM battery		Importer
	Medical Device		Switzerland Authorised Representative
	UK Conformity Assessed		Single Patient Multiple Use
	Australian Regulatory Compliance Mark		Single Sterile Barrier System
	(France) Product is intended for recycling and should not be placed in the normal bin		(France) Product contains sharps
	Electronic Instructions for Use		Compatible with U-100 insulin only
	(France) Waste must be placed in Needle Box		(France) Product contains electronic sharps
	(France) Waste must be placed in electronic sharps Purple Box		(France) Pharmacy collection available
	(France) Packaging intended for recycling		

Medical Device Regulation

This device complies with EU Medical Device Regulation 2017/745.

EU General Data Protection Regulation (GDPR)

Insulet complies with the EU 2016/679 General Data Protection Regulation.

Device Complaints

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorised representative and to your national authority.

Contact details for the manufacturer can be found on the back cover of this document. The contacts of national competent authorities (Vigilance Contact Points) and further information can be found on the following European Commission website: https://ec.europa.eu/health/md_sector/contact_en

If you have a problem with your System, contact Customer Care using the information on the provided Contact Card.

EU Authorised Representative details



Contact Person: The Complaints Officer

Address: Insulet Netherlands B.V., WTC Utrecht Stadsplateau 7, Suite 7.06, 3521 AZ Utrecht, The Netherlands

TEL: +31 308 990 670

E-mail: ECRep@insulet.com

Omnipod DASH® System Notice Concerning Interference

The Omnipod DASH® Insulin Management System (both the Pod and the PDM) complies with Part 15 of the Federal Communications Commission (FCC) Rules. Operation is subject to the following two conditions:

1. These devices may not cause harmful interference.
2. These devices must accept any interference received, including interference that may cause undesirable operation.

Caution: Changes or modifications not expressly approved by Insulet Corporation could void the user's authority to operate the equipment.

Both the Pod and the PDM generate, use and can radiate radio frequency energy, and may cause harmful interference to the radio communications of other devices. There are no guarantees that interference will not occur in a particular installation. If the Omnipod DASH® System does cause harmful interference to radio and television reception, the interference may be corrected by one of the following measures:

- Move or relocate the Omnipod DASH® System.
- Increase the distance between the Omnipod DASH® System and the other device that is emitting or receiving interference.

Insulet Corporation declares that the Omnipod DASH® System is in compliance with the essential requirements and other relevant provisions of the Radio Equipment Directive (2014/53/EU). The full Declaration of Conformity can be found at the following web address: http://omnipod.com/Red_Doc. This ISM device complies with Canadian ICES-003 and IC-RSS-210.

Electromagnetic Compatibility

The information contained in this section (such as separation distances) is, in general, specifically written with regard to the Omnipod DASH® System. The numbers provided will not guarantee faultless operation but should provide reasonable assurance of such. This information may not be applicable to other medical electrical equipment; older equipment may be particularly susceptible to interference.

General notes

Medical electrical equipment requires special precautions regarding electromagnetic compatibility (EMC) and needs to be installed and put into service according to the EMC information provided in this document and the instructions for use. If the Omnipod DASH® System fails due to electromagnetic disturbances, you may need to replace it.

Portable and mobile radio frequency (RF) communications equipment can affect the functioning of medical electrical equipment.

Caution: Cables and accessories not specified within the instructions for use are not authorised. Using other cables or accessories may adversely impact safety, performance and electromagnetic compatibility (increased emission and decreased immunity).

Care should be taken if the Omnipod DASH® System is used adjacent to other electrical equipment; if adjacent use is inevitable, such as in working environments, the Omnipod DASH® System should be observed to verify normal operation in this setting.

The Omnipod DASH® System communicates by low level RF energy. As with all RF receivers, the potential for disturbance exists, even with equipment that complies with FCC and CISPR emissions requirements.

The Omnipod DASH® System communicates with the following characteristics:

Frequency: 2.400-2.480 GHz, digitally modulated, with an effective radiated power of 1.14 mW

The Omnipod DASH® System complies with the immunity requirements of the general standard for electromagnetic compatibility, IEC 60601-1-2.

The Omnipod DASH® System has demonstrated immunity to both radio-frequency identification (RFID) systems and Electronic Article Surveillance (EAS) systems. Testing has been performed in accordance with AIM 7351731 to demonstrate this immunity.

Caution: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Omnipod DASH® System. Otherwise, degradation of the performance of this equipment could occur.

Electromagnetic Emissions

This equipment is intended for use in the electromagnetic environment specified below. The user of this equipment should ensure that it is used in such an environment.

Emissions	Compliance according to	Electromagnetic environment
RF Emissions (CISPR 11)	Group 2	The Pod and the PDM emit low-level electromagnetic energy (RF) in order to communicate. Although unlikely, nearby electronic equipment may be affected.
CISPR B Emissions Classification	Class B	The System is suitable for use in all establishments including domestic establishments.

Electromagnetic Immunity

The System is intended for use in the electromagnetic environment specified below. You should observe these requirements when using the System.

Immunity against	IEC 60601-1-2 test level	Compliance level (of this device)	Electromagnetic environment
ElectroStatic Discharge, ESD (IEC 61000-4-2)	contact discharge: ± 8 kV air discharge: ± 15 kV	± 8 kV ± 15 kV	If floors are covered with synthetic material, try to avoid electrostatic discharges.
Power frequency magnetic fields 50/60 Hz (IEC 61000-4-8)	30 A/m	400 A/m	Suitable for most environments. Magnetic field strengths in excess of 400 A/m would be unlikely except in close proximity to industrial magnetic devices.
Radiated RF (IEC 61000-4-3)	10 V/m at 80 MHz–2.7 GHz	10 V/m	Suitable for most environments. Keep portable RF communications equipment at least 30 cm (12 inches) away from the Omnipod DASH® System.

Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflections from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (mobile/wireless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast, and TV broadcast, cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the equipment is used exceeds the applicable RF compliance level above, the equipment should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orientating or relocating the equipment.

Warranty for the Omnipod DASH® Insulin Management System Personal Diabetes Manager and Pods (all countries except for Canada and the United States)

WARRANTY COVERAGE

Warranty Coverage for the Omnipod DASH® System Personal Diabetes Manager

Subject to the terms and conditions below, Insulet Netherlands BV ('Insulet') warrants to you, the original recipient of the Omnipod DASH® Insulin Management System ('Omnipod® System') for use in the country in which you purchased or received the Personal Diabetes Manager ('PDM'), that, if Insulet determines that your PDM has a defect in material or workmanship while utilised under normal use and conditions during the period of four (4) years from the date of purchase (or receipt where purchased on your behalf), Insulet will either repair or replace, at its sole discretion, the PDM.

This four-year (4) warranty period applies only to new PDMs and, in the event that the PDM is repaired or replaced, the warranty period shall not be extended or reset.

Warranty Coverage for the Omnipod® System Pods

Subject to the terms and conditions below, Insulet warrants to you, the original recipient of the Omnipod® System, that if Insulet determines that an Omnipod® System Pod ('Pod') you have received has a defect in material or workmanship while utilised under normal use and conditions during the period of twenty-four (24) months from the date of manufacture and seventy-two (72) hours from the time of activation, Insulet will either repair or replace, at its sole option, that Pod. To be eligible for replacement, the activation of the Pod must fall within both time periods (i.e. occur on or before the expiry date printed on the label with a manufacture date no more than twenty-four (24) months before and on or before a time no more than seventy-two (72) hours before you notify Insulet of the claim).

This twenty-four (24) month and seventy-two (72) hour warranty period applies only to new Pods and, in the event a Pod is repaired or replaced, the warranty period shall not be extended or reset.

WARRANTY TERMS AND CONDITIONS

This Warranty applies only to PDMs and Pods that were originally sold for use in the country in which you purchased or received the applicable product (the 'Territory'). Insulet will only dispatch repaired or replaced PDMs and Pods and provide Warranty services within the Territory.

Claim Procedure

To be eligible to claim under this Warranty, you must notify Insulet of the claimed defect with the PDM or the Pod within the applicable warranty period by calling the applicable Insulet Customer Care number found on our website or in the product Technical User Guide. For a claim involving the PDM, you must provide the PDM serial number and a description of the claimed defect. For a claim involving a Pod, you must provide the Pod lot number and a description of the claimed defect. You may also be required to verify the date of purchase (or receipt where purchased on your behalf) of the PDM and/or the Pod and the time that you activated the Pod.

Your failure to follow any of the above steps may result in the denial of coverage under this Warranty.

Unless Insulet elects to repair the Pod or the PDM (which may include, but is not limited to, a repair kit or replacement part(s) Insulet provides) or refers you to a third-party repairer, you must obtain Insulet's authorisation prior to returning the Pod or the PDM to Insulet. The Pod or PDM must be properly packaged and returned to Insulet according to the instructions provided in the Return Merchandise Authorisation (RMA) Kit, which will be sent to you by Insulet. With a prior authorisation, Insulet will pay all reasonable packaging and postage charges, where applicable, incurred in shipping the Pod or the PDM to Insulet under this Warranty. For the avoidance of doubt, this Warranty does not cover repairs performed or replacements provided by any person or entity other than Insulet, except those performed or provided by third parties to which you were explicitly referred by Insulet.

Proof of Purchase, Receipt or Activation

In order to verify the date of purchase (or receipt where purchased on your behalf) or, in the case of a Pod, the time of activation and determine if the claim under this Warranty is within the applicable warranty period, Insulet may require that you provide a valid proof of purchase, receipt or activation. Your failure to provide valid proof, as determined by Insulet, may result in the denial of coverage under this Warranty.

Exclusions

This Warranty covers only the original recipient and you cannot transfer or assign it with the sale, rental or other transfer of the PDM or of the Pods to any other person or entity.

This Warranty will apply only if the PDM or the Pod at issue has been used in accordance with the Omnipod® System Technical User Guide and/or other written instructions provided by Insulet. This Warranty does not apply if the PDM or the Pods have been:

- Altered, changed or modified by any person or entity other than Insulet or a third party authorised by Insulet;
- Opened, serviced or repaired by any person or entity other than Insulet or a third party authorised by Insulet;
- Damaged by an act of God or other 'force majeure'-like event;
- Damaged by misuse, abuse, negligence, accident, unreasonable use, or improper handling, care or storage;
- Damaged by wear and tear, causes unrelated to defective materials or workmanship (including without limited, unsuitable or faulty batteries) or other circumstances outside of the reasonable control of Insulet.

This Warranty does not apply to test strips, batteries, or other accessories or related products provided by third parties (e.g. data management tools, CGMs).

This Warranty does not extend to design defects (i.e. claims that the PDM or the Pods should have been designed in a different way).

Disclaimer of Implied Warranties and Limitation of Remedies

To the extent permitted by law in the Territory:

- **This Warranty and the remedies set out in it are the only warranties and remedies provided by Insulet to you in relation to the PDM and the Pods, and all other statutory and implied warranties are expressly excluded to the maximum extent permitted.**
- **Insulet, its suppliers, distributors, service providers and/or agents will not be liable for indirect, special, incidental or consequential damages caused by a defect in the PDM or a Pod, or by a breach of this Warranty, whether such claim is based in warranty, contract, tort or otherwise.**

Nothing in this Warranty is intended to exclude our liability for death or personal injury resulting from our negligence, for fraud or fraudulent misrepresentation, or for breach of your statutory rights in relation to the PDM or Pods.

Important Additional Provisions

This Warranty gives you specific legal rights. You may also have other statutory rights which vary by jurisdiction.

Your statutory rights are not affected by this Warranty.

Insulet does not warrant the suitability of the PDM or the Pods or the Omnipod® System for any specific person as health care and treatment are complex subjects requiring the services of qualified healthcare providers.

This Warranty is between you and Insulet. No other party has any rights to enforce any of its terms. Insulet may transfer its rights and obligations under this Warranty to another party without your consent.

If any provision of this Warranty is found to be invalid by any court, that provision will be deemed to be deleted from this Warranty and the validity of the remaining provisions will not be affected.

No Other Warranty or Agreement

Unless modified in writing and signed by both Insulet and you, the terms set out in this Warranty are the complete and exclusive understanding between Insulet and you, superseding all prior warranties and agreements, oral or written, and all other communications relating to any defect in, failure or other malfunction in a PDM, a Pod or an Omnipod® System. No employee, agent or other representative of Insulet or any other party is authorised to make any product warranty or agreement applicable to a PDM, a Pod or an Omnipod® System in addition to those made in the foregoing.

Consent to Disclaimer of Implied Warranties and the Limitation of Remedies

If you do not consent to and instead wish to reject the Disclaimer of Implied Warranties and the Limitation of Remedies included along with the Omnipod® System, please return any Omnipod® System products (including any PDM and Pod) to Insulet in exchange for a full refund. Failure to return such products shall constitute acknowledgement of and consent to the Disclaimer of Implied Warranties and the Limitation of Remedies.

Governing Law and Jurisdiction

This Warranty (and any non-contractual obligations arising out of or in connection with it) is governed by the laws of the country in which you purchased or received the PDM or Pods. Any court of competent jurisdiction in such country will have exclusive jurisdiction and venue for any dispute arising out of or in connection with this Warranty.

This page intentionally left blank.

Glossary

A1c (see Haemoglobin A1c)

Activation: The process of waking up a Pod and pairing it to a PDM so that the Pod only responds to commands from that PDM.

Advisory alarm: Intermittent vibrations or beeps accompanied by a message advising about an action that you may want to take to prevent a serious problem.

Aseptic technique: A method for maintaining sterilisation and preventing contamination.

Basal Programme: A daily schedule for continuous insulin delivery. It consists of one or more segments, each defining a basal rate, that together cover the 24-hour period from midnight to midnight.

Basal rate: A small amount of insulin that is delivered continuously over a period of time. Basal rates are specified in units per hour (U/hr).

Basal segment: The time period during which a specific basal rate is delivered.

BG Goal Range: A user-defined range of desired blood glucose values. This range is used in the history records to show which blood glucose values fall within this range.

Blood glucose/Blood glucose level: The amount of glucose, or sugar, in the blood.

Blood glucose meter: A device used to check blood glucose level.

Bluetooth® (see wireless communication)

Bolus Calculator: A feature that suggests meal and correction bolus doses based on your current blood glucose, the amount of carbohydrates you are about to eat, the insulin on board and several user-specific settings.

Bolus dose: A dose of insulin taken to correct an elevated blood glucose level (a correction bolus) or to cover carbohydrates in a meal or snack (a meal bolus).

Bolus preset: A bolus dose of insulin that is assigned a custom name and stored by the PDM for later use.

Calorie: A unit of measurement used to express the energy value of food. Calories come from the carbohydrate, protein and fat in food and drink.

Cannula: A small, thin tube inserted below the skin, which serves to introduce a liquid medication into the body.

Carbohydrate bolus (see meal bolus)

Carbohydrate (carbs): One of the three main energy sources in food. (The other two are protein and fat.) Foods that contain carbohydrates include starches, sugars, vegetables, fruits, and dairy products.

Correction bolus: An amount of insulin taken to compensate for high blood glucose levels.

Correct Above Threshold: The blood glucose level above which you would like to take insulin to reduce an elevated blood glucose.

Correction Factor (also known as sensitivity factor): A value that indicates how much one unit of insulin will lower your blood glucose. For example, if your Correction Factor is 50, one unit of insulin lowers your blood glucose by 50 mg/dL.

Deactivate: Preferred method for unpairing the PDM from the active Pod. Deactivation turns off insulin delivery in the Pod and then unpairs the PDM from that Pod.

Diabetes, diabetes mellitus: A condition characterised by hyperglycaemia (high blood glucose) resulting from the body's inability to use blood glucose for energy. In type 1 diabetes, the pancreas no longer makes insulin and therefore blood glucose cannot enter many cell types to be used for energy. In type 2 diabetes, either the pancreas does not make enough insulin or the body is unable to use insulin correctly.

Diabetic ketoacidosis (DKA): A serious condition in which extremely high blood glucose levels and a severe lack of insulin cause the body to break down fat and protein for energy. The breakdown of fat or protein releases ketones into the blood and urine. DKA can take hours or days to develop, with symptoms that include stomach pain, nausea, vomiting, fruity breath odour and rapid breathing.

Discard pod: The DISCARD POD option is offered if the PDM is unable to re-establish communication with the Pod after a communication error. This option allows the PDM to abandon that Pod and activate a new Pod. Note: A 'discarded' Pod may still be delivering insulin. Always remove a 'discarded' Pod from your body.

Duration of Insulin Action: The length of time that insulin remains active and available in your body after a bolus. This duration can vary greatly depending on the type of insulin you take.

Estimated bolus: After you confirm the amount of bolus that you want delivered, a bolus instruction is sent to the Pod. If the Pod is unable to send back confirmation of the amount of bolus actually delivered, the PDM estimates the amount delivered. This estimate is based on the expected delivery schedule. As soon as confirmation is received from the Pod, the PDM displays the actual (not estimated) bolus amount.

Extended bolus: The delivery of a meal bolus dose steadily over an extended period of time.

Fat: One of the three main energy sources in food. (The other two are carbohydrate and protein.) Fat contains 9 calories per gram. Foods high in fat include oils, margarine, salad dressings, red meat and whole milk dairy foods.

Fibre: The indigestible part of plant foods. Foods that are high in fibre include broccoli, beans, raspberries, squash, wholegrain bread and bran cereal. Fibre is a type of carbohydrate, but it does not raise blood glucose levels as other carbohydrates do.

Glucose: A simple sugar (also known as dextrose) used by the body for energy. Without insulin, many cells in the body cannot use glucose for energy.

Hazard alarm: A continuous sound and a screen message from the PDM or Pod indicating that an error has occurred or that insulin delivery has stopped. Alarms require your immediate attention.

Healthcare provider: A professional who practises medicine or teaches people how to manage their health.

Haemoglobin A1c (HbA1c): A test that measures a person's average blood glucose level over the last two to three months. Also called glycosylated haemoglobin, the test measures the amount of glucose that sticks to haemoglobin in red blood cells, which is proportional to the average amount of glucose in the blood over an extended time period.

Hyperglycaemia (high blood glucose): A higher-than-normal level of glucose in the blood; generally above 250 mg/dL.

Hypoglycaemia (low blood glucose): A lower-than-normal level of glucose in the blood; generally below 70 mg/dL.

Hypoglycaemia unawareness: A condition in which a person does not feel or recognise the symptoms of hypoglycaemia.

Infusing: Introducing a liquid substance under the skin into the body.

Infusion site: A place on the body where a Pod's cannula is inserted.

Insulin: A hormone that helps the body use glucose for energy. Insulin is made by cells in the pancreas called beta cells.

IC Ratio (Insulin to carbohydrate ratio): Number of grams of carbohydrate covered by one unit of insulin. For example, if your IC ratio is 1 to 15, then you need to deliver one unit of insulin to cover every fifteen grams of carbohydrate you eat.

Insulin on board (IOB) (active insulin): The amount of insulin that is still 'active' in the body from a previous bolus dose. The Bolus Calculator tracks IOB for you. The amount of time insulin remains 'on board' or 'active' depends on your duration of insulin action setting.

Ketoacidosis (see diabetic ketoacidosis)

Ketones: Acidic by-products that result from the breakdown of fat for energy. The presence of ketones indicates that the body is using stored fat and muscle (instead of glucose) for energy.

Manually-calculated bolus: A bolus amount of insulin chosen by you (not calculated by the Bolus Calculator).

Maximum basal rate: Upper limit for basal rates in a Basal Programme or temp basal.

Maximum bolus: The largest bolus that you can request from the PDM. The Bolus Calculator informs you if it calculates a bolus that is over this amount.

Minimum BG for Calcs: The minimum blood glucose reading at which the Bolus Calculator calculates a meal bolus. The Bolus Calculator is disabled below this value. 'For calcs' means 'for use in the Bolus Calculator's calculations'.

Meal bolus (also known as carbohydrate bolus): An amount of insulin administered before a meal or snack to ensure that blood glucose levels stay within the desired BG Goal Range after a meal.

Notifications: An on-screen reminder or informational message.

Occlusion: A blockage or interruption in insulin delivery.

Pairing: Linking two devices so that they can communicate wirelessly with each other. See also *sync*.

Podders: People with diabetes or caregivers of people with diabetes who use the Omnipod® system to manage their daily insulin needs.

Preset: A preset allows you to quickly enter a value that you use often.

Protein: One of the three main energy sources in food. (The other two are carbohydrate and fat.) Protein contains 4 calories per gram. Foods high in protein include meat, poultry, fish, legumes and dairy products.

Reverse Correction (negative correction): If your blood glucose level is below your Target BG, the Bolus Calculator uses your Correction Factor to reduce a portion of a meal bolus dose. This is an optional feature, which should be turned on or off according to the advice of your healthcare provider.

Segments: Defined periods of time during a 24-hour day. Segments are used to define basal segments in a Basal Programme, and also to define Target BG segments, IC ratio segments and Correction Factor segments.

Sensitivity factor (see Correction Factor)

Sync: The act of transferring information between two paired devices.

Target BG: The blood glucose level that the Bolus Calculator tries to achieve. You can define different Target BGs for different time periods. For example, you can have one Target BG before meals, a different one after meals and yet another at night.

Temp basal: A temporary basal rate that is used to cover predictable, short-term changes in basal insulin need. Temp basals are often used during exercise and for sick-day insulin adjustments.

Temp basal preset: An adjustment in a basal rate, in either % or U/hr, that can be assigned a custom name and stored by the PDM for later use.

Time segment (see segment)

Unconfirmed bolus: Occurs when you select the DISCARD POD option following a communication error during a bolus. In this case, the Pod was unable to send a confirmation to the PDM about how much of a bolus was actually delivered.

Wireless communication: Transfer of information without a physical connection between two devices. The PDM and Pod communicate over short distances using radio waves, which is referred to as wireless communication.

This page is intentionally left blank.

Index

A

About screen 18
activate
 Basal Programme 72
 Pod 37–50
 temp basal 69
 temp basal preset 71
active insulin. *See* insulin on board
adhesive 2, 37, 47
advisory alarms 116–117
air bubbles 41
airport security 167
alarms
 advisory 116–117
 Auto-off 114, 116
 empty reservoir 114
 hazard 114–115
 history records 83
 low reservoir 116
 occlusion 114
 overview 109
 PDM error 115
 Pod error 114
 Pod expired 114, 116
 responding to 110
 resume insulin 117
 sound and vibration pattern 112
 system error 115
 testing 97
 timing of 136
allergy to acrylic adhesive 37
aseptic technique 40
auto-off alarm 114, 116
 description 138
 setting 99

B

basal history records 86–92
Basal Programme
 about 139
 activate 72
 create 75
 delete 78
 edit 77
 override. *See* temp basal
 rename 77
 review 76
 suspend or resume 72
 switch 72
basal rate 139
 flow accuracy 180
 maximum, setting 104
 temporary. *See* temp basal
basal segment 139
Basal tab 12
battery, PDM
 charge 21, 131
 insert 132
 low alarm 117
 preserve 94
 type 129
BG Goal Range 104
BLE. *See* Bluetooth
blockage. *See* occlusion
blood glucose
 BG Goal Range 104
 HI and LO results 54
 history records 84–92
 Target BG 106, 147–162
blood glucose reading
 manual entry 51
 tags 53
 understanding the screen display 54

- Bolus Calculator
 - bolusing with 57
 - disabled 63, 146
 - sample calculations 151
 - settings 106–108
 - understanding 145–162
 - bolus, extended
 - based on carbs 57–63
 - blood on blood glucose 57
 - cancel 66
 - entering units of insulin 63–64
 - manually-calculated 63–64
 - progress 65
 - replace 66
 - setting 106
 - understanding 144
 - using the Bolus Calculator 57–63
 - bolus history records 84–92
 - bolus, immediate
 - based on carbs 57–63
 - blood on blood glucose 57
 - cancel 66
 - entering units of insulin 63–64
 - flow rate 180
 - history records 84–92
 - manually-calculated 63–64
 - progress 65
 - understanding 144
 - using the Bolus Calculator 57–63
 - bolus, missed. *See* missed bolus notification
 - bolus preset
 - create new 81
 - delete 82
 - edit 82
 - rename 82
 - using 63
 - Boot Mode 177
 - brightness, screen 94
- ## C
- cancel
 - bolus 66
 - temp basal 71
 - cannula 2, 48, 179
 - carbohydrate bolus. *See* Bolus Calculator
 - carb-to-insulin ratio. *See* IC Ratio
 - care
 - infusion site 49
 - PDM 127
 - Pod 125
 - change
 - Basal Programme 72
 - Pod 37–50
 - charging PDM battery 21, 131
 - check alarm function 97
 - check BG after bolus setting 100
 - cleaning
 - PDM 128
 - Pod 126
 - clock, AM/PM or 24-hour 26, 95
 - communication
 - failure 120
 - range 42, 135–137
 - confidence reminders
 - beeps 119
 - setting 101
 - confirmation messages 19
 - control solution
 - in history records 160
 - Correct Above threshold 106, 147
 - correction bolus. *See* Bolus Calculator
 - Correction Factor 108, 146
 - correction IOB 147, 150
 - CT scans 176
 - custom reminder
 - notification 118
 - setting 102
- ## D
- damaged PDM 129
 - Dashboard tab 12
 - data entry, how to 4
 - date and time 95–96
 - deactivate alarm 123
 - deactivate Pod 38
 - default settings 178
 - diabetic ketoacidosis 37, 174
 - diagnostic functions
 - check alarms 97

- reset PDM 97
- discard Pod 122, 137
- distance between Pod and PDM 42
- DKA. *See* diabetic ketoacidosis
- dropped PDM 129
- Duration of Insulin Action
 - sample calculations 150–151
 - setting 108

E

- electrical interference 128
- electrical safety 186
- electromagnetic compatibility 186
- emergency kit 166
- empty reservoir alarm 114
- entering numbers 5
- entering text 5
- estimated bolus 91
- exercise 176
- expiry, Pod 98, 118
- extended bolus
 - cancel 66
 - manually-calculated 64
 - progress 65
 - replace 66
 - setting 105, 106
 - understanding 144
 - using Bolus Calculator 61

F

- fill syringe, insulin 41
- first time PDM set-up 21
- flat rate (U/hr) setting
 - change setting 105
 - temp basal 140
- flight mode setting 93
- flow rate accuracy 180
- food bolus. *See* Bolus Calculator
- forgotten PIN 11

G

- glucagon kit 165, 169

H

- hazard alarms 114–115
- history records
 - alarms 83
 - and date change 92
 - and time change 92
 - blood glucose 86–92
 - carbs 86–92
 - insulin, basal and bolus 86–92
 - notifications 83
- holiday 166
- Home screen 11
- hospitalisation 176
- hyperglycaemia 171–174
 - avoiding 171
 - symptoms 171
 - treating 172
- hypoglycaemia 168–172
 - avoiding 169
 - symptoms 168
 - treating 169
 - unawareness 138

I

- IC ratio 107, 146
- illness 175
- immediate bolus. *See* bolus, immediate
- indications for use xii
- infection, infusion site 49, 163
- infusion site
 - avoid infections 49
 - guidelines for selection 44
 - inspection 48
 - preparation 46
- initial PDM set-up 21–36
- insulin
 - amount left in Pod 9
 - cleared for use xii
 - fill syringe 41
 - history records 86–92
 - rapid-acting vs long-acting 174
 - schedule. *See* Basal Program
 - storage 125
 - suspend and resume 73

insulin action. *See* Duration of Insulin Action
insulin on board 147, 150–153
insulin-to-carb ratio. *See* IC Ratio
interaction, PDM and Pod 135–138
IOB. *See* insulin on board

K

ketoacidosis. *See* diabetic ketoacidosis
ketones 174

L

Lock screen
 background 94
 message 10, 94
 unlocking 10
low battery
 alarm 117
 indicator 9
 recharging 131
lower limit, BG Goal Range 104
low reservoir
 advisory alarm 116
low reservoir setting 99

M

manual alarm deactivation 123
manual BG reading entry 51
manually-calculated bolus 63–64
map of Pod sites
 setting 98
 using 44
maximum basal rate setting 104
Maximum Bolus
 setting 105
 understanding 145
maximum insulin amount 41
meal bolus. *See* Bolus Calculator
meal IOB 145, 150
Menu 17–18
microwaves 128
Minimum BG for Calcs 107, 147
minimum insulin amount 41
missed bolus
 notification 118

 setting 100
mobile phones 176
model number, PDM xi
MRIs 176

N

navigation shorthand 8
new
 Basal Programme 75
 bolus preset 81
 temp basal preset 79
notifications 117–118
 BG 118
 custom reminder 118
 history records 83
 missed bolus 118
 No Pod 118
 overview 109
 Pod expiry 118
 responding to 110
numbers, entering 5

O

occlusion
 alarm 114
 detection 182
 preventing 47
ongoing bolus, modifying 66
operating temperature 127, 179, 181
orientation, Pod 47

P

PDM
 and water 127
 care 127
 diagram 3
 dropped or damaged 129
 electrical interference 128
 model number xi
 replacement 128
 reset 97
 screen time-out 94
 serial number 18
 settings 93, 93–108
 set-up 21–36

- specifications 181
 - unlock 10
 - PDM battery
 - alarm 117
 - charge 21
 - how to charge 131
 - low 117
 - PDM error alarm 115
 - PDM start-up
 - troubleshooting 177
 - per cent setting
 - change setting 106
 - temp basal 140
 - physical exertion 176
 - PIN
 - entry 10
 - forgotten 11
 - reset 95
 - playing sports 176
 - Pod
 - activation 37–50, 135
 - Auto-off 99, 138
 - care 125
 - change 37–50
 - check status 136
 - cleaning 126
 - deactivation 38
 - diagram 2
 - discard 137
 - expiry notification 118
 - expiry setting 98
 - fill with insulin 42
 - flow accuracy 180
 - flow rate 180
 - identify correct Pod 40
 - low reservoir setting 99
 - orientation 47
 - replace 37–50
 - safety checks xv
 - set-up 37–50
 - site selection 44, 47
 - specifications 179
 - storage 125
 - Pod error alarm 114
 - Pod expired alarm 114, 116
 - Pod Info tab 12
 - Pod site map 44, 98
 - prepare infusion site 46
 - preset, bolus
 - create 81
 - delete 82
 - edit 82
 - rename 82
 - using 63
 - preset, temp basal
 - create 79
 - delete 80
 - edit 80
 - rename 80
 - using 71
 - Product Support. *See* Customer Care
 - programme reminder setting 102
- ## R
- range, BG Goal 104
 - rapid-acting insulin xii, 174
 - recharge PDM battery 129
 - records. *See* history records
 - reminders. *See also* notifications
 - confidence 101
 - custom 102
 - programme 102, 119
 - replacement PDM
 - ordering 129
 - set-up 21
 - reservoir, Pod
 - filling 42
 - low advisory alarm setting 99
 - resume insulin delivery 72, 117
 - Reverse Correction 108, 147, 151
- ## S
- Safe Mode 177
 - safety
 - automatic checks xv, 43
 - electrical 186
 - scent of insulin 49
 - screen
 - brightness 94
 - Home screen 11
 - protector 4

- sensitivity 4
 - time-out 94
 - Select Boot Mode 177
 - sensitivity factor. *See* Correction Factor
 - serial number, PDM 18
 - settings
 - all PDM 93–108
 - Basal Programme 75
 - BG Goal Range 104
 - Bolus Calculator 106–108
 - bolus preset 81
 - check BG after bolus 100
 - confidence reminders 101
 - Correct Above 106
 - Correction Factor 108
 - custom reminders 102
 - date 95
 - Duration of Insulin Action 108
 - extended bolus configuration 106
 - flight mode 93
 - IC ratio 107
 - lock screen image 94
 - lock screen message 94
 - low reservoir 99
 - Maximum Basal Rate 104
 - Maximum Bolus 105
 - Minimum BG for Calcs 107, 147
 - missed bolus notification 100
 - PIN 95
 - Pod expiry 98
 - Pod site map 98
 - programme reminders 102
 - reset PDM 97
 - Reverse Correction 108
 - screen brightness 94
 - screen time-out 94
 - summary 178
 - Target BG 106
 - temp basal 105
 - temp basal preset 79
 - time 95–96
 - shorthand for navigation 8
 - sick days 175
 - site selection, Pod 44
 - sound or vibration 112
 - specifications, technical
 - PDM 181
 - Pod 179
 - sports 176
 - status bar 9
 - status check, Pod 136
 - storing PDM
 - location 127
 - specifications 181
 - storing Pod
 - location 125
 - specifications 179
 - suggested bolus calculator. *See* Bolus Calculator
 - supplies
 - obtaining xiv
 - PDM set-up 22
 - Pod set-up 40
 - travel 166
 - surgery 176
 - suspend insulin delivery 72, 143
 - swimming 126
 - switching Basal Programme 72
 - symbols on labels 183
 - symptoms
 - DKA 174
 - hyperglycaemia 171
 - hypoglycaemia 168
 - syringe, fill 41
 - system error alarm 115
- ## T
- tags, BG 53
 - Target BG 106, 147–162
 - temp basal
 - activate 69
 - cancel 71
 - create 69
 - setting 105
 - set to zero 70, 143
 - understanding 140–143
 - temp basal preset
 - activate 71
 - create 79
 - delete 80

- edit 80
- rename 80
- Temp Basal tab 13
- temperature
 - insulin 40, 125
 - PDM storage 127, 181
 - Pod 40, 179
- text, entering 5
- time and date settings 95–96
- time-out, PDM screen 94
- touchscreen 4
 - brightness 94
 - sensitivity 4
 - time-out 94
- travel 166–167
- Troubleshooting, PDM start-up 177

U

- U-100 insulin xii
- unconfirmed bolus 91
- unlock PDM 10
- upper limit, BG Goal Range 104
- USB cable 128
- user-calculated bolus. *See* manually-calculated bolus
- user ID. *See* PIN

V

- vibration or sound
 - advisory alarms 112
 - hazard alarms 112
 - notifications 113

W

- wake up PDM 10
- water
 - and the PDM 127
 - and the Pod 126

X

- X-rays 167, 176

My PDM Settings

Use these pages to keep track of your important PDM settings. Remember to update your information if you change or add PDM settings.

[illegible][illegible][illegible][illegible][illegible]

[illegible]

Duration of Insulin Action

Time that insulin remains 'active' in the body after a bolus _____ hrs

[illegible][illegible]

My notes

[illegible]



Insulet Corporation
100 Nagog Park
Acton, MA 01720 USA
1-800-591-3455 | 1-978-600-7850
omnipod.com



Pod shown without the necessary adhesive.

UK
CA
0086



CE
2797



PT-000271