


PDM TIPS AND REMINDERS


These tips are intended for use only with PDM models ATT400, DET400, DET450, DET456, ENT450, FRT400, FRT456, ITT456 and NLT450. The PDM model number is found on the back of the PDM next to the REF symbol.




How to view insulin records.

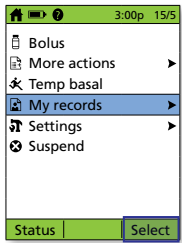

1. On the home screen, select **My records**.

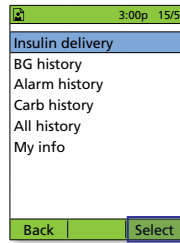

2. Select **Insulin delivery**.

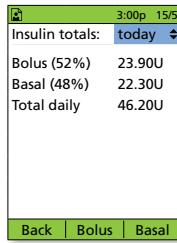


3. The PDM provides a summary of today's information including total boluses, total basal and total daily doses. Use the **Up/Down controller** buttons to view the summary for previous.


Important reminder

During the activation and priming of the Pod, the PDM and the Pod should be adjacent and in contact. Please refer to the Omnipod® Insulin Management System User Guide for a full set of instructions.


How to view multiple-day BG trends.

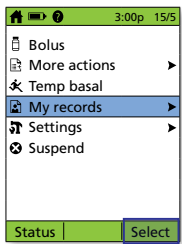


1. On the home screen, select **My records**.

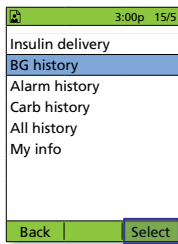

2. Select **BG history**.

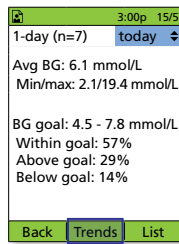


3. Press **Trends** to show the BG data for the past 7 days.

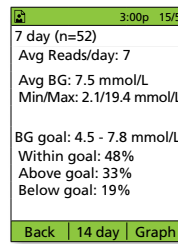


4. Continue pressing the **middle soft key** to show the trends for previous 14, 30, 60 and 90 days.

How to change existing basal rate.


NOTE: Insulin delivery must be suspended before changing basal rates.

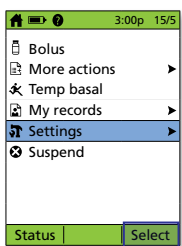


1. On the home screen, select **Settings**.

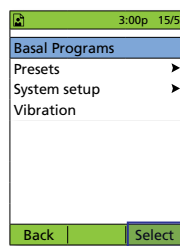


2. Select **Basal Programs**.

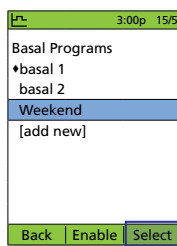


3. Select the **basal programme** to be edited. **On next screen**, select the segment to be edited and then, press **Edit**.

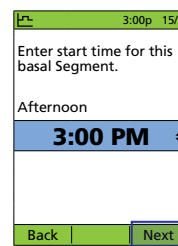


4. Enter the **start time** and then, press **Next** and repeat for **end time**.

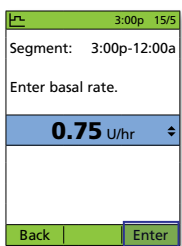

5. Enter the **basal rate** for the edited segment and then, press **Enter**.

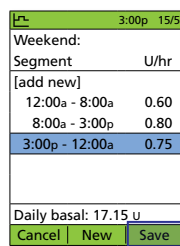

6. Press **Save**.

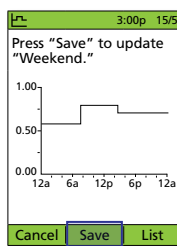


7. Press **Save**. For additional edits, repeat steps 3-8.


The values shown here are for illustrative purposes only. Actual screens may vary based on user settings and country. Consult with your healthcare professional before using these advanced features. Your healthcare professional can also provide you with your own personalised recommendations.

FreeStyle and related brand marks are trademarks of Abbott Diabetes Care Inc. in various jurisdictions and used with permission.


How to change correction factors.

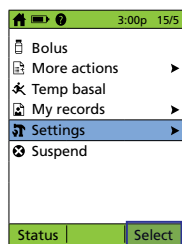


1. On the home screen, select **Settings**.

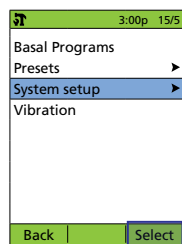


2. Select **System setup**.

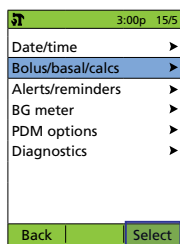


3. Select **Bolus/basal/calcs**.

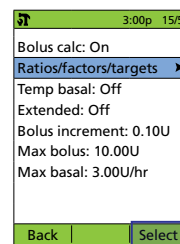


4. Select **Ratios/factors/targets**. **NOTE: Bolus calcs must be 'on'.**

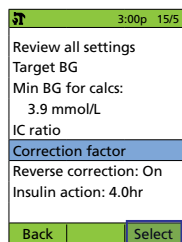

5. Select **Correction factor**.

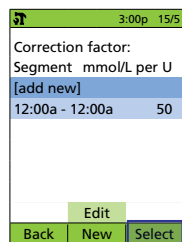


6. Select **add new** or choose **Segment** and hit **Edit**.
On the next screen, enter the **start time** and then, press **Next** and repeat for **end time**.

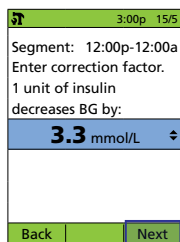


7. Enter the **correction factor** and then, press **Next**.

8. Repeat steps 6-7 to add or edit segments (up to 8 total segments). Press **Done** and then, **Save** when you are finished modifying the time segments.


How to change IC ratio or insulin action (duration).

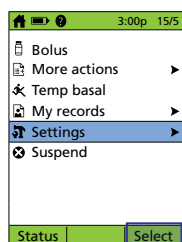


1. On the home screen, select **Settings**.

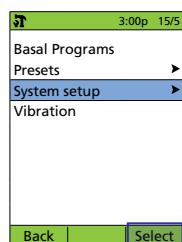

2. Select **System setup**.

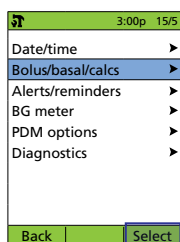


3. Select **Bolus/basal/calcs**.

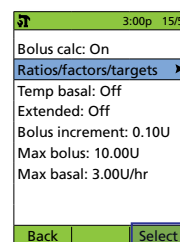


4. Select **Ratios/factors/targets**. **NOTE: Bolus calcs must be 'on'.**


To change IC ratio.

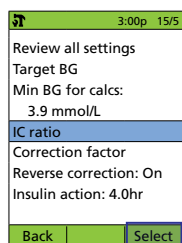

A. Select **IC ratio**.

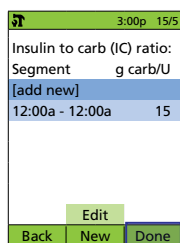


B. Select **add new** or choose **Segment** and hit **Edit**.
On the next screen, enter the **start time** and then, press **Next** and repeat for **end time**.

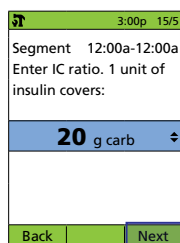


C. Enter **IC ratio** and then, press **Next**.
Press **Done** and then, **Save** when you are finished modifying the time segments.

To change insulin action.

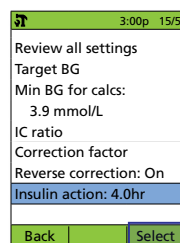

A. Select **Insulin action**.

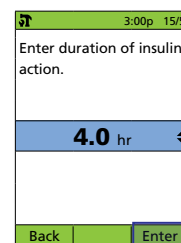

B. Use the **Up/Down controller** buttons to change the **duration of insulin action** and then, press **Enter**.