

# Omnipod® 5 System for iPhone

## TestFlight Instructions for Android/Controller Users



Thank you for taking part in the Pilot Program to use a Dexcom G7 Sensor with the Omnipod 5 App for iPhone. At present, the Omnipod 5 iPhone App is only available to use with Dexcom G6. To use Dexcom G7, you need to download the Omnipod 5 App using a unique method, which involves the TestFlight app. Once the Omnipod 5 App has been downloaded, you will need to go through first-time setup again. This means that you will lose your data and restart adaptivity.




### What is TestFlight?

Think of TestFlight as an early access version of the Apple App Store. It's a platform for downloading apps that are not yet publicly available, and it was created by Apple for this purpose.

**Note:** While TestFlight will work on iOS 14.0 and above, the **Omnipod 5 App requires iOS 17.2**. Update your phone to iOS 17.2 prior to downloading the Omnipod 5 App for iPhone.

## Downloading TestFlight


**Note:** Before you download the new Omnipod 5 App in TestFlight, make sure you **remove any previous TestFlight versions of the Omnipod 5 App** on your phone indicated by the yellow dot next to the name. **Do NOT delete the App Store version of the Omnipod 5 App for iPhone!**

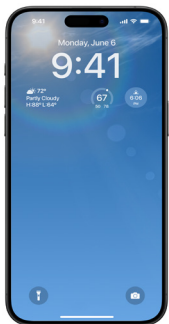


For the following steps, you **must use the iPhone** you plan to use with the Omnipod 5 App with!




You'll receive a personalized TestFlight invitation via email.

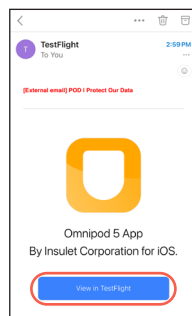


In the email, tap **View in TestFlight**. Your device's browser will open.

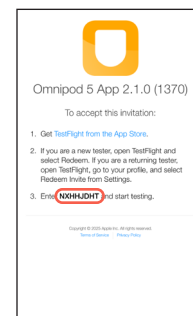


**Write down the Redeem Code.** You'll need to enter it later.

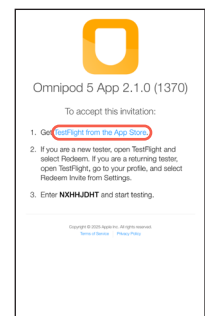


**Tap Get TestFlight from the App Store.**

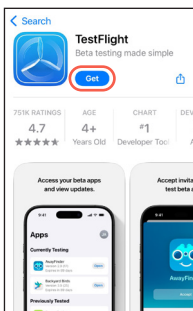


You'll be redirected to the Apple App Store. Tap the **download icon**.

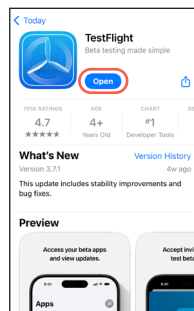

Once TestFlight has finished downloading, tap **Open**.

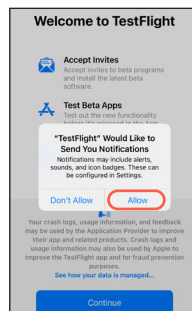

You'll be prompted to allow notifications. We recommend enabling them. Tap **Allow**.

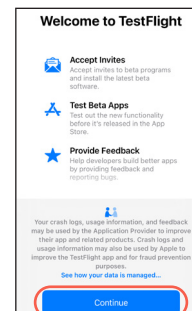


Carefully read TestFlight's terms and conditions. You must accept them to use the Omnipod 5 App. Tap **Continue**.

# Omnipod® 5 System for iPhone

## TestFlight Instructions for Android/Controller Users




### Redeeming the invitation and installing the Omnipod 5 App for iPhone

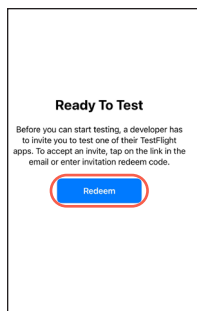


After you've accepted Testflight's Terms and Conditions, you will see this screen. Tap **Redeem**.

If you have tested an app on TestFlight before, you will need to tap on your profile in the top right corner and tap **Redeem Invite**.

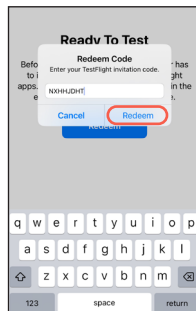


Enter the **Redeem Code** you wrote down earlier.


Tap **Redeem**.

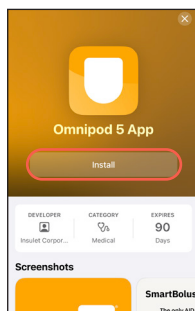

Tap **Install**.

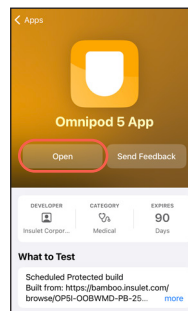


Once the Omnipod 5 App for iPhone has finished installing, tap **Open**.

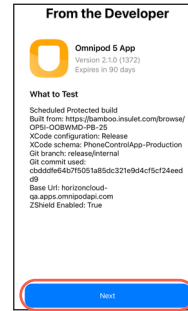

If prompted to allow Bluetooth, Notifications, and Critical Alerts tap **Allow**.

Then tap **Next**. You will now go through first-time setup to start insulin therapy.

### Updating the Omnipod 5 App for iPhone during the Pilot Program

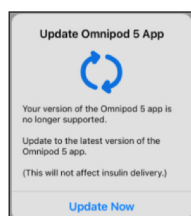

If the Omnipod 5 App for iPhone needs to be updated, you'll receive a notification to **Update Now**.

It is important that you use TestFlight to do the update. **Avoid uninstalling and reinstalling the app.**

Uninstalling the app will result in the loss of your settings, and you will have to complete the first-time setup again!

**For additional assistance,  
contact Product Support at 1-800-591-3455 Option 1.**