




Get Started Guide



Thank you for choosing the Omnipod® 5 Automated Insulin Delivery System, integrated with the leading sensor brands.*

Kickstart your journey with our
Get Started Guide for Omnipod 5.

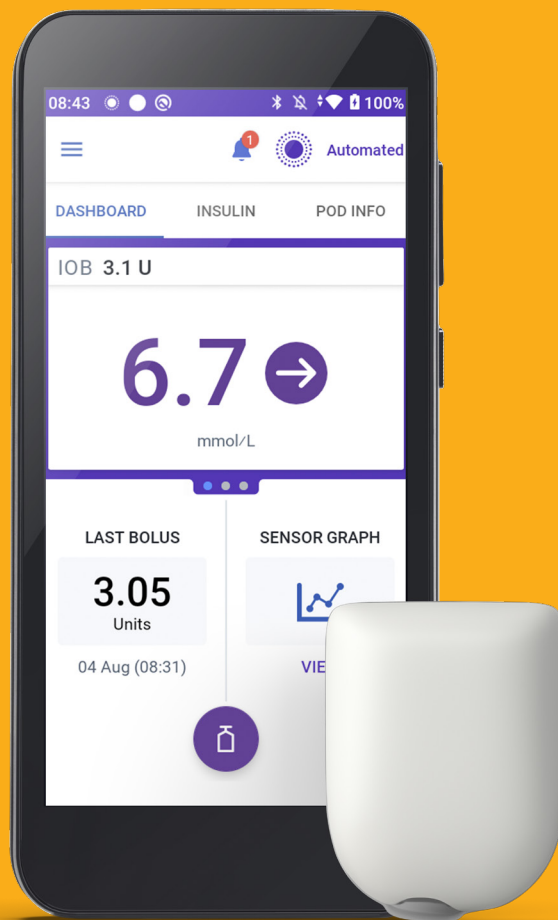

Pod shown without necessary adhesive.

*Compatible with the Dexcom G6, Dexcom G7 and FreeStyle Libre 2 Plus sensors. Each sold separately and requires separate prescription.

Omnipod 5 Onboarding

 **Before you get started on Omnipod 5, you must complete your Omnipod 5 onboarding online before your Omnipod 5 product training.**

Create an Omnipod ID and complete the consent screens, where we provide you with information about how your personal data is processed.

When you activate the Controller for the first time, you must enter your Omnipod ID and password.

Step 1 - Creating an Omnipod® ID

After your order is processed by Insulet, you will receive a “Complete Your Omnipod® 5 Onboarding Now” email. Open the email and select **Start Omnipod® 5 Onboarding** and log in with your or your dependent’s existing Omnipod ID.

If you did not receive an email:

1. Go to **www.omnipod.com/setup** or scan this QR code:
2. Select your country.



If you *do not* have an Omnipod ID

3a. Select Create Omnipod® ID.

- i. Fill in the form with your information, or the or the details of the dependant if you are acting as a parent or legal guardian. You will receive an email from Insulet to complete setting up your account.
- ii. Open the “Omnipod® ID set up nearly complete” email. Make sure you check your Junk or Spam folder if you do not see the email.
- iii. Select Set Up Omnipod® ID in the email. The link is valid for 24 hours.
- iv. Follow the on-screen instructions to review your information and set up your ID and password.
- v. Follow the on-screen instructions to set up two-factor authentication by email (required) or SMS text message (optional).
- vi. Enter the verification code sent by email or SMS text message to complete account set up.
- vii. Log in with your new Omnipod ID and password.
- viii. Follow the on-screen instructions to verify your account if logging in from a different device.

OR

If you *already* have an Omnipod ID

3b. Log in with your Omnipod ID and password.

Start Omnipod® 5 Onboarding

Sign in or create an Omnipod® ID for the person wearing the Pod.

Pod Wearer's Omnipod® ID*

Password*

Sign In

Forgot your Omnipod® ID or Password?

Don't have an Omnipod® ID yet?

Create Omnipod® ID

Create an Omnipod® ID to start Omnipod® 5 onboarding.

If you're a legal guardian setting up for your dependant, you must create an Omnipod® ID for your dependant.

ur Omnipod® ID

Check your email

We just sent instructions to set up your Omnipod® ID. If you don't see an email from us, please check your spam and junk folders.

The link expires in 24 hours, so be sure to use it soon.

OK

I agree to the Insulet Corporation's Terms of Use

Information below on the creation of my Omnipod® ID and consent to the processing of personal data, which may include health data. Such processing is necessary for the setup, use, and maintenance of my Omnipod® ID as described in the Insulet Privacy Policy both for myself, and if applicable, for my dependent.

As a customer, and, if applicable, their legal guardians, the option to create an Omnipod® ID is optional. We will verify in our systems whether you are an Insulet customer, or you are a legal guardian of a minor. You can use the Omnipod® ID to create an Omnipod® account, an online environment in which you can for yourself and, if applicable, for your dependent, manage privacy settings, arrange orders, and undertake certain product

Account verification

☒ Email

☐ Text Message (SMS)

Confirm

Parents and Legal Guardians

Make sure you create the Omnipod ID on behalf of the customer in your care. Select *I am a legal guardian for a dependent who will wear Omnipod® 5* at the top of the Create Omnipod® ID form.



The Omnipod ID:

- should be unique
- should be at least 6 characters long
- should not contain special characters (e.g. !#£%&*~@)
- should not contain blank spaces

Passwords:

- should be at least 8 characters long
- should include upper case, lower case, and number.
- should not include your (or the customer's) first name, last name, or Omnipod ID
- should only contain the following special characters (!#\$%+~<>@_)

Step 2 - Reading and validating Data Privacy consent

At Insulet, the safety and security of our users and products is paramount in everything we do. We are dedicated to making the lives of people with diabetes easier and simplifying diabetes management. Insulet respects the privacy of every one of our customers and is committed to the protection of their personal information. We have dedicated teams that are focused on keeping customer information safe from unauthorized access.

After setting up your account, you must review and consent to the following data privacy policies:

1. **Omnipod 5 Terms & Conditions** – Required.
2. **Omnipod 5 Consents** – Each type of consent must be agreed to individually:
 - **Product Use** – Required.
 - **Data Privacy Introduction** – Required.
 - **Product Research, Development and Improvement** – Optional
Select **Skip and Continue** to opt-out.
If you select **Agree and Continue**, a few optional questions display.

Step 3 - Linking your Omnipod account with a Glooko® account

Glooko is the Omnipod 5 data management platform that enables you to:

- See your glucose and insulin data.
- Share your data with your healthcare provider to support informed system adjustments.
- We recommend that you link your Omnipod ID to your Glooko account. If you do not have a Glooko account you can create one during setup by following these steps.
- Ask your healthcare provider for their clinic's ProConnect code to share your diabetes data.


ProConnect Code:



Link a Glooko account

After consenting to the data policies, the Omnipod 5 website prompts you to link your Glooko account.

1. Select **Link** on Omnipod 5.
2. Select **Continue** to allow Omnipod 5 to send you to Glooko to log in or create an account.
3. Within Glooko:
 - Select **Sign Up for Glooko** if you or the customer do not already have a Glooko account. Follow the on-screen instructions to create a Glooko account.
 - Select **Log In** if you or the customer already have a Glooko account.

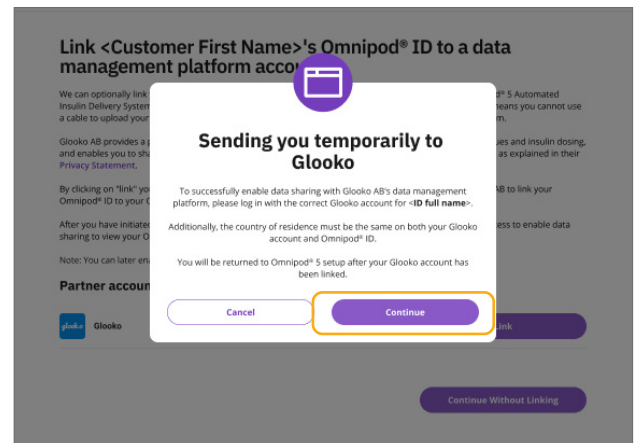


Share Glooko data with your healthcare provider

After you create an account and log in, Glooko prompts you to share your Omnipod 5 data with your medical team, which allows for better follow-up.

1. In the Glooko app, enter the **ProConnect Code** your healthcare provider provided.
2. Select **Share Data**.
3. Select the **You share data with Insulet** checkbox.
4. Select **Continue**. You have completed setting up Glooko, but you must return to Omnipod 5 to finish sharing your data.
5. Select **Return to Omnipod 5**.
6. Select **Agree** on the **Data Sharing with Glooko** consent.
7. Select **Continue**.

Omnipod 5 sends you a confirmation email that your onboarding is complete. Once you begin using the Omnipod 5 system, Omnipod 5 will share your data with your healthcare provider through Glooko.

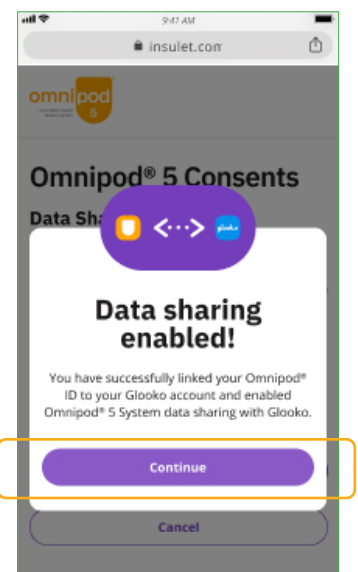

Congratulations on completing Omnipod® 5 onboarding.

Prepare for your Training Day

In preparation for starting on Omnipod 5 please follow guidance from your healthcare provider regarding any changes to current therapy, including any insulin therapy adjustment. You must be trained by your healthcare provider and/or Insulet Clinical team before you start on Omnipod 5.

Omnipod 5 Starter Kit

- 👉 **If you are receiving your training at home,** we will send you the Omnipod 5 Starter kit and box(es) of Omnipod 5 Pods. You will also need a vial of the rapid acting insulin[†] prescribed by your healthcare provider.

OR

- 👉 **If you are being trained in the hospital,** your Omnipod 5 Starter kit and box(es) of Omnipod 5 Pods will be there. Remember to take a vial of the rapid acting insulin[†] if you are using this already.

If you are expecting delivery of your Omnipod 5 Starter kit and Pods, and have not received these within 3 days of your scheduled training, please contact Customer Operations on **1800 954 075**.



Sensors

Dexcom sensor

- 👉 Please come to training wearing an active Dexcom G6 or Dexcom G7 sensor using the Dexcom app on a compatible smartphone and ensure your Dexcom receiver is switched off.*

FreeStyle Libre 2 Plus sensor

- 👉 Please ensure your healthcare provider has provided you with a prescription for FreeStyle Libre 2 Plus sensors.
- 👉 If you are currently using a FreeStyle Libre sensor, continue to wear this sensor when you attend your Omnipod 5 training.
- 👉 Please bring a new, unopened FreeStyle Libre 2 Plus sensor with you to the Omnipod 5 training.*

Insulin

- 👉 Remember to bring a vial of rapid-acting insulin[†] to your training.

*The FreeStyle Libre 2 Plus and the Dexcom G6 and G7 sensors are prescribed separately. The Dexcom G6 and G7 sensors must be used with the Dexcom mobile app. The Dexcom G6 and G7 receiver is not compatible.

[†]NovoLog[®]/NovoRapid[®], Humalog[®], Trurapi[®]/Truvelog/Insulin aspart Sanofi[®], Kirsty[®], and Admelog[®]/Insulin lispro Sanofi[®] are compatible with the Omnipod 5 System for use up to 72 hours (3 days).

Training Day Checklist

Checklist

- ☐ Have you created your Omnipod ID and password? It is important you remember your Omnipod ID and password as you will use this to log into the Omnipod 5 controller during your training.
- ☐ Have you completed your onboarding?
 - ☐ Have you accepted all mandatory consent where we provide you with information on the processing of your personal data?
 - ☐ (Optional) Did you complete linking your or your dependent's Omnipod ID with the Glooko account?
 - ☐ Did you see the 'Onboarding completed!' screen and did you get the confirmation email?
- ☐ Do you have a vial of rapid-acting insulin[†] for your training?
- ☐ Are you wearing an active Dexcom sensor using the Dexcom app on a compatible smartphone and ensured your Dexcom receiver is switched off?

OR

- ☐ Do you have a FreeStyle Libre 2 Plus unopened sensor ready to be activated at your training?

Omnipod ID

Omnipod ID:

Password:

Glooko account

Email (username):

Password:

Dexcom/ FreeStyle Libre 2 plus User ID


Username/email address:

Password:

ProConnect Code:



Additional Resources

 To be fully prepared for your Omnipod 5 training we encourage you to watch the '**How to Videos**' prior to your product training.



If you have any questions or concerns regarding Omnipod 5 not answered by the online resources, please contact the Omnipod team on:

1800 954 075*

Pod shown without necessary adhesive.

*Your call may be recorded for quality monitoring, and training purposes. Calls to 1800 numbers are free from local landlines but other networks or network carriers may charge for these calls.





**If you have any questions about your treatment,
please contact your diabetes team.**

Always read the label and follow the directions for use.

Omnipod 5 Automated Insulin Delivery System Important Safety Information.

The Omnipod 5 Automated Insulin Delivery System is indicated for use by individuals with Type 1 diabetes mellitus in persons 2 years of age and older. The Omnipod 5 System is intended for single patient, home use and requires supervision of the user's healthcare provider. Refer to the Omnipod® 5 Automated Insulin Delivery System User Guide and www.omnipod.com/safety for complete safety information including indications, contraindications, warnings, cautions, and instructions.

Warning: DO NOT start to use the Omnipod 5 System or change settings without adequate training and guidance from a healthcare provider. Initiating and adjusting settings incorrectly can result in over-delivery or under-delivery of insulin, which could lead to hypoglycaemia or hyperglycaemia.

©2025 Insulet Corporation. Omnipod, the Omnipod logo, Simplify Life and Podder are trademarks or registered trademarks of Insulet Corporation. All rights reserved. Dexcom, Dexcom G6, and Dexcom G7 are registered or unregistered trademarks of Dexcom, Inc. in the United States and/or other countries and are used with permission. The sensor housing, FreeStyle, Libre, and related brand marks are marks of Abbott and used with permission. Glooko is a trademark of Glooko, Inc. and used with permission. All other trademarks are the property of their respective owners. The use of third party trademarks does not constitute an endorsement of imply a relationship or other affiliation. Insulet Australia Pty Ltd, Level 16, Tower 2, Darling Park, 201 Sussex Street, Sydney, NSW, 2000. INS-OHS-02-2025-00373 V1.0