

# MORE THAN 80% TIME IN RANGE WITH OPTIMISED SETTINGS<sup>1</sup>



Of real-world Omnipod® 5 users with time in range (TIR) below 70%

**53%** do not frequently use the lowest Target Glucose setting<sup>2</sup>

## Lowering Target Glucose significantly improved TIR<sup>2</sup>




Real-world users showed a 12% time in range difference between the lowest and highest Target Glucose setting<sup>3</sup>

This was achieved with no clinically meaningful impact on time below range. <sup>3</sup>

### Target Glucose

8.3 mmol/L vs. 6.1 mmol/L

I:C Ratio: 1:9 | Correction Factor: 1.7 mmol/L

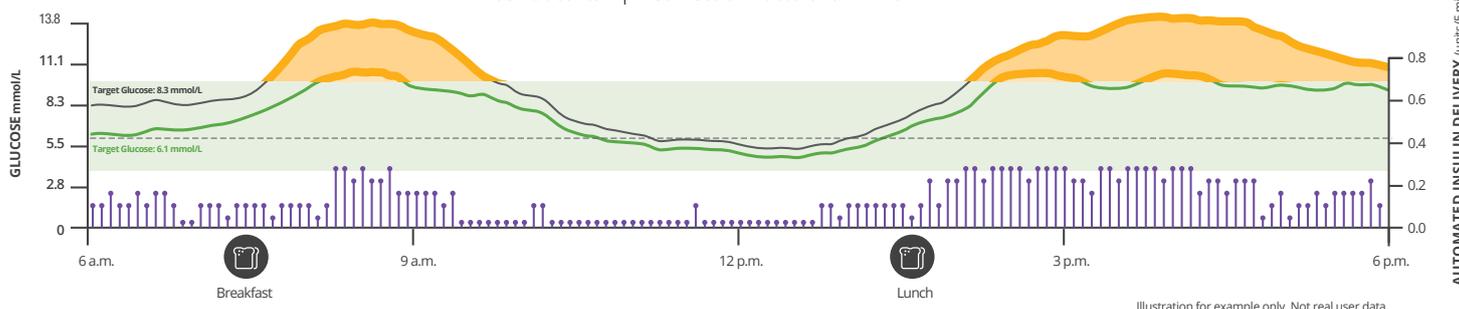

## Optimise settings



Target Glucose Setting: 6.1 mmol/L<sup>1</sup>




Insulin-to-carb ratio: I:C Ratio x TDI  $\leq$  350<sup>1</sup>



Correction factor: ISF x TDI  $\leq$  83<sup>1</sup>

1. Retrospective RWE data on file. 2025. Results shown for users with optimised settings, including sufficient CGM data ( $\geq 75\%$  of days with  $\geq 220$  readings),  $\geq 90\%$  time in Automated Mode,  $\geq 5$  bolus/day and an average Target Glucose of 6.1-6.4 mmol/L. Optimised settings: ISF x TDI = 83 mmol/L, I:C Ratio x TDI  $\leq 350$ . RF-062025-00014.

# Omnipod® 5 autocorrects for highs while helping to protect from lows.<sup>4,5</sup>

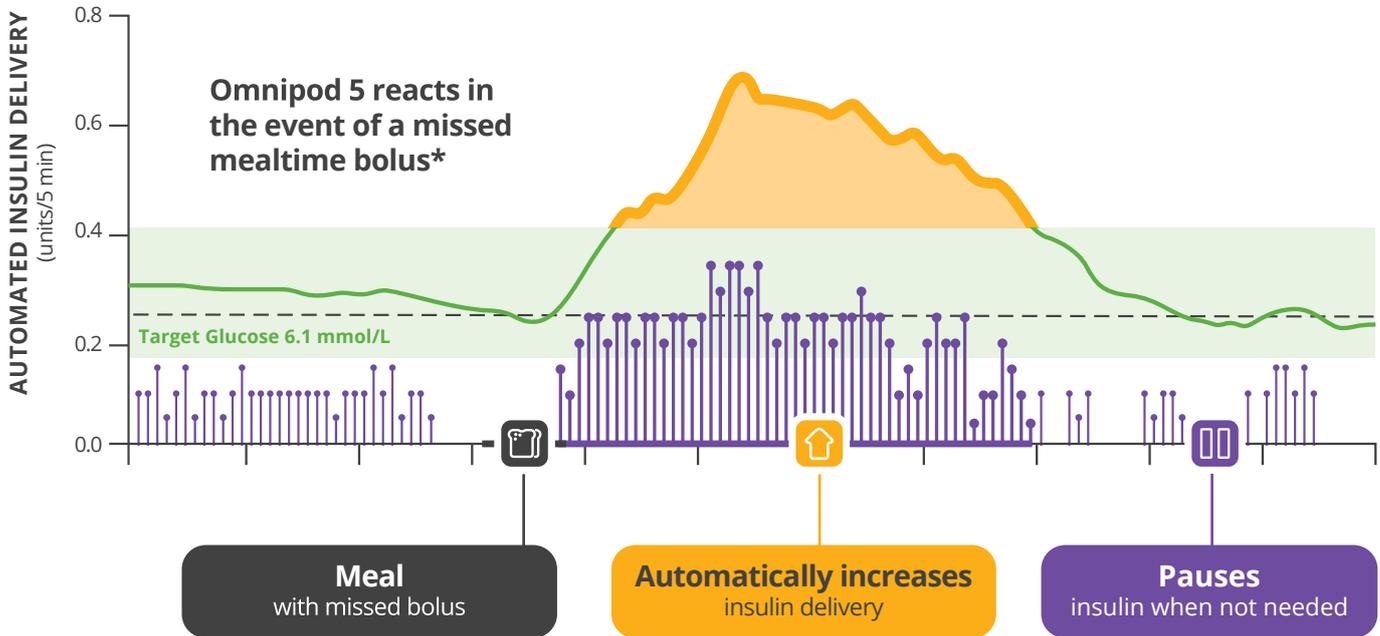

Refer to Cari Berget's "Practical considerations for using the Omnipod® 5 Automated Insulin Delivery System: Clinical experience from the United States and Europe" for tips on optimisation.



“Most people with T1D will benefit from AID therapy, and the **Omnipod 5 System has been found safe and effective in diverse clinical trial cohorts**, such as those transitioning from multiple daily injection (MDI) therapy and those with high baseline HbA1c, including those who may not count carbohydrates or bolus consistently.”  
—Berget C, et al.

\*Bolusing with the Omnipod 5 System is recommended for meals

References:  
1. Retrospective RWE data on file. 2025. Results shown for users with optimised settings including sufficient CGM data (≥75% of days with ≥220 readings), ≥90% time in Automated Mode, ≥5 bolus/day and an average Target Glucose of 6.1-6.4 mmol/L. Optimised settings: ISF x TDI ≤1500, I:C Ratio x TDI ≤350. RF-062025-00014. 2. Forlenza G, et al. Presented at: ATTD; March 19-22, 2025; Amsterdam, Netherlands. Impact of Lowering Target Glucose Setting with the Omnipod 5 Automated Insulin Delivery System: Real-world data from 403 type 1 diabetes (T1D) users who transitioned from the 8.3 mmol/L to 6.1 mmol/L Target Glucose. Omnipod 5 results based on users with ≥90 days CGM data and ≥75% of days with ≥220 readings available. Insulet Data on File. 06.27.25. RF-062025-00038. 3. Forlenza G, et al. Presented at: ATTD; March 19-22, 2025; Amsterdam, NL. Real-world data from 403 people with type 1 diabetes aged 2+ using the Omnipod 5 System who transitioned from the 8.3 mmol/L to 6.1 mmol/L Target Glucose. Each Target Glucose was used for a consecutive period of 14-90 days. Median time in Range (3.8-10 mmol/L) improved 11.8% (p<0.05). Median time (<-3.8 mmol/L) +0.23% (p<0.05). Omnipod 5 results based on users with ≥75% of days with ≥220 readings available. Insulet Data on file. 05.15.25. RF-042025-00013. 4. Brown S, et al. Diabetes Care. 2021;44:1630-1640. Prospective pivotal trial in 240 participants with T1D aged 6 - 70 yrs [adults/adolescents (n= 128; aged 14-70 yrs) children (n=112; aged 6-13.9 yrs)]. Study included a 14-day standard therapy (ST) phase followed by a 3-month Omnipod 5 hybrid closed-loop phase. Mean time >10 mmol/L in adults/adolescents and children, ST vs. 3-mo Omnipod 5: 32.4% vs. 24.7%; 45.3% vs. 30.2%, P<0.0001, respectively. Mean time < 3.9 mmol/L in adults/adolescents and children, ST vs. 3-mo Omnipod 5: 2.9% vs. 1.3%, P<0.0001; 2.2% vs. 1.8%, P=0.8153, respectively. Results measured by CGM. Study funded by Insulet. 5. Sherr JL, et al. Diabetes Care. 2022. 45(8):1907-1910. Prospective trial in 80 participants with T1D aged 2 - 5.9 yrs. Study included a 14-day standard therapy (ST) phase followed by a 3-month Omnipod 5 hybrid closed-loop (HCL) phase. Mean time >10 mmol/L in very young children (2 - 5.9yrs) as measured by CGM: ST

Always read the label and follow the directions for use.

Compatible glucose sensors sold separately and require a separate prescription.

Omnipod® 5 Automated Insulin Delivery System Important Safety Information.

The Omnipod® 5 Automated Insulin Delivery System is indicated for use by individuals with Type 1 diabetes mellitus in persons 2 years of age and older. The Omnipod® 5 System is intended for single patient, home use and requires supervision of the user's healthcare provider. The Omnipod 5 System is compatible with the following U-100 insulins: NovoLog®/ NovoRapid®, Humalog®, Trurapi®/Insulin aspart Sanofi®, Kirsty®, and Admelog®/Insulin lispro Sanofi®. Refer to the Omnipod® 5 Automated Insulin Delivery System User Guide and [www.omnipod.com/safety](http://www.omnipod.com/safety) for complete safety information including indications, contraindications, warnings, cautions, and instructions.

Warning: DO NOT start to use the Omnipod® 5 System or change settings without adequate training and guidance from a healthcare provider. Initiating and adjusting settings incorrectly can result in over-delivery or under-delivery of insulin, which could lead to hypoglycaemia or hyperglycaemia.

©2025 Insulet Corporation. Insulet, Omnipod, the Omnipod logo, Podder, Simplify Life and SmartAdjust are trademarks or registered trademarks of Insulet Corporation, in the United States and other various jurisdictions. All rights reserved. All other trademarks are the property of their respective owners. The use of third party trademarks does not constitute an endorsement or imply a relationship or other affiliation. Insulet Australia Pty Ltd, Level 16, Tower 2, Darling Park, 201 Sussex Street, Sydney, NSW, 2000. INS-OHS-10-2025-00009 v1.0

