

Omnipod® 5 System for iPhone

TestFlight Instructions for iPhone Users



Thank you for taking part in the Pilot Program to use a Dexcom G7 Sensor with the Omnipod 5 App for iPhone. At present, the Omnipod 5 iPhone App is only available to use with Dexcom G6. To use Dexcom G7, you need to download the Omnipod 5 App using a unique method, which involves the TestFlight app. The new Omnipod 5 App will replace your current one, but you will still be able to keep all of your data and adaptivity. After you download the new App, you will need to switch your Sensor to Dexcom G7 and set up a new Pod.



What is TestFlight?

Think of TestFlight as an early access version of the Apple App Store. It's a platform for downloading apps that are not yet publicly available, and it was created by Apple for this purpose.

Note: While TestFlight will work on iOS 14.0 and above, the **Omnipod 5 App requires iOS 17.2**. Update your phone to iOS 17.2 prior to downloading the Omnipod 5 App for iPhone.

Downloading TestFlight


Note: Before you download the new Omnipod 5 App in TestFlight, make sure you **remove any previous TestFlight versions of the Omnipod 5 App** on your phone indicated by the yellow dot next to the name. If you have the **App Store version of the Omnipod 5 App**, do **NOT** delete it.




For the following steps, you **must use the iPhone** you plan to use with the Omnipod 5 App with!




You'll receive a personalized TestFlight invitation via email.

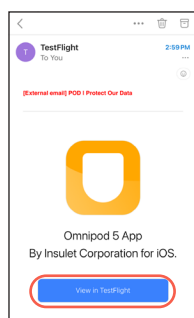


In the email, tap **View in TestFlight**. Your device's browser will open.

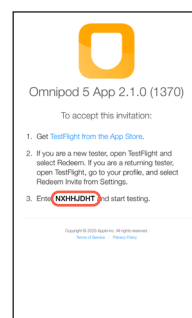


Write down the Redeem Code. You'll need to enter it later.

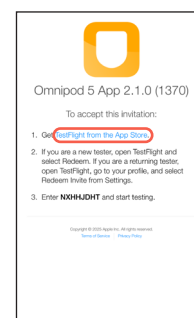


Tap **Get TestFlight from the App Store**.

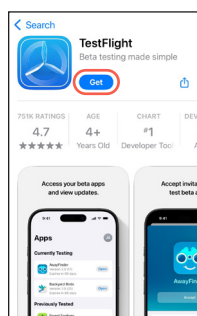


You'll be redirected to the Apple App Store. Tap **Get**.

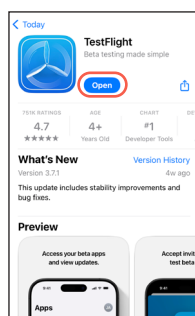

Once TestFlight has finished downloading, tap **Open**.

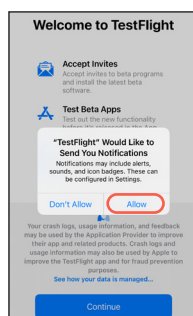

You'll be prompted to allow notifications. We recommend enabling them. Tap **Allow**.

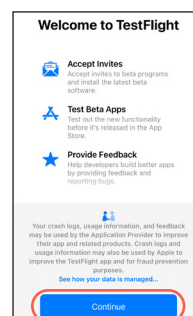


Carefully read TestFlight's terms and conditions. You must accept them to use the Omnipod 5 App. Tap **Continue**.

Omnipod® 5 System for iPhone

TestFlight Instructions for iPhone Users




Redeeming the invitation and updating the Omnipod 5 App for iPhone

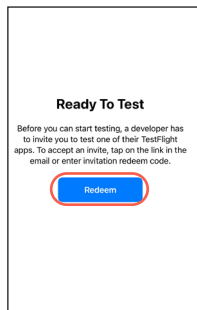


After you've accepted Testflight's Terms and Conditions, you will see this screen. Tap **Redeem**.

If you have tested an app on TestFlight before, you will need to tap on your profile in the top right corner and tap **Redeem Invite**.

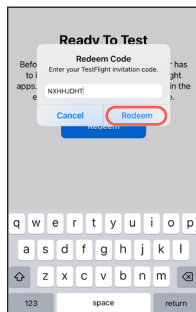



Enter the **Redeem Code** you wrote down earlier.

Tap **Redeem**.

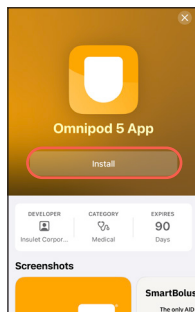

Tap **Install**.

You may receive this notification saying the App is already downloaded on your phone and you may lose data. Tap **Install**. **You will not lose your adaptivity!**

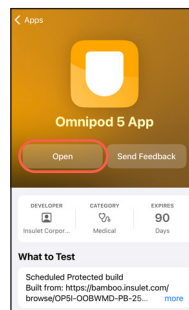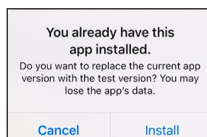

Once the Omnipod 5 App for iPhone has finished installing, tap **Open**.




If prompted to allow Bluetooth, Notifications, and Critical Alerts tap **Allow**.

Then tap **Next**. From the Home screen, tap **Menu** then **Manage Sensor**. Tap **Switch** and select Dexcom G7.

You will now need to **pair your Dexcom G7** and set up a new Pod.

Updating the Omnipod 5 App for iPhone during the Pilot Program

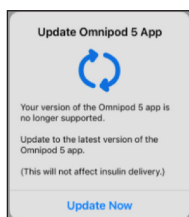

If the Omnipod 5 App for iPhone needs to be updated, you'll receive a notification to **Update Now**.

It is important that you use TestFlight to do the update. **Avoid uninstalling and reinstalling the app.**

Uninstalling the app will result in the loss of your settings, and you will have to complete the first-time setup again!

**For additional assistance,
contact Product Support at 1-800-591-3455 Option 1.**